# Aesculap® Acculan

|    |                                                                                 |
|----|---------------------------------------------------------------------------------|
| en | <b>Instructions for use/Technical description</b><br>Acculan Akku GA666, GA676  |
| de | <b>Gebrauchsanweisung/Technische Beschreibung</b><br>Acculan Akku GA666, GA676  |
| fr | <b>Mode d'emploi/Description technique</b><br>Acculan Akku GA666, GA676         |
| es | <b>Instrucciones de manejo/Descripción técnica</b><br>Acculan Akku GA666, GA676 |
| it | <b>Istruzioni per l'uso/Descrizione tecnica</b><br>Acculan Akku GA666, GA676    |
| pt | <b>Instruções de utilização/Descrição técnica</b><br>Acculan Akku GA666, GA676  |
| nl | <b>Gebruiksaanwijzing/Technische beschrijving</b><br>Acculan Akku GA666, GA676  |
| sv | <b>Bruksanvisning/Teknisk beskrivning</b><br>Acculan Akku GA666, GA676          |

|    |                                                                                   |
|----|-----------------------------------------------------------------------------------|
| fi | <b>Käyttöohje/Tekninen kuvaus</b><br>Acculan-akku GA666, GA676                    |
| et | <b>Kasutusjuhend/Tehniline kirjeldus</b><br>Acculan Aku GA666, GA676              |
| ru | <b>Инструкция по применению/Техническое описание</b><br>Acculan Akku GA666, GA676 |
| cs | <b>Návod k použití/Technický popis</b><br>Acculan Akku GA666, GA676               |
| pl | <b>Instrukcja użytkowania/Opis techniczny</b><br>Acculan Akku GA666, GA676        |
| sk | <b>Návod na použitie/Technický opis</b><br>Acculan Akku GA666, GA676              |
| tr | <b>Kullanım Kılavuzu/Teknik açıklama</b><br>Acculan Akku GA666, GA676             |

# Aesculap® Acculan

## Acculan Akku GA666, GA676

### Symbols on product and packages

|                                                                                    |                                                                                                                                                                                                                                                                                                             |
|------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|    | Caution, general warning symbol<br>Caution, see documentation supplied with the product                                                                                                                                                                                                                     |
|    | Follow the instructions for use                                                                                                                                                                                                                                                                             |
|    | Machine-readable, two-dimensional code<br>The code contains a unique serial number which can be used for electronic tracking of the individual instrument. The serial number is based on the global standard sGTIN (GS1).                                                                                   |
|    | Keep device away from fluids                                                                                                                                                                                                                                                                                |
|    | Serial number                                                                                                                                                                                                                                                                                               |
|  | Manufacturer and date of manufacture (year)                                                                                                                                                                                                                                                                 |
|  | Labelling of electrical and electronic devices according to the directive 2012/19/EU (WEEE), see Disposal                                                                                                                                                                                                   |
|  | The battery requires no maintenance. Since battery cells are subject to wear and ageing, Aesculap recommends replacing the battery cells every 12 months. See also stamp/label for the next recommended battery cell replacement date at the international B. Braun/Aesculap agency, see Technical Service. |
|  | Remove adhesive label before charging the battery for the first time                                                                                                                                                                                                                                        |
|  | Date of manufacture                                                                                                                                                                                                                                                                                         |
|  | Manufacturer's batch designation                                                                                                                                                                                                                                                                            |
|  | Manufacturer's article number                                                                                                                                                                                                                                                                               |
|  | Temperature limit<br>The temperature limit values to which the medical product can be safely subjected are designated.                                                                                                                                                                                      |

Humidity, limit  
Designates the humidity range to which the medical product can be safely subjected.

Air pressure, limit  
Designates the atmospheric pressure to which the medical product can be safely subjected.

### Contents

|      |                                                                                      |   |
|------|--------------------------------------------------------------------------------------|---|
| 1.   | Scope .....                                                                          | 2 |
| 2.   | General information .....                                                            | 3 |
| 2.1  | Use/intended use: .....                                                              | 3 |
| 2.2  | Main functions and design characteristics .....                                      | 3 |
| 2.3  | Indications .....                                                                    | 3 |
| 2.4  | Contraindications .....                                                              | 3 |
| 3.   | Safe handling .....                                                                  | 3 |
| 4.   | Product description .....                                                            | 3 |
| 4.1  | Scope of Supply .....                                                                | 3 |
| 4.2  | Components required to operate the device .....                                      | 3 |
| 4.3  | Operating principle .....                                                            | 4 |
| 5.   | Preparation .....                                                                    | 4 |
| 6.   | Working with the device .....                                                        | 4 |
| 6.1  | System set-up .....                                                                  | 4 |
| 6.2  | Functional test .....                                                                | 4 |
| 6.3  | Safe operation .....                                                                 | 4 |
| 7.   | Validated reprocessing procedure .....                                               | 4 |
| 7.1  | General safety information .....                                                     | 4 |
| 7.2  | General notes .....                                                                  | 4 |
| 7.3  | Preparing for cleaning .....                                                         | 4 |
| 7.4  | Cleaning/Disinfection .....                                                          | 4 |
| 7.5  | Wipe disinfection for electrical devices without sterilization ..                    | 5 |
| 7.6  | Inspection, maintenance and checks .....                                             | 5 |
| 8.   | Maintenance .....                                                                    | 6 |
| 9.   | Troubleshooting .....                                                                | 6 |
| 10.  | Storage and transport .....                                                          | 6 |
| 11.  | Accessories/Replacement Parts .....                                                  | 6 |
| 12.  | Technical Service .....                                                              | 6 |
| 13.  | Technical Data .....                                                                 | 6 |
| 13.1 | Classification according to directive 93/42/EWG .....                                | 6 |
| 13.2 | Performance data, information and standards .....                                    | 6 |
| 13.3 | Ambient conditions .....                                                             | 7 |
| 14.  | Disposal .....                                                                       | 7 |
| 15.  | Distributor in the US/Contact in Canada for product information and complaints ..... | 7 |

## 1. Scope

For item-specific instructions for use and information regarding material compatibility, see also the Aesculap Extranet at <https://extranet.bbraun.com>

## 2. General information

### 2.1 Use/intended use:

#### Task/Function

The Li-Ion batteries serve to supply energy to the Acculan machines and include the activation and control of the electric motors of the Acculan machines according to user commands.

#### Application Environment

The product fulfills the requirements of Type BF in accordance with IEC/DIN EN 60601-1.

For use in operating rooms, outside of the explosive hazard zone (e.g. areas with highly pure oxygen or anesthetic gases).

### 2.2 Main functions and design characteristics

| Art. no. | DC voltage | Capacity |
|----------|------------|----------|
| GA666    | 9.6 V      | 1.05 Ah  |
| GA676    | 9.6 V      | 1.95 Ah  |

### 2.3 Indications

The indications can be found in the instructions for use for the corresponding Acculan machines.

### 2.4 Contraindications

The contraindications can be found in the instructions for use for the corresponding Acculan machines.

## 3. Safe handling

#### CAUTION

Federal law restricts this device to sale by, or on order of a physician!

**WARNING**

**Risk of injury and damage to property due to improper handling of the product!**

- ▶ Comply with instructions for use for the Acculan 4 machines and the Acculan 3Ti machines.
- ▶ Follow the instructions for use of all products used.

**WARNING**

**Fire and explosion hazard due to battery poles shorted by fluids or metal components!**

- ▶ Do not short-circuit, open, crush or incinerate battery.

**CAUTION**

**Damage to or destruction of the batteries due to reprocessing!**

- ▶ Do not sterilize batteries.

- ▶ Inspect the new product after removing its transport packaging and prior to first use to ensure it is in good working order; charge with charger GA320 or GA677 and verify functionality.
- ▶ To prevent damage caused by improper setup or operation, and to not compromise the manufacturer warranty and liability:
  - Use the product according to these instructions for use.
  - Follow the safety and maintenance instructions.
  - Only combine Aesculap products with each other.
  - Follow the application advisories acc. to standard, see Extracts from relevant standards.

- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- ▶ Keep the instructions for use accessible for the user.
- ▶ Always adhere to applicable standards.
- ▶ Do not use the product if it is damaged or defective.
- ▶ Set aside the product if it is damaged.
- ▶ Store product at room temperature (Maximum recommended duration of storage is 3 months; keep out of direct sunlight).
- ▶ Keep product dry.
- ▶ Observe the instructions for use for the corresponding Acculan machines.
- ▶ Use battery regularly and always store in the charger.

## 4. Product description

### 4.1 Scope of Supply

| Art. no. | Designation                                        |
|----------|----------------------------------------------------|
| GA666    | Acculan NiMH battery short<br>– or –               |
| GA676    | Acculan NiMH battery                               |
| TA011867 | Instructions for use, Acculan battery GA666, GA676 |

### 4.2 Components required to operate the device

- Acculan 4 charger GA320 or Acculan 3Ti charger GA677
- Acculan 4 or Acculan 3Ti machine:

| Art. no. | Designation                          |
|----------|--------------------------------------|
| GA330    | Acculan 4 Drill and reamer           |
| GA331    | Acculan 4 Oscillating saw            |
| GA334    | Acculan 4 Reciprocating saw          |
| GA336    | Acculan 4 Oscillating saw straight   |
| GA337    | Acculan 4 Oscillating sternum saw    |
| GA340    | Acculan 4 Dermatome                  |
| GA341    | Acculan 4 Dermatome 0.1 mm           |
| GA344    | Acculan 4 Small drill                |
| GA668    | Acculan 3Ti Oscillating saw straight |
| GA670    | Acculan 3Ti Dermatome                |
| GA671    | Acculan 3Ti Small drill              |
| GA672    | Acculan 3Ti Drill and reamer         |
| GA673    | Acculan 3Ti Oscillating saw          |
| GA674    | Acculan 3Ti Reciprocating saw        |
| GT666    | Acculan 4 TPLO saw                   |
| GT668    | Acculan 3Ti TPLO saw                 |

### 4.3 Operating principle

The Acculan NiMH battery supplies the Acculan machine with voltage. The battery is charged in the Acculan 4 charger GA320 or Acculan 3Ti charger GA677. The charging process of a completely discharged battery can take up to 120 min.

The speed and rotational direction are set depending on the trigger setting of the machines:

- for saws and Dermatome: speed clockwise
- for drills: speed clockwise, counter-clockwise, oscillation and pilgrim step

## 5. Preparation

- Prior to use, inspect the product and its accessories for any visible damage.
- Use the products and their accessories only if they are in perfect technical condition.

## 6. Working with the device

### 6.1 System set-up

- See instructions for use for the respective Acculan machine and the Acculan 4 charger GA320 or Acculan 3Ti charger GA677.

### 6.2 Functional test

See the instructions for use for the respective Acculan machine.

### 6.3 Safe operation

**WARNING**

**Risk of injury and material damage due to inappropriate use of batteries!**

- Charge battery only in the Acculan 4 charger GA320 or Acculan 3Ti charger GA677.
- Insert battery correctly (Acculan machine and charger).

**WARNING**

**Risk of injury and material damage due to damaged batteries!**

- Check batteries for any damage before charging.
- Do not use or recharge damaged batteries.

- For proper loading and removal of the battery from the Acculan machine, see the instructions for use for the respective Acculan machine.

## 7. Validated reprocessing procedure

### 7.1 General safety information

#### Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for sterile processing.

#### Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

#### Note

Successful processing of this medical device can only be ensured if the processing method is first validated. The operator/sterile processing technician is responsible for this.

The recommended chemistry was used for validation.

#### Note

If there is no final sterilization, then a virucidal disinfectant must be used.

#### Note

For the latest information on reprocessing and material compatibility see also the Aesculap extranet at <https://extranet.bb Braun.com>

### 7.2 General notes

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Optical changes of materials, e.g. fading or discoloration of titanium or aluminum. For aluminum, the application/process solution only needs to be of pH >8 to cause visible surface changes.
- Material damage such as corrosion, cracks, fracturing, premature aging or swelling.
- Do not use metal cleaning brushes or other abrasives that would damage the product surface and could cause corrosion.

### 7.3 Preparing for cleaning

- Remove the battery from the Acculan machine.

### 7.4 Cleaning/Disinfection

Device-specific safety information on the reprocessing method

**DANGER**

**Risk of electric shock and fire hazard!**

- Do not use flammable or explosive cleaning or disinfecting solutions.
- Ensure that no fluids will penetrate the product.

**CAUTION**

**Damage to, or destruction of the product caused by mechanical cleaning/disinfection!**

- Only clean and disinfect the product manually.
- Do not sterilize the product under any circumstances.

**Risk of damage to the product due to inappropriate cleaning/disinfecting agents!**

- ▶ Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.
- ▶ Do not introduce any fluid onto the electrical contacts of the battery.

## 7.5 Wipe disinfection for electrical devices without sterilization

| Phase | Step              | D<br>[°C/°F] | t<br>[min] | Conc.<br>[%] | Water quality | Chemistry                  |
|-------|-------------------|--------------|------------|--------------|---------------|----------------------------|
| I     | Wipe disinfection | RT           | ≥ 1        | -            | -             | Meliseptol Wipes sensitive |

RT: Room temperature

### Phase I

- ▶ Remove any visible residues with a disposable disinfectant wipe.
- ▶ Wipe all surfaces of the optically clean product with a fresh, disposable disinfectant wipe.
- ▶ Observe the specified application time (1 min minimum).

## 7.6 Inspection, maintenance and checks

- ▶ Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functional and undamaged.
- ▶ Set aside the product if it is damaged.

## 8. Maintenance

The display "Battery replacement recommended" on the Acculan 3Ti charger GA677 (see TA022160) indicates when the battery should be replaced due to reduced capacity.

### Note

The "Battery replacement recommended" display on the Acculan 4 charger GA320 has no function in the NiMH battery GA666 and GA676.

No other maintenance measures must be performed on the battery. For corresponding services, contact your national B. Braun/Aesculap agency, see Technical Service.

## 9. Troubleshooting

- Replace product: have defective products repaired by Aesculap Technical Service, see Technical Service.

| Issue                 | Detection                                                                                                                                                 | Cause                        | Remedy                                                     |
|-----------------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------|------------------------------------------------------------|
| Battery not working   | No signal tone when inserting the battery into the machine                                                                                                | Battery defective            | Replace product.                                           |
| Battery quickly empty | Symbol "Battery replacement recommended" illuminates at the end of the charging process on the Acculan 3Ti charger GA677 (not on Acculan 4 charger GA320) | Battery cells at end of life | Have battery cells replaced by Aesculap Technical Service. |

## 10. Storage and transport

**WARNING**

**Risk of injury from burns and damage to product in case of short-circuit!**

- When storing the product, ensure that the product cannot short-circuit with other products or metal objects.

- In order to prevent damage during transport, ship product in the original packaging and in an outer carton.
- If the original packaging is no longer available, ensure the following when packaging the product:
  - The battery contacts are protected against short-circuit.
  - The packaging completely encloses the product.
  - The product is secured against movement within the packaging.
- Completely charge the product upon receipt and before extended periods of storage.

## 11. Accessories/Replacement Parts

| Art. no. | Designation                                        |
|----------|----------------------------------------------------|
| TA015514 | Instructions for use, battery GA666, GA676 (A4)    |
| TA011867 | Instructions for use, battery GA666, GA676 (flyer) |

## 12. Technical Service

**WARNING**

**Risk of injury and/or malfunction!**

- Do not modify the product.

Modifications carried out on medical technical equipment may result in loss of warranty rights and applicable licenses.

- For service and repairs, please contact your national B. Braun/Aesculap agency.

### Service addresses

Aesculap Technischer Service

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78532 Tuttlingen /Germany

Phone: +49 7461 95-1601

Fax: +49 7461 16-939

E-Mail: ats@aesculap.de

Or in the US:

Attn. Aesculap Technical Services

615 Lambert Pointe Drive

Hazelwood

MO, 63042

Aesculap Repair Hotline

Phone: +1 800 214-3392

Fax: +1 314 895-4420

Other service addresses can be obtained from the address indicated above.

## 13. Technical Data

### 13.1 Classification according to directive 93/42/EWG

| Art. no. | Designation     | Class |
|----------|-----------------|-------|
| GA666    | Acculan battery | I     |
| GA676    | Acculan battery | I     |

### 13.2 Performance data, information and standards

| Model                  | GA666                             | GA676                          |
|------------------------|-----------------------------------|--------------------------------|
| Cell type              | NiMH                              | NiMH                           |
| DC voltage             | 9.6 V                             | 9.6 V                          |
| Capacity               | 1.05 Ah                           | 1.95 Ah                        |
| Weight                 | approx. 0.304 kg                  | approx. 0.425 kg               |
| Dimensions (L x W x H) | approx.<br>120 mm x 43 mm x 50 mm | approx. 140 mm x 43 mm x 50 mm |
| EMC                    | IEC/DIN 60601-1-2                 | IEC/DIN 60601-1-2              |
| Conforming to standard | IEC/DIN 60601-1                   | IEC/DIN 60601-1                |

### 13.3 Ambient conditions

|                      | Operation            | Storage and transport                                                                               |
|----------------------|----------------------|-----------------------------------------------------------------------------------------------------|
| Temperature          | 10 °C to 40 °C       | -10 °C to 40 °C<br>Maximum recommended duration of storage is 3 months; keep out of direct sunlight |
| Relative humidity    | 30 % to 75 %         | 10 % to 90 %                                                                                        |
| Atmospheric pressure | 700 hPa to 1 060 hPa | 500 hPa to 1 060 hPa                                                                                |

#### Note

The atmospheric pressure of 800 hPa corresponds to a maximum operating altitude of 2 000 m.

## 14. Disposal

The user institution is obliged to process the product before its disposal, see chapter Validated processing procedure.

Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

The recycling pass can be downloaded from the Extranet as a PDF document under the respective article number. (The recycling pass includes disassembling instructions for the product, as well as information for proper disposal of components harmful to the environment.)

Products carrying this symbol are subject to separate collection of electrical and electronic devices. Within the European Union, disposal is taken care of by the manufacturer as a free-of-charge service.

- Detailed information concerning the disposal of the product is available through your national B. Braun/Aesculap agency, see Technical Service.

## 15. Distributor in the US/Contact in Canada for product information and complaints

Aesculap Inc.  
3773 Corporate Parkway  
Center Valley, PA, 18034,  
USA

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