en	Instructions for use/Technical description Sterile container system
USA	Note for U.S. users This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusa.com . If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.
de	Gebruiksaanweisung/Technische Beschreibung Sterilcontainer-System
fr	Mode d'emploi/Description technique Système de containers stériles
es	Instrucciones de manejo/Descripción técnica Sistema de contenedores estériles
it	Istruzioni per l'uso/Descrizione tecnica Sistema Sterilcontainer
pt	Instruções de utilização/Descrição técnica Sistema de contentores de esterilização
nl	Gebruiksaanwijzing/Technische beschrijving Steriele containersysteem
da	Brugsanvisning/Teknisk beskrivelse Sterilcontainersystem
nb	Bruksanvisning/Teknisk beskrivelse Sterilcontainer-system
sv	Bruksanvisning/Teknisk beskrivning Sterilcontainer-system
fi	Käyttöohje/Tekninen kuvaus Steriili säilytysjärjestelmä
ru	Инструкция по применению/Техническое описание Система контейнеров для стерилизации
cs	Návod k použití/Technický popis Systém sterilizačního kontejneru
pl	Instrukcja użytkowania/Opis techniczny System kontenerów sterylizacyjnych
sk	Návod na použitie/Technický opis Systém sterilných kontajnerov
tr	Kullanım Kılavuzu/Teknik açıklama Steril konteyner sistemi

Note
These short instructions for use apply to the components of the Aesculap AICON sterile container systems.

Note
For the safe use of the products, the instructions TA015495 for use must be observed.
If the most recent edition of the instructions for use is not available to you, please contact Aesculap.
You can also find the latest version of the instructions for use online at elFU.bbraun.com

Intended use

The Aesculap sterile container systems is a multi-use sterilization container. It serves as a sterile goods package to hold sterile goods and/or textiles during vacuum steam sterilization and to maintain sterility during storage and transport under proper healthcare facility conditions.
Users are trained experts in the area of hospital hygiene and preparation of medical devices or staff working under their instructions and supervision.

- ⚠ WARNING**
Risk of contamination of sterile materials from sterile containers that have not passed the function test!
The sealing of the sterile container and its germ barrier function will be compromised if the sterile container is combined with components from other manufacturers.
- ▶ Only combine Aesculap AICON sterile container accessories/replacement parts with each other.

Function checks

- ▶ Visually inspect all components of the sterile container for damage and correct functioning.
- ▶ See section 7.2. Function test in the instructions for use TA015495.

Cleaning and disinfection

- ⚠ CAUTION**
Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!
- ▶ Following the manufacturer's instructions, use cleaning and disinfecting agents
 - that are approved for use on materials such as aluminum, plastics and stainless steel,
 - that no stress cracks are caused for plastics (such as PPSU),
 - that softeners (such as in silicone) attack and lead to brittleness.
 - ▶ Observe specifications regarding concentration, temperature and exposure time.

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