Order number Pole Clamp Adapter kit: **34774382**
Artikelnummer Pole Clamp Adapter kit: **34774382**

1 HOW TO PROCEED / VORGEHENSWEISE

GB

Goal: The following procedures define how to update the PoleClamp SP with the clamp adapter.

Procedure:

1. Loose the screw and dismantle the Pole Clamp SP
2. Place the clamp adapter in the slot of the framework
3. Insert the bush in the framework
4. Place the washer (white polyamide)
5. Place the steel washer
6. Screw the framework and the pole clamp together
7. The necessary Drying time of Loctite

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2 IMPORTANT NOTES / WICHTIGE HINWEISE

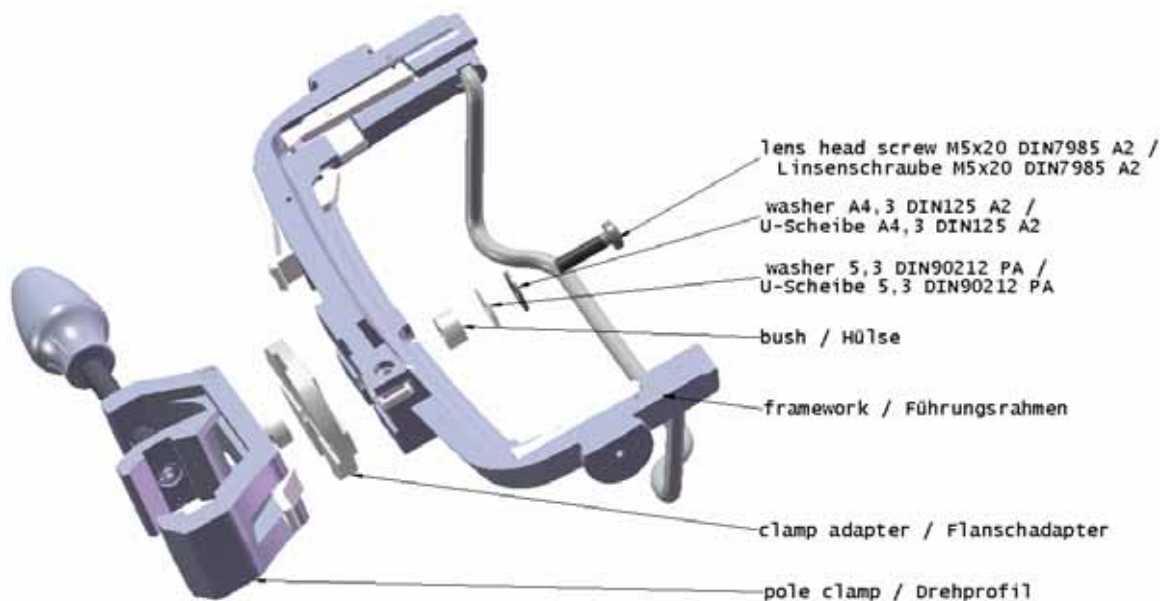
GB

- Take care of the general safety regulations
- The service technician is responsible for correctly executing the work
- The described example is the locking on the left side. All other lockings are reworked accordingly

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
Instruction:
Beschreibung:

1.

Loose the screw and dis-
mantle the Pole Clamp SP.

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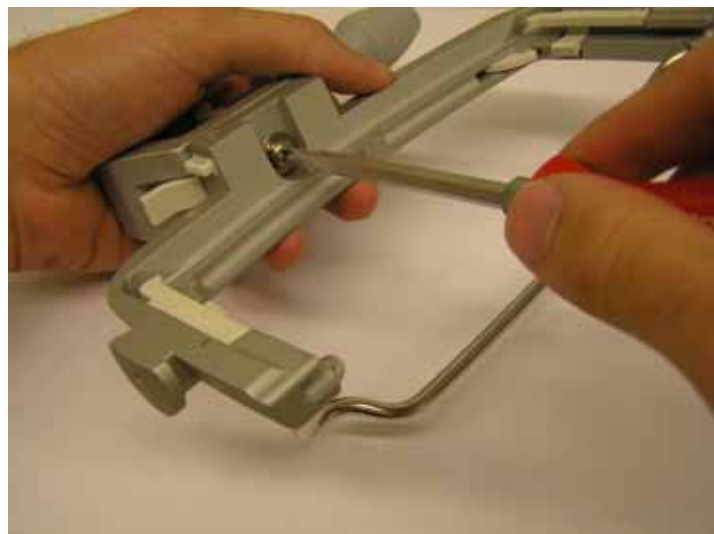
Illustration:
Darstellung:

2.

Place the clamp adapter in
the slot of the framework.
Orientate the clamp
adapter just like it is shown
on the picture.

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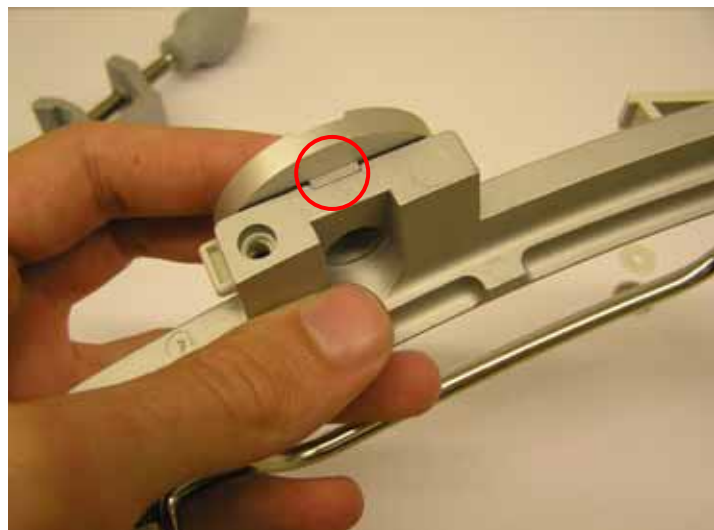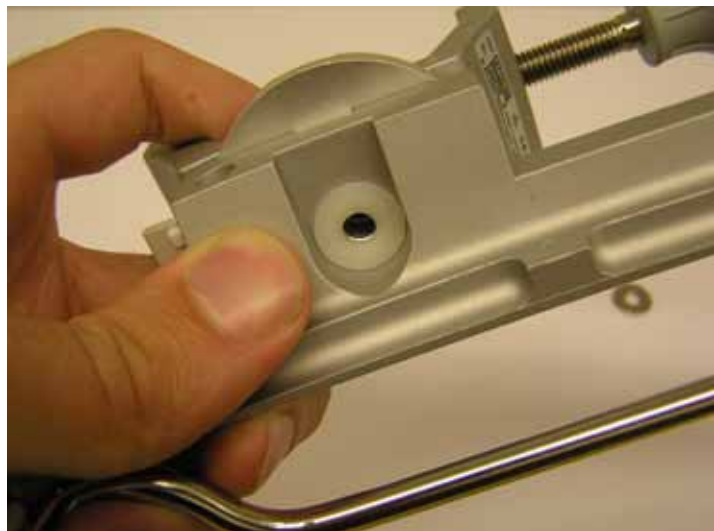
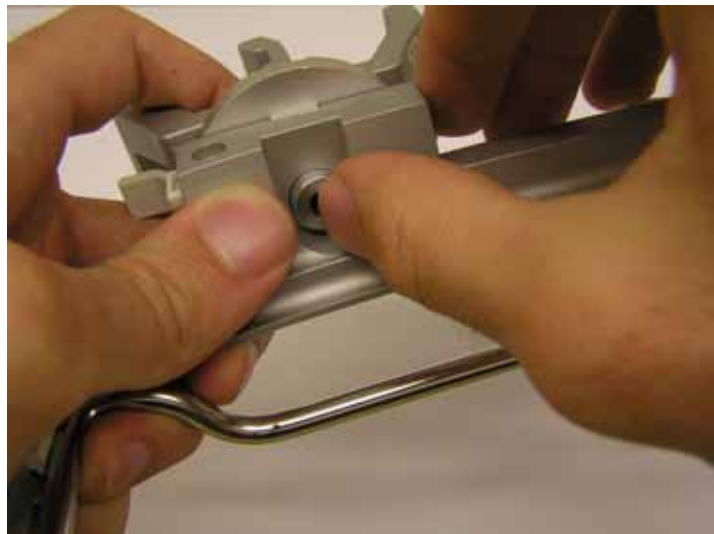
Place the steel washer.

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6.

Use Loctite 275 for thread
locking. Screw the
framework and the pole
clamp with the defined
torque of $0,8 \pm 0,1\text{Nm}$
together.

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Illustration:

Darstellung:

Pole Clamp SP Service Kit Pole Clamp Adapter kit

Instructions for Service

Serviceanweisung

Document: M001_000465

Date: **2009-08-19**

Version: **01**

Page: **6 / 7**

GB / DE

Instruction: Beschreibung:

7.

The necessary Drying time of
Loctite is 3 hours.

Loctite muss 3 Stunden
trocknen.

Illustration: Darstellung:

5 CAUTION! / ACHTUNG!

GB

Perform functional test after updating the Pole Clamp SP.

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Instruction:

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Function test:

Turn the pole clamp in all directions (4x90°) and check the correct locking.

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