GB	Instructions for use Bipolar coagulation forceps
D	Gebrauchsanweisung Bipolare Koagulations-Pinzetten
F	Mode d'emploi Pinces de coagulation bipolaires
E	Instrucciones de manejo Pinzas de coagulación bipolares
I	Istruzioni per l'uso Pinzette per coagulazione bipolari
RUS	Инструкция по применению Биполярные коагуляционные пинцеты

Aesculap AG
Am Aesculap-Platz
78532 Tuttlingen
Germany
Phone +49 7461 95-0
Fax +49 7461 95-2600
www.aesculap.de

TA-Nr. 012932 12/10 V4

CE marking according to directive 93/42/EEC
CE-Kennzeichnung gemäß Richtlinie 93/42/EWG
Marquage CE conforme à la directive 93/42/CEE
Identificación CE en conformidad con la directriz 93/42/CEE
Marchio CE conforme alla direttiva 93/42/CEE
Маркировка CE согласно директиве 93/42/ЕЭС

Technical alterations reserved
Technische Änderungen vorbehalten
Sous réserve de modifications techniques
Sujeto a modificaciones técnicas
Con riserva di modifiche tecniche
Мы оставляем за собой право вносить технические изменения

Symbols on product and packages

Symbol	Explanation
	CAUTION; general warning symbol Caution: See documentation supplied with the product

Intended use

Bipolar coagulation forceps are used for hemostatic coagulation in surgical procedures.

Safe handling and preparation

- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge or experience.
- Read, follow and keep the instructions for use.
- Use the product only in accordance with its intended use, see Intended use.
- Remove the transport packaging and thoroughly clean the new product, either by hand or by a mechanical process, prior to its initial sterilization.
- Store any new or unused products in a dry, clean and safe place.
- Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Replace any damaged components immediately with original spare parts.

WARNING

Danger of injury from burns and/or explosion from flammable gases! Application of the HF device according to its intended use can involve spark formation.

➤ Observe the safety notices in the instructions for use of the HF device.

WARNING

Thermal injuries to patients/users due to insufficient insulation of leads to active accessories

➤ Adjust the HF device to a setting appropriate to ensure that the maximum peak output voltage does not exceed the accessory voltage rating specified for the product.

➤ Adjust the HF power output to the intended operation. Take into account clinical experience or reference values.

➤ Select the lowest possible HF power output.

➤ Keep the product's contact surfaces clean during the operation. Wipe off encrusted tissue residues or body fluids, using a moistened swab.

The rated accessory voltage of the product is 600 Vp.

The accessory voltage rating must exceed or match the maximum peak output voltage at which the product is operated in combination with an appropriate HF device at an appropriate operating mode/setting (see IEC/ DIN EN 60601-2-2).

To avoid HF burns:

➤ The product's working tip must be in the user's field of vision whenever HF power is activated.

➤ Prior to activating the HF device, make certain that the working end of the product is not touching any electrically conductive accessories.

➤ Prior to each use, visually check the product for: damage to, or surface changes on, the insulation.

➤ Never put down the product on or next to the patient.

➤ When using accessories for endoscopy or laparoscopy applications, deactivate the automatic switch-on mode of the HF generator.

➤ Follow the instructions for use of the HF device.

Safe operation

WARNING

Risk of injury and/or malfunction!

➤ Always carry out a function check before using the product.

Validated processing procedure

Note
Adhere to national statutory regulations, international standards and guidelines, and local, clinical hygiene instructions for sterile processing.

Note
Mechanical processing should be preferred over manual cleaning because of the better and more reliable cleaning results of mechanical processing.

Note
Successful processing of this medical product can only be ensured if processing is performed through a validated processing procedure. The user/processor is responsible for the validation.

Note
Up-to-date information on processing can be found on the Aesculap Extranet at www.aesculap-extra.net

General notes

To avoid unnecessary, excessive contamination of the complete instrument tray during operations, take care that contaminated instruments are collected separately and not put back into the instrument tray.

Encrusted or fixated residues from surgery can make the cleaning process more difficult or ineffective, and can cause corrosion of stainless steels. Therefore the time interval between application and cleaning should not exceed 6 h, and neither potentially fixating pre-cleaning temperatures >45 °C nor any fixating disinfectants (active ingredient: aldehyde, alcohol) be used.

Excessive doses of neutralizers or basic detergents can cause chemical degradation and/or fading and obliteration of laser inscriptions on stainless steel surfaces, regarding visual reading and machine-readability of the inscriptions. Residues containing chlorine or chlorides e.g. in surgical residues, medicines, saline solutions and in the service water used for cleaning, disinfecting and sterilization will cause corrosion damage (pitting, stress corrosion) and result in the destruction of stainless steel products. To remove such residues, the products must be rinsed sufficiently with fully desalinated water and dried thoroughly.

Only use process chemicals that have been tested and approved (e.g. VAH/DGHM or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations. All process parameters specified by the chemical's manufacturer, such as temperatures, concentrations and exposure times, must be strictly observed. Failure to do so can result in the following problems:

- Optical deterioration of materials, e.g. fading or discoloration of titanium or aluminum surfaces. For aluminum, pH >8 in the application/process solution can already cause visible surface changes.
- Material damage, e.g. corrosion, cracks, fracturing, premature aging or swelling

➤ Clean the product immediately after use.

Immersion treatment in a 3 % H₂O₂ solution for approx. 5 minutes is a particularly effective and gentle method to dissolve encrustations from HF instruments. Subsequently, the debris can be removed by hand, with a medium-hard brush and/or in an ultrasonic cleaning bath. After that treatment, proceed through the normal steps of processing. For further detailed advice on hygienically safe and material-/value-preserving reprocessing, see www.a-k-l.org

Preparations at the place of use

- Rinse surfaces inaccessible to visual inspection, e.g. on products with hidden crevices or lumens or products with complex geometries, preferably with distilled water, using e.g. a disposable syringe.
- Remove visible surgical residues as completely as possible, using a lint-free wet wipe.
- Put the dry product into a closed disposal container and have it transferred to cleaning and disinfecting within 6 h.

Cleaning/Disinfecting

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

➤ Use cleaning/disinfecting agents according to manufacturers' instructions. The cleaning/disinfecting agents must

- be approved for e.g. aluminum, plastics and high-grade steel.
- not attack softeners (e.g. silicone).

➤ Observe specifications regarding concentration, temperature and exposure time.

➤ Carry out ultrasound cleaning:

- as an effective mechanical supplement to manual cleaning/disinfecting.
- as a pre-cleaning procedure for products with encrusted residues, in preparation for mechanical cleaning/ disinfecting.
- as an integrated mechanical support measure for mechanical cleaning/disinfecting.
- for additional cleaning of products with residues left after mechanical cleaning/disinfecting.

Manual cleaning/disinfecting

Manual ultrasonic cleaning and immersion disinfection

Stage	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Ultrasonic cleaning	RT (cold)	5	2	D-W	BBraun Stabimed; aldehyde-free, phenol-free and QAV-free
II	1 st Intermediate rinse	RT (cold)	1	-	D-W	-
III	Disinfecting	RT (cold)	15	2	D-W	BBraun Stabimed; aldehyde-free, phenol-free and QAV-free
IV	Final rinse	RT (cold)	1	-	FD-W	-
V	Drying	-	-	-	-	-

D-W: Drinking water
FD-W: Fully desalinated water (demineralized)
RT: Room Temperature

Stage I

- Clean the product in an ultrasound cleaning bath (frequency 35 kHz). Make sure that all accessible surfaces are moistened and acoustic shadows are avoided.
- Clean the product with a suitable cleaning brush until all visible residues have been removed from the surface.
- Brush through all surfaces not accessible to visual inspection, e.g. in products with hidden crevices, lumens or complex geometries, for at least 1 min or until no more residues can be removed, using a suitable cleaning brush.
- Mobilize non-rigid components, e.g. set screws, links, etc. during cleaning.
- After cleaning, thoroughly rinse through these components (at least five times) with the cleaning solution, using a disposable syringe (20 ml).
- Do not use metal cleaning brushes or other abrasives that would damage the product surfaces and could cause corrosion.

Stage II

- Rinse the product completely (all accessible surfaces) under running water.
- Allow water to drip off for a sufficient length of time.

Stage III

- Fully immerse the product in the disinfecting solution.
- Rinse lumens at least 5 times at the beginning of the exposure time, using a disposable syringe (20 ml) and an appropriate rinsing adapter. Make sure that all accessible surfaces are moistened.

Stage IV

- Rinse the product completely (all accessible surfaces) under running water.
- Rinse lumens at least 5 times, using a disposable syringe (20 ml) and an appropriate rinsing adapter.
- Allow water to drip off for a sufficient length of time.

Stage V

- Thoroughly dry the product with lint-free tissue or medical-quality filtered compressed air.

Mechanical cleaning/disinfecting with manual pre-cleaning

Note
The disinfector must be of tested and approved effectiveness (e.g. DGHM or FDA approval or CE mark according to DIN EN ISO 15883).

Note
Use appropriate (neutral, enzymatic and mild-alkaline) detergents for this product with aluminum components.

Note
For thermal disinfection, always use fully desalinated (demineralized) water. Ensure that Ao is >3 000 for the process.

Note
The disinfecter used for processing must be serviced and checked at regular intervals.

Manual pre-cleaning with ultrasound and brush

Stage	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Ultrasonic cleaning	RT (cold)	15	2	D-W	BBraun Stabimed; aldehyde-free, phenol-free and QAV-free
II	Irrigation	RT (cold)	1	-	D-W	-

D-W: Drinking water
RT: Room Temperature

Stage I

- Clean the product in an ultrasound cleaning bath (frequency 35 kHz). Make sure that all accessible surfaces are moistened and acoustic shadows are avoided.
- Clean the product with a suitable cleaning brush until all visible residues have been removed from the surface.
- Brush through all surfaces not accessible to visual inspection, e. g. in products with hidden crevices, lumens or complex geometries, for at least 1 min or until no more residues can be removed, using a suitable cleaning brush.
- Mobilize non-rigid components, e.g. set screws, links, etc. during cleaning.
- After cleaning, thoroughly rinse through these components (at least five times) with the cleaning solution, using a disposable syringe (20 ml).
- Do not use metal cleaning brushes or other abrasives that would damage the product surfaces and could cause corrosion.

Stage II

- Rinse the product completely (all accessible surfaces) under running water.

Mechanical neutral or mild-alkaline cleaning and thermal disinfection

Machine type: Single-chamber washer/disinfector without ultrasound

- Place the product on a tray that is suitable for cleaning (avoid rinsing blind spots).
- Connect components with lumens and channels directly to the rinsing port of the injector carriage.
- Keep working ends open during cleaning.

Stage	Step	T [°C/°F]	t [min]	Water quality	Chemical
I	Prerinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	Neutral: BBRAUN HELIMATIC CLEANER neutral, pH-neutral, process solution 0.5 % Mild-alkaline: concentrate, pH = 9.5 < 5 % anionic tensides - 0.5 % solution
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disinfecting	90/194	5	FD-W	-
V	Drying	-	-	-	According to disinfectant program

D-W: Drinking water
FD-W: Fully desalinated water (demineralized)

- Check visible surfaces for residues after mechanical cleaning/disinfecting.
- Repeat the cleaning process if necessary.

Inspection, maintenance and checks

CAUTION

Damage (metal seizure/friction corrosion) to the product caused by insufficient lubrication!

- Prior to function checks, lubricate moving parts (e.g. joints, pusher components and threaded rods) with maintenance oil suitable for the respective sterilization process (e.g. for steam sterilization: Aesculap STERILIT® I oil spray JG600 or STERILIT® I drip lubricator JG598).

- Allow the product to cool down to room temperature.
- After each complete cleaning, disinfecting and drying cycle, check that the product is: dry, clean, operational, and free of damage (e.g. broken insulation or corroded, loose, bent, broken, cracked, worn or fractured components).
- Dry the product if it is wet or moist.
- Repeat cleaning and disinfecting of products that still show impurities or contamination.
- Check the product for proper functioning.
- Immediately sort out damaged or inoperative products and have them sent to Aesculap Technical Service, see Technical Service.
- Check for compatibility with associated products.

Packaging

- Appropriately protect products with fine working tips.
- Sort the product into its appropriate storage device or put it on a suitable tray. Make certain that all cutting edges are protected.
- Package trays appropriately for the sterilization process (e.g. in Aesculap sterile containers).
- Ensure that the packaging provides sufficient protection against recontamination of the product during storage (DIN EN ISO 11607).

Sterilization method and parameters

- Make certain that all external and internal surfaces will be exposed to the sterilizing agent (e.g. by opening all valves and faucets).
- Validated sterilization process
 - Steam sterilization through fractionated vacuum process
 - Steam sterilizer according to DIN EN 285, validated according to DIN EN ISO 17665
 - Sterilization through fractionated vacuum process at 134 °C/holding time 5 min
- When sterilizing several products at the same time in one steam sterilizer: Make certain that the maximum load capacity of the steam sterilizer, as specified by the manufacturer, is not exceeded.

Storage

- Store sterile products in germ-proof packaging under dust protection in a dry, dark and temperature-controlled room.

Technical Service

WARNING

Risk of injury and/or malfunction!

- Do not modify the product.

➤ For service and repairs, please contact your national B. Braun/Aesculap agency.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 7461 95-1602
Fax: +49 7461 16-5621
E-mail: ats@aesculap.de
Other service addresses can be obtained from the address indicated above.

Disposal

- Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

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