D

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B | BRAUN

TA-Nr. 010021 01/09 Änd.-Nr. 32568

Your Aesculap bipolar coagulation forceps is a high-quality product, the proper handling of which is described below.

In order to preserve the utility value of the bipolar coagulation forceps, please read the instructions for use carefully and keep them safely.

General

The Aesculap bipolar coagulation forceps is a high-quality product which is used **in all areas of open surgery** for bipolar coagulation.

WARNING

The use of the instrument for purposes other than the intended one may lead to damage or destruction.

Notes on use

Each time before use, examine the instrument for

- correct function,
- surface damage,
- cracks.

Damaged or faulty instruments must no longer be used!

When the instruments are not in use, they must be kept in a

- dry,
 - clean and
 - protected place.
- The choice of the HF output power should be tailored specially to the intended operation. The power setting should therefore be based on the clinician's experience with respect to relevant clinical references or as a result of suitable training.
- The insulation of the coagulation forceps is designed for a maximum recurring rated voltage of 800 Vp. In the instructions for use of the HF unit, the maximum recurring output voltage is stated as a function of the HF modes and of the set dose. The coagulation forceps may be used only when the rated voltage specified for the instrument is equal to or greater than the preselected maximum output voltage of the HF unit.
- It is essential to follow the information and instructions of the respective manufacturer of the unit.
- The coagulation forceps with Aesculap round plug should be used exclusively with Aesculap bipolar cables (e.g. GK212, GK210, GN074 oder XF830).

Disinfection, cleaning, care and sterilization

- Before being used for the first time and each time after use, the instrument must be thoroughly and properly cleaned. This product should be manually cleaned by the usual cleaning methods.

Good results are guaranteed only with clean, bare metal contact surfaces.

- In the course of the operation, dried-on tissue residues or body fluids can be most reliably, conveniently and gently removed by continuous wiping with a moist swab.
- Thick crusts can be carefully rubbed off with cleaning brush GK299 or with a plastic brush.
- Spray disinfection may be carried out with all materials customarily used for disinfecting surgical instruments.
- Do not clean or store together with other hard or pointed instruments since this may damage the insulation.
- Do not bend apart the limbs of the forceps, since otherwise the coating may be destroyed.

- Before the sterilization, the insulation must be examined for damaged areas.
- For sterilization, we recommend steam sterilization at 2 bar gauge pressure (134 °C).
- The following treatments lead to faster wearing of the parts and should therefore be avoided:
- Cleaning with sharp objects (scalpel, scraper). This damages or destroys the insulation or the contact surface.
 - Cleaning in ultrasonic units and automatic cleaning machines (thermodisinfectors).
 - Placing in disinfection solutions.
 - Gas sterilization.

Maintenance and repairs

Service work, repairs or modifications to the product have to be carried out only by persons authorized by Aesculap. Only in this way will warranties and guarantees remain valid.

- If any repairs are needed, please send the product to:
- Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 7461 95-2700
Fax: +49 7461 16-2887
E-mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

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