

Aesculap Neurosurgery

KAIRison pneumatische Knochenstanze




KURZANLEITUNG


www.aesculap-neuro.com

- ! Gebrauchsanweisung lesen!
- ! Schalter auf OFF stellen!
- ! Druckluftversorgung von 8 bar zur Verfügung stellen!
- ! Druckluftschlauch an Versorgung anschließen!

I Vorbereitung

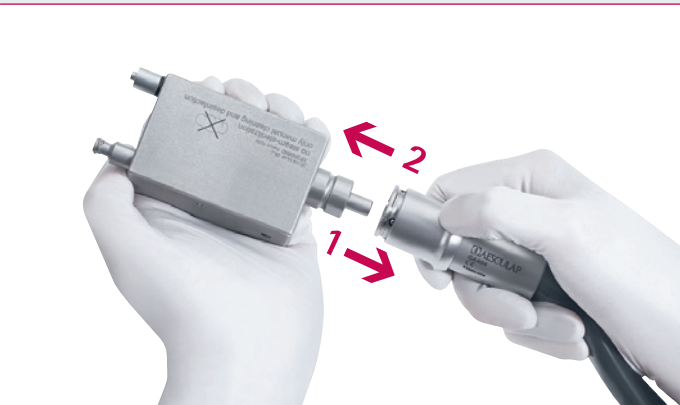

II Druckversorgung an Druckregler anschließen

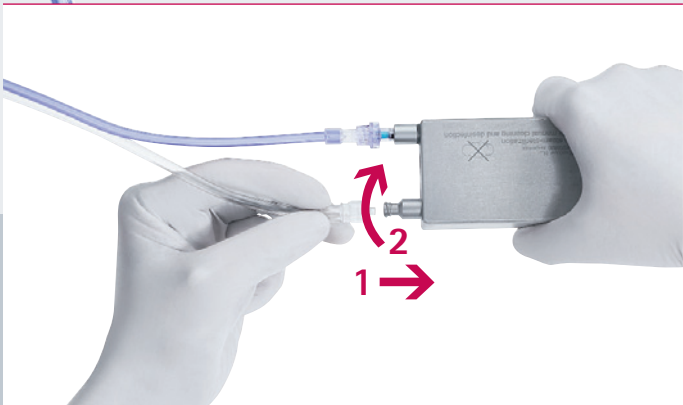

III Einmalschläuche mit Druckreduzierer verbinden (Luer-Lock)



IV Einmalschläuche an Handgriff anschließen




V Schaftposition überprüfen, Schaft in Griff einhängen



VI Schaft in Richtung Handgriff drehen bis dieser hörbar einrastet



VII Schalter auf ON

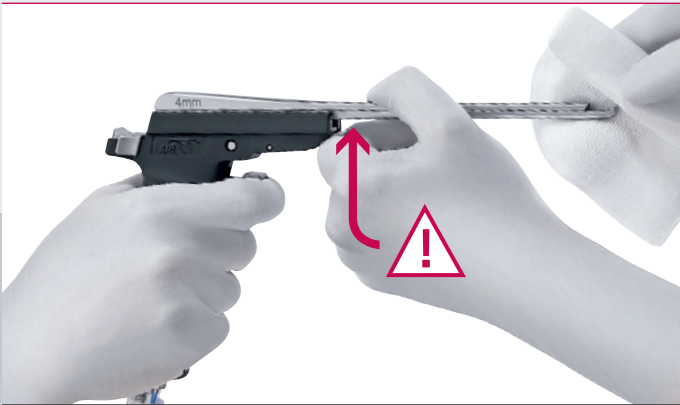

! Sicherheitshebel bei jeder Reinigung betätigen

Weitere Informationen finden Sie:



C24301 KAIRison pneumatische Knochenstanze



C25411 KAIRison Aufbereitung (Poster)

Breite	Stanzenschaft 235 mm	Stanzenschaft 285 mm
2 mm	FK882R	FK892R
3 mm	FK883R	FK893R
4 mm	FK884R	FK894R
5 mm	FK885R	FK895R
6 mm	FK886R	FK896R

Einzelkomponenten

FK899R	Pneumatischer Handgriff
FK898R	Druckregler
FK887R	Halterung für Druckregler
FK888SU	Einmal-Schlauchset
GA466	Aesculap-Träger Druckluftschlauch
GA468	Aesculap-Schrader Druckluftschlauch



Aesculap AG & Co. KG
Am Aesculap-Platz
78532 Tuttlingen
Deutschland
Telefon +49 7461 95-0
Fax +49 7461 95-2600
www.aesculap.de

Aesculap Neurosurgery

KAIRison pneumatic bone punch





SHORT GUIDE


www.aesculap-neuro.com

- Read instructions for use
- Set switch to OFF
- Have a compressed air supply of at least 8 bar available
- Connect compressed air hose to supply

I Preparation

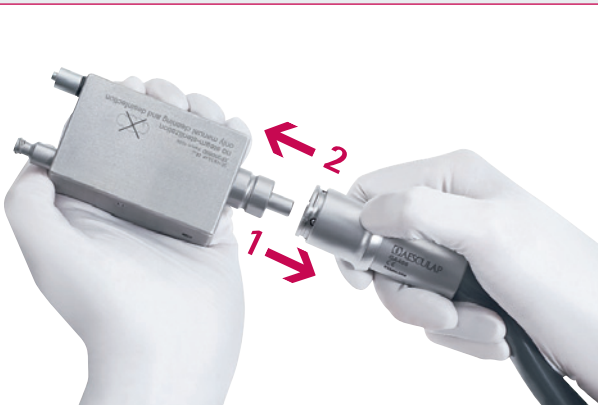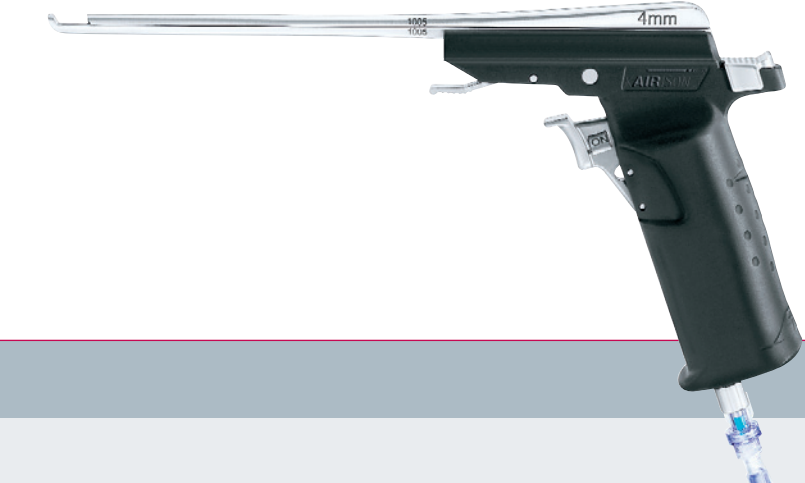

II Connect compressed air supply to pressure regulator

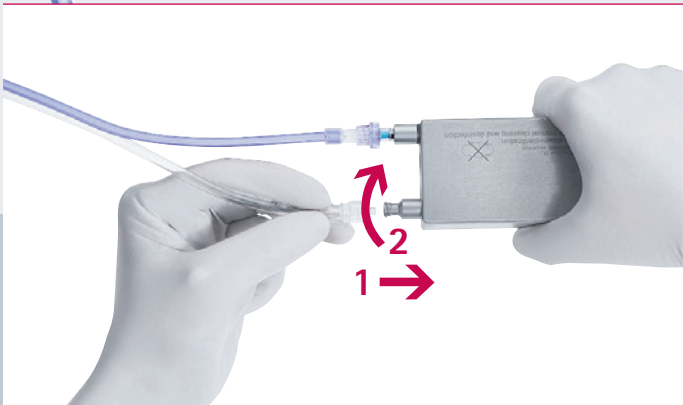

III Connect single use tubes to pressure regulator (Luer Lock)



IV Connect single use tubes to handle




V Check shaft position, hook shaft into handle



VI Turn shaft towards handle until handle clicks audibly into place



VII Turn switch ON

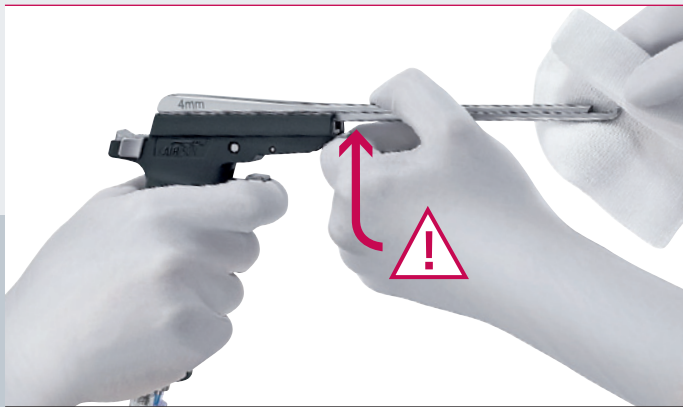

! Activate safety lever each time punch is cleaned during use

For more information please refer to our brochures:



C24302 KAIRison pneumatic bone punch



C25411 KAIRison reprocessing (poster)

Width	Shaft length 235 mm	Shaft length 285 mm
2 mm	FK882R	FK892R
3 mm	FK883R	FK893R
4 mm	FK884R	FK894R
5 mm	FK885R	FK895R
6 mm	FK886R	FK896R

Individual components

- FK899R Pneumatic punch handle
- FK898R Pressure reducer
- FK887R Punch bracket pressure reducer
- FK888SU Disposable tubes
- GA466 Pneumatic hose with Aesculap-Dräger connection
- GA468 Pneumatic hose with Aesculap-Schrader connection



Aesculap AG & Co. KG
Am Aesculap-Platz
78532 Tuttlingen
Germany
Phone +49 7461 95-0
Fax +49 7461 95-2600
www.aesculap.de