- D** Gebrauchsanweisung
Mikro-Instrumente
- GB** Instructions for use
Micro-instruments
- USA** Micro-instruments
- F** Mode d'emploi
Micro-instruments
- E** Instrucciones de manejo
Microinstrumental
- I** Istruzioni per l'uso
Microstrumenti
- P** Instruções de utilização
Microinstrumentos
- NL** Gebruiksaanwijzing
Micro-instrumenten

Legende

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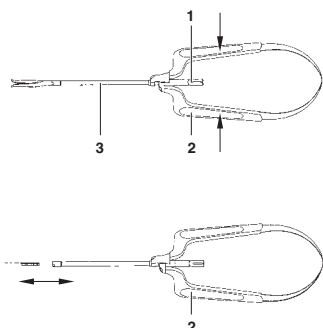
TA-Nr. 010327 04/09 Änd.-Nr. 25927/32568

Legend

- 1 Adjustment knob
- 2 Handle stems
- 3 Jaw part mounting (on handle)
- 4 Jaw part with notched rod

Intended use

The micro-instruments are intended for use in neurosurgery, and are specially designed for use in tight spaces. They can be applied to the cutting, grasping, dissecting, etc. of vessels and tissues.

Available sizes

The micro-instruments are available as complete instruments or as separate, combinable components. Various jaw parts (e.g., straight scissors, curved scissors, and forceps) can be used with the suitable handles.

Complete information about products currently available can be found in AESCULAP brochure C 74511.

Safe handling and preparation**CAUTION**

Federal law restricts this device to sell by or on order of a physician!

- Read the instructions for use and keep them in a safe place.
- Use the instrument only in accordance with professional standards and practices, see Applications.
- Clean the new instrument either manually or mechanically prior to the initial sterilization.
- Store the new or unused instrument in a dry, clean and safe place.
- Inspect the instrument after each cleaning and disinfecting cycle to be sure it is clean, functioning properly, not damaged, has e.g. intact insulation and does not have any loose, bent, broken, cracked, worn, or fractured components.
- Do not use the instrument if it is damaged or defective. Replace damaged parts immediately with original spare parts.
- Set aside the instrument immediately if it is damaged.
- Prior to each use, inspect the instrument for: loose, bent, broken, cracked, worn, or fractured components.

Safe operation

Sharp cutting edges and tips on the instruments can cause injury to user and patient!

- Handle instruments with care.

Disassembling

- Turn the adjustment knob **1** at 180° until it engages (i.e., clicks into place).
- Hold the handle stems **2** in one hand and at the same hold the jaw part with notched rod **4** in the other hand.
- Pull the jaw part mounting **3** and jaw part with notched rod **4** apart.
- Turn the adjustment knob **1** back to its original position.

Assembling

- Ensure that the bridge of the jaw part with notched rod **4** and the groove in the jaw part mounting **3** are in the same axis when the handle stems are in closed position.
- Push the jaw part mounting **3** and jaw part with notched rod **4** together until the notched rod engages with an audible click.

Check functionality:

- Push the handle stems **2** all the way together.
Check to ensure that the jaw part functions properly.

Care and handling

- Clean the entire handle with the cleaning brush provided (see Manual cleaning/Disinfecting).
- The jaw part with notched rod can be cleaned either manually or mechanically.

After each use

- Clean the contaminated instrument as soon as possible.
- Leave hinged instruments open at a 90° angle.
- When carrying out mechanical cleaning, avoid watermarks by placing instruments in wire baskets suitable for cleaning.
- Disassemble the instrument, see Disassembling.
- Remove debris from the instrument while it is dry.
- If the instrument is cleaned while wet, use an active cleaning disinfectant. Prior to mechanical cleaning and disinfecting, rinse the instrument thoroughly in clear running water.
- If ultrasound is needed, carry out the procedure according to the user instructions provided by the manufacturer of the device:
 - as an effective mechanical supplement to manual cleaning.
 - for initial cleaning of instruments with dried debris on them prior to mechanical cleaning.
- Perform manual or mechanical cleaning. Follow the manufacturer's instructions.

Manual cleaning, disinfecting

- Place the instrument in a suitable active cleaning and disinfectant solution in such a way that all surfaces, interior surfaces, lumen, and openings are covered. Follow the disinfectant manufacturer's instructions.
- After disinfecting chemically, always rinse the items thoroughly in plenty of clear running water. Follow the disinfectant manufacturer's instructions.
- Remove encrusted material with a soft nylon brush. Do not use harsh cleaning agents or metal brushes.
- Clean lumina and channels with a soft round nylon brush.

Note

Use brushes that are an appropriate diameter for the lumen they are being used to clean.

- Carry out the final rinse in distilled or fully demineralized water.
- Dry the instrument with a soft, absorbent, lint-free cloth.
- Dry lumen and channels with compressed air.

Mechanical cleaning, disinfecting

- When choosing a program for mechanical cleaning, take into account the materials in the instruments to be cleaned (e.g. stainless instrument-grade steel, aluminum). Follow the instructions provided by the manufacturer of the device.
- Carry out the final rinse in distilled or fully demineralized water.
- Be sure the drying phase is sufficiently long.
- Remove the instrument from the apparatus as soon as the program is completed.

Control

- Allow instruments to cool down to room temperature.
- Moving parts such as hinges and latches should be sprayed with a sterilizable and moisture-permeable lightweight machine oil (such as AESCULAP STERILIT Spray JG 600 or maintenance oil JG 598).
- Inspect the instrument after each cleaning and disinfecting cycle to be sure it is: clean, functioning properly, not damaged, has intact insulation and does not have any loose, bent, broken, cracked, worn, or fractured components.
- Set aside and replace the instrument if it is damaged or defective.

Storage

- Keep instruments with fine working tips and/or microsurgical instruments in appropriate storage units.

Sterilization method and parameters

- Sterilize with steam, taking note of the following:
The sterilization has to be done according to a validated steam sterilization procedure (e.g. in a sterilizer in conformance with EN 285/ANSI/AAMI/ISO 11134-1993, ANSI/AAMI ST46-1993, and validated in conformance with EN 554/ISO 13683). In case of application of the fractionated vacuum procedure the sterilization has to be carried out for a minimum of 5 minutes at 134 °C and at 2 bar pressure.

Sterilization for the US-market:

- Sterilization of the device may be accomplished by steam or ethylene oxide (EtO) gas.
- AESCULAP does not recommend the device be sterilized by "Flash" or chemical sterilization.
- Surgical instruments may also be placed within an AESCULAP rigid sterilization container (sterile container) for processing under generally accepted hospital in-use conditions.

The recommended sterilization parameters are as follows:

Sterilization method	Temp.	Minimum exposure time	
		Wrapped	In a sterile container system
Pre-vacuum	270–275 °F	4 min	4 min
Gravity	250–254 °F	15 min	40 min
	270–275 °F	10 min	30 min
Ethylene Oxide (EtO)	125–130 °F	105 min with 12 % EO, 88 % FREON; 45–75 % chamber humidity, reaction of 6 hours	

Repairs

Repairs may be carried out only by persons authorized to do so by AESCULAP. Only in this way will warranties and guarantees remain valid.

- If any repairs are needed, please send the device to:
AESCULAP Technischer Service
Am Aesculap-Platz
78532 Tuttlingen/Germany
Phone: +49 (7461) 95 27 00
Fax: +49 (7461) 16 28 87
E-mail: ats@aesculap.de

Or in the US:
AESCULAP Inc.
Attn. AESCULAP Technical Services
615 Lambert Pointe Drive
Hazelwood
MO, 63042-2609
AESCULAP Repair Hotline
Phone: +1 (800) 214-3392
Fax: +1 (314) 895-4420

Other service addresses can be obtained from the address indicated above.

Distributor in the US

AESCULAP Inc.
3773 Corporate Parkway
Center Valley, PA, 18034

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