B

BRAUN

SHARING EXPERTISE

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany

Phone +49 (0) 7461 95-0 | Fax +49 (0) 7461 95-26 00 | www.aesculap.com

Aesculap® – a B. Braun brand

TA013139 2020-03 V6 Change No. 61877

CE

Aesculap®

Sterile container system

1. Scope

These short instructions for use apply to BASISVARIOPrimeLinePrimeLine Pro mini, dental and optical containers.

- For article specific instructions for use and material compatibility and lifetime information, see B. Braun eIFU at eifu.bbraun.com

2. Intended use

The Aesculap sterile container systems is a multi-use sterilization container. It serves as a sterile goods package to hold sterile goods and/or textiles during sterilization and to maintain sterility during storage and transport under proper hospital conditions.

Users are trained experts in the area of hospital hygiene and preparation of medical devices or staff working under their instructions and supervision.

The Aesculap sterile container system is suitable for steam sterilization. The optics container is also suitable for sterilization in ethylene oxide.

Separately labeled sterile containers (Sterile Container S) with perforated lids and perforated container bottoms are also suitable for sterilization with hydrogen peroxide in a Sterrad® 100 S, Sterrad® 200, Sterrad® NX, Sterrad® 100NX, Steris® V-Pro®1 and Steris® V-Pro®1 Plus sterilizer.

Note

Sterrad® 100 SSterrad® 200Sterrad® NXSterrad® 100 NXASP.

<Fixed-Notion>Sterrad® 200</Fixed-Notion>V-Pro® 1<Fixed-Notion>Sterrad® 200</Fixed-Notion> andV-Pro® 1 Plusare trademarks of Steris

⚠ WARNING

Risk of contamination due to possible leaks in the container in the event of the Aesculap sterile containers being combined with components from other manufacturers.

The function of the Aesculap sterile containers as a germ barrier during transport and storage of the sterile goods can no longer be guaranteed by Aesculap.

- Only combine Aesculap sterile container products with each other.

3. Functional test

- Visually inspect all components of the sterile container for damage and correct functioning.
- Check both sides of the PrimeLine plastic lid for cracks.
- Function check information can be found in our instructions for use TA015110-EN and function check poster C63302.

4. Cleaning and disinfection

⚠ WARNING

Danger of corrosion and stress cracks due to improper cleaning and disinfecting!

- When cleaning and disinfecting, observe the statements made by the manufacturer of cleaning and disinfecting agent being used.
- Only perform pH-neutral or mildly alkaline cleaning.

⚠ WARNING

Also, for PrimeLine:

- Do not use solvents such as acetone or benzine.
- Carry out the final rinse using only distilled or demineralized water.
- Ensure that no residues from the cleaning process remain on the lid.
- Do not use rinsing agents.

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