
- (GB)** Instructions for use/Technical description
(USA) Aesculap clip applicator forceps
(D) Gebrauchsanweisung/Technische Beschreibung
Aesculap Clip-Anlegezangen
(F) Mode d'emploi/Description technique
Pinces de pose pour clips Aesculap
(E) Instrucciones de manejo/Descripción técnica
Pinza aplicadora de clips Aesculap
(I) Istruzioni per l'uso/Descrizione tecnica
Aesculap Pinze applicatrici per clip
(P) Instruções de utilização/Descrição técnica
Pinças de aplicação de clips Aesculap
(NL) Gebruiksaanwijzing/Technische beschrijving
Aesculap Clipapplicatietangen
(S) Bruksanvisning/Teknisk beskrivning
Aesculap Clipstänger
(RUS) Инструкция по применению/Техническое описание
Зажим для наложения клипсов Aesculap
(CZ) Návod k použití/Technický popis
Aesculap Kleště k zakládání svorek
(PL) Instrukcja użytkowania/Opis techniczny
Kleszcze do klipsów marki Aesculap
(SK) Návod na používanie/Technický opis
Aesculap zakladacie kliešte na klipy
(TR) Kullanım Kılavuzu/Teknik açıklama
Aesculap klips yerleştirme penseleri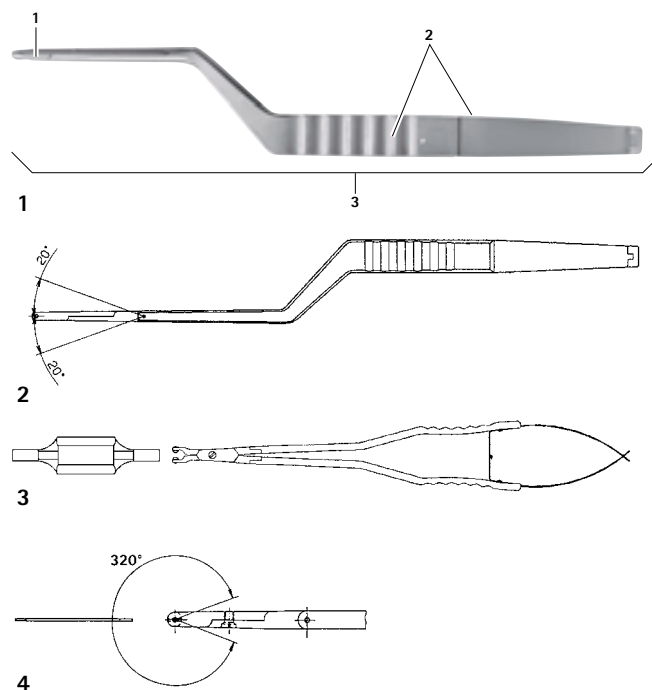

B | BRAUN
SHARING EXPERTISE

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 (0) 7461 95-0 | Fax +49 (0) 7461 95-26 00 | www.aesculap.com

Aesculap – a B. Braun company

TA-Nr. 013089 11/12 V6 Änd.-Nr. 46258

CE 0123 – DIR 93/42/EEC

Technical alterations reserved

Aesculap®

Aesculap clip applier forceps

Legend

- 1 Jaw part
- 2 Branches left/right
- 3 Clip applier forceps

Symbols on product and packages

Caution, general warning symbol
Caution, see documentation supplied with the product

Applicable to

- For item-specific instructions for use and information on material compatibility, see also the Aesculap Extranet at www.extranet.bbraun.com

Intended use

Aesculap clip applier forceps are used for holding and applying Aesculap YASARGIL Phynox and titanium aneurysm clips and Aesculap KOPITNIK AVM microclips in neurosurgical procedures.

Note

Aesculap In the following, YASARGIL aneurysm clips and Aesculap KOPITNIK AVM microclips are referred to as "clips".

Available sizes

Detailed information on available sizes can be found in the main Neurosurgery Catalog and in the Yasargil Aneurysm Clips brochures.

All clip applier forceps are marked according to size (Mini or Standard, Long or AVM micro) and clip material (Phynox or titanium) so that their correct application with clips of the appropriate size and material is ensured.

Safe handling and preparation

CAUTION

Federal law restricts this device to sale by, or on order of a physician!

- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Read, follow, and keep the instructions for use.
- Use the product only in accordance with its intended use, see Intended use.
- Remove the transport packaging and clean the new product, either manually or mechanically, prior to its initial sterilization.
- Store any new or unused products in a dry, clean, and safe place.
- Prior to each use, inspect the product for loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Replace any damaged components immediately with original spare parts.

Safe operation

CAUTION

Damage to, imprecise function and incorrect closing force of the clips due to use of incorrect applier forceps!

- Use Aesculap clip applier forceps only.
- Use clips only with the appropriate Aesculap clip applier forceps (see labels).

- Use clips and clip applier forceps only in the following combinations:
 - Titanium Standard clips with the clip applier forceps for Titanium Standard clips
 - Phynox Standard clips with the clip applier forceps for Phynox Standard clips
 - Titanium Mini clips with the clip applier forceps for Titanium Mini clips
 - Phynox Mini clips with the clip applier forceps for Phynox Mini clips
 - Phynox Long clips with the clip applier forceps for Phynox Long clips
 - AVM clips with the applier forceps for AVM clips

Locks

Note

Some clip applier forceps variants are fitted with a lock to engage during clip transfer.

The clip applier forceps may be fitted with one of the following locks:

- Permanent lock
- Button-controlled lock

Operating the permanent lock

Note

With the clip applier forceps with permanent lock, the lock is always active and cannot be switched off.

- Insert the clip into jaw piece 1.
- Compress branches left/right 2 until the lock engages.
The clip is in transfer position.
- Move clip applier forceps 3 with the clip into the operating field.
- To release the lock, slightly compress branches left/right 2.
- Apply the clip.

Operating the button-controlled lock

Note

The clip applier forceps with button-controlled lock can be used with or without the locking function. The locking function of the button-controlled lock needs to be activated deliberately by pressing the lock button.

Transferring the clip and engaging the lock

- Insert the clip into jaw piece 1.
- Push the lock button and simultaneously compress branches left/right 2.
The lock is engaged and the clip is slightly clamped in jaw piece 1.

Releasing the lock

- Compress branches left/right 2.
Jaw piece 1 of clip applier forceps 3 closes and the clip opens.
The lock automatically returns to the unlocked position.

Detaching the clip applier forceps from the clip

- Release the pressure on branches left/right 2.
Jaw piece 1 of clip applier forceps 3 opens and the clip closes.
- Detach clip applier forceps 3 from the clip.

Vario clip applier forceps

The Aesculap Vario clip applier forceps offer an additional rotary function for positioning of the clip in the clip applier forceps.

The maximum range of rotation at the working end of the Aesculap Vario clip applier forceps is 20° in each direction, starting from the middle position, see Fig. 2.

The adjusting plates of the Aesculap Vario clip applier forceps can be used for parallel alignment of the applier jaws, see Fig. 3.

The adjusting plates allow swiveling the clamping jaws by 320°, see Fig. 4.

Validated reprocessing procedure

General safety instructions

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for sterile processing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

Mechanical reprocessing should be favored over manual cleaning as it gives better and more reliable results.

Note

Successful processing of this medical device can only be ensured if the processing method is first validated. The operator/sterile processing technician is responsible for this.

The recommended chemistry was used for validation.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

For the latest information on reprocessing and material compatibility see also the Aesculap extranet at www.extranet.bbraun.com

The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

General information

Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixating pre-cleaning temperatures >45 °C nor fixating disinfecting agents (active ingredient: aldehydes/alcohols) should be used.

Excessive measures of neutralizing agents or basic cleaners may result in a chemical attack and/or to fading and the laser marking becoming unreadable visually or by machine for stainless steel.

Residues containing chlorine or chlorides e.g. in surgical residues, medicines, saline solutions and in the service water used for cleaning, disinfection and sterilization will cause corrosion damage (pitting, stress corrosion) and result in the destruction of stainless steel products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Optical changes of materials, e.g. fading or discoloration of titanium or aluminum. For aluminum, the application/process solution only needs to be of pH >8 to cause visible surface changes.
- Material damage such as corrosion, cracks, fracturing, premature aging or swelling.
- Do not use metal cleaning brushes or other abrasives that would damage the product surfaces and could cause corrosion.
- Further detailed advice on hygienically safe and material-/value-preserving reprocessing can be found at www.a-k-i.org, link to Publications, Red Brochure – Proper maintenance of instruments.

Disassembling the product before carrying out the reprocessing procedure

- Open up instruments with hinges.

Preparations at the place of use

- If applicable, rinse non-visible surfaces preferably with deionized water, with a disposable syringe for example.
- Remove any visible surgical residues to the extent possible with a damp, lint-free cloth.
- Transport the dry product in a sealed waste container for cleaning and disinfection within 6 hours.

Cleaning/disinfection

Product-specific safety notes on the reprocessing procedure

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

- Use cleaning and disinfecting agents according to the manufacturer's instructions which
 - are approved for (e.g. aluminum, plastics, high-grade steel),
 - do not attack softeners (e.g. in silicone).
- Observe specifications regarding concentration, temperature and exposure time.
- Do not exceed the maximum permitted cleaning temperature of 55 °C.

- Do not use oxidizing chemicals (e.g. H₂O₂), which could cause bleaching/layer loss of the product.
- Use suitable cleaning/disinfecting agents if the product is put away in a wet condition. To prevent foam formation and reduced effectiveness of the process chemicals: Prior to mechanical cleaning and disinfection, rinse the product thoroughly with running water.
- Clean and disinfect microsurgical products mechanically if they can be placed securely in the machine or on the positioning aids.

Validated cleaning and disinfection procedure

Validated procedure	Specific requirements	Reference
Manual cleaning with immersion disinfection	Cleaning brush 20 ml disposable syringe Keep working ends open for cleaning. When cleaning instruments with movable hinges, ensure that these are in an open position and, if applicable, move the joint while cleaning. Drying phase: Use a lint-free cloth or medical compressed air	Chapter Manual cleaning/disinfection and sub-chapter: Chapter Manual cleaning with immersion disinfection
Mechanical alkaline cleaning and thermal disinfection	Place the instrument in a tray that is suitable for cleaning (avoiding rinsing blind spots). Place instruments in the tray with their hinges open.	Chapter Mechanical cleaning/disinfecting and sub-chapter: Chapter Mechanical alkaline cleaning and thermal disinfecting

Manual cleaning/disinfection

- Prior to manual disinfecting, allow water to drip off for a sufficient length of time to prevent dilution of the disinfecting solution.
- After manual cleaning/disinfection, check visible surfaces visually for residues.
- Repeat the cleaning /disinfection process if necessary.

Manual cleaning with immersion disinfection

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Disinfecting cleaning	RT (cold)	>15	2	D–W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Intermediate rinse	RT (cold)	1	–	D–W	–
III	Disinfection	RT (cold)	15	2	D–W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
IV	Final rinse	RT (cold)	1	–	FD-W	–
V	Drying	RT	–	–	–	–
D–W:	Drinking water					
FD–W:	Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)					
RT:	Room temperature					
*Recommended:	BBraun Stabimed					

- Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- Fully immerse the product in the cleaning/disinfectant for at least 15 min. Ensure that all accessible surfaces are moistened.
- Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- Drain any remaining water fully.

Phase III

- Fully immerse the product in the disinfectant solution.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- Rinse lumens at least 5 times at the beginning of the exposure time using an appropriate disposable syringe. Ensure that all accessible surfaces are moistened.

Phase IV

- Rinse/flush the product thoroughly (all accessible surfaces).
- Mobilize non-rigid components, such as set screws, joints, etc. during final rinse.
- Rinse lumens with an appropriate disposable syringe at least five times.
- Drain any remaining water fully.

Phase V

- Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air), see Validated cleaning and disinfection procedure.

Mechanical cleaning/disinfecting

Note
The cleaning and disinfection device must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note
The cleaning and disinfection device used for processing must be serviced and checked at regular intervals.

Mechanical alkaline cleaning and thermal disinfecting

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemical/Note
I	Prerinse	<25/77	3	D–W	–
II	Cleaning	55/131	10	FD-W	Concentrate, alkaline: pH = 13 <5 % anionic surfactant 0.5 % working solution pH = 11*
III	Intermediate rinse	>10/50	1	FD-W	–
IV	Thermal disinfecting	90/194	5	FD-W	–
V	Drying	–	–	–	According to the program for cleaning and disinfection device

D–W:

Drinking water

FD–W:

Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)

*Recommended:

BBraun Helimatic Cleaner alkaline

- Check visible surfaces for residues after mechanical cleaning/disinfecting.

Inspection, maintenance and checks

CAUTION

Damage (metal seizure/friction corrosion) to the product caused by insufficient lubrication!

Prior to function checks, lubricate moving parts (e.g. joints, pusher components and threaded rods) with maintenance oil suitable for the respective sterilization process (e.g. for steam sterilization: Aesculap STERILIT® I oil spray JG600 or STERILIT® I drip lubricator JG598).

- Allow the product to cool down to room temperature.
- After each complete cleaning, disinfecting and drying cycle, check that the instrument is dry, clean, operational, and free of damage (e.g. broken insulation or corroded, loose, bent, broken, cracked, worn, or fractured components).
- Dry the product if it is wet or damp.
- Repeat cleaning and disinfection of products that still show impurities or contamination.
- Check that the product functions correctly.
- Immediately put aside damaged or inoperative products and send them to Aesculap Technical Service, see Technical Service.
- Check for compatibility with associated products.

Packaging

- Appropriately protect products with fine working tips.
- Store products with ratchet locks fully opened or locked no further than in the first notch.
- Pack trays appropriately for the intended sterilization process (e.g. in sterile Aesculap containers).
- Ensure that the packaging provides sufficient protection against recontamination of the product during storage.

Steam sterilization

- Check to ensure that the sterilizing agent will come into contact with all external and internal surfaces (e.g. by opening any valves and faucets).
- Validated sterilization process

Steam sterilization through fractionated vacuum process

Steam sterilizer according to DIN EN 285 and validated according to DIN EN ISO 17665

Sterilization using fractionated vacuum process at 134 °C/holding time 5 min
- When sterilizing several instruments at the same time in a steam sterilizer, ensure that the maximum load capacity of the steam sterilizer specified by the manufacturer is not exceeded.

Sterilization for the US market

- Aesculap advises against sterilizing the device by flash sterilization or chemical sterilization.
- Sterilization may be accomplished by a standard prevacuum cycle in a steam autoclave.

To achieve a sterility assurance level of 10⁻⁶, Aesculap recommends the following parameters:

Aesculap Orga Tray/Sterile container (perforated bottom) Minimum cycle parameters*			
Sterilization method	Temp.	Time	Minimum drying time
Prevacuum	270 °F/275 °F	4 min	20 min

*Aesculap has validated the above sterilization cycle and has the data on file. The validation was accomplished in an Aesculap sterile container cleared by FDA for the sterilization and storage of these products. Other sterilization cycles may also be suitable, however individuals or hospitals not using the recommended method are advised to validate any alternative method using appropriate laboratory techniques. Use an FDA cleared accessory to maintain sterility after processing, such as a wrap, pouch, etc.

WARNING for the US market

If this device is/was used in a patient with, or suspected of having Creutzfeldt–Jakob Disease (CJD), the device cannot be reused and must be destroyed due to the inability to reprocess or sterilize to eliminate the risk of crosscontamination.

Storage

- Store sterile products in germ-proof packaging, protected from dust, in a dry, dark, temperature-controlled area.

Technical Service

Risk of injury and/or malfunction!
► Do not modify the product.

► For service and repairs, please contact your national B. Braun/Aesculap agency.
Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 (7461) 95-1602
Fax: +49 (7461) 16-5621
E-Mail: ats@aesculap.de

Or in the US:
Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood
MO, 63042

Aesculap Repair Hotline
Phone: +1 (800) 214-3392
Fax: +1 (314) 895-4420

Other service addresses can be obtained from the address indicated above.

Distributor in the US/Contact in Canada for product information and complaints

3773 Corporate Parkway
Center Valley, PA, 18034,
USA

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