Aesculap® Acculan

(GB) Instructions for use/Technical description

(USA) Acculan 4 Li-Ion battery short GA346

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Acculan 4 Li-Ion battery short GA346

Symbols on product and packages

	Caution, general warning symbol Caution, see documentation supplied with the product
	Follow the instructions for use
	Machine-readable, two-dimensional code The code contains a unique serial number which can be used for electronic tracking of the individual instrument. The serial number is based on the global standard sGTIN (GS1).
	Keep device away from fluids
	Serial number
	Manufacturer and date of manufacture (year)
	Labelling of electrical and electronic devices according to the directive 2012/19/EU (WEEE), see Disposal
	Date of manufacture
	Manufacturer's batch designation
	Manufacturer's article number
	Temperature limit The temperature limit values to which the medical product can be safely subjected are designated.
	Humidity, limit Designates the humidity range to which the medical product can be safely subjected.
	Air pressure, limit Designates the atmospheric pressure to which the medical product can be safely subjected.

1. Scope

For item-specific instructions for use and information regarding material compatibility, see also the Aesculap Extranet at <https://extranet.bbraun.com>

2. General information

2.1 Use/intended use:

Task/Function

The Li-Ion batteries serve to supply energy to the Acculan machines and include the activation and control of the electric motors of the Acculan machines according to user commands.

Application Environment

The product fulfills the requirements of Type BF in accordance with IEC/DIN EN 60601-1.

For use in operating rooms, outside of the explosive hazard zone (e.g. areas with highly pure oxygen or anesthetic gases).

2.2 Main functions and design characteristics

DC voltage	Energy content
9.9 V	10,89 Wh

The Li-Ion battery GA346 is a replacement for the NiMH battery GA666

2.3 Indications

The indications can be found in the instructions for use for the corresponding Acculan machines.

2.4 Contraindications

The contraindications can be found in the instructions for use for the corresponding Acculan machines.

3. Safe handling

CAUTION

Federal law restricts this device to sale by, or on order of a physician!

WARNING

Risk of injury and damage to property due to improper handling of the product!

- ▶ Comply with instructions for use for the Acculan 4 machines and the Acculan 3Ti machines.
- ▶ Follow the instructions for use of all products used.

WARNING

Fire, combustion and explosion hazard due to battery poles shorted by fluids or metal components!

- ▶ Do not short-circuit, open, crush or incinerate battery.

CAUTION

Damage to or destruction of the batteries due to reprocessing!

- ▶ Do not sterilize batteries.

- ▶ Inspect the new product after removing its transport packaging and prior to first use to ensure it is in good working order; charge with charger GA320 and verify functionality.
- ▶ To prevent damage caused by improper setup or operation, and to not compromise the manufacturer warranty and liability:
 - Use the product according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
 - Follow the application advisories acc. to standard, see Extracts from relevant standards.
- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- ▶ Keep the instructions for use accessible for the user.
- ▶ Always adhere to applicable standards.
- ▶ Do not use the product if it is damaged or defective.
- ▶ Set aside the product if it is damaged.
- ▶ Store product at room temperature (Maximum recommended duration of storage is 365 days; keep out of direct sunlight).
- ▶ Keep product dry.
- ▶ Observe the instructions for use for the corresponding Acculan machines.

4. Product description

4.1 Scope of Supply

Art. no.	Designation
GA346	Acculan 4 Li-Ion battery short
TA014537	instructions for use Acculan 4 Li-Ion battery short

4.2 Components required to operate the device

- Acculan 4 charger GA320
- Acculan 4 or Acculan 3Ti machine:
 - GA340
 - GA341
 - GA344
 - GT666
 - GA670
 - GA671
 - GT668
 - GA675

4.3 Operating principle

The Acculan 4 Li-Ion battery supplies the Acculan machines with short handle with voltage. The battery is charged in the Acculan 4 charger GA320. The charging process of a completely discharged battery can take up to 120 min.

The speed and rotational direction are set depending on the trigger setting of the machines:

- for saws and Dermatome: speed clockwise
- for drills: speed clockwise, counter-clockwise, oscillation and pilgrim step

5. Preparation

- ▶ Prior to use, inspect the product and its accessories for any visible damage.
- ▶ Use the products and their accessories only if they are in perfect technical condition.

Aesculap® Acculan

Acculan 4 Li-Ion battery short GA346

6. Working with the device

6.1 System set-up

- ▶ See instructions for use for the respective Acculan machine and the Acculan 4 charger GA320.

6.2 Functional test

See the instructions for use for the respective Acculan machine.

6.3 Safe operation

WARNING

Risk of injury and material damage due to inappropriate use of batteries!

- ▶ Charge battery only in the Acculan 4 charger GA320.
- ▶ Insert battery correctly (Acculan machine and charger).

WARNING

Risk of injury and material damage due to damaged batteries!

- ▶ Check batteries for any damage before charging.
- ▶ Do not use or recharge damaged batteries.

- ▶ For proper loading and removal of the battery from the Acculan machine, see the instructions for use for the respective Acculan machine.

Note

Successful processing of this medical device can only be ensured if the processing method is first validated. The operator/sterile processing technician is responsible for this.

The recommended chemistry was used for validation.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

For the latest information on reprocessing and material compatibility see also the Aesculap extranet at <https://extranet.bb Braun.com>

7.2 General notes

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Optical changes of materials, e.g. fading or discoloration of titanium or aluminum. For aluminum, the application/process solution only needs to be of pH >8 to cause visible surface changes.
- Material damage such as corrosion, cracks, fracturing, premature aging or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surface and could cause corrosion.

7.3 Preparing for cleaning

- ▶ Remove the battery from the Acculan machine.

7.4 Cleaning/Disinfection

Device-specific safety information on the reprocessing method

DANGER

Risk of electric shock and fire hazard!

- ▶ Do not use flammable or explosive cleaning or disinfecting solutions.
- ▶ Ensure that no fluids will penetrate the product.

CAUTION

Damage to, or destruction of the product caused by mechanical cleaning/disinfection!

- ▶ Only clean and disinfect the product manually.
- ▶ Do not sterilize the product under any circumstances.

7. Validated reprocessing procedure

7.1 General safety information

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for sterile processing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Risk of damage to the product due to inappropriate cleaning/disinfecting agents!

- ▶ Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.
- ▶ Do not introduce any fluid onto the electrical contacts of the battery.

7.5 Wipe disinfection for electrical devices without sterilization

Phase	Step	D [°C/°F]	t [min]	Conc. [%]	Water quality	Chemistry
I	Wipe disinfection	RT	≥1	-	-	Meliseptol Wipes sensitive

RT: Room temperature

Phase I

- ▶ Remove any visible residues with a disposable disinfectant wipe.
- ▶ Wipe all surfaces of the optically clean product with a fresh, disposable disinfectant wipe.
- ▶ Observe the specified application time (1 min minimum).

7.6 Inspection, maintenance and checks

- ▶ Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functional and undamaged.
- ▶ Set aside the product if it is damaged.

8. Maintenance

The display "Battery replacement recommended" on the Acculan 4 charger GA320 (see TA014534/TA014535) indicates when the battery should be replaced due to reduced capacity.

No other maintenance measures must be performed on the battery.

For corresponding services, contact your national B. Braun/Aesculap agency, see Technical Service.

9. Troubleshooting

- ▶ Replace product: have defective products repaired by Aesculap Technical Service, see Technical Service.

Issue	Detection	Cause	Remedy
Battery not working	No signal tone when inserting the battery into the machine	Battery defective	Replace product.
Battery quickly empty	Symbol "Battery replacement recommended" illuminates at the end of the charging process on the charger GA320	Battery cells at end of life	Have battery cells replaced by Aesculap Technical Service.

Aesculap® Acculan

Acculan 4 Li-Ion battery short GA346

10. Storage and transport

WARNING

Risk of injury from burns and damage to product in case of short-circuit!

- ▶ **When storing the product, ensure that the product cannot short-circuit with other products or metal objects.**

- ▶ In order to prevent damage during transport, ship product in the original packaging and in an outer carton.
- ▶ If the original packaging is no longer available, ensure the following when packaging the product:
 - The battery contacts are protected against short-circuit.
 - The packaging completely encloses the product.
 - The product is secured against movement within the packaging.

Li-Ion batteries must be identified with corresponding warning notices on the shipment packaging when sending through the mail.

- ▶ Identify product with warning notice when shipping:
 - UN3480, if only batteries are in the shipment packaging.
 - UN3481, if machines and batteries are in the shipment packaging.

Due to the regulations of the International Air Transport Association (IATA), this product may not be returned by air shipment.

- ▶ Ship product only by ground/sea transport.

Never ship defective batteries.

- ▶ Dispose of defective products, see Disposal.

Note

Batteries with normal wear (decreased capacity) that have no evidence of damage are not considered defective. These batteries can be shipped for a service case to Aesculap Technical Service.

Due to transport regulations, the product is charged only approx. 30 % and has a limited storage life of 12 months. After the first complete charge, the product can be stored for 24 months without recharging.

- ▶ Completely charge the product upon receipt and before extended periods of storage.

11. Accessories/Replacement Parts

Art. no.	Designation
TA014536	Instructions for use, Acculan 4 Li-Ion battery short GA346 (A4)
TA014537	Instructions for use, Acculan 4 Li-Ion battery short GA346 (leaflet)

12. Technical Service

WARNING

Risk of injury and/or malfunction!

- ▶ **Do not modify the product.**

Modifications carried out on medical technical equipment may result in loss of warranty rights and applicable licenses.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency.

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen /Germany

Phone: +49 7461 95-1602

Fax: +49 7461 16-5621

E-Mail: ats@aesculap.de

Or in the US:

Attn. Aesculap Technical Services

615 Lambert Pointe Drive

Hazelwood

MO, 63042

Aesculap Repair Hotline

Phone: +1 800 214-3392

Fax: +1 314 895-4420

Other service addresses can be obtained from the address indicated above.

13. Technical Data

13.1 Classification according to directive 93/42/EWG

Art. no.	Designation	Class
GA346	Acculan 4 Li-Ion battery short	I

13.2 Performance data, information and standards

Model	Acculan 4 Li-Ion battery short GA346
Cell type	Li-Ion
DC voltage	9.9 V
Energy content	10,89 Wh
Weight	approx. 0.265 kg
Dimensions (L x W x H)	approx. 120 mm x 43 mm x 50 mm
EMC	IEC/DIN 60601-1-2
Conforming to standard	IEC/DIN 60601-1

13.3 Ambient conditions

	Operation	Storage and transport
Temperature	10 °C to 40 °C	-10 °C to 40 °C Maximum recommended duration of storage is 365 days; keep out of direct sunlight
Relative humidity	30 % to 75 %	10 % to 90 %
Atmospheric pressure	700 hPa to 1 060 hPa	500 hPa to 1 060 hPa

Note

The atmospheric pressure of 700 hPa corresponds to a maximum operating altitude of 3 000 m.

14. Disposal

The user institution is obliged to process the product before its disposal, see chapter Validated processing procedure.

Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

The recycling pass can be downloaded from the Extranet as a PDF document under the respective article number. (The recycling pass includes disassembling instructions for the product, as well as information for proper disposal of components harmful to the environment.)

Products carrying this symbol are subject to separate collection of electrical and electronic devices. Within the European Union, disposal is taken care of by the manufacturer as a free-of-charge service.

- Detailed information concerning the disposal of the product is available through your national B. Braun/Aesculap agency, see Technical Service.

15. Distributor in the US/Contact in Canada for product information and complaints

Aesculap Inc.
3773 Corporate Parkway
Center Valley, PA, 18034,
USA

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