

Histoacryl® Flexible


Improved closure of surgical incisions



Closure Technologies

Histoacryl® Flexible

Improved closure of surgical incisions

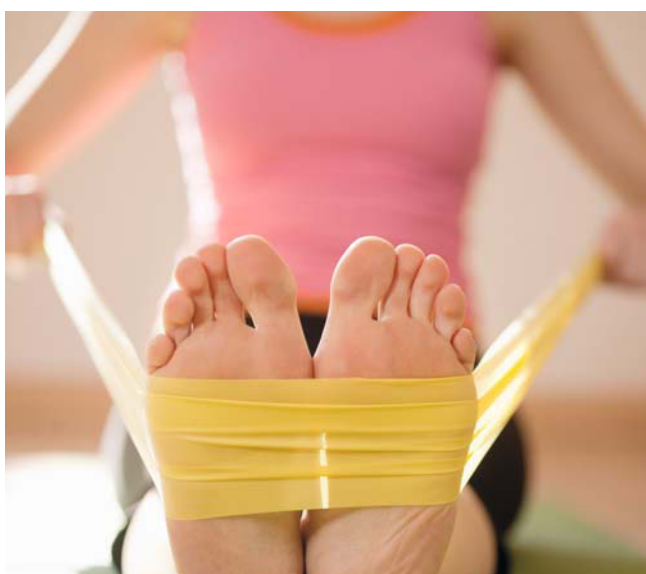

NEW!

New Histoacryl® Flexible

The new generation of Histoacryl® glue has been designed to close and protect surgical wounds.

Due to its new formulation, Histoacryl® Flexible is specially suitable for long incisions:


- ▶ **Flexibility**
Allows closure of incisions up to 25 cm¹
- ▶ **Microbial barrier**
Polymerized Histoacryl® Flexible adhesive films are an effective microbial barrier²
- ▶ **Ease of use**
Ready to use product storable at temperatures below 25°C
- ▶ **Accuracy**
Special tip permits a better control and improved application
- ▶ **Excellent cosmetic results**
Cyanoacrylate based topical skin adhesives yield excellent cosmetic results^{3,4,5,6}

How to use


- ▶ Just a small amount is necessary to provide an effective wound closure⁷
- ▶ The polymerization of Histoacryl® Flexible starts immediately
- ▶ Edges must be held together for just 30 seconds⁷



Open the blister and take out the application tip

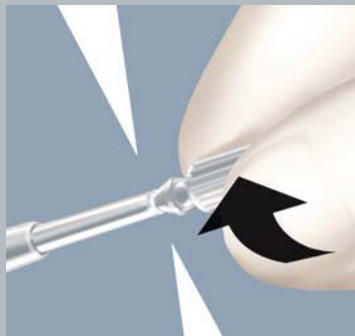

Open the ampoule by twisting off the ribbed tip

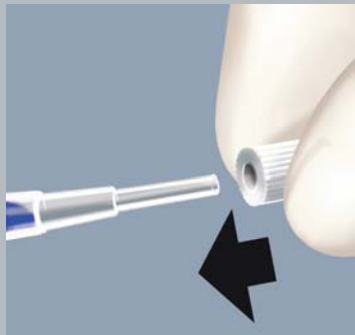

Attach the tip to opened ampoule



Apply the glue in a thin layer to the approximated wound edges and hold in apposition for 30 seconds

Histoacryl® Flexible

Improved closure of surgical incisions




Improved formula. Improved outcomes.

Compared to classical Histoacryl®

The new formula of Histoacryl® Flexible provides:

- ▶ **Closure of longer incisions**
Histoacryl® Flexible can be used to close surgical incisions of up to 25 cm¹
- ▶ **Enhanced flexibility**
Histoacryl® Flexible adhesive films adapt easier three-dimensionally⁸
- ▶ **More comfort**
Histoacryl® Flexible generates less heat during polymerization⁹
- ▶ **More adaptable**
Histoacryl® Flexible provides a more flexible coverage⁸
- ▶ **Easier to apply**
Histoacryl® Flexible contains an applicator tip that permits the easy creation of longer layers onto the surgical incision

Figure 1:
Comparison of flexibility (bending cycles) of
Histoacryl® Flexible vs. classical Histoacryl®

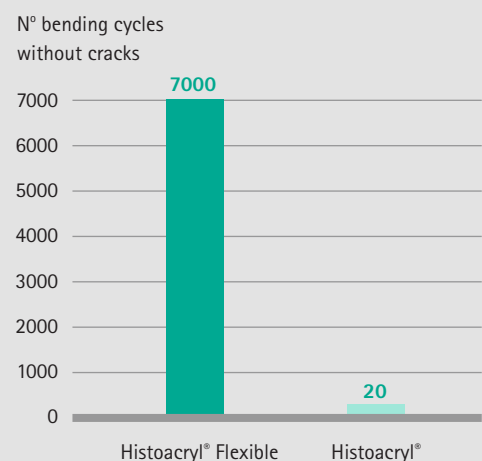


Compared to leading competitors in the market

The new Histoacryl® Flexible:

► Is stronger

Peel strength significantly higher ($p < 0.001$) for Histoacryl® Flexible than for Dermabond® High Viscosity⁹


Figure 2:
Peel strength comparison of Histoacryl® Flexible
vs. Dermabond® High Viscosity

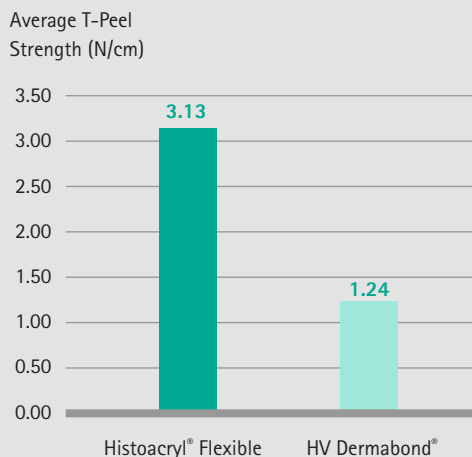

► Enhanced visibility

Histoacryl® Flexible is more intensely coloured than Dermabond® Advanced, thus allowing a better visualization during application of the product even if just one single layer is applied¹⁰

Figure 3:
Comparison of colour intensity of Histoacryl® Flexible vs.
Dermabond® Advanced

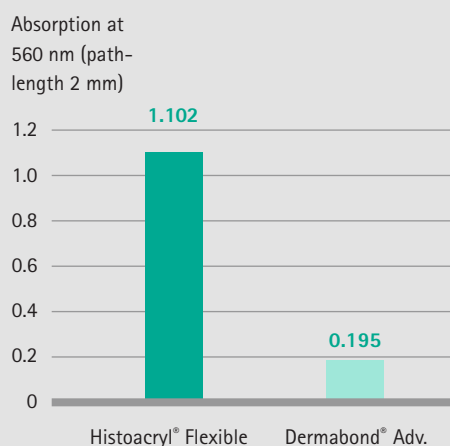

In addition to all these advantages, in vitro tests showed that Histoacryl® Flexible provided an **100 % effective antimicrobial barrier**^{*11} against microbial penetration for **7 days** and for the following bacteria²: *Staphylococcus aureus*, *Staphylococcus epidermidis*, *Escherichia coli*, *Pseudomonas aeruginosa*, *Enterococcus faecium*, *Brevundimonas diminuta* and *Candida albicans*

*In vitro results may not be representative of antimicrobial barrier properties in vivo.

Histoacryl® Flexible

Improved closure of surgical incisions



Skin Closure

Histoacryl® Flexible is suitable to close and protect the skin of small and long incisions* in a broad variety of surgical disciplines such as plastic surgery (mammoplasties, abdominoplasties, hand surgery), general surgery (inguinal hernia repair, colectomies), cardiovascular surgery (valve repair, CABG), etc.

*Subcutaneous suturing might be necessary according to the surgeon's criteria.

Ordering information

Code	Description
1051250P	5 ampoules of Histoacryl® Flexible (0.5 mL) and 5 tips per box
1051260P	10 ampoules of Histoacryl® Flexible (0.5 mL) and 10 tips per box



References

- ¹ Data on file, in vitro test with pork skin.
- ² Data on file, in vitro test according to: Bhende S, Rothenburger S, Spangler DJ, Dito M. In vitro assessment of microbial barrier properties of Dermabond topical skin adhesive. Surg Infect (Larchmt). 2002 Fall;3(3):251-7.
- ³ Amiel GE, Sukhotnik I, Kawar B, Siplovich L. Use of N-butyl-2-cyanoacrylate in elective surgical incisions – longterm outcomes. J Am Coll Surg. 1999 Jul;189(1):21-5.
- ⁴ Barnett P, Jarman FC, Goodge J, Silk G, Aickin R. Randomised trial of Histoacryl blue tissue adhesive glue versus suturing in the repair of paediatric lacerations. J Paediatr Child Health. 1998 Dec;34(6):548-50.
- ⁵ Simon HK, McLario DJ, Bruns TB, Zempsky WT, Wood RJ, Sullivan KM. Long-Term appearance of lacerations repaired using a tissue adhesive. Pediatrics. 1997 Feb;99(2):193-5.
- ⁶ Ellis DA, Shaik A. The ideal tissue adhesive in facial plastic and reconstructive surgery. J Otolaryngol. 1990 Feb;19(1):68-72.
- ⁷ Histoacryl® Flexible Instructions for Use.
- ⁸ Data on file, in vitro test in the style of the mandrel bend test ASTM D522 – 93 a (2008).
- ⁹ Data on file, in vitro test, internal SOP: AAVS 108 version 3.
- ¹⁰ Data on file, in vitro test according to ASTM F2256 – 05(2010).
- ¹¹ Data on file, determined by UV-spectrometry.



B. Braun Surgical SA | Carretera de Terrassa, 121 | 08191 Rubí | Spain
Phone +34 93 5 86 62 00 | Fax +34 93 6 99 63 30 | www.bbraun.com

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 7461 95-0 | Fax +49 7461 95-26 00 | www.aesculap.com

Aesculap – a B. Braun company

The main product trademark 'Aesculap' is a registered trademark of Aesculap AG.
The product trademark 'Histoacryl' is a registered trademark of B. Braun Melsungen AG.
'Dermabond' is a registered trademark of Ethicon.

Subject to technical changes. All rights reserved.
This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.