

Aesculap[®] Lyoplast[®] Onlay

Dura Substitution – Fast. Easy. Versatile. Reliable.



Aesculap Neurosurgery

Aesculap dura
substitution –
the next generation

Lyoplant® Onlay

Dura Substitution – Fast. Easy. Versatile. Reliable.

FAST

- Time-saving Onlay application
- Familiar use

EASY

- Convenient handling
- Thin material
- Excellent elasticity and flexibility
- Good adaptability to the defect and surrounding anatomical structures

VERSATILE

- Onlay or suturable application
- Suitable for cranial and spinal cases
- One dura substitute for many indications

RELIABLE

- Excellent biocompatibility
- Liquid-tight closure
- Reduces cortical adhesion
- Integrates with the body's own connective tissue cells
- Good tensile strength
- High suture pull-out strength




Lyoplant® Onlay is a biological, absorbable dura substitution consisting of a bilayer membrane, designed to provide high ease of use.

The product stands out due to the **fast** application, the **easy** handling, the **versatile** usage in terms of fulfilling the surgeons' needs and preferences and the **reliable** treatment for the patient.

It allows for a simple Onlay application with the possibility to incorporate suture fixation if necessary.


Biological bilayer membrane

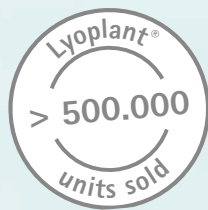

The **two layers** are not chemically crosslinked. The close connection between the two layers as well as the production of the materials itself are obtained by a very gentle lyophilisation (freeze-drying) process.

The **first layer** is a highly purified collagen element that is produced from bovine pericardium. It is the same strong and tear-proof material used for our well-established suturable dura substitution Lyoplant®.

Selling more than 500.000 units of Lyoplant® over now many years shows the proven trust in this product.

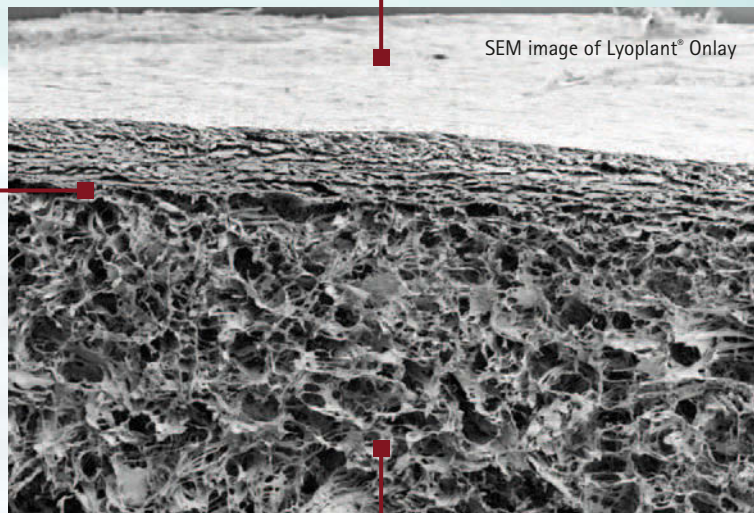


SEM image of Lyoplant® Onlay

The **second layer** is also a highly purified collagen element, made from bovine split hide. The fleece-like spongy quality of this layer allows the implant to adhere to the dura around the defect. Thus, Lyoplant® Onlay can be applied as an Onlay simply by laying the implant on the dura. This possibility of a suture-less closure of the dura defect can save valuable OR time.

Configurations

Sizes		Content	Art. No.
2.5 x 2.5 cm	1" x 1"	1 piece	106 7010
5.0 x 5.0 cm	2" x 2"	1 piece	106 7020
2.5 x 7.5 cm	1" x 3"	1 piece	106 7030
7.5 x 7.5 cm	3" x 3"	1 piece	106 7040
10.0 x 12.5 cm	4" x 5"	1 piece	106 7050

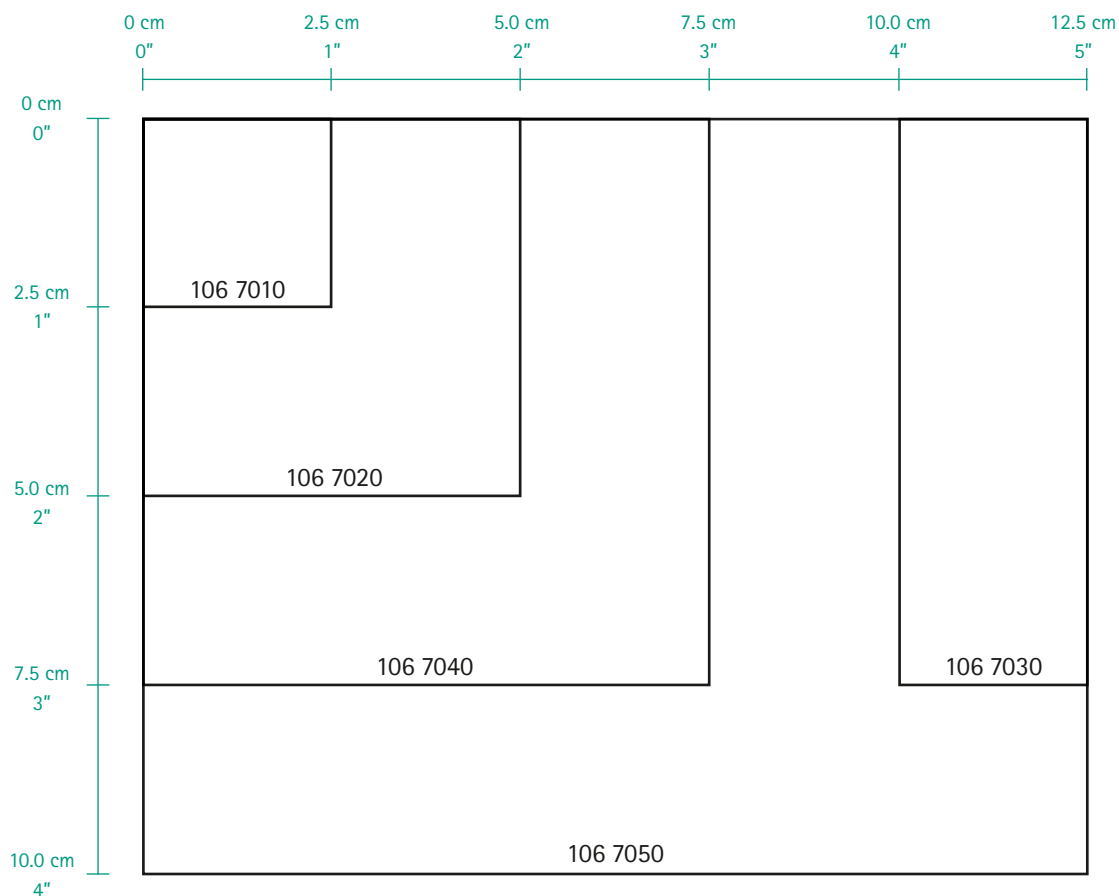

Mode of Application



CUT

- Lyoplast® Onlay can be cut in the required shape and size easily.
- Onlay technique:
The implant should overlap the dura defect by approx. 1 cm to ensure a high level of adhesion and a liquid-tight seal.
- Suturing:
The implant should be cut as closely as possible to the defect size.

REHYDRATE

- Ensure that the fleece-like, porous side (labeled 'DURA SIDE') is facing the dura. Which side has to face the dura should be identified before rehydration.
- Prior to implantation, Lyoplast® Onlay is placed in sterile saline solution or in another isotonic solution to ensure better suppleness and flexibility.

APPLY

- Onlay technique:
The implant simply has to be laid flat against the defect edges, ensuring that it is not taut.
- Suturing:
If required and if considered necessary by the user, Lyoplast® Onlay can be sutured in place. The implant should be fixed with non-absorbable suture material (polyester, polypropylene), using atraumatic round-bodied needles.
- The implant can be sealed with fibrin glue.



Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 7461 95-0 | Fax +49 7461 95-26 00 | www.aesculap.com

Aesculap – a B. Braun company

The main product trademark 'Aesculap' and the product trademark "Lyoplant" are registered trademarks of Aesculap AG.

Subject to technical changes. All rights reserved.
This brochure may only be used for the exclusive purpose of obtaining information about our products. Reproduction in any form partial or otherwise is not permitted.

Brochure No. C90102

0615/1.0/2