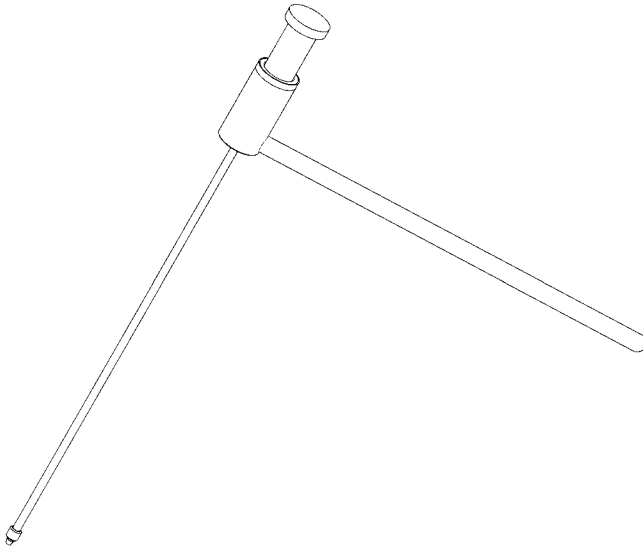
(D)

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MACS^{TL}-Kirschnerdraht-Instrumentarium

(GB)

Instructions for use

(USA)

MACS^{TL} Kirschner wire instrument set

(F)

Mode d'emploi

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6 X-ray marking

7 Handle

8 Kirschner wire removal instrument

9 Insertion instrument for removing Kirschner wire

10 MACS^{TL} insertion instrument set with screwed-on Kirschner wire removal instrument

Symbols on product

	Sterilization using irradiation
	This product is not for reuse in terms of its proper applications as defined by the manufacturer
	Use by
	Attention, see instructions for use

Intended use

The MACS^{TL} Kirschner wire instrument set is used for positioning the MACS^{TL} anchoring screws in the vertebral body. The position and direction of the anchoring screws can be determined by positioning a Kirschner wire under X-ray control. The insertion instrument itself is not visualized by radiography.

System components

Designation	Art. no.
Kirschner wire set in sterile packaging (2 Kirschner wires with threading and 1 removal instrument)	FW 343 S
Kirschner wire insertion instrument, in unsterile packaging	FW 330 R

Safe handling and preparation

CAUTION

Federal law restricts this device to sale by or on order of a physician!

Applying the MACS^{TL} Kirschner wire instrument set requires precise knowledge of spine stabilization and the biomechanical situation at the spine.

The surgical application of the MACS^{TL} spine system is described, in detail, in the respective O.R. manual.

- Read the O.R. manual for the MACS^{TL} Kirschner wire instrument set thoroughly, prior to first application.
- Inform the patient of the advantages and drawbacks of using the MACS^{TL} Kirschner wire instrument set.
- Read and follow the instructions for use and keep them in a safe place.
- Use the product only in accordance with its intended use, see Intended use.
- Clean the new product either manually or mechanically prior to the initial sterilization.
- Store any new or unused products in a dry, clean and safe place.
- Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functioning properly, not damaged, has intact insulation and does not have any loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

- Replace any damaged components immediately with original spare parts.
- Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.

Kirschner wire set

The product has been gamma-sterilized and packaged sterilely.

The product is not intended for reuse.

- Do not use products that are in open or damaged sterile packaging.
- Do not re-sterilize the product.
- Do not use the product after its use-by date.
- Use the Kirschner wire only as a temporary aid for installing the anchoring screws.
- Do not bend the Kirschner wire.
- Do not use the Kirschner wire in cases of severe osteopenia.
- Take account of the position of the anchoring screws when positioning the Kirschner wire in the bone.

Safe operation

WARNING

Risk of injury through contaminated cannulation of the MACS^{TL} insertion instrument set!

- Only use a MACS^{TL} insertion instrument set that has been reprocessed correctly.

WARNING

Risk of injury from improper application!

- Apply Kirschner wires in such a way that nerve roots, the spinal cord as well as blood vessels and organs are not injured in any way!

Prerequisites:

- Kirschner wire insertion instrument mounted
 - Plunger 4 pushed forward as far as it will go
 - New Kirschner wire has been prepared
- Insert the threading of the Kirschner wire 2 into the insertion instrument.
 - Screw the Kirschner wire lightly into the plunger 4.

A

B

- Position the insertion instrument parallel to the x-rays under radiographic control.

The insertion instrument is in parallel alignment with the direction of the X-ray as soon the metal housing shaft is projected as a point in the center of the X-ray marking 6, A. If the insertion instrument is not in proper alignment, the metal shaft is projected as a line, B.

The depth of penetration of the Kirschner wire is shown by the marking lines on the stop 5. The marking lines are 5 mm apart from each other. The stop with the marking lines 5 serves to limit maximum depth of penetration to 20 mm.

- Insert the Kirschner wire by lightly tapping on the plunger 4 until the desired depth of penetration is achieved.

CAUTION

The Kirschner wire will twist if the insertion instrument is tilted!

- Do not tilt the insertion instrument and Kirschner wire assembly.
- To unscrew the Kirschner wire insertion instrument: Rotate the plunger 4 in a counterclockwise direction.
- Keep the Kirschner wire vertical when pulling it out: do not allow it to tilt.

- Screw in the anchoring screws via the Kirschner wire, using the special MACS^{TL} insertion instrument set (see instructions for use of the MACS^{TL} insertion instrument set).
- Introduce removal instrument **8** into the cannulated MACS^{TL} insertion instrument set and screw it onto the Kirschner wire.
- Pull out the Kirschner wire with removal instrument **8** through the cannulated MACS^{TL} insertion instrument set.

Disassembling

- Pull the plunger **4** out of the housing **3**.
- Unscrew the handle **7** from the housing **3**.

Assembling

- Insert the plunger **4** into the housing **3**.
- Screw the handle **7** onto the housing **3**.

Care and handling

Possible damage to, or destruction of, the threads on the Kirschner wires and of the removal instrument, caused by cleaning, disinfecting or sterilizing!

- **Do not clean, disinfect, sterilize or reuse the MACS^{TL} Kirschner wire set.**

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of the products.

After each use

- Reprocess the contaminated product as soon as possible.
- For mechanical cleaning and disinfecting, place products in a suitable wire basket (avoid water-marks).
- Disassemble separable products, see Disassembling.
- Process the product preferably when dry.

- If it is processed while wet, use an active cleaning disinfectant. Prior to mechanical cleaning and disinfecting, rinse the product thoroughly in clear running water.
- If ultrasound is needed, carry out the procedure according to the instructions provided by the manufacturer of the apparatus:
 - as an effective mechanical supplement to manual cleaning.
 - for pre-cleaning of products with dried debris prior to mechanical cleaning.
 - for subsequent cleaning of products that still have dried debris on them after mechanical cleaning.
- Products with fine working tips must be cleaned and disinfected manually.
- Follow the manufacturer's instructions.

Manual cleaning, disinfecting

- Place the product in a suitable active cleaning and disinfectant solution in such a way that all surfaces, interior surfaces, lumen, and openings are covered. Follow the disinfectant manufacturer's instructions.
- After disinfecting chemically, rinse the product thoroughly in plenty of clear running water. Follow the disinfectant manufacturer's instructions.
- Remove encrusted materials with a soft nylon brush. Do not use harsh cleaning agents or metal brushes.
- Clean lumen and channels with a soft round nylon brush of the appropriate diameter.
- Carry out the final rinse in distilled or fully demineralized water.
- Dry the product with a soft, absorbent, lint-free cloth.
- Dry lumen and channels with compressed air.

Mechanical cleaning, disinfecting

- Reprocess the products using cleaning-optimized cycles. Follow the instructions provided by the manufacturer of the device.
- Carry out the final rinse in distilled or fully demineralized water.
- Be sure the drying phase is sufficiently long.
- Remove the product from the machine immediately after the cycle ends.

Control

- Allow the product to cool down to room temperature.
- Spray moving parts such as hinges and latches with a sterilizable and moisture-permeable lubricant (such as AESCULAP STERILIT spray JG 600 or maintenance oil JG 598).
- Assemble separable products, see Assembling.
- Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functioning properly, not damaged, has intact insulation and does not have any loose, bent, broken, cracked, worn, or fractured components.
- Set aside the product if it is damaged.

Storage

- Store the product with fine working tip in an appropriate support.

Sterilization method and parameters

- Sterilize with steam, taking note of the following: Sterilization has to be performed in a validated steam sterilization process (e.g. in a sterilizer according to EN 285/ANSI/AAMI/ISO 11134-1993, ANSI/AAMI ST46-1993, validated according to EN 554/ISO 13683). In case of application of the fractionated vacuum procedure the sterilization has to be carried out for a minimum of 5 minutes at 134 °C and at 2 bar pressure.

Sterilization for the US market:

- Sterilization of the device may be accomplished by steam.
- Aesculap does not recommend the device be sterilized by "Flash" or chemical sterilization.
- Surgical instruments may also be placed within an Aesculap rigid sterilization container (sterile container) for processing under generally accepted hospital in-use conditions.

The recommended sterilization parameters are as follows:

Sterilization method	Temp.	Minimum exposure time	
		Wrapped	In a sterile container system
Pre-vacuum	270—275 °F	4 min	4 min
Gravity	250—254 °F	15 min	40 min
	270—275 °F	10 min	30 min

WARNING for the US market

If this device is/was used in a patient with, or suspected of having Creutzfeldt-Jakob Disease (CJD), the device cannot be reused and must be destroyed due to the inability to reprocess or sterilize to eliminate the risk of crosscontamination!

Repairs

Service work, repairs or modifications to the product have to be carried out only by persons authorized by Aesculap. Only in this way will warranties and guarantees remain valid.

- If any repairs are needed, please send the product to:

AESCULAP Technischer Service
Am Aesculap-Platz
78532 Tuttlingen/Germany
Phone: +49 (7461) 95 16 02
Fax: +49 (7461) 16 28 87
E-mail: ats@aesculap.de

Or in the US:

AESCULAP Inc.
Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood
MO, 63042-2609
Aesculap Repair Hotline
Phone: +1 (800) 214-3392
Fax: +1 (314) 895-4420

Other service addresses can be obtained from the address indicated above.

Distributor in the US/Contact in Canada for product information and complaints

AESCULAP Inc.
3773 Corporate Parkway
Center Valley, PA, 18034,
USA

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Aesculap AG