Aesculap® Acculan

en Instructions for use/Technical description

Battery GA666, GA676

USA Note for U.S. users

This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusa.com. If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.

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Battery GA666, GA676

Symbols on product and packages

	Caution Observe important safety information such as warnings and precautions in the instructions for use.
	Two-dimensional machine readable code The code contains a unique serial number which can be used for electronic tracking of the individual instrument. The serial number is based on the global standard sGTIN (GS1).
	Manufacturer
	Date of manufacture
	Follow the instructions for use
	Labeling of electrical and electronic devices pursuant to directive 2002/96/EG (WEEE)
	Manufacturer's batch name
	Manufacturer serial number
	Manufacturer order number
	Temperature limits during transport and storage
	Air humidity limits during transport and storage
	Atmospheric pressure limits during transport and storage
Rx only	According to US federal law, this product may only be sold by a physician or by the order of a physician
	CE mark according to Regulation (EU) 2017/745
	Medical device
	Keep dry
	The battery is maintenance-free. Battery cells are subject to ageing and wear. Therefore, the battery cells have to be replaced after a certain period of use. The stamp provides a general point of reference. The "Battery replacement recommended" indicator on the charger is the definitive indicator for a replacement.

Remove the sticker before the initial charging of the battery

Contents

1.	About this document	3
1.1	Scope	3
1.2	Warnings	3
2.	General information	3
2.1	Use/intended use:	3
2.2	Main functions and design characteristics	3
2.3	Indications	3
2.4	Contraindications	3
3.	Safe handling	3
4.	Product description	3
4.1	Scope of Supply	3
4.2	Components required to operate the device	4
4.3	Operating principle	4
5.	Preparation	4
6.	Working with the device	4
6.1	System set-up	4
6.2	Functional test	4
6.3	Safe operation	4
7.	Processing process	4
7.1	General safety information	4
7.2	General notes	5
7.3	Reusable products	5
7.4	Preparing for cleaning	5
7.5	Cleaning/Disinfection	5
7.6	Wipe disinfection of electrical equipment	5
7.7	Inspection, maintenance and checks	5
8.	Maintenance	6
9.	Troubleshooting	6
10.	Storage and transport	6
11.	Accessories/Replacement Parts	6
12.	Technical Service	6
13.	Technical Data	6
13.1	Classification in accordance with Regulation (EU) 2017/745	6
13.2	Performance data, information and standards	6
13.3	Ambient conditions	7
14.	Disposal	7

1. About this document

1.1 Scope

These instructions for use apply to the products listed below:

Art. no.	Name
GA666	Acculan battery
GA676	Acculan battery

- For instructions for use of specific items and information on material compatibility, see B. Braun eIFU at eifu.bbraun.com

1.2 Warnings

Warnings indicate risks to patients, users, and/or the product that could develop while using the product. Warnings are marked as follows:

DANGER

Indicates a potential danger. If it is not prevented, death or severe injuries may result.

WARNING

Indicates a potential danger. Minor or moderate injuries may result if this is not prevented.

CAUTION

Indicates a potential damage to property. The product may become damaged if it is not prevented.

2. General information

2.1 Use/intended use:

Task/Function

The rechargeable batteries supply power to the Acculan units and contain the circuitry required to activate and control the electric motors of the Acculan units in accordance with operator commands.

Application Environment

The product fulfills the requirements of Type BF in accordance with IEC/DIN EN 60601-1.

For use in operating rooms, outside of the explosive hazard zone (e.g. areas with highly pure oxygen or anesthetic gases).

2.2 Main functions and design characteristics

Art. no.	DC voltage	Capacity
GA666	9.6 V	1.05 Ah
GA676	9.6 V	1.95 Ah

2.3 Indications

The indications are listed in the instruction for use of the relevant Acculan units.

2.4 Contraindications

The contraindications are listed in the instruction for use of the relevant Acculan units.

3. Safe handling

WARNING

Risk of injury and damage to property due to improper handling of the product!

- Comply with instructions for use for the Acculan 4 machines and the Acculan 3Ti machines.
- Follow the instructions for use of all products used.

WARNING

Fire and explosion hazard due to battery poles shorted by fluids or metal components!

- Do not short-circuit, open, crush or incinerate battery.

CAUTION

Damage to or destruction of the batteries due to reprocessing!

- Do not sterilize batteries.
- Inspect the new product after removing its transport packaging and prior to first use to ensure it is in good working order; fully charge using charger GA320 or GA677 and verify functionality.
- To prevent damage caused by improper setup or operation, and to not compromise the manufacturer warranty and liability:
 - Use the product according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
 - Follow the application advisories acc. to standard, see Extracts from relevant standards.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.
- Do not use the product if it is damaged or defective.
- Set aside the product if it is damaged.
- Charge product fully after 3 months in storage.
- Keep product dry.
- Observe the instructions for use for the corresponding Acculan machines.

Note

The user must report all serious incidents occurring in connection with the product to the manufacturer and to the competent authority of the country in which the user is registered.

4. Product description

4.1 Scope of Supply

Art. no.	Designation
GA666	Acculan NiMH battery short
	– or –
GA676	Acculan NiMH battery
TA011867	Instructions for use, Acculan battery GA666, GA676

4.2 Components required to operate the device

- Acculan 4 charger GA320 or Acculan 3Ti charger GA677
- Acculan 4 or Acculan 3Ti machine:

Art. no.	Name
GA330	Acculan 4 Drill and reamer
GA331	Acculan 4 Oscillating saw
GA334	Acculan 4 Reciprocating saw
GA336	Acculan 4 Oscillating saw straight
GA337	Acculan 4 Oscillating sternum saw
GA340	Acculan 4 Dermatome
GA341	Acculan 4 Dermatome 0.1 mm
GA344	Acculan 4 Small drill
GA668	Acculan 3Ti Oscillating saw straight
GA670	Acculan 3Ti Dermatome
GA671	Acculan 3Ti Small drill
GA672	Acculan 3Ti Drill and reamer
GA673	Acculan 3Ti Oscillating saw
GA674	Acculan 3Ti Reciprocating saw
GT666	Acculan 4 TPLO saw
GT668	Acculan 3Ti TPLO saw

4.3 Operating principle

The Acculan NiMH battery supplies the Acculan machine with voltage. The battery is charged in the Acculan 4 charger GA320 or Acculan 3Ti charger GA677. The charging process of a completely discharged battery can take up to 120 min.

The speed and rotational direction are set depending on the trigger setting of the machines:

- for saws and Dermatome: speed clockwise
- for drills: speed clockwise, counter-clockwise, oscillation and pilgrim step

5. Preparation

- Prior to use, inspect the product and its accessories for any visible damage.
- Use the products and their accessories only if they are in perfect technical condition.

6. Working with the device

6.1 System set-up

- See instructions for use for the respective Acculan machine and the Acculan 4 charger GA320 or Acculan 3Ti charger GA677.

6.2 Functional test

See the instructions for use for the respective Acculan machine.

6.3 Safe operation

⚠ WARNING

Risk of injury and material damage due to inappropriate use of batteries!

- Charge battery only in the Acculan 4 charger GA320 or Acculan 3Ti charger GA677.
- Insert battery correctly (Acculan machine and charger).

⚠ WARNING

Risk of injury and material damage due to damaged batteries!

- Check batteries for any damage before charging.
- Do not use or recharge damaged batteries.

⚠ CAUTION

Total battery failure can occur if the batteries are left in the charger while it is disconnected from the power supply!

- Always keep the charger connected to the power supply. Indicator light for readiness for service must be lit.
- Always remove batteries from the charger when it is disconnected from the power supply.

⚠ CAUTION

Loss of capacity/performance due to prolonged storage of discharged batteries!

- Leave the batteries in the functional charger after they have been successfully charged (trickle charging).
- For proper loading and removal of the battery from the Acculan machine, see the instructions for use for the respective Acculan machine.

7. Processing process

7.1 General safety information

Note

Observe national statutory regulations, national and international standards and directives, and any proprietary hygiene instructions for processing.

Note

The applicable national regulations concerning the reprocessing of devices must be observed for patients having Creutzfeldt-Jakob disease (CJD), suspected of having CJD, or possible variants of CJD.

Note

It should be noted that successful processing of this medical device can only be guaranteed following prior validation of the processing method. The operator/processor shall be responsible.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

See also B. Braun eIFU at eifu.bbraun.com for current information on processing and compatibility of material

7.2 General notes

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Optical changes of materials, e.g. fading or discoloration of titanium or aluminum. For aluminum, the application/process solution only needs to be of pH >8 to cause visible surface changes.
- Material damage such as corrosion, cracks, fracturing, premature aging or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surface and could cause corrosion.

7.3 Reusable products

The service life of the product is limited by damage, normal wear, type and duration of the application, handling, storage and transportation of the product.

Careful visual and functional testing prior to next use is the best way to identify a malfunctioning product.

7.4 Preparing for cleaning

- ▶ Remove the battery from the Acculan machine.

7.5 Cleaning/Disinfection

Device-specific safety information on the reprocessing method

⚠ DANGER

Risk of electric shock and fire hazard!

- ▶ Do not use flammable or explosive cleaning or disinfecting solutions.
- ▶ Ensure that no fluids will penetrate the product.

⚠ CAUTION

Damage to, or destruction of the product caused by mechanical cleaning/disinfection!

- ▶ Only clean and disinfect the product manually.
- ▶ Do not sterilize the product under any circumstances.

⚠ CAUTION

Risk of damage to the product due to inappropriate cleaning/disinfecting agents!

- ▶ Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.
- ▶ Do not introduce any fluid onto the electrical contacts of the battery.

7.6 Wipe disinfection of electrical equipment

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemistry
I	Cleaning	RT	0.5	-	-	17 % 1-Propanol. 0.23 % Didecyltrimethylammonium chloride*
II	Wipe disinfection	RT	≥3	-	-	17 % 1-Propanol, 0.23 % Didecyltrimethylammonium chloride*

RT: Room temperature

* Recommended: Meliseptol® Wipes sensitive (B. Braun)

Phase I

- ▶ Remove any visible residues with a disposable disinfectant wipe.

Phase II

- ▶ Wipe all surfaces of the optically clean product with a fresh, disposable disinfectant wipe.
- ▶ Observe contact time (at least 3min).

7.7 Inspection, maintenance and checks

- ▶ Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functional and undamaged.
- ▶ Set aside the product if it is damaged.

8. Maintenance

The display "Battery replacement recommended" on the Acculan 3Ti charger GA320 (see TA014535) indicates when the battery should be replaced due to reduced capacity.

No other maintenance measures must be performed on the battery.

For corresponding services, contact your national B. Braun/Aesculap agency, see Technical Service.

9. Troubleshooting

- Replace product: have defective products repaired by Aesculap Technical Service, see Technical Service.

Issue	Detection	Cause	Remedy
Battery not working	No signal tone when inserting the battery into the machine	Battery broken	Replace product.
Battery quickly empty	Symbol "Battery replacement recommended" illuminated on the charger at the end of the charging process	Battery cells at end of life	Have battery cells replaced by Aesculap Technical Service.

10. Storage and transport

⚠ WARNING

Risk of injury from burns and damage to product in case of short-circuit!

- **When storing the product, ensure that the product cannot short-circuit with other products or metal objects.**
- In order to prevent damage during transport, ship product in the original packaging and in an outer carton.
- If the original packaging is no longer available, ensure the following when packaging the product:
 - The battery contacts are protected against short-circuit.
 - The packaging completely encloses the product.
 - The product is secured against movement within the packaging.
- Completely charge the product upon receipt and before extended periods of storage.

11. Accessories/Replacement Parts

Art. no.	Designation
TA015514	Instructions for use, battery GA666, GA676 (A4)
TA011867	Instructions for use, battery GA666, GA676 (flyer)

12. Technical Service

⚠ DANGER

Danger to life of patients and users in case of malfunctions and/or failure of protective measures!

- **Do not perform any servicing or maintenance work under any circumstances while the product is being used on a patient.**

⚠ CAUTION

Modifications of medical devices may result in a loss of potential guarantee/warranty claims and forfeiture of applicable licenses.

- **Do not modify the product.**
- **Contact national B. Braun/Aesculap representative for service and repair.**

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 (7461) 95 -1601

Fax: +49 (7461) 14 -939

E-Mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

13. Technical Data

13.1 Classification in accordance with Regulation (EU) 2017/745

Art. no.	Designation	Class
GA666	Acculan battery	I
GA676	Acculan battery	I

13.2 Performance data, information and standards

	GA666	GA676
Cell type	NiMH	NiMH
DC voltage	9.6 V	9.6 V
Capacity	1.05 Ah	1.95 Ah
Weight	approx. 0.304 kg	approx. 0.425 kg
Dimensions (L x W x H)	approx. 120 mm x 43 mm x 50 mm	approx. 140 mm x 43 mm x 50 mm
EMC	IEC/DIN 60601-1-2	IEC/DIN 60601-1-2
Conforming to standard	IEC/DIN 60601-1	IEC/DIN 60601-1

13.3 Ambient conditions

	Operation	Transport and storage
Temperature	10 °C to 40 °C	-10 °C to 40 °C Maximum recommended duration of storage is 3 months; keep out of direct sunlight
Relative humidity	30 % to 75 %	10 % to 90 %
Atmospheric pressure	700 hPa to 1 060 hPa	500 hPa to 1 060 hPa

Note

The atmospheric pressure of 700 hPa correlates with a maximum operating altitude of 3 000 m.

14. Disposal

⚠ WARNING

Risk of infection from contaminated products!

- Observe national regulations when disposing of or recycling the product, its components, and their packagings.

Note

The operator must process the product before disposal, see Processing process.

The recycling pass can be downloaded from the Extranet as a PDF document under the respective article number. (The recycling pass includes disassembling instructions for the product, as well as information for proper disposal of components harmful to the environment.) Products carrying this symbol are subject to separate collection of electrical and electronic devices. Within the European Union, disposal is taken care of by the manufacturer as a free-of-charge service.

- For questions regarding the disposal of the product, please contact your local B. Braun/Aesculap representative, see Technical Service.

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13.1 (AB) 2017/745 yönetmeliği uyarınca sınıflandırma