AdTec® needle holder

GB

Instructions for use

USA

AdTec® needle holder

D

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AdTec® needle holder

F

Mode d'emploi

AdTec® needle holder

E

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AdTec® needle holder

I

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AdTec® needle holder

P

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AdTec® needle holder

NL

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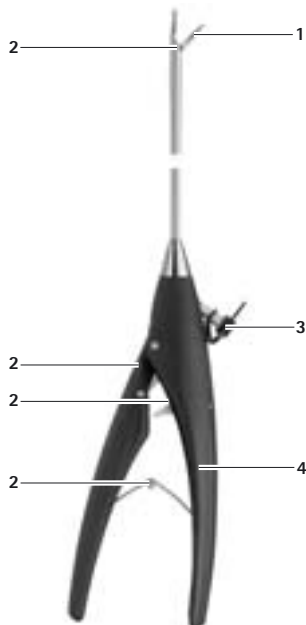
AdTec® needle holder

DK

Brugsanvisning

AdTec® needle holder

B | BRAUN

SHARING EXPERTISE

Aesculap AG

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78532 Tuttlingen

Germany

Phone +49 7461 95-0

Fax +49 7461 95-2600

www.aesculap.de

TA-Nr. 011172 07/11 V3 Änd.-Nr. 32163

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Jaw piece
- 2

Greasing points
- 3

Luer lock connector for cleaning channel
- 4

Handle

Intended use

The AdTec® needle holders are used for grasping and suturing in laparoscopic operations.
The applier forceps PL406R is used for fixating the counter-pressure plate of a circular stapler.

Available instruments

The AdTec® needle holders available to order are described in brochure C004 and at www.endoscopy-catalog.com

Safe handling and preparation

CAUTION

Federal law restricts this device to sale by or on order of a physician!

- Read, follow and keep the instructions for use.
- Use the product only in accordance with its intended use, see Intended use.
- Clean the new product either manually or mechanically prior to the initial sterilization.
- Store any new or unused products in a dry, clean and safe place.
- Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Replace any damaged components immediately with original spare parts.
- To avoid damage to the working tip: Carefully insert the product through the working channel (e.g. trocar).

Safe operation

Closing the jaw piece

- To close jaw piece 1 and grasp objects without engaging the lock on handle 4, slightly compress the handle.

Engaging the lock

- To lock handle 4 and jaw piece 1, fully compress the handle until the lock engages automatically with a click.
The instrument is locked now.

Releasing the lock

- Fully compress handle 4 until the lock is released automatically with a click.

Note

After releasing the lock, the handle must be opened half-way at least before the lock can be engaged again.

Validated processing procedure

Note

Observe all relevant national regulations and standards with regard to processing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of the products.

Note

Mechanical processing should be preferred over manual cleaning because of the better and more reliable cleaning results of mechanical processing.

Note

Successful processing of this medical product can only be ensured through a validated processing procedure. The user/processor is responsible for the validation.

Due to process tolerances, manufacturer's specifications can only serve as guide values for the processing procedures applied by the individual user.

Note

Up-to-date information on processing can be found on the Aesculap Extranet at www.aesculap-extra.net

General notes

Encrusted or fixated residues from surgery can make the cleaning process more difficult or ineffective, and can cause corrosion of stainless steels. Consequently the time interval between application and processing should not exceed 6 h, and neither should fixating pre-cleaning temperatures > 45 °C be applied nor any fixating disinfecting agents (active ingredient: aldehyde, alcohol) be used.

Excessive doses of neutralizers or basic detergents can cause chemical degradation and/or fading of the laser engraving on stainless steel.

Residues containing chlorine or chlorides, as found e.g. in surgical residues, tinctures, medicines, saline solutions, service water used for cleaning, and in cleaning/disinfecting agents, cause corrosion damage (pitting, stress corrosion) and thereby destruction of stainless steel products. To remove such residues, the products must be rinsed sufficiently with fully desalinated water and dried thoroughly.

Only use process chemicals recommended by the chemicals' manufacturers as effective cleaning/disinfecting agents compatible with the materials used in the respective product. Strictly observe all application instructions, e.g. regarding temperatures, concentrations, exposure times, etc. Failure to do so can result in the following problems:

- Material damage, e.g. corrosion, cracks, fracturing, premature aging or swelling.
- Further detailed advice on hygienically safe and material-/value-preserving reprocessing can be found at www.a-k-i.org, Publications, Red Brochure – Proper maintenance of instruments.

Preparations at the place of use

- Open jointed products.
- Remove the sealing cap from the Luer lock connector.
- Remove visible residues as completely as possible, using a lint-free, single-use cleaning tissue.

Preparation prior to cleaning

- Have the product dry in a disposal container and ready for immediate cleaning and disinfecting within 30 min after use.

Cleaning/Disinfecting

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

- Use cleaning and disinfecting agents according to manufacturer's instructions. The cleaning and disinfecting agents must
– be approved for plastic and high-grade steel,
– not attack softeners (e.g. silicone).
- Observe specifications regarding concentration, temperature and exposure time.

Manual cleaning/disinfecting

- Clean hinged or jointed products in open and closed positions.
- Check visible surface for residues after manual cleaning/disinfecting.
- Repeat the cleaning process if necessary.

Manual cleaning with immersion disinfecting and cleaning with brush

Stage	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Disinfecting cleaning	RT (cold)	15	2	D–W	BBraun Stabimed; aldehyde phenol and QAV free; pH = 9
II	Intermediate rinse	RT (cold)	1	–	D–W	–
III	Final rinse	RT (cold)	0.5	–	FD–W	–
IV	Drying	RT	–	–	–	–

D–W: Drinking water
FD–W: Fully desalinated water (demineralized)
RT: Room Temperature

Stage I

- Fully immerse the product in the disinfecting solution. Make certain that all accessible surfaces are moistened.
- Clean the product under running tap water, using a suitable cleaning brush if necessary, until all visible residues have been removed from the surface.
- Brush through all surfaces that are not accessible to visual inspection, e.g. in products with hidden crevices, lumens or complex geometries, for at least 5 min or until no more residues can be removed. Mobilize non-rigid components, e.g. set screws, links, etc. during cleaning.
- After that, thoroughly rinse (at least 5 times) with the cleaning solution these difficult to access areas of the product, using a single-use syringe (20 ml).
- To avoid the risk of corrosion, do not use a metal brush or other abrasives, which would damage the surfaces, for cleaning the product.

Stage II

- Rinse the product completely (all accessible surfaces) under running water.

Stage III

- Carry out a full rinse of the product (all accessible surfaces) under running water.
- Allow water to drip off for a sufficient length of time.

Stage IV

- Dry the product completely with compressed air.

Mechanical cleaning/disinfecting with manual pre-cleaning

Manual pre-cleaning with brush

Stage	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Disinfecting cleaning	RT (cold)	15	2	D–W	BBraun Stabimed; aldehyde phenol and QAV free; pH = 9
II	Intermediate rinse	RT (cold)	1	–	D–W	–

D–W: Drinking water
RT: Room Temperature

Stage I

- Fully immerse the product in the disinfecting solution. Make certain that all accessible surfaces are moistened.
- Clean the product with a suitable cleaning brush until all visible residues have been removed from the surface.
- Brush through all surfaces that are not accessible to visual inspection, e.g. in products with hidden crevices, lumens or complex geometries, for at least 5 min or until no more residues can be removed. Mobilize non-rigid components, e.g. set screws, links, etc. during cleaning.
- After that, thoroughly rinse (at least 5 times) with the cleaning solution these difficult to access areas of the product, using a single-use syringe (20 ml).
- To avoid the risk of corrosion, do not use a metal brush or other abrasives that would damage the surfaces for cleaning the product.

Stage II

- Rinse the product completely (all accessible surfaces) under running water.

Mechanical alkaline cleaning and thermal disinfecting

Machine type: Single-chamber cleaning/disinfecting machine without ultrasound

- Place the product on a tray that is suitable for cleaning (avoid rinsing blind spots).
- Connect components with lumens and channels directly to the injector carriage attachment.
- Store products with hinges or joints on the tray in such a way that the joints are kept open.

Stage	Step	T [°C/°F]	t [min]	Water quality	Chemical
I	Prerinse	< 25/77	3	D–W	–
II	Cleaning	55/131	10	FD–W	– Concentrate, alkaline: pH = 10.9 – < 5 % anionic tensides – 1 % solution: pH = 10.5
III	Neutralization	20/68	2	FD–W	– Concentrate, acid: pH = 2.6 Basis: Citric acid – 1 % solution: pH = 3.0
IV	Intermediate rinse	70/158	1	FD–W	–
V	Thermal disinfecting	94/201	10	FD–W	–
VI	Drying	90/194	40	–	–

D–W: Drinking water
FD–W: Fully desalinated water (demineralized)

Control, care and inspection

- Allow the product to cool down to room temperature.
- Lightly lubricate moving parts (e.g. joints and links) with a maintenance oil suitable for the respective sterilization process (e.g. Aesculap STERILIT® spray JG600 or maintenance oil JG598).
- Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functioning properly, not damaged, has intact insulation and does not have any loose, bent, broken, cracked, worn, or fractured components.
- Check for compatibility with associated products.
- Set aside the product if it is damaged.

Packaging

- Sort the product into its appropriate storage device or put it on a suitable tray.
- Package trays appropriately for the sterilization process (e.g. in Aesculap sterile containers).
- Pack the product in such a way that the packaging will prevent recontamination of the product in the period between reprocessing and reuse.

Sterilization method and parameters

- Make certain that all external and internal surfaces will be exposed to the sterilizing agent (e.g. by opening all valves and faucets).
- Validated sterilization process
 - Steam sterilization through fractionated vacuum process
 - Steam sterilizer acc. to EN 285/ANSI/AAMI/ISO 11134-1993, ANSI/AAMI ST 46-1993, validated acc. to EN 554/ISO 13683
 - Sterilization through fractionated vacuum process at 134 °C/holding time 5 min
- When sterilizing several products at the same time in one steam sterilizer: Make certain that the maximum allowable load capacity of the steam sterilizer, as specified by the manufacturer, is not exceeded.

Sterilization for the US market

- Sterilization of the device may be accomplished by steam.
- Aesculap does not recommend the device be sterilized by "Flash" or chemical sterilization.
- Surgical instruments may also be placed within an Aesculap rigid sterilization container (sterile container) for processing under generally accepted hospital in-use conditions.

The recommended sterilization parameters are as follows:

Sterilization method	Temp.	Minimum exposure time	
		Wrapped	In a sterile container system
Pre-vacuum	270—275 °F	4 min	4 min

WARNING for the US market

If this device is/was used in a patient with, or suspected of having Creutzfeldt–Jakob Disease (CJD), the device cannot be reused and must be destroyed due to the inability to reprocess or sterilize to eliminate the risk of cross-contamination!

Storage

- Store processed products in germ-proof packaging in a dust-protected, dry, dark and temperature-controlled room.

Technical Service

For service, maintenance or repairs, please contact your national B. Braun/Aesculap agency.
Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 7461 95-2700
Fax: +49 7461 16-2887
E-mail: ats@aesculap.de

Or in the US:
Aesculap Inc.
Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood, MO 63042
Aesculap Repair Hotline
Phone: +1 800 214-3392
Fax: +1 314 895-4420
Other service addresses can be obtained from the address indicated above.

Accessories/Spare parts

Designation	Art. no.
Sealing cap for Luer Lock connector (pack of 20)	EJ751251

Distributor in the US/Contact in Canada for product information and complaints

Aesculap Inc.
3773 Corporate Parkway
Center Valley, PA 18034
USA

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