

How to use the cassette pack

1. Take off the protective cap, which is intended for protection during transport.



2. Cut off pearl at the end of the thread.



3. Pull out approximately 10 cm of suture material, cut it off and discard. The suture inside the bottle is now ready to use.



4. To preserve the contents put on the metal cap after use.



All the suture materials presented in cassette pack are also available in a wide variety of needles, suture sizes and lengths. Please contact your local Distributor for more information about the very comprehensive product range of B. Braun Surgical.

Veterinary Sutures for large animals



B. Braun Surgical



B. Braun Surgical has a complete veterinary suture range to suit the specific needs of large animals surgery.

Distributed by:

B | BRAUN
AESCULAP



B. Braun Surgical GmbH
D-34209 Melsungen/Germany
Contact-Address:
AESCULAP AG & CO. KG
P.O. Box 40
78501 Tuttlingen/Germany
Phone+49 (7461) 95-0
Fax +49 (7461) 95-2600

B | BRAUN

The veterinary cassette pack range of B. Braun Surgical

For many years, the cassette range of B. Braun Surgical has been recognised by Veterinary Surgeons throughout the world as offering:

- a full range of suture materials of consistent high quality
- a secure, convenient and economical suture delivery system

Wet Catgut plain

Natural absorbable suture made of pure collagen.
Presented in 30 ml. solution with isopropyl alcohol content.

- excellent quality with high knot-pull tensile strength
- good knot holding due to special surface finish

Wet Catgut chrom

Natural absorbable suture made of pure collagen and treated with chromic salts to retard the absorption process.
Presented in 30 ml. solution with isopropyl alcohol content.

- excellent quality with high knot-pull tensile strength
- good knot holding due to special surface finish.

Supramid

Synthetic non-absorbable pseudo-monofilament of polyamide.
Presented in 30 ml. solution with isopropyl alcohol content.

- good tensile strength
- smooth tissue passage
- good tissue compatibility
- exceptional handling qualities because of the combined characteristics of monofilament and polyfilament thread.



Dafilon®

- Synthetic non-absorbable monofilament of polyamide
- good knot-pull tensile strength
 - secure knot holding
 - smooth tissue passage
 - good tissue compatibility

Synthofil®

- Synthetic non-absorbable braided and coated polyester.
- high knot-pull tensile strength
 - long term tensile strength retention
 - excellent knot security
 - good tissue compatibility




Silkam

- Natural non-absorbable braided and coated suture of silk filaments.
- excellent knot security
 - exceptional handling characteristics



Steelex®

- Non-absorbable monofilament or polyfilament of stainless steel.
- extremely high knot-pull tensile strength
 - excellent tissue compatibility

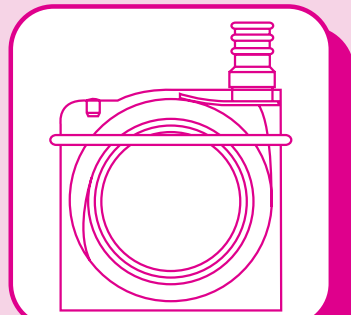


Recommended for:

- muscle
- intestine
- uterus
- skin
- ligature

USP: 6/0 - 6

Length: 100m - 25m

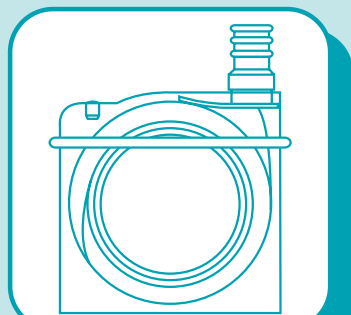


Recommended for:

- muscle
- intestine
- uterus
- skin
- ligature

USP: 6/0 - 6

Length: 100m - 25m

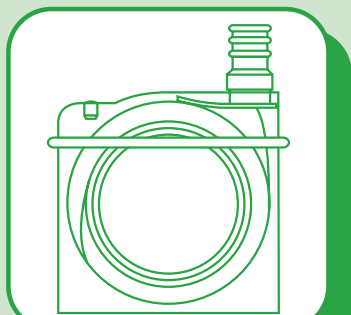

Recommended for:

- hernia
- intestine
- skin
- fascia

USP: 6/0 - 6


Length: 100m - 25m

Recommended for:

- skin

USP: 6/0 - 4

Length: 100m - 25m

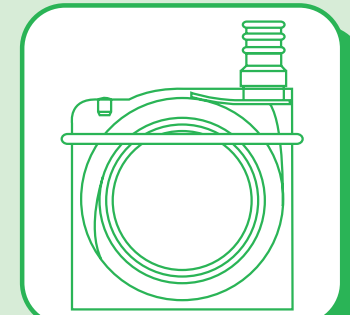


Recommended for:

- orthopaedics
- skin

USP: 6/0 - 4

Length: 100m - 25m

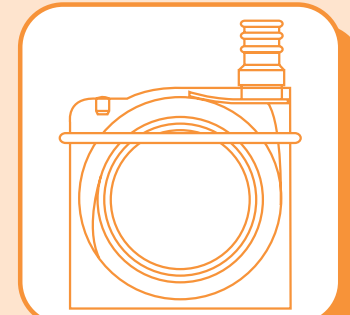


Recommended for:

- skin
- muscle

USP: 6/0 - 7

Length: 100m - 25m

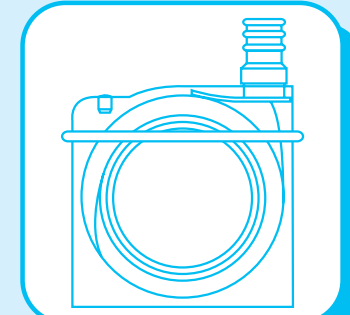

Recommended for:

- orthopaedics

USP: 5/0 - 3

Length: 50m - 20m

