

AESCULAP® STERILE GOODS MANAGEMENT

PROPER MAINTENANCE OF INSTRUMENTS

I SAFE HANDLING

- Use before sterilization only
- Evaporative cooling effect during use!
- Store in a dry place (JG598: avoid direct sun exposure / UV radiation)
- Do not use after the expiry date
- Medical devices must be reprocessed according to the instructions for use before STERILIT® is applied

II REMARKS

STERILIT® carries the CE label and fulfills the fundamental requirements in compliance with MDR 2017/745

III STERILIT® – MAINTENANCE OILS

- Base: paraffin/white oil
- Reduces friction and protects against fretting corrosion
- Physiological harmlessness checked according to ISO 10993
- Suitable for steam sterilization methods in conformance with ISO 17665-1

IV STEAM PENETRATION

The special formula ensures that the steam penetrates the oil film during the sterilization process, even in areas difficult to access

V PROPERTIES

In accordance with the requirements of the Instrument Preparation Working Group (A.K.I.), STERILIT® possesses the following properties:

- Steam penetrable oil film
- Oil film does not interfere with the effectiveness of the sterilization
- Lubricates and protects against corrosion
- Silicone-free, no staining or crust formation on instrument surfaces

i INFORMATION

Further information can be found in the "Reprocessing of Instruments to Retain Value" brochure.

This is available for download at www.a-k-i.org

Details on conducted tests and explanatory notes are available on demand. Please contact your AESCULAP® representative.

JG600 STERILIT® OIL SPRAY

Including spray nozzles, AESCULAP® oil spray, in spray cans, for instrument oiling prior to sterilization.




JG598 STERILIT® DRIP FEED OILER

AESCULAP® instrument oil, for drip feed oiling of joints, ratches, etc. prior to sterilization.



SCISSORS

Oiling a scissor joint

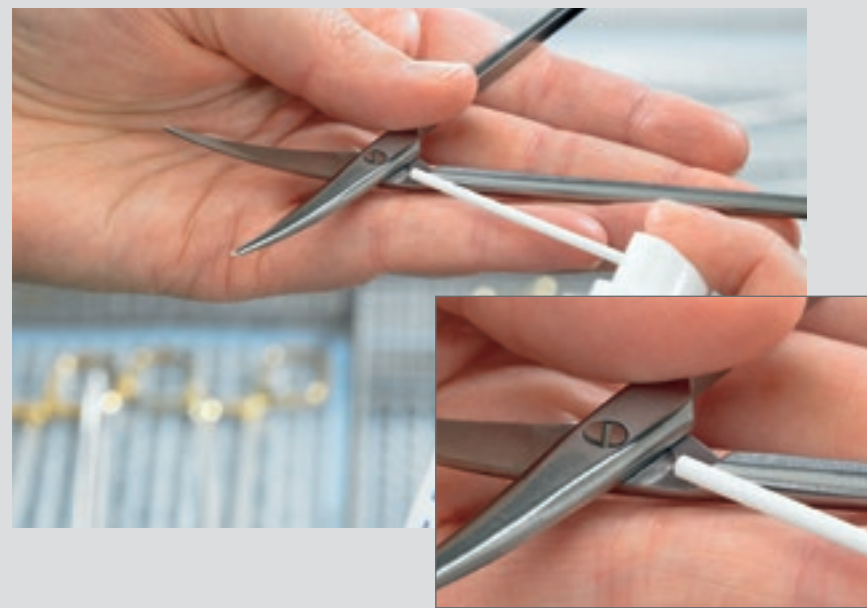

NEEDLE HOLDERS

Oiling a box joint



CLAMPS

Oiling a box joint



FORCEPS

Oiling screws on an articulated instrument



MICRO INSTRUMENTS

Oiling a micro needle holder



ENDOSCOPIC NEEDLE HOLDERS

Oiling a ratchet



BONE RONGEURS

Oiling a bone rongeur



BONE RONGEURS

Oiling a bone rongeur

