
- GB** **Instructions for use**  
MINOP®/Paediscscope single-use introducer for neuro-endoscopy
- USA** **Gebruiksaanwijzing**  
MINOP®/Paediscscope Einmal-Introducer für Neuro-Endoskopie
- D** **Mode d'emploi**  
Intubateur à usage unique MINOP®/Paediscscope pour neuro-endoscopie
- F** **Instrucciones de manejo**  
Introductor desechable MINOP®/Paediscscope para neuroendoscopia
- E** **Istruzioni per l'uso**  
Introduttore monouso per neuro-endoscopia MINOP®/Paediscscope
- I** **Instruções de utilização**  
MINOP®/introdutor descartável Paediscscope para endoscopia neurológica
- P** **Gebruiksaanwijzing**  
MINOP®/Paediscscope wegwerp-introducer voor neuro-endoscopie
- NL** **Bruksanvisning**  
MINOP®/Paediscscope engångs-introducer för neuro-endoskopi
- S** **Инструкция по применению**  
MINOP®/Paediscscope одноразовый интродьюсер для нейроэндоскопии
- RUS** **Návod k použití**  
MINOP®/Paediscscope –zavaděč k jednomu použití pro neuroendoskopii
- CZ** **Instrukcja użytkowania**  
Jednorazowy instrument do wprowadzania MINOP®/Paediscscope do neuroendoskopii
- PL** **Návod na používanie**  
MINOP®/pediskop jednorazový vkladáč pre neuroendoskopiu
- SK** **Kullanım Kılavuzu**  
MINOP®/Paediscscope nevro endoskopi için tek kullanımlık uygulayıcı
- TR**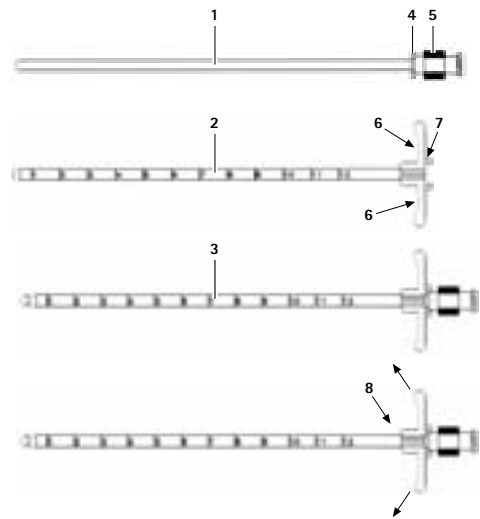

Aesculap®  
MINOP®/Paediscop single-use introducer for neuro-endoscopy

Legend

- 1 Obturator
- 2 Sheath
- 3 Obturator and sheath assembled and interlocked
- 4 Interlock flange
- 5 Release locking knob
- 6 Sheath handles
- 7 Set screws
- 8 Handles

Symbols on product and packages

|  |                                                                                         |
|--|-----------------------------------------------------------------------------------------|
|  | Sterilization using ethylene oxide                                                      |
|  | Not for reuse in intended applications as defined by the manufacturer                   |
|  | Use by                                                                                  |
|  | Caution, general warning symbol<br>Caution, see documentation supplied with the product |

Applicable to

- For item-specific instructions for use and information on material compatibility, see also the Aesculap Extranet at [www.extranet.bbraun.com](http://www.extranet.bbraun.com)

Intended use

The single-use introducer serves the creation and preservation of a temporary access canal to the patient's ventricular system. For the adjustment to different entry depths, the single-use introduce is constructed in such a way that the sheath covering can be cleaved axially and the sheath halves can be hinged away right up to the cranial surface. The single-use introducer consists of two main components: Obturator and sheath, which are interlocked.

Available sizes

| Single-use introducer | Size (interior part of sheath Ø) |
|-----------------------|----------------------------------|
| FH603SU               | 10 French (3.2 mm)               |
| FH604SU               | 19 French (6.1 mm)               |

| Single-use introducer | For use in conjunction with (typical combination) |
|-----------------------|---------------------------------------------------|
| FH603SU               | PaediScope PF010A                                 |
| FH604SU               | MINOP® 6 mm trocar FF399R                         |

*Note*  
These products are LATEX-FREE.

Safe handling and preparation

CAUTION  
Federal law restricts this device to sale by, or on order of a physician!

Infection hazard for patients and/or users and impairment of product functionality due to reuse. Risk of injury, illness or death due to contamination and/or impaired functionality of the product!

► Do not reprocess the product.

- The product is EO sterilized and sterile packed.  
The product must not be reused.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
  - Read, follow, and keep the instructions for use.
  - Use the product only in accordance with its intended use, see Intended use.
  - Do not use products from open or damaged sterile packaging.
  - Prior to each use, inspect the product for loose, bent, broken, cracked or fractured components.
  - Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
  - Do not re-sterilize the product.
  - Do not use the product after its use-by date.

Safe operation

- Turn the interlock flange 4 of the obturator, thereby interlocking the obturator into the set screws 7 of the sheath, in order to ensure that the obturator 1 is interlocked into the sheath 2. Compare image 3. The tip of the obturator protrudes from the sheath. At the end of the sheath, there is no visible gap between the obturator and the shaft.
- With the aid of the scale on the side of the obturator, determine the depth to which the introducer should be inserted.
- Insert the single-use introducer to the desired depth and unlock and remove the obturator. When doing this, ensure that the obturator remains sterile.
- Carefully insert the endoscope with pushing forward the sheath. It can be visually confirmed with the endoscope that the sheath is positioned correctly.
- If the sheath is not in the desired position, carefully remove the endoscope and the sheath.
- Insert the obturator into the sheath again and interlock.
- Repeat the ventricular insertion. After removing the obturator, the sheath end should remain in the ventricular system. This reduced the risk of the tip causing hemorrhage when the endoscope is pushed forwards.

The round sheath end may cause hemorrhages!

► Under no circumstances push the sheath forwards into the brain without the obturator being interlocked in the correct position.

► Lower the obturator completely and interlock it in the sheath.

- Two abacterial individuals are required for fixing the sheath.
- One person must hold the sheath depth steady.
- The second person holds the sheath handles 6 securely in place.
- As soon as the handles 8 are separated, the sheath splits lengthways.
- Split the sheath to just above the cranial bone.
- Fix the split sheath securely to the cranium by sewing or tacking both sheath halves. This creates the access canal to the ventricle.
- When introducing objects through the sheath, attention must always be paid to ensuring that the sheath does not move. Therefore, objects that barely fit through the sheath should not be inserted.

Validated reprocessing procedure

Single-use products

Risk of infection for patients and/or users and impairment of product functionality due to reuse. Risk of injury, illness or death due to contamination and/or impaired functionality of the product!

► Do not reprocess the product!

Storage

- Store sterile packed single-use products dust-protected in a dry, dark and temperature-controlled room.

Technical Service

Risk of injury and/or malfunction!

► Do not modify the product.

- For service and repairs, please contact your national B. Braun/Aesculap agency. Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

**Service addresses**  
Aesculap Technischer Service  
Am Aesculap-Platz  
78532 Tuttlingen / Germany  
Phone: +49 (7461) 95-1602  
Fax: +49 (7461) 16-5621  
E-Mail: [ats@aesculap.de](mailto:ats@aesculap.de)  
Or in the US:  
Attn. Aesculap Technical Services  
615 Lambert Pointe Drive  
Hazelwood  
MO, 63042  
Aesculap Repair Hotline  
Phone: +1 (800) 214-3392  
Fax: +1 (314) 895-4420  
Other service addresses can be obtained from the address indicated above.

Distributor in the US/Contact in Canada for product information and complaints

3773 Corporate Parkway  
Center Valley, PA, 18034,  
USA

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