MR Safety Information
MR Conditional

only for US-Users

If you have questions, please contact:

AESCLAP Inc.
3773 Corporate Parkway
Center Valley, PA 18034
Phone: +1-800-282-9000
Web: www.aesculapusa.com

The *proGAV 2.0* valve is MR Conditional.

Non-clinical testing demonstrated that the *proGAV 2.0* valve is MR Conditional.

A patient with this device can be safely scanned immediately after implantation in an MR system meeting the following conditions:

- Static magnetic field of 3 Tesla or less
- Maximum spatial gradient magnetic field of 1,400-Gauss/cm (14.0 T/m) or less
- Maximum MR system reported, whole body averaged specific absorption rate (SAR) of 4 W/kg (First Level Controlled Operating Mode)
- Do not take the *proGAV 2.0 Tools* into the MR environment. They are MR Unsafe.

Under the scan conditions defined above, the *proGAV 2.0* valve is expected to produce a maximum temperature rise of + 3°C after 15 minutes of continuous scanning.

In non-clinical testing, the image artifact caused by the device extends approximately 20 mm from the *proGAV 2.0* valve when imaged with a gradient echo pulse sequence and a 3-Tesla MR system.

To confirm that the valves setting has not been altered by exposure to the MRI scanner, the pressure setting of the adjustable unit can be checked with the *proGAV 2.0 Compass*.

Patients should register the conditions under which the implant can be scanned safely with the Medic Alert Foundation (www.medicalert.org) or equivalent organization.

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