


Surecan® Safety II


Power-injectable safety non-coring needles for Access Ports

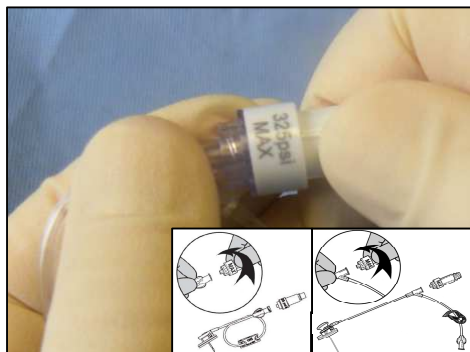


Fully tighten all connections or needle-less connectors before use.



Desinfect the area around the port with an antiseptic using a circular motion, one minutes contact is necessary.
Prepare the dressing pack using an aseptic technique. Put on gloves.
Disinfect the site for a second time and allow to dry.

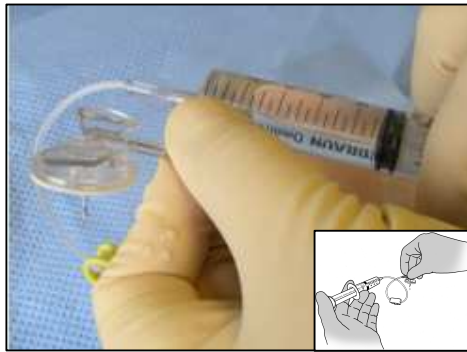


Attach a 10 mL syringe containing normal saline (NaCl 0.9%) to the proximal luer connection of the Surecan® Safety II infusion set.
Grasp the handle and remove the needle cover. Prime the infusion set.

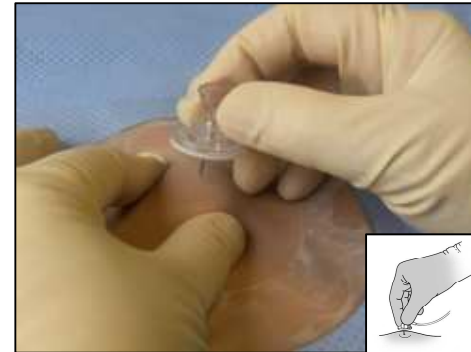


Locate and hold the access port firmly between two fingers. Hold the wings and insert the needle, at right angles to the skin, until the back of the titanium chamber is felt. Do not use excessive force as this could damage the needle tip, which then could damage the septum.




The needle should lie flat on the skin.
The needle wings are clipped to the base to reduce movement of the needle in the port.
Always check for blood reflux to confirm the patency of the catheter and correct positioning of the needle.

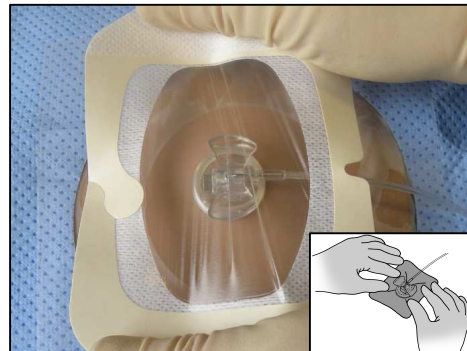

Place the tape over the wings to stabilise if necessary.
Apply a clear dressing over the needle and port site.
Attach the infusion line to the extension tube and start to infuse.

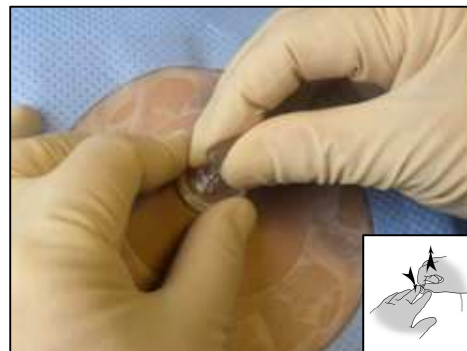

When the treatment is completed, flush the port according to the institutions protocol.
To remove the needle, stabilise the port by securely holding the needle base with two fingers. Firmly pull the wings up until the green point is visible.

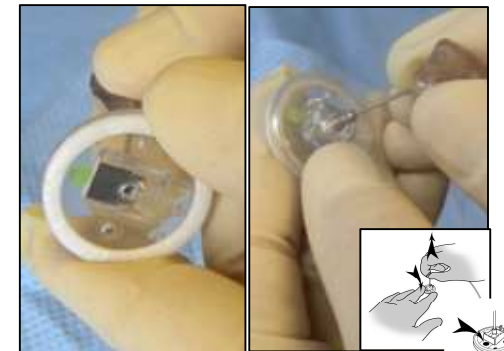

The needle is now locked in the safety position.
Dispose of the needle in a sharps container. (green "point" visible)

B | BRAUN
SHARING EXPERTISE