- (GB)** Instructions for use/Technical description
S⁴ Element instruments
- (USA)** S⁴ Element-Instrumente
- (D)** Gebrauchsanweisung/Technische Beschreibung
S⁴ Element-Instrumente
- (F)** Mode d'emploi/Description technique
Instruments S⁴ Element
- (E)** Instrucciones de manejo/Descripción técnica
Instrumental S⁴ Element
- (I)** Istruzioni per l'uso/Descrizione tecnica
Strumenti S⁴ Element
- (P)** Instruções de utilização/Descrição técnica
Instrumentos S⁴ Element
- (NL)** Gebruiksaanwijzing/Technische beschrijving
S⁴ Element-Instrumenten
- (S)** Bruksanvisning/Teknisk beskrivning
S⁴ Element-instrument
- (RUS)** Инструкция по применению/Техническое описание
Инструменты S⁴ Element
- (CZ)** Návod k použití/Technický popis
Nástroje S⁴ Element
- (PL)** Instrukcja użytkowania/Opis techniczny
Instrumenty S⁴ Element
- (SK)** Návod na použitie/Technický opis
Nástroje S⁴ Element
- (TR)** Kullanım Kılavuzu/Teknik açıklama
S⁴ Element-Araçları


B | BRAUN
SHARING EXPERTISE

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 (0) 7461 95-0 | Fax +49 (0) 7461 95-26 00 | www.aesculap.com

Aesculap – a B. Braun company

TA-Nr. 013272 2017-12 V6 Änd.-Nr. 57993

CE 0482 – DIR 93/42/EEC

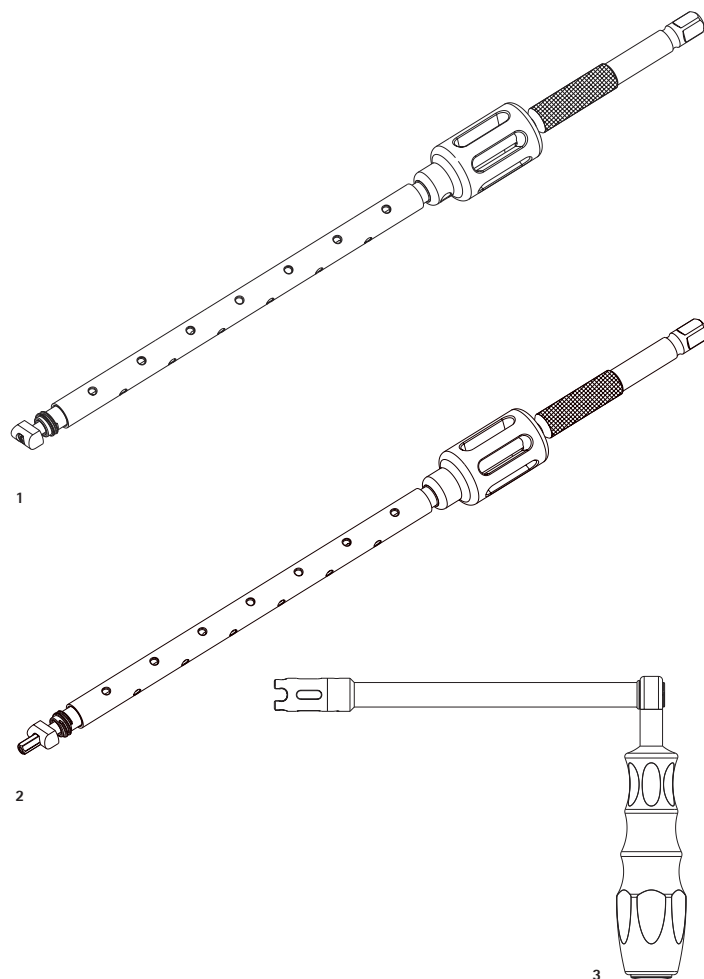Aesculap®

S⁴ Element instruments

Legend

- 1 S⁴ Element monoaxial screwdriver (FW276R)
- 2 S⁴ Element polyaxial screwdriver (FW277R)
- 3 S⁴ Element counter torque (FW283R)

Symbols on product and packages

	Caution, general warning symbol Caution, see documentation supplied with the product
	Date of manufacture

Applicable to

- For item-specific instructions for use and information on material compatibility, see also the Aesculap Extranet at <https://extranet.bbraun.com>

Intended use

The S⁴ Element instruments are used for the implantation of S⁴ Element implants.

Safe handling and preparation

CAUTION

Federal law restricts this device to sale by, or on order of a physician!

	Risk of injury caused by incorrect operation of the product! ► Adhere to the instructions for use of any medical device.
--	--

- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Read, follow, and keep the instructions for use.
- Use the product only in accordance with its intended use, see Intended use.
- Remove the transport packaging and clean the new product, either manually or mechanically, prior to its initial sterilization.
- Store any new or unused products in a dry, clean, and safe place.
- Prior to each use, inspect the product for loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Replace any damaged components immediately with original spare parts.
- To avoid damage to the working end: Carefully insert the product through the working channel (e.g. trocar).

Safe operation

	Risk of injury and/or malfunction! ► Always carry out a function check prior to using the product.
	Damage to the implant through use of the incorrect screwdriver! ► Use monoaxial screws only with monoaxial screwdriver FW276R. ► Use polyaxial screws only with polyaxial screwdriver FW277R.

- Note**
Do not use screwdriver FW277R or FW276R to apply bending forces or leverage.
- Note**
Do not overtighten the threaded shaft of FW277R or FW276R or tighten with mechanical aids.
- Note**
Do not use screwdriver FW277R/FW276R with torque wrenches or in combination with locking screws.
- Note**
The tips of the screwdrivers FW277R and FW276R are brittle and should not be used as a lever.
- Note**
Instrument parts detached from the instrument body should be removed from the patient because they are not implantable.

Validated reprocessing procedure

General safety instructions

- Note**
Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for sterile processing.
- Note**
For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.
- Note**
Mechanical reprocessing should be favored over manual cleaning as it gives better and more reliable results.
- Note**
Successful processing of this medical device can only be ensured if the processing method is first validated. The operator/sterile processing technician is responsible for this.
- The recommended chemistry was used for validation.

- Note**
If there is no final sterilization, then a virucidal disinfectant must be used.
- Note**
For up-to-date information about reprocessing and material compatibility, see also the Aesculap Extranet at <https://extranet.bbraun.com>
- The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

General information

- Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixating pre-cleaning temperatures >45 °C nor fixating disinfecting agents (active ingredient: aldehydes/alcohols) should be used.
- Excessive measures of neutralizing agents or basic cleaners may result in a chemical attack and/or to fading and the laser marking becoming unreadable visually or by machine for stainless steel.
- Residues containing chlorine or chlorides e.g. in surgical residues, medicines, saline solutions and in the service water used for cleaning, disinfection and sterilization will cause corrosion damage (pitting, stress corrosion) and result in the destruction of stainless steel products. These must be removed by rinsing thoroughly with demineralized water and then drying.
- Additional drying, if necessary.
- Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:
- Optical changes of materials, e.g. fading or discoloration of titanium or aluminum. For aluminum, the application/process solution only needs to be of pH >8 to cause visible surface changes.
 - Material damage such as corrosion, cracks, fracturing, premature aging or swelling.
 - Do not use metal cleaning brushes or other abrasives that would damage the product surfaces and could cause corrosion.
 - Further detailed advice on hygienically safe and material-/value-preserving reprocessing can be found at www.a-k-l.org, link to Publications, Red Brochure – Proper maintenance of instruments.

Preparations at the place of use

- If applicable, rinse non-visible surfaces preferably with deionized water, with a disposable syringe for example.
- Remove any visible surgical residues to the extent possible with a damp, lint-free cloth.
- Transport the dry product in a sealed waste container for cleaning and disinfection within 6 hours.

Cleaning/disinfection

Product-specific safety notes on the reprocessing procedure

	Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures! ► Use cleaning and disinfecting agents according to the manufacturer's instructions which – are approved for plastic materials, – do not attack softeners (e.g. in silicone). ► Observe specifications regarding concentration, temperature and exposure time.
--	---

- Use suitable cleaning/disinfecting agents if the product is put away in a wet condition. To prevent foam formation and reduced effectiveness of the process chemicals: Prior to mechanical cleaning and disinfection, rinse the product thoroughly with running water.
- Carry out ultrasound cleaning:
 - as an effective mechanical supplement to manual cleaning/disinfecting.
 - as a pre-cleaning procedure for products with encrusted residues, in preparation for mechanical cleaning/disinfecting.
 - as an integrated mechanical support measure for mechanical cleaning/disinfecting.
 - for additional cleaning of products with residues left after mechanical cleaning/disinfecting.
- Clean and disinfect microsurgical products mechanically if they can be placed securely in the machine or on the positioning aids.

Validated cleaning and disinfection procedure

Validated procedure	Specific requirements	Reference
Manual cleaning with immersion disinfection ■ Counter torque: – FW283R	■ Cleaning brush: e.g. TA011944, TE654202 or GK469200 ■ 20 ml disposable syringe ■ When cleaning products with movable hinges, ensure that these are in an open position and, if applicable, move the joint while cleaning. ■ Drying phase: Use a lint-free cloth or medical compressed air	Chapter Manual cleaning/disinfection and sub-chapter: ■ Chapter Manual cleaning with immersion disinfection
Manual cleaning with ultrasound and immersion disinfection ■ Screwdriver: – FW276R – FW277R	■ Cleaning brush: e.g. TA011944, TE654202 or GK469200 ■ 20 ml disposable syringe ■ Keep working ends open for cleaning. ■ When cleaning products with movable hinges, ensure that these are in an open position and, if applicable, move the joint while cleaning. ■ Drying phase: Use a lint-free cloth or medical compressed air	Chapter Manual cleaning/disinfection and sub-chapter: ■ Chapter Manual cleaning with ultrasound and immersion disinfection
Manual pre-cleaning with brush and subsequent mechanical alkaline cleaning and thermal disinfection ■ Counter torque: – FW283R	■ Cleaning brush: e.g. TA011944, TE654202 or GK469200 ■ 20 ml disposable syringe ■ Place the product in a tray that is suitable for cleaning (avoiding rinsing blind spots). ■ Connect components with lumens and channels directly to the rinsing port of the injector carriage. ■ Place products in the tray with their hinges open.	Chapter Mechanical cleaning/disinfection with manual pre-cleaning and sub-chapter: ■ Chapter Manual pre-cleaning with a brush ■ Chapter Mechanical alkaline cleaning and thermal disinfecting

Validated procedure	Specific requirements	Reference
Manual pre-cleaning with ultrasound and brush, and subsequent mechanical alkaline cleaning and thermal disinfection	- Cleaning brush: e.g. TA011944, TE654202 or GK469200 20 ml disposable syringe - Place the product in a tray that is suitable for cleaning (avoiding rinsing blind spots). - Connect components with lumens and channels directly to the rinsing port of the injector carriage. - Place products in the tray with their hinges open.	Chapter Mechanical cleaning/disinfection with manual pre-cleaning and sub-chapter: - Chapter Manual pre-cleaning with ultrasound and brush - Chapter Mechanical alkaline cleaning and thermal disinfecting

Manual cleaning/disinfection

Manual cleaning with immersion disinfection

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Disinfecting cleaning	RT (cold)	>15	2	D–W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Intermediate rinse	RT (cold)	1	–	D–W	–
III	Disinfection	RT (cold)	15	2	D–W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
IV	Final rinse	RT (cold)	1	–	FD–W	–
V	Drying	RT	–	–	–	–

D–W: Drinking water
FD–W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)
RT: Room temperature
*Recommended: BBraun Stabimed

- Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- Fully immerse the product in the cleaning/disinfectant for at least 15 min. Ensure that all accessible surfaces are moistened.
- Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- Drain any remaining water fully.

Phase III

- Fully immerse the product in the disinfectant solution.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- Rinse lumens at least 5 times at the beginning of the exposure time using an appropriate disposable syringe. Ensure that all accessible surfaces are moistened.

Phase IV

- Rinse/flush the product thoroughly (all accessible surfaces).
- Mobilize non-rigid components, such as set screws, joints, etc. during final rinse.
- Rinse lumens with an appropriate disposable syringe at least five times.
- Drain any remaining water fully.

Phase V

- Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air), see Validated cleaning and disinfection procedure.

Manual cleaning with ultrasound and immersion disinfection

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Ultrasonic cleaning	RT (cold)	>15	2	D–W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Intermediate rinse	RT (cold)	1	–	D–W	–
III	Disinfection	RT (cold)	15	2	D–W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
IV	Final rinse	RT (cold)	1	–	FD–W	–
V	Drying	RT	–	–	–	–

D–W: Drinking water
FD–W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)
RT: Room temperature
*Recommended: BBraun Stabimed

- Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- Clean the product in an ultrasonic cleaning bath (frequency 35 kHz) for at least 15 min. Ensure that all accessible surfaces are immersed and acoustic shadows are avoided.
- Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- Drain any remaining water fully.

Phase III

- Fully immerse the product in the disinfectant solution.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- Rinse lumens at least five times at the beginning of the exposure time with an appropriate disposable syringe. Ensure that all accessible surfaces are moistened.

Phase IV

- Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as set screws, joints, etc. during final rinse.
- Rinse lumens with an appropriate disposable syringe at least five times.
- Drain any remaining water fully.

Phase V

- Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air), see Validated cleaning and disinfection procedure.

Mechanical cleaning/disinfection with manual pre-cleaning

Note

The cleaning and disinfection device must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note

The cleaning and disinfection device used for processing must be serviced and checked at regular intervals.

Manual pre-cleaning with a brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Disinfectant cleaning	RT (cold)	>15	2	D–W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Rinsing	RT (cold)	1	–	D–W	–

D–W: Drinking water
RT: Room temperature
*Recommended: BBraun Stabimed

- Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- Fully immerse the product in the cleaning/disinfectant for at least 15 min. Ensure that all accessible surfaces are moistened.
- Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.

Manual pre-cleaning with ultrasound and brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Ultrasonic cleaning	RT (cold)	>15	2	D–W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Rinsing	RT (cold)	1	–	D–W	–

D–W: Drinking water
RT: Room temperature
*Recommended: BBraun Stabimed

- Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- Clean the product in an ultrasonic cleaning bath (frequency 35 kHz) for at least 15 min. Ensure that all accessible surfaces are immersed and acoustic shadows are avoided.
- Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.

Mechanical alkaline cleaning and thermal disinfecting

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemical
I	Prerinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	Concentrate, alkaline: - pH = 13 <5 % anionic surfactant 0.5 % working solution - pH = 11*
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disinfecting	90/194	5	FD-W	-
V	Drying	-	-	-	According to the program for cleaning and disinfection device

D-W: Drinking water
FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)
*Recommended: BBraun Helimatic Cleaner alkaline

- ▶ Check visible surfaces for residues after mechanical cleaning/disinfecting.

Inspection, maintenance and checks

CAUTION

Damage (metal seizure/friction corrosion) to the product caused by insufficient lubrication!

- ▶ Prior to function checks, lubricate moving parts (e.g. joints, pusher components and threaded rods) with maintenance oil suitable for the respective sterilization process (e.g. for steam sterilization: Aesculap STERILIT® I oil spray JG600 or STERILIT® I drip lubricator JG598).

- ▶ Allow the product to cool down to room temperature.
- ▶ After each complete cleaning, disinfecting and drying cycle, check that the product is dry, clean, operational, and free of damage (e.g. broken insulation or corroded, loose, bent, broken, cracked, worn, or fractured components).
- ▶ Dry the product if it is wet or damp.
- ▶ Repeat cleaning and disinfection of products that still show impurities or contamination.
- ▶ Check that the product functions correctly.
- ▶ Immediately put aside damaged or inoperative products and send them to Aesculap Technical Service, see Technical Service.
- ▶ Check for compatibility with associated products.

Packaging

- ▶ Place the product in its holder or on a suitable tray. Ensure that all cutting edges are protected.
- ▶ Pack trays appropriately for the intended sterilization process (e.g. in Aesculap sterile containers).
- ▶ Ensure that the packaging provides sufficient protection against recontamination of the product during storage.

Steam sterilization

- ▶ Check to ensure that the sterilizing agent will come into contact with all external and internal surfaces (e.g. by opening any valves and faucets).
- ▶ Validated sterilization process
 - Steam sterilization using fractionated vacuum process
 - Steam sterilizer according to DIN EN 285 and validated according to DIN EN ISO 17665
 - Sterilization using fractionated vacuum process at 134 °C/holding time 5 min
- ▶ When sterilizing several products at the same time in a steam sterilizer, ensure that the maximum load capacity of the steam sterilizer specified by the manufacturer is not exceeded.

Sterilization for the US market

- Aesculap advises against sterilizing the device by flash sterilization or chemical sterilization.
- Sterilization may be accomplished by a standard prevacuum cycle in a steam autoclave.

To achieve a sterility assurance level of 10⁻⁶, Aesculap recommends the following parameters:

Aesculap Orga Tray/Sterile container (perforated bottom) Minimum cycle parameters*			
Sterilization method	Temp.	Time	Minimum drying time
Prevacuum	270 °F/275 °F	4 min	20 min

*Aesculap has validated the above sterilization cycle and has the data on file. The validation was accomplished in an Aesculap sterile container cleared by FDA for the sterilization and storage of these products. Other sterilization cycles may also be suitable, however individuals or hospitals not using the recommended method are advised to validate any alternative method using appropriate laboratory techniques. Use an FDA cleared accessory to maintain sterility after processing, such as a wrap, pouch, etc.

WARNING for the US market

If this device is/was used in a patient with, or suspected of having Creutzfeldt-Jakob Disease (CJD), the device cannot be reused and must be destroyed due to the inability to reprocess or sterilize to eliminate the risk of crosscontamination.

Storage

- ▶ Store sterile products in germ-proof packaging, protected from dust, in a dry, dark, temperature-controlled area.

Technical Service

WARNING

Risk of injury and/or malfunction!

- ▶ Do not modify the product.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 (7461) 95-1602
Fax: +49 (7461) 16-5621
E-Mail: ats@aesculap.de

Or in the US:
Aesculap Implant Systems LLC
Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood
MO, 63042
Aesculap Repair Hotline
Phone: +1 (800) 214-3392
Fax: +1 (314) 895-4420
Other service addresses can be obtained from the address indicated above.

Disposal

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

Distributor in the US/Contact in Canada for product information and complaints

Aesculap Implant Systems LLC
3773 Corporate Parkway
Center Valley, PA, 18034,
USA

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