- GB** **Instructions for use/Technical description**
S⁴ Element Pedicle Screw System – Minimally Invasive Surgery Instrumentation (FW755R, FW762R)
- D** **Gebrauchsanweisung/Technische Beschreibung**
S⁴ Element Pedikelschraubensystem – Instrumentarium für die minimal-invasive Chirurgie (FW755R, FW762R)
- F** **Mode d'emploi/Description technique**
Système de vis pédiculaires S⁴ Element – instrumentation de chirurgie mini-invasive (FW755R, FW762R)
- E** **Instrucciones de manejo/Descripción técnica**
Sistema de tornillos pediculares S⁴, instrumental quirúrgico mínimamente invasivo (FW755R, FW762R)
- I** **Istruzioni per l'uso/Descrizione tecnica**
S⁴ Sistema di viti pedicolari Element – Strumentario chirurgico mini-invasivo (FW755R, FW762R)
- P** **Instruções de utilização/Descrição técnica**
Sistema de Parafusos Pediculares S⁴ Elemento – Instrumentos Cirúrgicos Minimamente Invasivos (FW755R, FW762R)
- NL** **Gebruiksaanwijzing/Technische beschrijving**
S⁴ Element pedikelschroefstelsel – instrumentarium voor minimaal invasieve chirurgie (FW755R, FW762R)
- S** **Bruksanvisning/Teknisk beskrivning**
S⁴ Element av pedikelskruvsystem – Instrument för minimalinvasiv kirurgi (FW755R, FW762R)
- RUS** **Инструкция по применению/Техническое описание**
Система педикулярных винтов S⁴ Element – Инструментарий для минимально инвазивной хирургии (FW755R, FW762R)
- CZ** **Návod k použití/Technický popis**
S⁴ Element systém pedikulárních šroubů – instrumentace pro miniinvasivní chirurgii (FW755R, FW762R)
- PL** **Instrukcja użytkowania/Opis techniczny**
S⁴ Element System śrub przeznasadowych – Instrumentarium do mini-inwazyjnych zabiegów chirurgicznych (FW755R, FW762R)
- SK** **Návod na používanie/Technický opis**
Systém pediklových skrutiek s prvkom S⁴ – minimálne invazívny chirurgický nástroj (FW755R, FW762R)
- TR** **Kullanım Kılavuzu/Teknik açıklama**
S⁴ Element Pedikül Vida Sistemi – Minimal İnvazif Cerrahi Aletler (FW755R, FW762R)

1

2

B | BRAUN
SHARING EXPERTISE

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 (0) 7461 95-0 | Fax +49 (0) 7461 95-26 00 | www.aesculap.com

Aesculap – a B. Braun company

TA-Nr. 013865 02/13 V6

CE – DIR 93/42/EEC

Technical alterations reserved

Aesculap®

S⁴ Element Pedicle Screw System – Minimally Invasive Surgery Instrumentation (FW755R, FW762R)

Legend

- 1 S⁴ Element MIS Insertion Key (FW755R)
- 2 S⁴ Element MIS Rod Persuader (FW762R)

Symbols on product and packages

	Caution, general warning symbol Caution, see documentation supplied with the product
	Date of manufacture

Applicable to

- ▶ For item-specific instructions for use and information on material compatibility, see also the Aesculap Extranet at www.extranet.bbraun.com

Intended use

The instruments FW755R and FW762R are intended to be used to insert and manipulate S⁴ Pedicle Screw System family implants in spinal fusion patients.

Safe handling and preparation

- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- ▶ Read, follow, and keep the instructions for use.
- ▶ Use the product only in accordance with its intended use, see Intended use.
- ▶ Remove the transport packaging and clean the new product, either manually or mechanically, prior to its initial sterilization.
- ▶ Store any new or unused products in a dry, clean, and safe place.
- ▶ Prior to each use, inspect the product for loose, bent, broken, cracked, worn, or fractured components.
- ▶ Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- ▶ Replace any damaged components immediately with original spare parts.

Safe operation

	Risk of injury and/or malfunction! ▶ Always carry out a function check prior to using the product.
--	--

Validated reprocessing procedure

General safety instructions

Note
Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for sterile processing.

Note
For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note
Mechanical reprocessing should be favored over manual cleaning as it gives better and more reliable results.

Note
Successful processing of this medical device can only be ensured if the processing method is first validated. The operator/sterile processing technician is responsible for this.
The recommended chemistry was used for validation.

Note
If there is no final sterilization, then a virucidal disinfectant must be used.

Note
For the latest information on reprocessing and material compatibility see also the Aesculap extranet at www.extranet.bbraun.com
The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

General information

Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixing pre-cleaning temperatures >45 °C nor fixing disinfecting agents (active ingredient: aldehydes/alcohols) should be used.

Excessive measures of neutralizing agents or basic cleaners may result in a chemical attack and/or to fading and the laser marking becoming unreadable visually or by machine for stainless steel.

Residues containing chlorine or chlorides e.g. in surgical residues, medicines, saline solutions and in the service water used for cleaning, disinfection and sterilization will cause corrosion damage (pitting, stress corrosion) and result in the destruction of stainless steel products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- ▶ Optical changes of materials, e.g. fading or discoloration of titanium or aluminum. For aluminum, the application/process solution only needs to be of pH >8 to cause visible surface changes.
- ▶ Material damage such as corrosion, cracks, fracturing, premature aging or swelling.

- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surfaces and could cause corrosion.
- ▶ Further detailed advice on hygienically safe and material-/value-preserving reprocessing can be found at www.a-k-i.org, link to Publications, Red Brochure – Proper maintenance of instruments.

Preparations at the place of use

- ▶ If applicable, rinse non-visible surfaces preferably with deionized water, with a disposable syringe for example.
- ▶ Remove any visible surgical residues to the extent possible with a damp, lint-free cloth.
- ▶ Transport the dry product in a sealed waste container for cleaning and disinfection within 6 hours.

Cleaning/disinfection

Product-specific safety notes on the reprocessing procedure

DANGER

Risk to patients!
▶ FW755R must only be cleaned mechanically.

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!
▶ Use cleaning and disinfecting agents according to the manufacturer's instructions which
– are approved for e.g. aluminum, plastics, high-grade steel,
– do not attack softeners (e.g. in silicone).
▶ Observe specifications regarding concentration, temperature and exposure time.
▶ Do not exceed the maximum allowable washing temperature of 90 °C.

Validated cleaning and disinfection procedure

Validated procedure	Specific requirements	Reference
Manual cleaning with immersion disinfection ■ FW762R	<ul style="list-style-type: none"> ■ Cleaning brush: e.g. TA011944, TE654202, GK469200 ■ Disposable syringe 20 ml ■ When cleaning products with movable hinges, ensure that these are in an open position and, if applicable, move the joint while cleaning. ■ Drying phase: Use a lint-free cloth or medical compressed air 	Chapter Manual cleaning/disinfection and sub-chapter: ■ Chapter Manual cleaning with immersion disinfection
Manual pre-cleaning with ultrasound and brush, and subsequent mechanical alkaline cleaning and thermal disinfection ■ FW755R ■ FW762R	<ul style="list-style-type: none"> ■ Cleaning brush: e.g. TA011944, TE654202, GK469200 ■ Disposable syringe 20 ml ■ Place the product in a tray that is suitable for cleaning (avoiding rinsing blind spots). ■ Connect components with lumens and channels directly to the rinsing port of the injector carriage. 	Chapter Mechanical cleaning/disinfection with manual pre-cleaning and sub-chapter: ■ Chapter Manual pre-cleaning with ultrasound and brush ■ Chapter Mechanical alkaline cleaning and thermal disinfection

Manual cleaning/disinfection

- ▶ Prior to manual disinfecting, allow water to drip off for a sufficient length of time to prevent dilution of the disinfecting solution.
- ▶ After manual cleaning/disinfection, check visible surfaces visually for residues.
- ▶ Repeat the cleaning /disinfection process if necessary.

Manual cleaning with immersion disinfection

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Disinfecting cleaning	RT (cold)	>15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Intermediate rinse	RT (cold)	1	-	D-W	-
III	Disinfection	RT (cold)	15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
IV	Final rinse	RT (cold)	1	-	FD-W	-
V	Drying	RT	-	-	-	-

D-W: Drinking water
FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)
RT: Room temperature
*Recommended: BBraun Stabimed

- ▶ Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- ▶ Fully immerse the product in the cleaning/disinfectant for at least 15 min. Ensure that all accessible surfaces are moistened.
- ▶ Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- ▶ If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- ▶ Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- ▶ Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- ▶ Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- ▶ Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- ▶ Drain any remaining water fully.

Phase III

- ▶ Fully immerse the product in the disinfectant solution.
- ▶ Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.
- ▶ Rinse lumens at least 5 times at the beginning of the exposure time using an appropriate disposable syringe. Ensure that all accessible surfaces are moistened.

Phase IV

- ▶ Rinse/flush the product thoroughly (all accessible surfaces).
- ▶ Mobilize non-rigid components, such as set screws, joints, etc. during final rinse.
- ▶ Rinse lumens with an appropriate disposable syringe at least five times.
- ▶ Drain any remaining water fully.

Phase V

- ▶ Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air), see Validated cleaning and disinfection procedure.

Mechanical cleaning/disinfection with manual pre-cleaning

Note

The cleaning and disinfection device must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note

The cleaning and disinfection device used for processing must be serviced and checked at regular intervals.

Manual pre-cleaning with ultrasound and brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Ultrasonic cleaning	RT (cold)	>15	2	D-W	Aldehyde-free, phenol-free, and QUAT-free concentrate, pH ~ 9*
II	Rinsing	RT (cold)	1	-	D-W	-

D-W: Drinking water

RT: Room temperature

*Recommended: BBraun Stabimed

- ▶ Note the information on appropriate cleaning brushes and disposable syringes, see Validated cleaning and disinfection procedure.

Phase I

- ▶ Clean the product in an ultrasonic cleaning bath (frequency 35 kHz) for at least 15 min. Ensure that all accessible surfaces are immersed and acoustic shadows are avoided.
- ▶ Clean the product with a suitable cleaning brush in the solution until all discernible residues have been removed from the surface.
- ▶ If applicable, brush through non-visible surfaces with an appropriate cleaning brush for at least 1 min.
- ▶ Mobilize non-rigid components, such as set screws, links, etc. during cleaning.
- ▶ Thoroughly rinse through these components with the cleaning disinfectant solution (at least five times), using a disposable syringe.

Phase II

- ▶ Rinse/flush the product thoroughly (all accessible surfaces) under running water.
- ▶ Mobilize non-rigid components, such as set screws, joints, etc. during rinsing.

Mechanical alkaline cleaning and thermal disinfecting

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemical
I	Prerinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	<ul style="list-style-type: none">■ Concentrate, alkaline:<ul style="list-style-type: none">- pH = 13- <5 % anionic surfactant■ 0.5 % working solution<ul style="list-style-type: none">- pH = 11*
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disinfecting	90/194	5	FD-W	-
V	Drying	-	-	-	According to the program for cleaning and disinfection device

D-W: Drinking water

FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)

*Recommended: BBraun Helimatic Cleaner alkaline

- ▶ Check visible surfaces for residues after mechanical cleaning/disinfecting.

Inspection, maintenance and checks

Damage (metal seizure/friction corrosion) to the product caused by insufficient lubrication!

- ▶ Prior to function checks, lubricate moving parts (e.g. joints, pusher components and threaded rods) with maintenance oil suitable for the respective sterilization process (e.g. for steam sterilization: Aesculap STERILIT® I oil spray JG600 or STERILIT® I drip lubricator JG598).

- ▶ Allow the product to cool down to room temperature.
- ▶ After each complete cleaning, disinfecting and drying cycle, check that the product is dry, clean, operational, and free of damage (e.g. broken insulation or corroded, loose, bent, broken, cracked, worn, or fractured components).
- ▶ Dry the product if it is wet or damp.

- ▶ Repeat cleaning and disinfection of products that still show impurities or contamination.
- ▶ Check that the product functions correctly.
- ▶ Immediately put aside damaged or inoperative products and send them to Aesculap Technical Service, see Technical Service.
- ▶ Check for compatibility with associated products.

Packaging

- ▶ Place the product in its holder or on a suitable tray. Ensure that all cutting edges are protected.
- ▶ Pack trays appropriately for the intended sterilization process (e.g. in Aesculap sterile containers).
- ▶ Ensure that the packaging provides sufficient protection against recontamination of the product during storage.

Steam sterilization

- ▶ Check to ensure that the sterilizing agent will come into contact with all external and internal surfaces (e.g. by opening any valves and faucets).
- ▶ Validated sterilization process
 - Steam sterilization through fractionated vacuum process
 - Steam sterilizer according to DIN EN 285 and validated according to DIN EN ISO 17665
 - Sterilization using fractionated vacuum process at 134 °C/holding time 5 min
- ▶ When sterilizing several products at the same time in a steam sterilizer, ensure that the maximum load capacity of the steam sterilizer specified by the manufacturer is not exceeded.

Storage

- ▶ Store sterile products in germ-proof packaging, protected from dust, in a dry, dark, temperature-controlled area.

Technical Service

Risk of injury and/or malfunction!

- ▶ Do not modify the product.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency. Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 (7461) 95-1602

Fax: +49 (7461) 16-5621

E-Mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

Disposal

- ▶ Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

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