GB

USA

Instructions for use

S<sup>4</sup> Spinal System – Torque wrenches/Break-off bit

D

F

E

I

P

NL

CZ

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S<sup>4</sup> Spinal System – Drehmomentschlüssel/Abbrechbit

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GB

USA

S<sup>4</sup> Spinal System – Torque wrenches/Break-off bit

Symbols on product

| Symbol                                                                            | Explanation                         |
|-----------------------------------------------------------------------------------|-------------------------------------|
|  | Attention, see instructions for use |

- Intended use**  
The S<sup>4</sup> Spinal System – torque wrenches are used to tighten the S<sup>4</sup> Spinal System – clamping screws with a defined torque:
- 10 Nm for tightening the clamping screw in the screw head/hook head (torque wrench FW168R for break-off bit)
  - 4 Nm for tightening the clamping screw in the transverse connector (torque wrench FW207R)

Safe handling and preparation

- Caution*  
*Federal law restricts this device to sale by or on order of a physician!*
- Read and follow the instructions for use and keep them in a safe place.
  - Read and follow the O.R. manual and keep it in a safe place.
  - Use the product only in accordance with its intended use, see Intended use.
  - Clean the new product either manually or mechanically prior to the initial sterilization.
  - Store any new or unused products in a dry, clean and safe place.
  - Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functioning properly, not damaged, has intact insulation and does not have any loose, bent, broken, cracked, worn, or fractured components.
  - Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
  - Replace any damaged components immediately with original spare parts.
  - Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.

- Safe operation**  
The torque wrenches FW168R for break-off bit and FW207R display torque when measuring.
- Before using the torque wrenches FW168R for break-off bit and FW207R check that the arrows point at the zero mark.
  - To avoid reading errors, always read off the torque display in parallel.

CAUTION

**Risk of damage to the implant by overwinding of the clamping screw!**

- Slowly tighten the clamping screw using the torque wrench FW168R for break-off bit, until the arrow points at 10 Nm.

CAUTION

**Risk of damage to the torque wrench FW168R for break-off bit due to improper usage!**

- To undo a tightened clamping screw always use screw driver FW193R.

CAUTION

**Damage to the countering instrument due to improper usage!**

- Always slide the countering instrument over the screw head onto the rod as far as it will go. In doing so, make sure that the rod entirely engages into the slot at the working tip.
- Make sure that the rod protrudes on both sides of the countering instrument.

WARNING

**Risk of injury/correction loss due to improper mounting of the clamping screw!**

- Set the clamping screw in place correctly.
- Make certain that rods are correctly positioned on the floor of the groove.
- When tightening the clamping screw always apply counter-pressure using the countering instrument intended for this purpose.
- Break off the flanks only when the clamping screw has been fully tightened.

WARNING

**Risk of injury due to improper mounting of the transverse connector!**

- When tightening the clamping screw make sure that the transverse connector completely bears on the rods.
- Fully tighten the clamping screw at the transverse connector using the torque wrench FW207R (4 Nm).

- Tighten the clamping screws, turning the torque wrench evenly in a clockwise direction.  
In doing so, make sure that the torque wrench is turned without jerk.
- The break-off bit protects the implant from excessive forces if the torque wrench FW168R is overturned.
- If the break-off bit breaks due to excessive forces, undo the defective break-off bit and send it in for repair.
  - Always screw on the break-off bit in such a way that the union nut of the break-off bit and the shaft of the torque wrench fit closely.

Care and handling

*Note*  
*For patients with Creutzfeldt-Jakob Disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of the products.*

After each use

- 

CAUTION

**Damage to the implant due to improper cleaning of the torque wrench FW168R for break-off bit!**

  - Always disassemble the break-off bit for cleaning and sterilization of the torque wrench.
- Reprocess the contaminated product as soon as possible.
  - For mechanical cleaning and disinfecting, place products in a suitable wire basket (avoid watermarks).
  - Process the product preferably when dry.
  - If it is processed while wet, use an active cleaning disinfectant. Prior to mechanical cleaning and disinfecting, rinse the product thoroughly in clear running water.
  - Do not use high-grade chlorine bleaching, as it might provoke pitting corrosion.
  - Carry out ultrasound treatment, if required, according to the user instructions provided by the manufacturer of the ultrasound equipment:
    - as an effective mechanical supplement to manual cleaning.
    - for pre-cleaning of products with dried debris prior to mechanical cleaning.
    - for subsequent cleaning of products that still have dried debris on them after mechanical cleaning.
  - Follow the manufacturer's instructions.

- Manual cleaning, disinfecting**
- Place the product in a suitable active cleaning disinfectant in such a way that all surfaces, interior surfaces, lumen, and openings are covered. Follow the disinfectant manufacturer's instructions.
  - After disinfecting chemically, rinse the product thoroughly in plenty of clear running water. Follow the disinfectant manufacturer's instructions.
  - Remove encrusted materials with a soft nylon brush. Do not use harsh cleaning agents or metal brushes.
  - Clean lumen and channels with a soft round nylon brush of the appropriate diameter.
  - Carry out the final rinse in distilled or fully demineralized water.
  - Dry the product with an absorbent and lint-free cloth.
  - Dry lumen and channels with compressed air.

- Mechanical cleaning, disinfecting**
- Reprocess the products using cleaning-optimized cycles. Follow the instructions provided by the manufacturer of the device.
  - Carry out the final rinse in distilled or fully demineralized water.
  - Be sure the drying phase is sufficiently long.
  - Remove the product from the machine immediately after the cycle ends.

- Control**
- Allow the product to cool down to room temperature.
  - Spray moving parts (such as hinges and latches) as well as the break-off bit with a sterilizable and moisture-permeable lightweight maintenance oil such as Aesculap Sterilit Spray JG 600 or maintenance oil JG 598).
  - Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functioning properly, not damaged, has intact insulation and does not have any loose, bent, broken, cracked, worn, or fractured components.
  - Set aside the product if it is damaged.

- Storage**
- Store the product in suitable storage aids.

- Sterilization method and parameters**
- Sterilize with steam, taking note of the following:  
The sterilization has to be done according to a validated steam sterilization procedure (e.g. in a sterilizer according to EN 285/ANSI/AAMI/ISO 11134-1993, ANSI/AAMI ST46-1993 and validated according to EN 554/ISO 13683). In case of application of the fractionated vacuum procedure the sterilization has to be carried out for a minimum of 5 minutes at 134 °C and at 2 bar pressure.
- Chemical residues which are not fully eliminated might cause staining.
- Be sure the drying phase is sufficiently long to avoid excessive moisture and resulting watermarks on the product.

- Sterilization for the US market**
- Sterilization of the device may be accomplished by steam.
  - Aesculap does not recommend the device be sterilized by "Flash" or chemical sterilization.
  - Surgical instruments may also be placed within an Aesculap rigid sterilization container (sterile container) for processing under generally accepted hospital in-use conditions.

The recommended sterilization parameters are as follows:

| Sterilization method | Temp.      | Minimum exposure time |                               |
|----------------------|------------|-----------------------|-------------------------------|
|                      |            | Wrapped               | In a sterile container system |
| Pre-vacuum           | 270–275 °F | 4 min                 | 4 min                         |

*Warning for the US market*  
*If this device is/was used in a patient with, or suspected of having Creutzfeldt-Jakob Disease (CJD), the device cannot be reused and must be destroyed due to the inability to reprocess or sterilize to eliminate the risk of cross-contamination!*

- Maintenance**  
The torque wrenches are calibrated by the manufacturer to 10 Nm or 4 Nm, respectively.
- If the arrow does not point at the zero mark with the torque wrench unstressed, it is damaged and must not be used in no account.
- Replace the damaged torque wrench.

- Repairs**  
Service work, repairs or modifications to the product must be carried out by personnel authorized by Aesculap only. Only in this way will warranties and guarantees remain valid.
- If any repairs are needed, please send the product to:  
Aesculap Technischer Service  
Am Aesculap-Platz  
78532 Tuttlingen / Germany  
Phone: +49 7461 95 27 00  
Fax: +49 7461 16 28 87  
E-mail: ats@aesculap.de  
Or in the US:  
Aesculap Inc.  
Attn. Aesculap Technical Services  
615 Lambert Pointe Drive  
Hazelwood  
MO, 63042  
Aesculap Repair Hotline  
Phone: +1 800 214-3392  
Fax: +1 314 895-4420

Other service addresses can be obtained from the address indicated above.

- Distributor in the US/Contact in Canada for product information and complaints**
- Aesculap Inc.  
3773 Corporate Parkway  
Center Valley, PA, 18034,  
USA

D

S<sup>4</sup> Spinal System – Drehmoment-schlüssel/Abbrechbit

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