GB	Instructions for use
USA	Sterile cover for Unitrac
D	Gebrauchsanweisung
	Sterilüberzug für Unitrac
F	Mode d'emploi
	Revêtement stérile pour Unitrac
E	Instrucciones de manejo
	Envoltorio estéril para Unitrac
I	Istruzioni per l'uso
	Rivestimento sterile per Unitrac
P	Instruções de utilização
	Resguardo esterilizado para Unitrac
NL	Gebruiksaanwijzing
	Steriele hoes voor Unitrac
DK	Brugsanvisning
	Sterilt overtræk til Unitrac
S	Bruksanvisning
	Sterilt hölje för Unitrac
CZ	Návod k použití
	Sterilní potah pro Unitrac

B | BRAUN

SHARING EXPERTISE

Aesculap AG
Am Aesculap-Platz
78532 Tuttlingen
Germany
Phone +49 7461 95-0
Fax +49 7461 95-2600
www.aesculap.de

TA-Nr. 013666 07/11 V5

CE marking according to directive 93/42/EEC
CE-Kennzeichnung gemäß Richtlinie 93/42/EWG
Marquage CE conforme à la directive 93/42/CEE
Identificación CE en conformidad con la directriz 93/42/CEE
Marchio CE conforme alla direttiva 93/42/CEE
Símbolo CE, em conformidade com a Directiva 93/42/CEE
CE-certificering conform richtlijn 93/42/EEG
CE-märkning iht. retningslinje 93/42/EEC
CE-märkning i enlighet med direktiv 93/42/EEG
CE-označení podle směrnice 93/42/EHS

Technical alterations reserved
Technische Änderungen vorbehalten
Sous réserve de modifications techniques
Sujeto a modificaciones técnicas
Con riserva di modifiche tecniche
Salvo alterações técnicas
Technische wijzigingen voorbehouden
Retten til tekniske ændringer forbeholdes
Med reservation för eventuella tekniska ändringar
Technické změny vyhrazeny

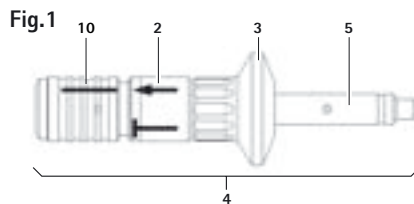
Fig.2

Fig.3

Fig.4

Fig.5

Fig.6

Fig.7

Fig.8

Fig.9

Fig.10

Fig.11

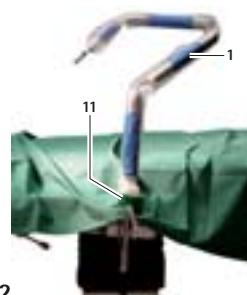
Fig.12

Legend

- 1 UNITRAC retractor arm
- 2 Mount - Quick-release coupling
- 3 Lock nut
- 4 UNITRAC Quick-release coupling
- 5 Coupling piece
- 6 Mount
- 7 Sterile cover
- 8 Carton blank
- 9 Adhesive strips for fixating the sterile cover
- 10 Sleeve
- 11 Four adhesive strips for fixating hoses and cables

Symbols on product and packages

Symbol	Explanation
	Sterilization using ethylene oxide
	Not for reuse in intended applications as defined by the manufacturer
	Use by
	Caution, general warning symbol Caution, see documentation supplied with the product
	Date of manufacture

Intended use

The sterile cover is used together with the quick-release coupling RT020R as a sterile cover for the UNITRAC retractor arm RT040R.

Safe handling and preparation

CAUTION

Federal law restricts this device to sale by or on order of a physician!

Infection hazard for patients and/or users and impairment of product functionality due to reuse. Risk of injury, illness or death due to contamination and/or impaired functionality of the product!

➤ Do not reprocess the product.

The product is EO sterilized and sterile packed.
The product must not be reused.

- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Read, follow, and keep the instructions for use.
- Use the product only for its intended purpose, see Intended use.
- Do not use products from open or damaged sterile packaging.
- Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Do not use the product after its use-by date.

Safe operation

- Anchor UNITRAC retractor arm **1** to the OP table track at maximum extension length (vertical), see **Fig. 1**.
- Turn lock nut **3** on UNITRAC quick-release coupling **4** all the way down in the direction of mount **2**, see **Fig. 2**.
- Insert UNITRAC quick-release coupling **4** with male coupling piece **5** into mount **6** on the UNITRAC retractor arm **1** until it audibly and visibly locks, see **Fig. 3**.
- Turn lock nut **3** on UNITRAC quick-release coupling **4** in the opposite direction until just before the end, see **Fig. 4**.
- Pull sterile cover **7** from above over UNITRAC quick-release coupling **4** and UNITRAC retractor arm **1**, see **Fig. 5**.
- Tear off carton blank **8**, which hangs when pulling over lock nut **3**, and dispose of it, see **Fig. 6**.
- Pull sterile cover **7** over UNITRAC retractor arm **1** until it can be fixated with adhesive strip **9** between lock nut **3** and mount **2**, see **Fig. 7**.
- Turn sleeve **10** clockwise to avoid unintentional opening of UNITRAC quick-release coupling **4**, see **Fig. 2**.
- Turn lock nut **3** tightly to full stop with mount **6** in order to guarantee a connection between UNITRAC quick-release **4** and UNITRAC retractor arm **1** that is free of clearance and wobbling, see **Fig. 9**
- Pull sterile cover **7** completely over UNITRAC retractor arm **1**, see **Fig. 10** and **Fig. 11**.
- As needed, use the four adhesive strips **11** to fixate hoses and/or cables on UNITRAC retractor arm **1** such that they do not disrupt the OP procedure, see **Fig. 12**.

Validated reprocessing procedure

Single-use products

Infection hazard for patients and/or users and impairment of product functionality due to reuse. Risk of injury, illness or death due to contamination and/or impaired functionality of the product!

➤ Do not reprocess the product.

Storage

- Store sterile packed single-use products dust-protected in a dry, dark and temperature-controlled room.

Technical Service

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 7461 95-1602
Fax: +49 7461 16-5621
E-Mail: ats@aesculap.de

Or in the US:
Aesculap Inc.
Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood, MO 63042
Aesculap Repair Hotline
Phone: +1 800 214-3392
Fax: +1 314 895-4420

Other service addresses can be obtained from the address indicated above.

Disposal

- Adhere to national regulations when disposing of or recycling the product, its components and its packaging!
- Incineration of the product, its components and its packaging produces harmless residues.

Distributor in the US/Contact in Canada for product information and complaints

Aesculap Inc.
3773 Corporate Parkway
Center Valley, PA 18034
USA

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