


AESCULAP® ZERLEGBARE KERRISON KNOCHENSTANZEN

1. | DEMONTAGE

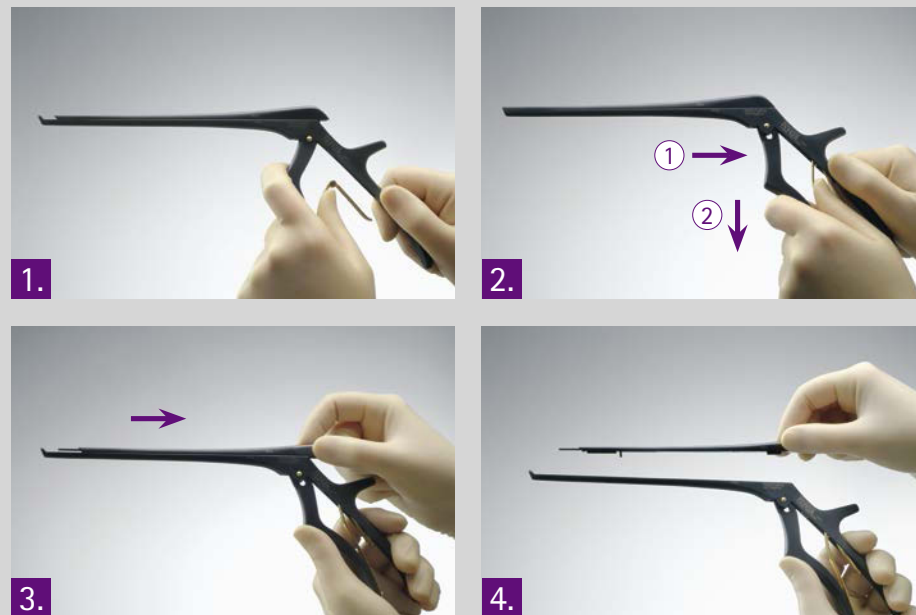


- Feder aus dem äußeren Griffteil aushängen.
- Griff komplett schließen und inneres Griffteil nach unten ziehen.
- Schiebeteil nach hinten abziehen.

2. | REINIGUNG



Bitte beachten Sie die Angaben in der Gebrauchsanweisung (TA010668).
→ <https://extranet.bbraun.com>

3. | MONTAGE

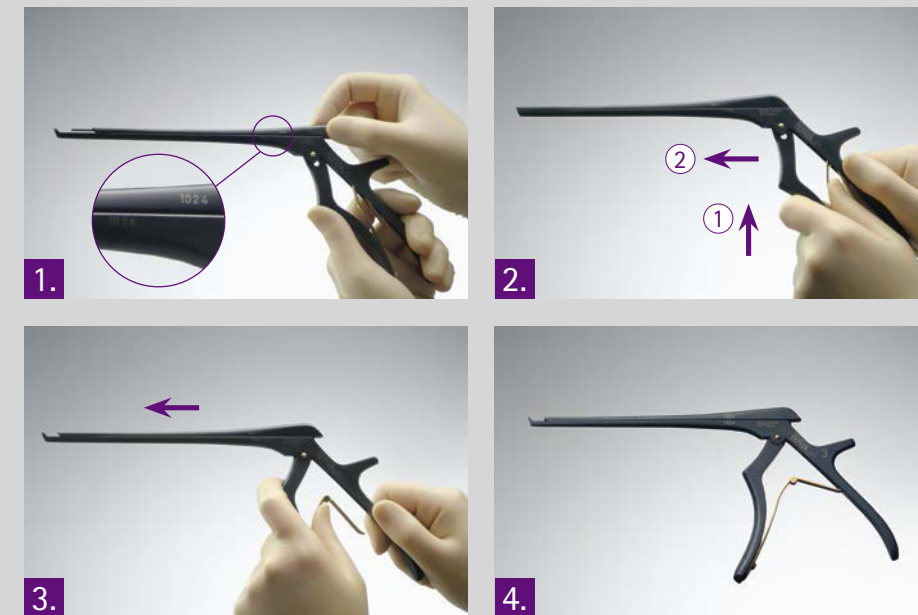

- Bitte den Paarungscode beachten.
- Griff komplett schließen, Schiebeteil aufsetzen und nach vorne schieben.
- Inneres Griffteil nach oben drücken und Griff öffnen.
- Feder in das äußere Griffteil einhängen.

4. | PFLEGE




Bewegliche Teile (z.B. Gelenk, Schiebeteillinflächen und Führungsbereiche) mit dampfdurchlässigem Sterilit® Pflegeöl JG598 gut ölen.

5. | FUNKTIONSKONTROLLE



Weitere Informationen erhalten Sie beim Aesculap Technischen Service
☎ Tel. 07461 95-1602
✉ repair.instruments@aesculap.de

6. | LAGERUNG

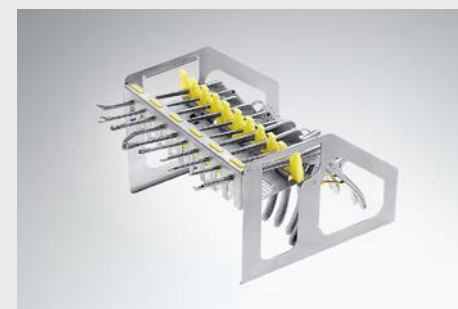

Instrument in geeigneter Lagerungshilfe (z. B. Lagerungsrahmen für Knochenstanzen JF120R) lagern.

7. | STERILISATION



Bitte beachten Sie die Angaben in der Gebrauchsanweisung (TA010668).
→ <https://extranet.bbraun.com>

WEITERE INFORMATIONEN
FINDEN SIE IN UNSEREN
PROSPEKTEN:

- Instrumenten Aufbereitung richtig gemacht
- C84801 Knochenstanzen
- C81901 Aesculap InOS® – Instrumenten Organisations-system
- C63401 Aesculap validierte Aufbereitungsverfahren


C22911 0417/0.75/3

AESCULAP® – a B. Braun brand

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Deutschland
Tel. 07461 95-0 | Fax 07461 95-2600 | www.aesculap.de

AESCULAP® DETACHABLE KERRISON BONE PUNCHES

1. | DISASSEMBLY

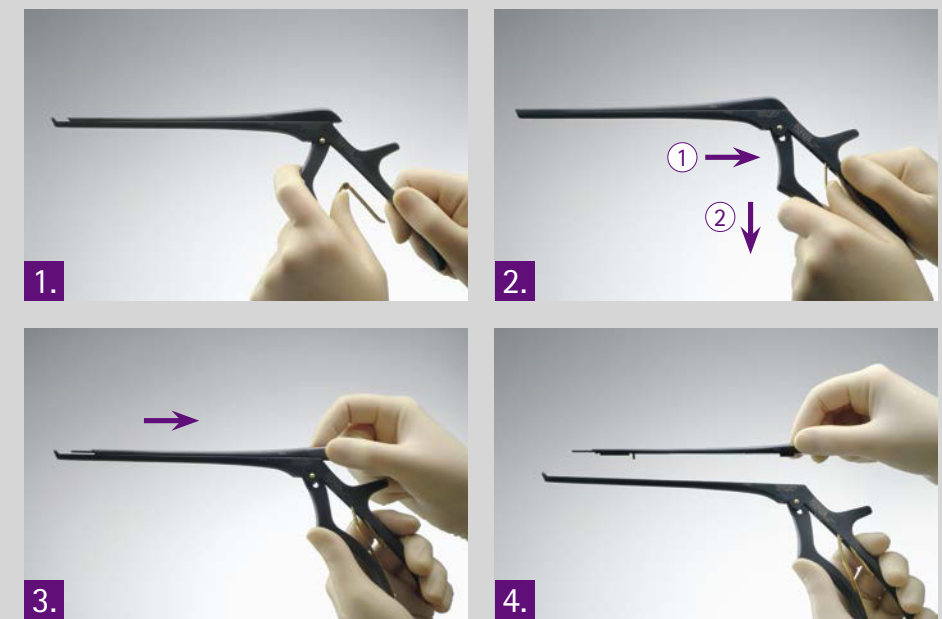


- Remove spring from outer part of handle.
- Close handle completely and pull down the inner part of the handle.
- Remove sliding part backwards.

2. | CLEANING



Please note the instructions for use (TA010668).
→ <https://extranet.bbraun.com>

3. | ASSEMBLY

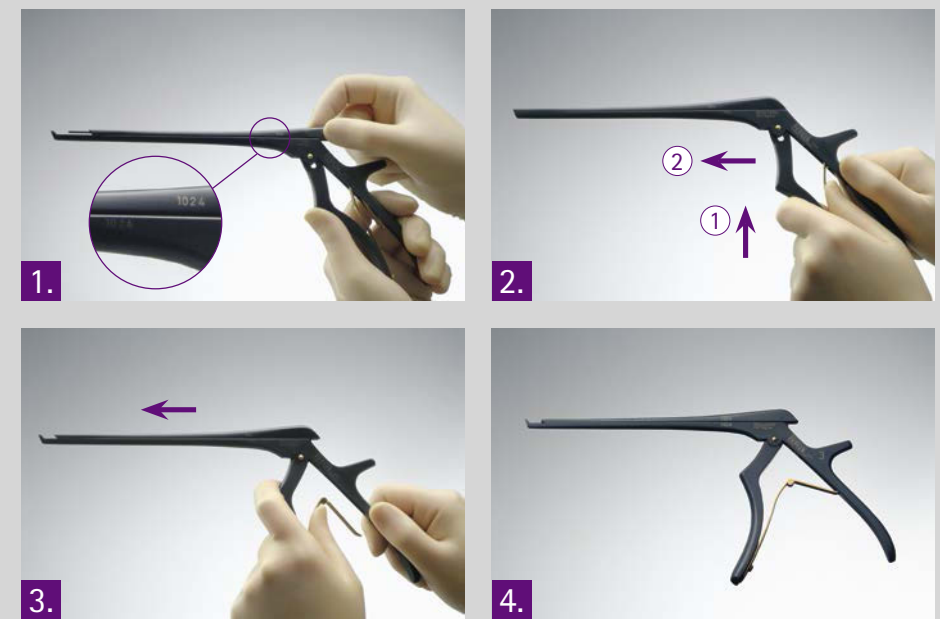

- Please note the pairing code.
- Put on sliding part and push it forward.
- Press up inside part of handle and open handle.
- Insert spring into outer part of handle.

4. | MAINTENANCE



Moving parts (hinges, latches and contact area of sliding part) should be lubricated with a sterilizable and moisture-permeable light weight machine oil such as Sterilit® maintenance oil JG598.

5. | FUNCTION CONTROL



For further information please contact the Aesculap Technical Service
☎ Phone +49 7461 95-1602
✉ repair.instruments@aesculap.de

6. | STORAGE



Keep instrument in appropriate storage tray (e.g. storage tray for bone punches JF120R).

7. | STERILIZATION



Please note the instructions for use (TA010668).
→ <https://extranet.bbraun.com>

FOR FURTHER INFORMATION
PLEASE SEE OUR BROCHURES:

- Proper Maintenance of Instruments
- C84802 Bone Punches
- C81902 Aesculap InOS® – Instrument Organisation System
- C63402 Validated Reprocessing Procedures

C22911 0417/0.75/3

AESCULAP® – a B. Braun brand

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 7461 95-0 | Fax +49 7461 95-2600 | www.aesculap.com