Rongeur

- GB

USA

D

F

E

I

CZ
- Instructions for use

Rongeur

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Rongeur

Mode d'emploi

Rongeur

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Rongeur

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Rongeur

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Rongeur

Intended use

The Rongeur are used for the ablation of soft tissue (e.g. intervertebral disk tissue, mucosa, ethmoidal cells, cell tissue, nucleus tissue, sequester material) in neurosurgery and spine surgery.

The Rongeur are not suitable for the ablation of bones or adjacent bone structures, for which purpose Aesculap recommends using bone punches because of their superior mechanical strength.

Available sizes

The Rongeur are available in various:

- jaw lengths
- jaw angulations
- blade orientations
- jaw widths
- jaw shapes
- working lengths of the pusher component
- handle designs (branch or ring)

Detailed information about the available sizes/variants can be found in our brochures.

Safe handling and preparation

CAUTION

Federal law restricts this device to sale by or on order of a physician!

- Read, follow and keep the instructions for use.
- Use the product only in accordance with its intended use, see Intended use.
- Clean the new product either manually or mechanically prior to the initial sterilization.
- Store any new or unused products in a dry, clean and safe place.
- Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Replace any damaged components immediately with original spare parts.

Safe operation

CAUTION

Damage to, or destruction of the jaws caused by overstraining!

- Use Rongeur only according to their intended use (i.e. for removing soft tissues).
- Do not cut or lever out bones or adjacent bone structures.
- Avoid overstraining the jaws by tearing off or holding and levering of tissue, or by sideways strains due to twisting.
- Do not use Rongeur with blunt cutting edges. Send blunt Rongeur back for repair.

Processing

Note
Observe all relevant national regulations and standards concerning processing.

Note
For patients with Creutzfeldt-Jakob Disease (CJD), suspected CJD or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of the products.

Note
Up-to-date information on processing can be found on the Aesculap Extranet at www.aesculap-extra.net.

Preparations

- Pre-clean product if necessary.
- Put away the product in dry condition.
- Use suitable cleaning/disinfecting agents if the product is put away in wet condition. Prior to mechanical cleaning and disinfecting, rinse the product thoroughly with running water.

Cleaning/Disinfecting

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

- Only use a cleaning/disinfecting agent that has been approved for surgical steel and follow the manufacturer's instructions.

Note
This product cannot be disassembled. It is cleaned and disinfected in assembled condition.

- Use cleaning/disinfecting agents that are suitable for the product. Always follow the manufacturer's instructions with regard to concentration, temperature and exposure time.
- Avoid encrustation of residues/proteins (e.g. caused by aldehyde/alcohol).
- Only use bactericidal, fungicidal and virucidal disinfecting agents.
- Carry out ultrasound cleaning:
 - as an effective mechanical supplement to manual cleaning/disinfection.
 - to prepare products with encrusted debris for mechanical cleaning/disinfection.
 - as an integrated mechanical support measure for mechanical cleaning/disinfection.
 - as an aftertreatment for products that are still dirty after mechanical cleaning/disinfection.
- Preferably use thermal disinfecting processes.

Mechanical cleaning/disinfecting

- Pre-cleaning
- Thoroughly pre-rinse under running water.
 - Use a cleaning brush.
 - Carry out the final rinse under running water.

- Cleaning
- Place the product on a tray that is suitable for cleaning (to avoid rinsing blind spots).
 - Store any products with hinges or joints on the wire basket in such a way that the joints are open.
 - Ensure that water can flow out of openings.
 - Process the product in a cleaning/disinfecting device. Follow the instructions provided by the manufacturer of the machine.
 - Carry out processing cycle:
 - Use an appropriate alkaline cleaning/disinfecting agent according to its manufacturer's instructions.
 - Do not exceed the maximum washing temperature of 55 °C.
 - Wash the product for at least 10 min.
 - Neutralize if necessary.
 - Carry out intermediate rinse for at least 1 min.
 - Carry out intensive final rinse with distilled, demineralized or fully desalinated water.
 - Carry out thermal disinfection: Rinse for 10 min at 93 °C with distilled, demineralized or fully desalinated water.
 - Complete the program with a drying phase of at least 20 min at a temperature of not more than 110 °C.
 - After completion of the mechanical cleaning/disinfecting cycle, inspect surfaces, cavities, lumens, and openings for visible debris.
 - Clean manually if necessary.

Manual cleaning/disinfecting

- Use an appropriate alkaline cleaning/disinfecting agent according to its manufacturer's instructions.
- Immerse the product in the cleaning/disinfecting agent in such a way that all surfaces, cavities, lumens, and openings are covered.
- After the end of the disinfection period, thoroughly rinse the product under running water, making sure that water flows through every lumen and channel and all blind holes are repeatedly filled and drained.
- Clean any products with hinges or joints in open and closed positions.

- Remove encrusted debris with a soft nylon brush. Do not use harsh cleaning agents or metal brushes.
- Clean lumens, channels and blind holes with soft round plastic brushes of fitting diameter.
- Carry out an intensive final rinse with distilled, demineralized or fully desalinated water.
- Inspect surfaces, cavities, lumens and openings for visible debris. If necessary repeat the cleaning/disinfection process.
- Use a lint-free cloth or a compressed-air gun for drying the product.
- Make certain that lumens, channels and blind holes are dried, too.

Control, care and inspection

- Allow the product to cool down to room temperature.
- Lightly lubricate moving parts such as hinges and latches with a sterilizable, steam-permeable and tissue-compatible maintenance oil (e.g. Aesculap Sterilit spray JG600 or maintenance oil JG598).
- Inspect the product after each cleaning and disinfecting cycle to be sure it is clean, functioning and cutting properly, not damaged, and does not have any loose, bent, broken, cracked, worn, or fractured components.
- Set aside the product if it is damaged.
- If the cutting edge is blunt, send the product back for repair, see Technical Service.

Packaging

- Appropriately protect products with fine working tips.
- Sort the product into its appropriate storage device or put it on a suitable tray. Make certain that all cutting edges are protected. Observe the weight limit for each tray/container.
- Package trays appropriately for the sterilization process (e.g. in Aesculap sterile containers).
- Ensure that the packaging will prevent recontamination of the product in the period between reprocessing and reuse.

Sterilization method and parameters

- Make certain that all external and internal surfaces of the product will be exposed to the sterilizing agent.
- Sterilize with steam, observing the following rules: Sterilization has to be performed in a validated steam sterilization process (e.g. in a sterilizer according to EN 285/ANSI/AAMI/ISO 11134-1993, ANSI/AAMI ST46-1993, validated according to EN 554/ISO 13683). For the fractionated vacuum procedure, the sterilization has to be carried out for a minimum of 5 minutes at 134 °C and at 2 bar pressure.
- When sterilizing several products at the same time in one steam sterilizer: Make sure that the maximum allowable load capacity of the steam sterilizer, as specified by the manufacturer, is not exceeded.

Sterilization for the US market

- Sterilization of the device may be accomplished by steam.
- Aesculap does not recommend the device be sterilized by "Flash" or chemical sterilization.
- Surgical instruments may also be placed within an Aesculap rigid sterilization container (sterile container) for processing under generally accepted hospital in-use conditions.

The recommended sterilization parameters are as follows:

Sterilization method	Temp.	Minimum exposure time	
		Wrapped	In a sterile container system
Pre-vacuum	270–275 °F	4 min	4 min

WARNING for the US market

If this device is/was used in a patient with, or suspected of having Creutzfeldt-Jakob Disease (CJD), the device cannot be reused and must be destroyed due to the inability to reprocess or sterilize to eliminate the risk of cross-contamination!

Storage

- Store processed products under conditions as germ-free as possible, in a dry, dark, cool and dust-protected room.

Technical Service

For service, maintenance or repairs, contact your national B. Braun/Aesculap agency. Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 7461 95-2700
Fax: +49 7461 16-2887
E-mail: ats@aesculap.de

Or in the US:
Aesculap Inc.
Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood, MO 63042
Aesculap Repair Hotline
Phone: +1 800 214-3392
Fax: +1 314 895-4420
Other service addresses can be obtained from the address indicated above.

Distributor in the US/Contact in Canada for product information and complaints

AESCULAP Inc.
3773 Corporate Parkway
Center Valley, PA 18034
USA

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