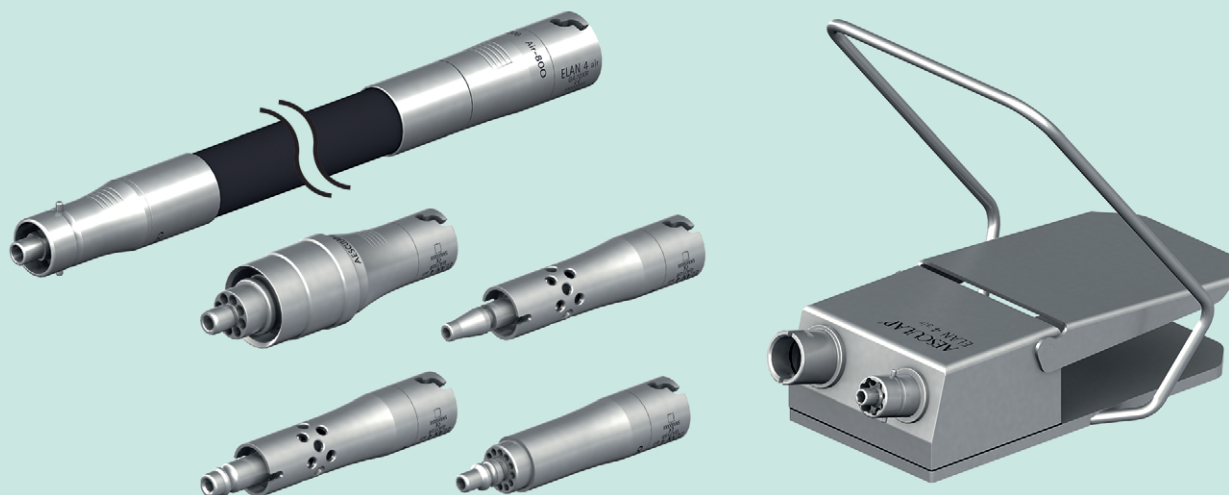
en Instructions for use/Technical description

ELAN 4 air foot control GA708, ELAN 4 air hose GA702R/GA703R, ELAN 4 air wall adapter GA710R to GA715R

USA Note for U.S. users

This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusa.com. If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.

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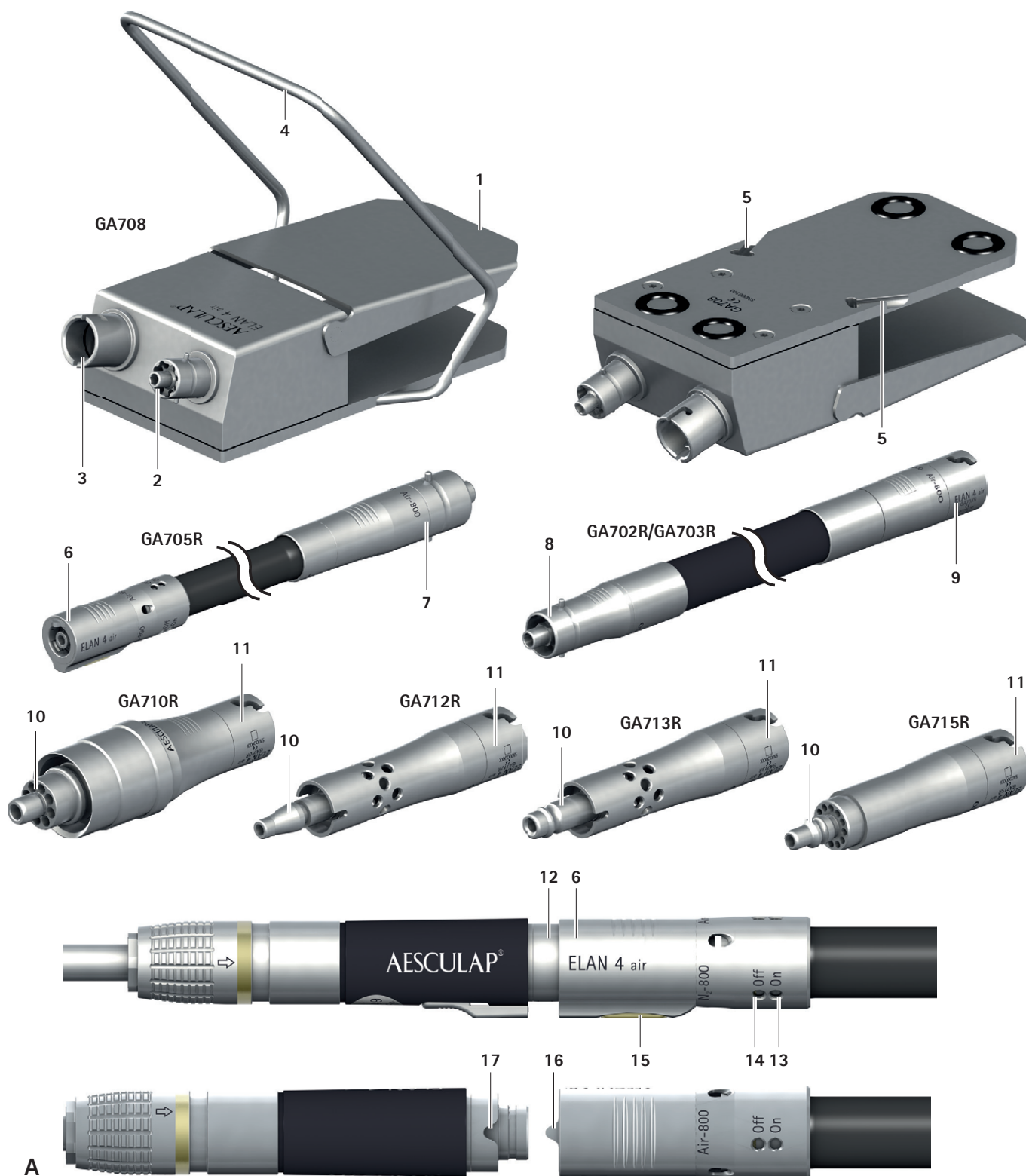
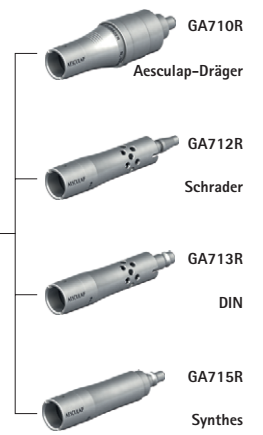
Aesculap®

ELAN 4 air foot control GA708, ELAN 4 air wall hose GA702R/GA703R,
ELAN 4 air wall adapter GA710R to GA715R

Legend

ELAN 4 air foot control GA708

- 1 Pedal
- 2 Connection for wall hose
- 3 Connection for motor hose
- 4 Bracket
- 5 Adapter for bracket

ELAN 4 air motor hose for foot control GA705R

- 6 Connection for applied part
- 7 Connection for foot control

ELAN 4 air wall hose GA702R/GA703R

- 8 Connection for wall adapter
- 9 Connection for foot control

ELAN 4 air wall adapter GA710R/GA712R/GA713R/GA715R

- 10 Connection for compressed-air source
- 11 Connection for wall hose

Coupling ELAN 4 air applied part with ELAN 4 air motor hose for foot control GA705R

- 12 Connection for motor hose of applied part
- 13 Field of vision "On"
- 14 Field of vision "Off"
- 15 Release button
- 16 Nib on motor hose
- 17 Groove on connection for motor hose

Symbols on product and packages

	Caution Observe important safety information such as warnings and precautions in the instructions for use.
 YYYY-MM	Maintenance indicator Date of the next recommended servicing (year-month) by the international B. Braun/Aesculap agency, see Technical Service
	Two-dimensional machine readable code The code contains a unique serial number that can be used for electronically tracking individual instruments. The serial number is based on the worldwide sGTIN (GS1) standard.
	Manufacturer
	Date of manufacture
	Non-sterile medical device
	Manufacturer batch ID
	Manufacturer's serial number
	Manufacturer's order number
	Temperature limits during transport and storage
	Air humidity limits during transport and storage
	Atmospheric pressure limits during transport and storage
Rx only	According to US federal law, this product may only be sold by a physician or on the orders of a physician
	CE mark according to Regulation (EU) 2017/745
	Medical device

Contents

1.	About this document	3
1.1	Scope	3
1.2	Warnings	3
2.	General information	3
2.1	Intended use	3
2.2	Application Environment	3
2.3	Indications	3
2.4	Absolute contraindications	3
2.5	Relative contraindications	4
3.	Safe handling	4
4.	Product description	4
4.1	Scope of supply	4
4.2	Components required for operation	4
4.3	Operating principle	4
5.	Preparation	5
5.1	Compressed air supply/compressed air hoses	5
5.2	Operation with central gas supply according to DIN EN ISO 5359	5
5.3	Operation with compressed-gas cylinders according to DIN EN ISO 11117	5
5.4	First use of foot control	5
6.	Working with the product	5
6.1	System set-up	5
6.2	Function checks	6
6.3	Safe operation	6
7.	Processing procedure	6
7.1	General safety notes	6
7.2	General information	7
7.3	Reusable products	7
7.4	Preparations at the place of use	7
7.5	Cleaning/disinfection	7
7.6	Wipe disinfection	7
7.7	Inspection, maintenance and checks	7
8.	Maintenance	7
8.1	ELAN 4 air foot control, ELAN 4 air wall adapter	7
8.2	ELAN 4 air wall hose	7
9.	Troubleshooting list	8
10.	Technical Service	10
11.	Accessories/Spare parts	10
12.	Technical data	10
12.1	Classification in accordance with Regulation (EC) 2017/745 ..	10
12.2	ELAN 4 air foot control GA708	10
12.3	ELAN 4 air wall hose 3m GA702R	10
12.4	ELAN 4 air wall hose 5m GA703R	10
12.5	ELAN 4 air wall adapter Aesculap-Dräger GA710R	10
12.6	ELAN 4 air wall adapter Schrader with diffuser GA712R	10
12.7	ELAN 4 air wall adapter DIN with diffuser GA713R	10
12.8	ELAN 4 air wall adapter Synthes GA715R	10
12.9	Ambient conditions	11
13.	Disposal	11

1. About this document

1.1 Scope

These instructions for use apply to the products listed below:

Art. no.	Designation
GA708	ELAN 4 air foot control
GA702R	ELAN 4 air wall hose, 3m
GA703R	ELAN 4 air wall hose, 5m
GA710R	ELAN 4 air Wall adapter Aesculap Dräger
GA712R	ELAN 4 air wall adapter Schrader with diffuser
GA713R	ELAN 4 air wall adapter DIN with diffuser
GA715R	ELAN 4 air wall adapter Synthes

► For article-specific instructions for use and material compatibility information, see B. Braun eIFUs at eifu.bbraun.com

1.2 Warnings

Warnings indicate hazards to patients, users and/or products that may occur during the use of the product. Warnings are marked as follows:

DANGER

Indicates a potential danger. If it is not prevented, death or severe injuries may result.

WARNING

Indicates a potential danger. If not prevented, minor or moderate injuries may result.

CAUTION

Indicates a potential damage to property. If not prevented, the product may be damaged.

2. General information

2.1 Intended use

The ELAN 4 air foot control GA708, the ELAN 4 air wall hoses GA702R and GA703R, ELAN 4 air wall adapters GA710R, GA712R, GA713R and GA715R are accessories of the ELAN 4 air motor system.

The foot control is used by the user to control the motor speed of the connected ELAN 4 air applied part, provided that a hand control is not being used.

The ELAN 4 air wall hose connects the ELAN 4 air foot control to the ELAN 4 air wall adapters.

The ELAN 4 air wall adapters connect the ELAN 4 air wall hose to the compressed air supply unit.

2.2 Application Environment

The product is used in non-sterile areas.

Location of foot control: On the floor

2.3 Indications

The type and area of application depend on the applied parts and tools selected.

2.4 Absolute contraindications

The product is not licensed for use on the central nervous system or central circulatory system.

2.5 Relative contraindications

The safe and effective use of the products greatly depends on influences which can only be controlled by the user. Therefore the specifications provided represent framework conditions only.

The clinically successful use of the product is dependent on the knowledge and experience of the surgeon. The surgeon must decide on which bodily structures to use the product and take into account the safety and warning information contained in these instructions for use.

3. Safe handling

DANGER

Danger of fire and explosion when operating devices that run on oxygen!

- ▶ Never operate the pneumatic motor with oxygen!
- ▶ Only use compressed air or nitrogen to power pneumatic motors.

WARNING

Risk of injury and material damage if this product is not used as intended!

- ▶ Use the product only for its intended purpose.

WARNING

Risk of injury and damage to property due to improper handling of the product!

This product is part of the ELAN 4 air motor system.

- ▶ Follow the instructions for use of all products used.
- General risk factors associated with surgical procedures are not described in this documentation.
- It is the operating surgeon's responsibility to ensure that the surgical procedure is performed correctly.
- The operating surgeon must have a thorough understanding of both the hands-on and conceptual aspects of the established operating techniques.
- ▶ Clean the new product after removing its transport packaging.
- ▶ Prior to use, check that the product is in good working order.
- ▶ To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- ▶ Keep the instructions for use accessible for the user.
- ▶ Always adhere to applicable standards.
- ▶ Use the product within the specified pressure range only.

Note

The user agrees to report all serious incidents occurring in connection with the product to the manufacturer and to the competent authority of the country in which the user is registered.

4. Product description

4.1 Scope of supply

ELAN 4 air foot control

Art. no.	Name
GA708	ELAN 4 air foot control
TA014450	Instructions for use for ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R
TA014782	Supplement ELAN 4 air motor system

ELAN 4 air wall hose

Art. no.	Name
GA702R	ELAN 4 air wall hose, 3 m
GA703R	– or – ELAN 4 air wall hose, 5m
TA014454	Instructions for use for ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R (leaflet)

ELAN 4 air wall adapter

Art. no.	Name
	One of the following wall adapters:
GA710R	ELAN 4 air wall adapter Aesculap Dräger
GA712R	ELAN 4 air wall adapter Schrader with diffuser
GA713R	ELAN 4 air wall adapter DIN with diffuser
GA715R	ELAN 4 air wall adapter Synthes

4.2 Components required for operation

- ELAN 4 air wall adapter GA710R, GA712R, GA713R or GA715R
- ELAN 4 air wall hose GA702R or GA703R
- ELAN 4 air foot control GA708
- ELAN 4 air motor hose GA705R
- ELAN 4 air applied part
- Tool

4.3 Operating principle

ELAN 4 air foot control

By activating the pedal 1 of the ELAN 4 air foot control, the user controls the motor speed of the connected ELAN 4 air applied part. The pedal activates a valve of the ELAN 4 air foot control.

When the pedal is in its initial position, the valve is closed and the motor of the applied part does not run. When the pedal is in its end position, the valve is open and the motor of the applied part runs at maximum speed. When the pedal is released, the ELAN 4 air motor hose is vented.

ELAN 4 air wall hoses and ELAN 4 air wall adapters

The ELAN 4 air wall hoses and ELAN 4 air wall adapters guide compressed air from the compressed-air source to the foot control.

The wall hoses and wall adapters have a fast-lock coupling. This coupling prevents the escape of compressed air during detachment of the parts.

The wall hoses have an exit-air recycling system. The supply hose is embedded in an additional exit air hose, which takes the exit air away from the operating field.

There are two exit air recycling systems:

- Aesculap-Dräger and Synthes wall adapters with complete exit-air recycling: The exit air is channeled back from the O.R. into the central gas supply system. In this way, air turbulences and adverse effects on the climate and hygiene in the O.R. are avoided.
- Schrader and DIN wall adapters without complete exit-air recycling: The exit air enters the O.R. through a diffuser on the wall adapter.

5. Preparation

Non-compliance with the following instructions will preclude all responsibility and liability in this respect on the part of Aesculap.

- ▶ When setting up and operating the product, adhere to
 - national regulations for installation and operation,
 - national legal regulations pertaining to compressed air supply,
 - National regulations on fire and explosion protection
- ▶ Before using the product and its accessories, check to be sure that neither are damaged.
- ▶ Use the products and their accessories only if they are in perfect technical condition.

5.1 Compressed air supply/compressed air hoses

⚠ WARNING

Damage to persons and/or property due to the placing of the hose!
The user may injure the patient with an unintentional movement due to shock.

- ▶ Be careful not to damage the compressed-air hose.
- ▶ Ensure annual repair.

⚠ WARNING

Operating pressures in excess of 10bar will damage the motor system!

- ▶ Set an operating pressure of between 6 bar and 10 bar.

⚠ CAUTION

Impaired performance of the pneumatic motor/bursting of damaged compressed-air hoses!

- ▶ Lay compressed-air hoses only through unobstructed areas.
- ▶ Do not squeeze or block open hose ends or diffusers.
- ▶ Do not coil up compressed-air hoses too tightly (minimum coil diameter 20cm).
- ▶ Do not kink, squeeze or clamp compressed-air hoses.
- ▶ Verify that the compressed-air hoses are not damaged (e.g. by scalpels) or squeezed with sharp objects (e.g. cloth clips).

5.2 Operation with central gas supply according to DIN EN ISO 5359

The gas supply must provide clean, dry air and must be set as follows:

Nominal operating pressure	8 ±2 bar
Maximum permissible operating pressure	10 bar
For exhaust air recirculation, the maximum permissible velocity pressure in the exhaust line during motor operation	0.5bar to 0.7 bar
Minimum gas-supply delivery	300 l/min

5.3 Operation with compressed-gas cylinders according to DIN EN ISO 11117

The following conditions must be met:

- Compressed air or nitrogen is used
- An adequate stock of full gas cylinders is in place
- Cylinder pressure reduced to operating pressure of 6 bar to 10 bar through a pressure reducer (e.g., GA099)
- Operating pressure has been read from the scale on the device and set with the T-screw.

Note

If the system is operated with compressed gas cylinders, the exit air is not removed from the O.R. It emerges from the ELAN 4 wall adapter, away from the operating field.

5.4 First use of foot control

Mounting/dismounting the bracket

- ▶ Insert bracket 4 on one side into the adapter for bracket 5.
- ▶ Widen the bracket 4 and insert it fully into the other adapter for bracket 5.
- ▶ Disassembly: Press bracket 4 together by the two lower horizontal ends and remove bracket 4 from the adapters for bracket 5.

6. Working with the product

⚠ WARNING

Risk of injury and/or malfunction!

- ▶ Always carry out a function check prior to using the product.

⚠ WARNING

Risk of injury and material damage due to inadvertent activation of the applied part!

- ▶ Secure application parts that are not actively used against inadvertent activation (off position).

⚠ WARNING

Dropping may damage the product!

- ▶ Use the products only if they are in a flawless technical condition, see function check

6.1 System set-up

Note

Operating elements on the system components of the ELAN 4 air motor system are labeled with a gold marking.

Connecting the accessories

Do not use accessories in combinations that are not mentioned in these instructions for use.

- ▶ Follow the instructions for use of individual accessories.
- ▶ Please address your B. Braun/Aesculap partner or Aesculap Technical Service with any inquiries in this respect; for a contact address, see Technical Service.

Connecting wall adapter to compressed-air source

- ▶ Couple connection for compressed-air source 10 of the wall adapter with the compressed-air source.

Connecting the wall adapter to the foot control

- ▶ Attach the connection for wall adapter 8 of the wall hose to the connection for wall hose 11 of the wall adapter and couple the bayonet lock.

Connecting the wall hose to the foot control

- Attach the connection for foot control **9** of the wall hose to the connection for wall hose **2** on the foot control and couple the bayonet lock.

Connecting the motor hose to the foot control

- Attach the connection for foot control **7** of the motor hose to the connection for motor hose **3** on the foot control and couple the bayonet lock.

Connecting the applied part to the motor hose

- Attach the connection for motor hose of the applied part **12** to the connection for applied part **6** of the motor hose, see Fig. A. When doing so, ensure that the nib **16** on the motor hose is aligned with the groove **17** on the connection for motor hose **12**.

The applied part engages. When the field of vision is "Off" **14** on the motor hose, a gold-colored marking is visible.

Note

The applied part attached to this motor hose is only ready for operation when the release button is in "On" position.

Releasing the applied part for operation ("On" position)

- Activate the release button **15** on the motor hose and push the connection for applied part **6** onto the applied part, see Fig. A.
The applied part engages. When the field of vision is "On" **13** on the motor hose, a gold-colored marking is visible.

Locking the applied part ("Off" position)

- Activate the release button **15** on the motor hose and detach the motor hose from the applied part.
The applied part engages. When the field of vision is "Off" **14** on the motor hose, a gold-colored marking is visible.

Disconnecting the applied part from the motor hose

- Activate the release button **15** on the motor hose and pull at the connection for applied part **6** to disconnect the motor hose from the applied part.

Disconnecting the motor hose from the foot control

- Disconnect the bayonet lock of the connection for foot control **7** of the motor hose and detach motor hose from foot control.

Disconnecting the wall hose from the foot control

- Disconnect the bayonet lock of the connection for foot control **9** of the wall hose and detach wall hose from foot control.

Disconnecting the wall hose from the wall adapter

- Turn the connection for wall adapter **8** of the wall hose into the middle position of the bayonet lock to ensure pressure equalization and prevent any loud bangs.
- Disconnect the wall hose fully from the wall adapter.

Disconnecting the wall adapter from the compressed-air source

- Uncouple the connection for compressed-air source **10** of the wall adapter from the compressed-air source.

6.2 Function checks

- Check the ELAN 4 air motor system with a trial run prior to each application.
- Make certain that the applied part and foot control are in good working order.
- Make certain that all compressed-air hoses and the wall adapter are correctly connected and leak-tight.
- Check the secure connection of all products used.
- Make certain that the applied part runs at maximum speed when the foot control pedal is pushed all the way down.
- Do not use a damaged or broken product. Immediately set aside any damaged products!

6.3 Safe operation

Operation of the applied part is only possible if:

- the applied part is released ("On" position)
- a foot control or hand control is connected.

Activating the applied part via foot control

- Press the pedal **1** on the ELAN 4 air foot control.
The motor of the applied part runs.
- Adjusting speed: Press down on the pedal **1** to a greater or lesser extent.
- To select maximum speed: Press down on the pedal **1** as far as it will go.

7. Processing procedure

7.1 General safety notes

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for reprocessing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD, or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

It should be noted that successful reprocessing of this medical device can only be guaranteed following prior validation of the reprocessing method. The operator/reprocessing technician is responsible for this.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

See also B. Braun eIFUs at eifu.bbraun.com for up-to-date information on processing and material compatibility

The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

7.2 General information

Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixating pre-cleaning temperatures >45 °C nor fixating disinfecting agents (active ingredient: aldehydes/alcohols) should be used.

Excessive neutralizing agents or basic cleaners may result in a chemical attack and/or fading and the laser marking becoming unreadable either visually or by machine.

Residues containing chlorine or chlorides, e.g., in surgical residues, medicines, saline solutions, and in the service water used for cleaning, disinfection, and sterilization, will cause corrosion damage (pitting, stress corrosion) and result damage to metallic products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Perform additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Material damage such as corrosion, cracks, fracturing, premature aging, or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surface and could cause corrosion
- ▶ For detailed information on hygienically safe and material-friendly/careful processing, please refer to www.a-k-i.org Header "AKI-Brochures", "Red brochure".

7.6 Wipe disinfection

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Wipe disinfection	RT	≥ 1	-	-	17 % Propan-1-ol, 0.23 % Didecyltrimethylammonium chloride*

RT: Room temperature

* Meliseptol® wipes sensitive (BBraun) are recommended

Phase I

- ▶ Remove any visible residues with a disposable disinfectant wipe.
- ▶ Wipe all surfaces of the optically clean product with a fresh, disposable disinfectant wipe.
- ▶ Observe the specified application time (1 min minimum).
- ▶ After manual cleaning/disinfection, check visible surfaces for residue and repeat the cleaning/disinfection process as required.

7.7 Inspection, maintenance and checks

- ▶ Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.

Note

Aesculap additionally recommends the occasional spraying of moving parts (such as knobs, couplings) with the Aesculap STERILIT Power Systems Oil spray.

- ▶ Set aside the product if it is damaged.

7.3 Reusable products

The service life of the product is limited by damage, normal wear, type and duration of the application, handling, storage and transportation of the product.

Careful visual and functional testing prior to next use is the best way to identify a malfunctioning product.

7.4 Preparations at the place of use

- ▶ Separate the products from each other immediately after use.
- ▶ Remove any visible surgical residues as much as possible with a damp, lint-free cloth.
- ▶ Carry out non-fixating/NaCl-free pre-cleaning immediately after use.
- ▶ Send the product in a dry condition for cleaning and disinfection within 6 hours.

7.5 Cleaning/disinfection

Product-specific safety instructions for the reprocessing procedure

⚠ CAUTION

Damage to, or destruction of the product caused by mechanical cleaning/disinfection!

- ▶ Only clean and disinfect the product manually.
- ▶ Do not sterilize the product under any circumstances.

⚠ CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents!

- ▶ Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.

8. Maintenance

8.1 ELAN 4 air foot control, ELAN 4 air wall adapter

To ensure reliable operation, the product must be maintained at least once every two to three years.

For technical service, please contact your national B. Braun/Aesculap agency, see Technical Service.

8.2 ELAN 4 air wall hose

To ensure reliable operation, the product must be maintained in accordance with the maintenance labeling or at least once a year.

e.g. 2016-07

For technical service, please contact your national B. Braun/Aesculap agency, see Technical Service.

9. Troubleshooting list

Fault	Cause	Detection	Remedy
Applied part not running	No compressed air	Motor coupling in "off" position	Activate applied part for operation ('on' position).
		Hoses and wall adapters not correctly coupled	Check the coupling points.
		No compressed air at the wall outlet	Check the building technology
		Gas cylinder empty (pressure gauge set to "0")	Change cylinder.
		Valve on cylinder closed	Open valve.
	Applied part defective	Tool not moving	Replace applied part.
Disconnecting/connecting the applied part from/with the motor hose not possible (change from "On" to "Off" position and vice versa)	Foot control defective	Pedal not moving	Replace foot control.
	Do not disengage the release button on the motor hose between connection/separation and activation/locking of the applied part	Coupling processes cannot be performed	Disengage the release button on the motor hose between connection/separation and activation/locking of the applied part and re-actuate.
		Motor coupling defective	Replace applied part and motor hose.
Applied part runs without activating the foot control	Foot control defective	Applied part runs without being activated by the foot control	Replace foot control.
Insufficient power	Blunt tool	Cutting edge worn	Use a new tool.
		Cutting edge is clogged up, e.g. due to insufficient rinsing	Use a new tool.
	The craniotome burr is not centrally aligned with the dura guard hook	Dura guard bracket bent Poor progress during craniotomy	Use new dura guard, follow instructions for use (TA014473/TA014474).
	Applied part defective	Insufficient performance of applied part	Follow instructions for use of the applied part (processing, maintenance).
			Replace applied part.
		Steep rise in temperature after a short period of time	Follow instructions for use of the applied part (processing, maintenance).
	Air resistance in hoses and wall adapter too high	Use of extension hoses	Replace applied part.
			Only use one wall hose.
		Hoses bent or squashed	Lay hoses only through unobstructed areas.
		No free exhaust air outlet at the diffuser	Free up the openings on the diffuser.
	Air supply too low	Operating pressure too low	Adjust the operating pressure to min. 6 bar.
		Pressure regulator on the cylinder defective	Check pressure regulator.
		Leaks in the hose/hose coupling	Replace hose.
	Accessory broken	Large amount of heat generated after a short time	Observe the instructions for use of accessories.
			Replace accessory.

Fault	Cause	Detection	Remedy
Applied part is getting hot	Excessive use	The applied part becomes hot Loud operating noise/vibration	Follow instructions for use of the applied part (interval operation).
	Improper processing		Follow instructions for use of the applied part (reprocessing, maintenance)
	Shaft of applied part bent		Replace applied part.
	Applied part defective		Replace applied part.
	Accessory defective		Observe the instructions for use of accessories. Replace accessory.
High speed applied part vibrates excessively	Shaft of applied part bent	Excessive running noises, vibrations	Replace applied part.
	Applied part defective		Replace applied part.
	Reprocessing carried out incorrectly		Follow instructions for use of the applied part (processing, maintenance).
Tool cannot be coupled or uncoupled	ELAN 4 air Highspeed: Automatic tool lock blocked	Highspeed tool doesn't engage	Activate slide for tool lock and hold, then couple tool.
	ELAN 4 air Highspeed: Slider for tool lock locked	Applied part released for operation ("On" position)	Block applied part ('off' position).
	Incompatible tool	Incorrect tool	Select an appropriate tool according to the instructions for use of the applied part.
	Tool connection or coupling deformed, defective	Tool is difficult to couple/uncouple or cannot be coupled/uncoupled	Use a new tool. Replace applied part.
Attachment for GA749 (craniotome) cannot be coupled or uncoupled	Automatic attachment lock blocked	Attachment does not click into place	Pull back unlocking sleeve and hold, then couple the attachment.
	Unlocking sleeve blocked	Applied part released for operation ("On" position)	Lock the applied part ("Off" position).
	Attachment coupling defective	Attachment is difficult to couple/uncouple or cannot be coupled/uncoupled	Use a new attachment. Replace applied part.
	Attachment coupling is dirty	Attachment is difficult to couple or cannot be coupled	Clean the attachment or use a new attachment. Clean the applied part.
Rotary dura guard GB947R hard to turn	Bearing dirty or worn	Rotary dura guard is hard to move	Follow the instructions for use (TA014473/TA014474) (reprocessing, maintenance).
			Replace rotary dura guard.
Hoses/wall adapter/foot control cannot be coupled or uncoupled	Hoses/wall adapters/foot control not compatible	Hoses/wall adapter are difficult to couple/uncouple or cannot be coupled/uncoupled	Select an appropriate hose/wall adapter/foot control according to the instructions for use.
	Hose coupling deformed, defective		Replace hose/wall adapter/foot control.
Rinsing adapter cannot be adapted	Rinsing adapter not compatible	Rinsing adapter cannot be adapted	Use matching rinsing adapter
	Port on applied part deformed		Replace applied part.
	Screw thread on rinsing adapter deformed/defective		Replace rinsing adapter.
Oil spray adapter cannot be attached	Oil spray adapter not compatible	Oil spray adapter cannot be attached	Use the matching oil spray adapter.

10. Technical Service

DANGER

Danger to life of patients and users in case of malfunctions and/or failure of protective measures!

- Do not perform any servicing or maintenance work under any circumstances while the product is being used on a patient.

CAUTION

Modifications to medical equipment may result in loss of guarantee/warranty claims and any approvals.

- Do not modify the product.
- For service and repairs, please contact your national B. Braun/Aesculap representative.

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 (7461) 95 -1601

Fax: +49 (7461) 14 -939

E-Mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

11. Accessories/Spare parts

Art. no.	Name
GA099	Pressure reducer with quick-locking coupling
GA708203	Bracket
GA702R	ELAN 4 air wall hose, 3m
GA703R	ELAN 4 air wall hose, 5m
GA705R	ELAN 4 air motor hose 3m for foot control
GA710R	ELAN 4 air wall adapter Aesculap Dräger
GA712R	ELAN 4 air wall adapter Schrader with diffuser
GA713R	ELAN 4 air wall adapter DIN with diffuser
GA715R	ELAN 4 air wall adapter Synthes
GB600	STERILIT Power Systems oil spray
TA014450	ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R (A4 for folder)
TA014454	ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R (leaflet)

12. Technical data

12.1 Classification in accordance with Regulation (EC) 2017/745

Art. no.	Designation	Class
GA708	ELAN 4 air foot control	I
GA702R	ELAN 4 air wall hose, 3m	I
GA703R	ELAN 4 air wall hose, 5m	I
GA710R	ELAN 4 air wall adapter Aesculap-Dräger	I
GA712R	ELAN 4 air wall adapter Schrader with diffuser	I
GA713R	ELAN 4 air Wall adapter DIN with diffuser	I
GA715R	ELAN 4 air Wall adapter Synthes	I

12.2 ELAN 4 air foot control GA708

Weight	2.1 kg ±5 %
Dimensions (L x W x H)	235 mm x 105 mm x 60 mm ±5 % For mounted bracket: 235 mm x 160 mm x 192 mm ±5 %

12.3 ELAN 4 air wall hose 3m GA702R

Weight	0.7 kg ±10 %
Dimensions (Ø x L)	26 mm x 3 000 mm ±5 %

12.4 ELAN 4 air wall hose 5m GA703R

Weight	1.06 kg ±10 %
Dimensions (Ø x L)	26 mm x 5 000 mm ±5 %

12.5 ELAN 4 air wall adapter Aesculap-Dräger GA710R

Weight	290 g ±5 %
Dimensions (Ø x L)	37 mm x 121 mm ±5 %

12.6 ELAN 4 air wall adapter Schrader with diffuser GA712R

Weight	154 g ±10 %
Dimensions (Ø x l)	26 mm x 126 mm ±5 %

12.7 ELAN 4 air wall adapter DIN with diffuser GA713R

Weight	156 g ±10 %
Dimensions (Ø x l)	26 mm x 121 mm ±5 %

12.8 ELAN 4 air wall adapter Synthes GA715R

Weight	150 g ±10 %
Dimensions (Ø x L)	26 mm x 110 mm ±5 %

12.9 Ambient conditions

	Operation	Transport and storage
Temperature	10 °C to 27 °C	-10 °C to 50 °C
Relative humidity	30 % to 75 %	10 % to 90 %
Atmospheric pressure	700 hPa to 1 060 hPa	500 hPa to 1 060 hPa

13. Disposal

WARNING

Risk of infection from contaminated products!

- Observe national regulations when disposing of or recycling the product, its components, and their packaging.

Note

The operator must process the product before disposal, see Processing procedure.

- In case of questions regarding the disposal of the product, contact your local B. Braun/Aesculap representative, see Technical Service.

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12.1 (AB) 2017/745 yönetmeliği uyarınca sınıflandırma