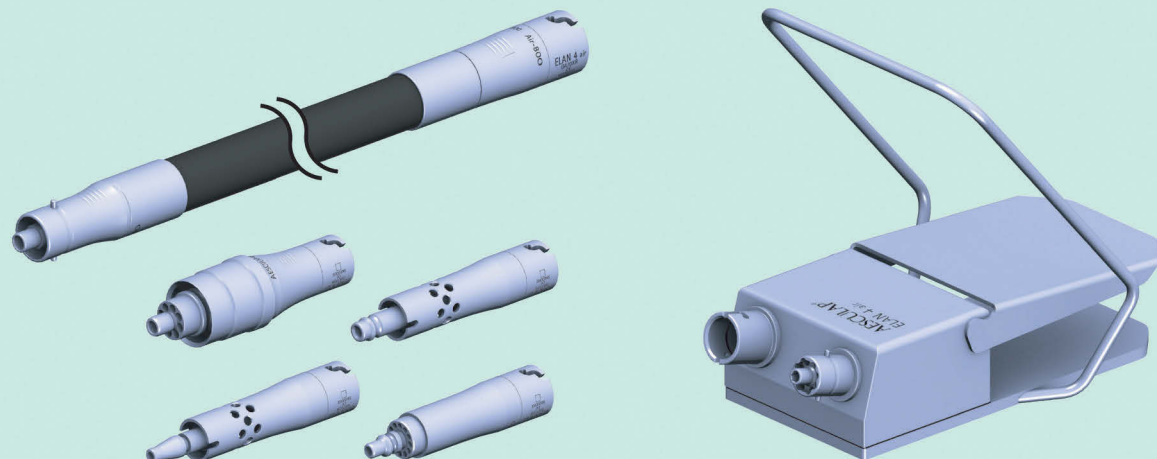
(GB) Instructions for use/Technical description

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
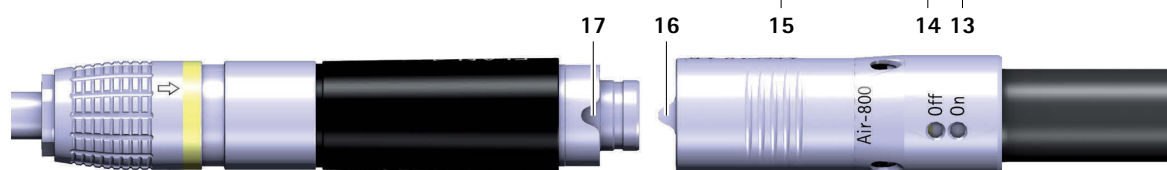
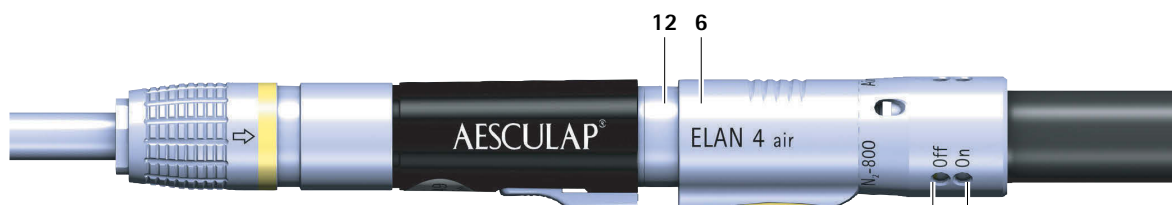
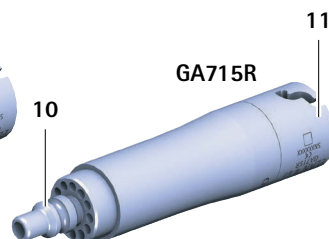
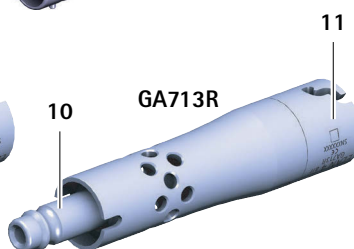
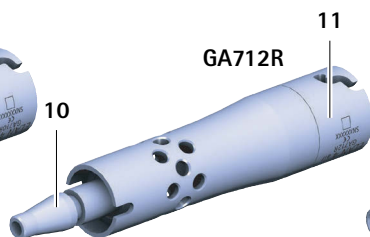
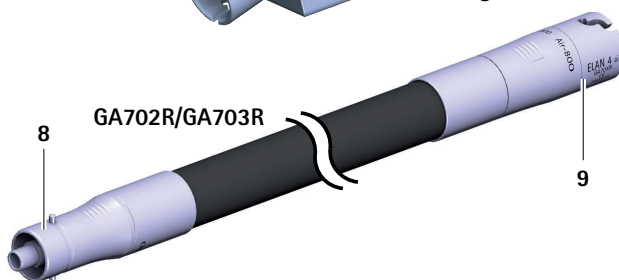
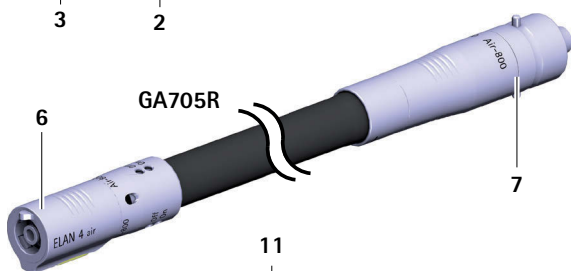
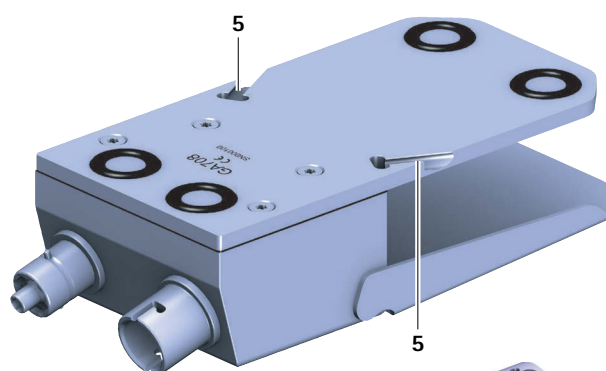
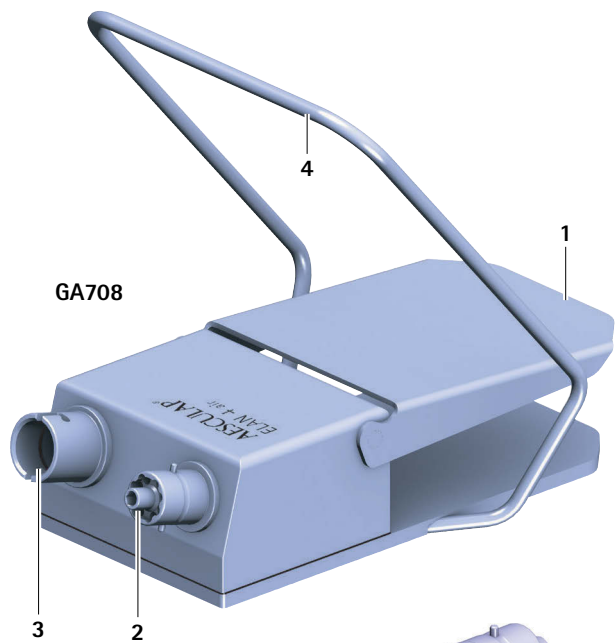
A

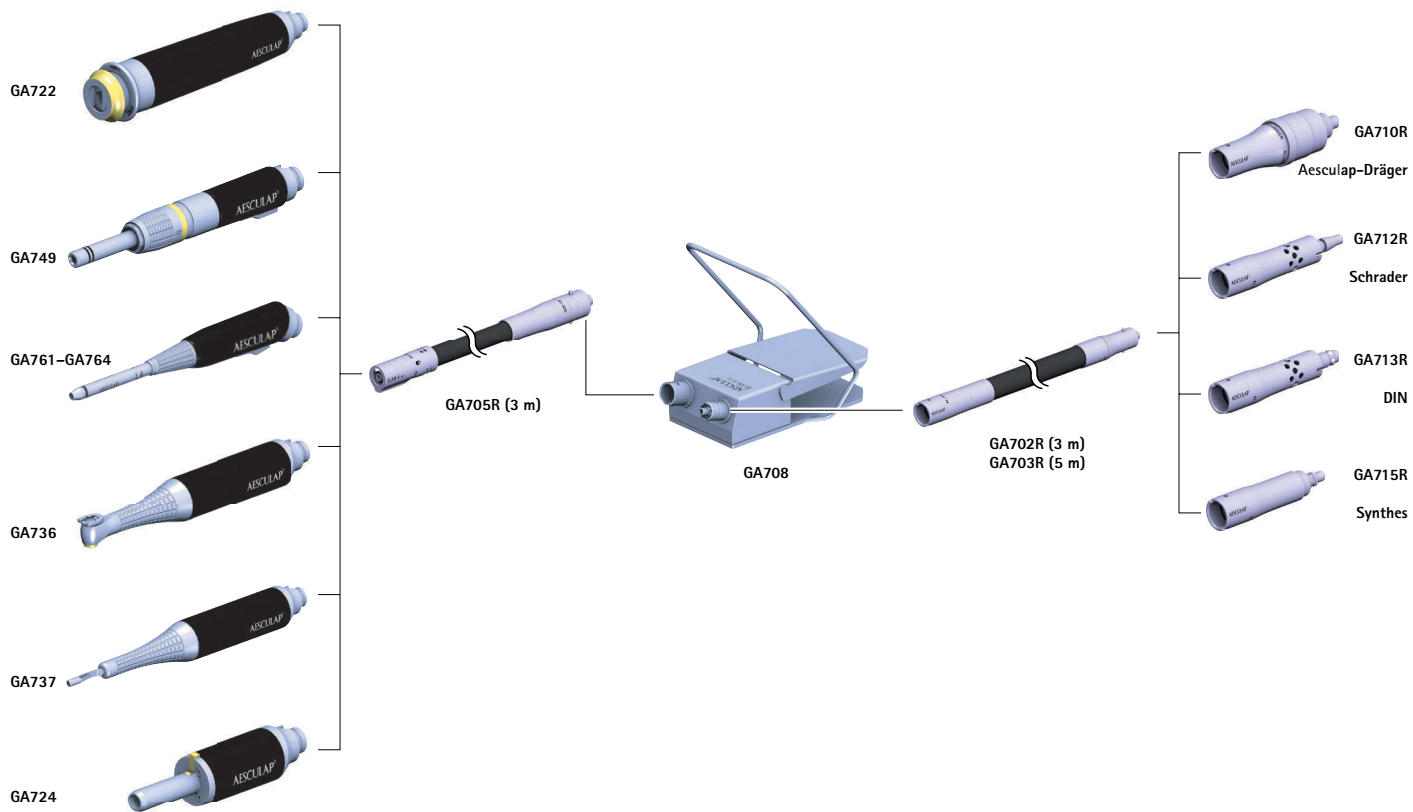Aesculap®

ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R,
ELAN 4 air wall adapters GA710R to GA715R

Legend

ELAN 4 air foot control GA708

- 1 Pedal
- 2 Connection for wall hose
- 3 Connection for motor hose
- 4 Bracket
- 5 Adapter for bracket

ELAN 4 air motor hose for foot control GA705R

- 6 Connection for applied part
- 7 Connection for foot control

ELAN 4 air wall hose GA702R/GA703R

- 8 Connection for wall adapter
- 9 Connection for foot control

ELAN 4 air wall adapter GA710R/GA712R/GA713R/GA715R

- 10 Connection for compressed-air source
- 11 Connection for wall hose

Coupling ELAN 4 air applied part with ELAN 4 air motor hose for foot control GA705R

- 12 Connection for motor hose of applied part
- 13 Field of vision "On"
- 14 Field of vision "Off"
- 15 Release button
- 16 Nib on motor hose
- 17 Groove on connection for motor hose

Symbols on product and packages

	Caution Observe important safety information such as warnings and precautions in the instructions for use.
 YYYY-MM	Maintenance label Recommended date of next service (format: YYYY-MM) by international B. Braun-/Aesculap agents, see Technical Service
	Machine-readable, two-dimensional code The code contains a unique serial number which can be used for electronic tracking of the individual instrument. The serial number is based on the global standard sGTIN (GS1).
	Date of manufacture
	Manufacturer's batch designation
	Manufacturer's serial number
	Manufacturer's article number
	Temperature limits during transport and storage
	Air humidity limits during transport and storage
	Atmospheric pressure limits during transport and storage

Contents

1.	Applicable to	3
2.	Intended use	3
2.1	Task/function within system	3
2.2	Application Environment	3
2.3	Indications	3
2.4	Contraindications	4
3.	Safe handling	4
4.	Product description	4
4.1	Scope of supply	4
4.2	Components required for operation	5
4.3	Operating principle	5
5.	Preparation and setup	5
5.1	Compressed-air supply	5
5.2	Operation with central gas supply according to DIN EN ISO 5359	5
5.3	Operation with compressed-gas cylinders according to DIN EN ISO 11117	6
5.4	Compressed-air hoses	6
5.5	First use of foot control	6
6.	Working with the foot control, wall hose and wall adapter ...	6
6.1	Function checks	6
6.2	System set-up	6
6.3	Safe operation	7
7.	Validated reprocessing procedure	7
7.1	General safety notes	7
7.2	General information	7
7.3	Preparations at the place of use	7
7.4	Cleaning/disinfection	8
7.5	Wipe disinfection	8
7.6	Inspection, maintenance and checks	8
8.	Maintenance	8
8.1	ELAN 4 air foot control, ELAN 4 air wall adapter	8
8.2	ELAN 4 air wall hose	8
9.	Troubleshooting list	9
10.	Technical Service	12
11.	Accessories/Spare parts	12
12.	Technical data	12
12.1	Classification acc. to Directive 93/42/EEC	12
12.2	ELAN 4 air foot control GA708	12
12.3	ELAN 4 air wall hose 3m GA702R	13
12.4	ELAN 4 air wall hose 5m GA703R	13
12.5	ELAN 4 air wall adapter Aesculap-Dräger GA710R	13
12.6	ELAN 4 air wall adapter Schrader with diffuser GA712R	13
12.7	ELAN 4 air wall adapter DIN with diffuser GA713R	13
12.8	ELAN 4 air wall adapter Synthes GA715R	13
12.9	Ambient conditions	13

13.	Disposal	13
14.	Distributor in the US/Contact in Canada for product information and complaints	13

1. Applicable to

- For item-specific instructions for use and information on material compatibility, see also the Aesculap Extranet at <https://extranet.bbraun.com>

2. Intended use

2.1 Task/function within system

The ELAN 4 air foot control, the ELAN 4 air wall hoses and the ELAN 4 air wall adapters are accessories of the ELAN 4 air motor system.

The foot control can be used by the user to control the motor speed of the connected ELAN 4 air applied part, provided that a hand control is not being used.

The ELAN 4 air wall hose connects the ELAN 4 air foot control to the ELAN 4 air wall adapters.

The ELAN 4 air wall adapters connect the ELAN 4 air wall hose to the compressed air supply unit.

2.2 Application Environment

Application in the unsterile area

Sterile separation takes place at the motor hose.

Installation place of the foot control	On the floor
--	--------------

2.3 Indications

Modes of application	Separating, removing and shaping hard tissue, cartilage and similar, as well as for drilling, reaming and sawing bone and bone replacement materials.
Surgical discipline/areas of application	Neurosurgery, ENT surgery, OMF surgery, orthopedics and trauma surgery.

Note

The type and area of application depend on the applied parts and tools selected.

ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R

2.4 Contraindications

The ELAN 4 air motor system is not authorized for use in the central nervous system or central circulatory system.

Note

The clinical success of the use of the ELAN 4 air motor system is dependent on the knowledge and experience of the surgeon. The surgeon must decide which structures it is sensible to treat and take into account the safety and warning information contained in these instructions for use.

3. Safe handling

CAUTION

Federal law restricts this device to sale by, or on order of a physician!

DANGER

Danger of fire and explosion when operating devices that run on oxygen!

- ▶ Never operate the pneumatic motor with oxygen!
- ▶ Only use compressed air or nitrogen to power pneumatic motors.

WARNING

Risk of injury and material damage due to inappropriate use of the product!

- ▶ Use the product only in accordance with the intended purpose.

WARNING

Risk of injury and damage to property due to improper handling of the product!

This product is part of the ELAN 4 air motor system.

- ▶ Follow the instructions for use of all products used.

- General risk factors associated with surgical procedures are not described in this documentation.
- It is the operating surgeon's responsibility to ensure that the surgical procedure is performed correctly.
- The operating surgeon must have a thorough understanding of both the hands-on and conceptual aspects of the established operating techniques.
- ▶ Clean the new product after removing its transport packaging.
- ▶ Prior to use, check that the product is in good working order.
- ▶ To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.

- ▶ Keep the instructions for use accessible for the user.
- ▶ Always adhere to applicable standards.
- ▶ Use the product within the specified pressure range only.

4. Product description

4.1 Scope of supply

ELAN 4 air foot control

Art. no.	Designation
GA708	ELAN 4 air foot control
TA014450	Instructions for use for ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R
TA014782	Supplement for ELAN 4 air motor system

ELAN 4 air wall hose

Art. no.	Designation
GA702R	ELAN 4 air wall hose 3 m
	–or–
GA703R	ELAN 4 air wall hose 5 m
TA014454	Instructions for use for ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R (leaflet)

ELAN 4 air wall adapter

Art. no.	Designation
GA710R	One of the following wall adapters:
GA712R	ELAN 4 air wall adapter Aesculap-Dräger
GA713R	ELAN 4 air wall adapter Schrader with diffuser
GA715R	ELAN 4 air wall adapter Synthes
TA014453	Supplement for ELAN 4 air wall adapters

4.2 Components required for operation

- ELAN 4 air wall adapter GA710R, GA712R, GA713R or GA715R
- ELAN 4 air wall hose GA702R or GA703R
- ELAN 4 air foot control GA708
- ELAN 4 air motor hose GA705R
- ELAN 4 air applied part
- Tool

4.3 Operating principle

ELAN 4 air foot control

By activating the pedal 1 of the ELAN 4 air foot control, the user controls the motor speed of the connected ELAN 4 air applied part. The pedal activates a valve of the ELAN 4 air foot control.

When the pedal is in its initial position, the valve is closed and the motor of the applied part does not run. When the pedal is in its end position, the valve is open and the motor of the applied part runs at maximum speed. When the pedal is released, the ELAN 4 air motor hose is vented.

ELAN 4 air wall hoses and ELAN 4 air wall adapters

The ELAN 4 air wall hoses and ELAN 4 air wall adapters guide compressed air from the compressed-air source to the foot control.

The wall hoses and wall adapters have a fast-lock coupling. This coupling prevents the escape of compressed air during detachment of the parts.

The wall hoses have an exit-air recycling system. The supply hose is embedded in an additional exit air hose, which takes the exit air away from the operating field.

There are two exit air recycling systems:

- Aesculap-Dräger and Synthes wall adapters with complete exit-air recycling: The exit air is channeled back from the O.R. into the central gas supply system. In this way, air turbulences and adverse effects on the climate and hygiene in the O.R. are avoided.
- Schrader and DIN wall adapters without complete exit-air recycling: The exit air enters the O.R. through a diffuser on the wall adapter.

5. Preparation and setup

Non-compliance with the following instructions will preclude all responsibility and liability in this respect on the part of Aesculap.

- ▶ When setting up and operating the product, adhere to
 - national regulations for installation and operation,
 - national legal regulations pertaining to compressed air supply,
 - National regulations on fire and explosion protection
- ▶ Before using the product and its accessories, check to be sure that neither are damaged.
- ▶ Use the products and their accessories only if they are in perfect technical condition.

5.1 Compressed-air supply

WARNING

Damage to persons and/or property due to the placing of the hose!

The user may injure the patient with an unintentional movement due to shock.

- ▶ Be careful not to damage the compressed-air hose.
- ▶ Ensure annual repair.

WARNING

Operating pressures in excess of 10bar will damage the motor system!

- ▶ Set an operating pressure of between 6 bar and 10 bar.

5.2 Operation with central gas supply according to DIN EN ISO 5359

The gas supply must provide clean, dry air and must be set as follows:

Nominal operating pressure	8 ± 2 bar
Maximum allowable operating pressure	10 bar
Maximum allowable backup pressure in exit air conduit while motor is operating (with exit air recycling)	0.5 bar to 0.7 bar
Minimum gas-supply delivery	300 l/min

5.3 Operation with compressed-gas cylinders according to DIN EN ISO 11117

The following conditions must be met:

- Compressed air or nitrogen is used
- An adequate stock of full gas cylinders is in place
- Cylinder pressure reduced to operating pressure of 6 bar to 10 bar through a pressure reducer (e.g., GA099)
- Operating pressure has been read from the scale on the device and set with the T-screw.

Note

If the system is operated with compressed gas cylinders, the exit air is not removed from the O.R. It emerges from the ELAN 4 wall adapter, away from the operating field.

Aesculap®

ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R

5.4 Compressed-air hoses

Impaired performance of the pneumatic motor/bursting of damaged compressed-air hoses!

- ▶ Lay compressed-air hoses only through unobstructed areas.
- ▶ Do not squeeze or block open hose ends or diffusers.
- ▶ Do not coil up compressed-air hoses to tightly (minimum coil diameter 20cm).
- ▶ Do not kink, squeeze or clamp compressed-air hoses.
- ▶ Verify that the compressed-air hoses are not damaged (e.g. by scalpels) or squeezed with sharp objects (e.g. cloth clips).

5.5 First use of foot control

Mounting/dismounting the bracket

- ▶ Insert bracket 4 on one side into the adapter for bracket 5.
- ▶ Widen the bracket 4 and insert it fully into the other adapter for bracket 5.
- ▶ Disassembly: Press bracket 4 together by the two lower horizontal ends and remove bracket 4 from the adapters for bracket 5.

6. Working with the foot control, wall hose and wall adapter

6.1 Function checks

- ▶ Check the ELAN 4 air motor system with a trial run prior to each application.
- ▶ Make certain that the applied part and foot control are in good working order.
- ▶ Make certain that all compressed-air hoses and the wall adapter are correctly connected and leak-tight.
- ▶ Check the secure connection of all products used.
- ▶ Make certain that the applied part runs at maximum speed when the foot control pedal is pushed all the way down.

6.2 System set-up

Note

Operating elements on the system components of the ELAN 4 air motor system are labeled with a gold marking.

Connecting wall adapter to compressed-air source

- ▶ Couple connection for compressed-air source 10 of the wall adapter with the compressed-air source.

Connecting the wall adapter to the foot control

- ▶ Attach the connection for wall adapter 8 of the wall hose to the connection for wall hose 11 of the wall adapter and couple the bayonet lock.

Connecting the wall hose to the foot control

- ▶ Attach the connection for foot control 9 of the wall hose to the connection for wall hose 2 on the foot control and couple the bayonet lock.

Connecting the motor hose to the foot control

- ▶ Attach the connection for foot control 7 of the motor hose to the connection for motor hose 3 on the foot control and couple the bayonet lock.

Connecting the applied part to the motor hose

- ▶ Attach the connection for motor hose of the applied part 12 to the connection for applied part 6 of the motor hose, see Fig. A. When doing so, ensure that the nib 16 on the motor hose is aligned with the groove 17 on the connection for motor hose 12.
The applied part engages. When the field of vision is "Off" 14 on the motor hose, a gold-colored marking is visible.

Note

The applied part attached to this motor hose is only ready for operation when the release button is in "On" position.

Releasing the applied part for operation ("On" position)

- ▶ Activate the release button 15 on the motor hose and push the connection for applied part 6 onto the applied part, see Fig. A.
The applied part engages. When the field of vision is "On" 13 on the motor hose, a gold-colored marking is visible.

Locking the applied part ("Off" position)

- ▶ Activate the release button 15 on the motor hose and detach the motor hose from the applied part.
The applied part engages. When the field of vision is "Off" 14 on the motor hose, a gold-colored marking is visible.

Disconnecting the applied part from the motor hose

- ▶ Activate the release button 15 on the motor hose and pull at the connection for applied part 6 to disconnect the motor hose from the applied part.

Disconnecting the motor hose from the foot control

- ▶ Disconnect the bayonet lock of the connection for foot control 7 of the motor hose and detach motor hose from foot control.

Disconnecting the wall hose from the foot control

- ▶ Disconnect the bayonet lock of the connection for foot control **9** of the wall hose and detach wall hose from foot control.

Disconnecting the wall hose from the wall adapter

- ▶ Turn the connection for wall adapter **8** of the wall hose into the middle position of the bayonet lock to ensure pressure equalization and prevent any loud bangs.
- ▶ Disconnect the wall hose fully from the wall adapter.

Disconnecting the wall adapter from the compressed-air source

- ▶ Uncouple the connection for compressed-air source **10** of the wall adapter from the compressed-air source.

Note

It should be noted that successful reprocessing of this medical device can only be guaranteed following prior validation of the reprocessing method. The operator/reprocessing technician is responsible for this.

The recommended chemistry was used for validation.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

For up-to-date information about reprocessing and material compatibility, see also the Aesculap Extranet at <https://extranet.bbraun.com>

The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

6.3 Safe operation

WARNING

Risk of injury and/or malfunction!

- ▶ Always carry out a function check prior to using the product.

WARNING

Risk of injury and material damage due to inadvertent activation of the applied part!

- ▶ Secure applied parts that are not actually used against inadvertent activation (off position).

Operation of the applied part is only possible if:

- the applied part is released ("On" position)
- a foot control or hand control is connected.

Activating the applied part via foot control

- ▶ Press the pedal **1** on the ELAN 4 air foot control.
The motor of the applied part runs.
- ▶ Adjusting speed: Press down on the pedal **1** to a greater or lesser extent.
- ▶ To select maximum speed: Press down on the pedal **1** as far as it will go.

7. Validated reprocessing procedure

7.1 General safety notes

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for reprocessing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD, or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

7.2 General information

Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixating pre-cleaning temperatures >45 °C nor fixating disinfecting agents (active ingredient: aldehydes/alcohols) should be used.

Excessive neutralizing agents or basic cleaners may result in a chemical attack and/or fading and the laser marking becoming unreadable either visually or by machine.

Residues containing chlorine or chlorides, e.g., in surgical residues, medicines, saline solutions, and in the service water used for cleaning, disinfection, and sterilization, will cause corrosion damage (pitting, stress corrosion) and result damage to metallic products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Perform additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Material damage such as corrosion, cracks, fracturing, premature aging, or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surface and could cause corrosion
- ▶ For further detailed information on hygienically safe and material-preserving/value-preserving reprocessing, see www.a-k-i.org, link to Publications, Red Brochure – Proper maintenance of instruments.

7.3 Preparations at the place of use

- ▶ Separate the products from each other immediately after use.
- ▶ Remove any visible surgical residues as much as possible with a damp, lint-free cloth.
- ▶ Carry out non-fixating/NaCl-free pre-cleaning immediately after use.
- ▶ Send the product in a dry condition for cleaning and disinfection within 6 hours.

Aesculap®

ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R,
ELAN 4 air wall adapters GA710R to GA715R

7.4 Cleaning/disinfection

Product-specific safety instructions for the reprocessing procedure

CAUTION

Damage to, or destruction of the product caused by mechanical cleaning/disinfection!

- Only clean and disinfect the product manually.
- Do not sterilize the product under any circumstances.

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents!

- Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.

7.5 Wipe disinfection

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Wipe disinfection	RT	≥ 1	-	-	Meliseptol HBV wipes 50 % Propan-1-ol

RT: Room temperature

Phase I

- Remove any visible residues with a disposable disinfectant wipe.
- Wipe all surfaces of the optically clean product with a fresh, disposable disinfectant wipe.
- Observe the specified application time (1 min minimum).

7.6 Inspection, maintenance and checks

- Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.
- Set aside the product if it is damaged.

8. Maintenance

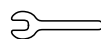
8.1 ELAN 4 air foot control, ELAN 4 air wall adapter

To ensure reliable operation, the product must be maintained at least once every two to three years.

For technical service, please contact your national B. Braun/Aesculap agency, see Technical Service.

8.2 ELAN 4 air wall hose

To ensure reliable operation, the product must be maintained in accordance with the maintenance labeling or at least once a year.

e.g. 2016-07

For technical service, please contact your national B. Braun/Aesculap agency, see Technical Service.

9. Troubleshooting list

Malfunction	Cause	Detection	Remedy
Applied part not running	No compressed air	Motor coupling in "Off" position	Release the applied part for operation ("On" position).
		Hoses and wall adapters not correctly coupled	Check the coupling points.
		No compressed air at the wall outlet	Check the building technology.
		Gas cylinder empty (Pressure Gage set to "0")	Change cylinder.
		Valve on cylinder closed	Open valve.
	Applied part defective	Tool not moving	Have applied part repaired by manufacturer.
Disconnecting/connecting the applied part from/with the motor hose not possible (change from "On" to "Off" position and vice versa)	Foot control defective	Pedal not moving	Have foot control repaired by manufacturer.
	Release button on the motor hose is not released between connecting/disconnecting and releasing/locking the applied part	Coupling processes cannot be performed	Release the release button on the motor hose between connecting/disconnecting and releasing/locking the applied part and activate again.
			Have applied part or motor hose repaired by manufacturer.
Applied part runs without activating the foot control	Foot control defective	Applied part runs without activating the foot control	Have foot control repaired by manufacturer.

Aesculap®

ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R

Malfunction	Cause	Detection	Remedy
Insufficient power	Blunt tool	Cutting edge worn	Use a new tool.
		Cutting edges coated with substances, e.g. from too little rinsing	Use a new tool.
	No central alignment of the craniotome cutter to the dura guard bracket	Dura guard bracket bent	Use new dura guard, follow instructions for use (TA014473/TA014474).
		Poor progress during craniotomy	Have dura guard bracket repaired by manufacturer
	Applied part defective	Insufficient power of applied part	Follow instructions for use of the applied part (reprocessing, maintenance).
			Have applied part repaired by manufacturer.
		Large amount of heat generated after a short time	Follow instructions for use of the applied part (reprocessing, maintenance)
	Air resistance in hoses and wall adapter too high	Use of extension hoses	Use only one wall hose.
		Hoses kinked or crushed	Lay hoses only through unobstructed areas.
		No free escape of exit air at diffuser	Free up the openings on the diffuser.
Applied part is getting hot	Air supply too low	Operating pressure too low	Adjust the operating pressure to min. 6 bar.
		Pressure reducer at cylinder defective	Check the pressure reducer.
		Leak in hose/hose coupling	Have hose repaired by manufacturer.
	Excessive use	Applied part heats up Loud running noise/vibration	Follow instructions for use of the applied part (interval operation).
	Reprocessing carried out incorrectly		Follow instructions for use of the applied part (reprocessing, maintenance).
	Shaft of applied part bent		Have applied part repaired by manufacturer.
	Applied part defective		Have applied part repaired by manufacturer.

Malfunction	Cause	Detection	Remedy
Hightspeed applied part vibrates excessively	Shaft of applied part bent	Excessive running noises, vibrations	Have applied part repaired by manufacturer.
	Applied part defective		Have applied part repaired by manufacturer.
	Reprocessing carried out incorrectly		Follow instructions for use of the applied part (reprocessing, maintenance).
Tool cannot be coupled or uncoupled	ELAN 4 air Hightspeed: Automatic tool lock blocked	Hightspeed tool doesn't engage	Activate slide for tool lock and hold, then couple tool.
	ELAN 4 air Hightspeed: Slider for tool lock locked	Applied part released for operation ("On" position)	Lock the applied part ("Off" position).
	Incompatible tool	Wrong tool	Select an appropriate tool according to the instructions for use of the applied part.
	Tool connection or coupling deformed, defective	Tool is difficult to couple or cannot be coupled	Use a new tool. Have applied part repaired by manufacturer.
Attachment for GA749 (craniotome) cannot be coupled or uncoupled	Automatic attachment lock blocked	Attachment doesn't engage	Pull back unlocking sleeve and hold, then couple the attachment.
	Unlocking sleeve blocked	Applied part released for operation ("On" position)	Lock the applied part ("Off" position).
	Attachment coupling defective	Attachment is difficult to couple or cannot be coupled	Use a new attachment. Have applied part repaired by manufacturer.
	Attachment coupling is dirty	Attachment is difficult to couple or cannot be coupled	Clean the attachment or use a new attachment. Clean the applied part.
Turnable dura guard GB947R is difficult to turn	Bearing dirty or worn	Turnable dura guard is difficult to move	Follow the instructions for use (TA014473/TA014474) (reprocessing, maintenance).
			Have turnable dura guard repaired by manufacturer.
Hoses/wall adapters/foot control cannot be coupled/uncoupled	Hoses/wall adapters/foot control not compatible	Hoses/wall adapters are difficult to couple/uncouple or cannot be coupled/uncoupled	Select an appropriate hose/wall adapter/foot control according to the instructions for use.
	Hose coupling deformed, defective		Have hose/wall adapter/foot control repaired by manufacturer.

ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R

10. Technical Service

Danger to life of patients and users if the product malfunctions and/or protective measures fail or are not used!

- ▶ **Do not perform any servicing or maintenance work under any circumstances while the product is being used on a patient.**
- ▶ **Do not modify the product.**

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency.
- ▶ For safe transport and dispatch, use the original packaging.

Note

This packaging can be procured from the national B. Braun/Aesculap agencies.

Service addresses

Aesculap Technischer Service
Am Aesculap-Platz
78532 Tuttlingen / Germany
Phone: +49 (7461) 95 -1601
Fax: +49 (7461) 14 -939
E-Mail: ats@aesculap.de
Or in the US:
Aesculap Inc.
Attn. Aesculap Technical Services
615 Lambert Pointe Drive
Hazelwood
MO, 63042 USA
Aesculap Repair Hotline
Phone: +1 (800) 214 -3392
Fax: +1 (314) 895 -4420

Other service addresses can be obtained from the address indicated above.

11. Accessories/Spare parts

Art. no.	Designation
GA708203	Bracket
GA702R	ELAN 4 air wall hose 3m
GA703R	ELAN 4 air wall hose 5m
GA705R	ELAN 4 air motor hose 3m for foot control
GA710R	ELAN 4 air wall adapter Aesculap-Dräger
GA712R	ELAN 4 air wall adapter Schrader with diffuser
GA713R	ELAN 4 air wall adapter DIN with diffuser
GA715R	ELAN 4 air wall adapter Synthes
TA014450	ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R (A4 for ring binder)
TA014454	ELAN 4 air foot control GA708, ELAN 4 air wall hoses GA702R/GA703R, ELAN 4 air wall adapters GA710R to GA715R (leaflet)

12. Technical data

12.1 Classification acc. to Directive 93/42/EEC

Art. no.	Designation	Class
GA708	ELAN 4 air foot control	I
GA702R	ELAN 4 air wall hose 3m	I
GA703R	ELAN 4 air wall hose 5m	I
GA710R	ELAN 4 air wall adapter Aesculap-Dräger	I
GA712R	ELAN 4 air wall adapter Schrader with diffuser	I
GA713R	ELAN 4 air wall adapter DIN with diffuser	I
GA715R	ELAN 4 air wall adapter Synthes	I

12.2 ELAN 4 air foot control GA708

Weight	2.1 kg ± 10 %
Dimensions (L x W x H)	235 mm x 105 mm x 60 mm ± 10 % For mounted bracket: 235 mm x 160 mm x 192 mm ± 10 %

12.3 ELAN 4 air wall hose 3m GA702R

Weight	0.7 kg $\pm$ 10 %
Dimensions ($\varnothing$ x L)	26 mm x 3 000 mm $\pm$ 10 %

12.4 ELAN 4 air wall hose 5m GA703R

Weight	1.06 kg $\pm$ 10 %
Dimensions ($\varnothing$ x L)	26 mm x 5 000 mm $\pm$ 10 %

12.5 ELAN 4 air wall adapter Aesculap-Dräger GA710R

Weight	290 g $\pm$ 1 0%
Dimensions ($\varnothing$ x L)	37 mm x 121 mm $\pm$ 10%

12.6 ELAN 4 air wall adapter Schrader with diffuser GA712R

Weight	154 g $\pm$ 10 %
Dimensions ($\varnothing$ x L)	26 mm x 126 mm $\pm$ 10%

12.7 ELAN 4 air wall adapter DIN with diffuser GA713R

Weight	156 g $\pm$ 10 %
Dimensions ($\varnothing$ x L)	26 mm x 121 mm $\pm$ 10 %

12.8 ELAN 4 air wall adapter Synthes GA715R

Weight	150 g $\pm$ 10 %
Dimensions ($\varnothing$ x L)	26 mm x 110 mm $\pm$ 10%

12.9 Ambient conditions

	Operation	Storage and transport
Temperature	10 °C to 27 °C	-10 °C to 50 °C
Relative humidity	30 % to 75 %	10 % to 90 %
Atmospheric pressure	700 hPa to 1 060 hPa	500 hPa to 1 060 hPa

13. Disposal

Note

The user institution is obliged to process the product before its disposal, see Validated reprocessing procedure.

- Adhere to national regulations when disposing of or recycling the product, its components and its packaging.
- Detailed information concerning the disposal of the product is available through your national B. Braun/Aesculap agency, see Technical Service.

14. Distributor in the US/Contact in Canada for product information and complaints

Aesculap Inc.
3773 Corporate Parkway
Center Valley, PA, 18034,
USA

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