en	Supplement ELAN 4 Dura guard/Holding sleeve
USA	Note for U.S. users This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusa.com . If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.
de	Beiblatt ELAN 4 Duraschutz/Haltehülse
fr	Notice d'accompagnement Protecteur de dure-mère / douille de préhension ELAN 4
es	Hoja adjunta ELAN 4 Casquillo de retención Duraschutz
it	Allegato ELAN 4 Protezione dura madre/bussola di tenuta
pt	Adenda Proteção da dura-máter/bucha fixadora ELAN 4
nl	Bijlage ELAN 4-durabeschermer/borghuls
sv	Bifogat blad ELAN 4 duraskydd/hållarhylsa
et	Lisa ELAN 4 Dura-kaitse/kinnitusliitmik
ru	Вкладыш Защита твердой мозговой оболочки/фиксирующая втулка ELAN 4
cs	Příloha Ochrana dura mater/přidržná objímka ELAN 4
pl	Załącznik ELAN 4 Zabezpieczenie opony twardej / tuleja mocująca
sk	Príloha ELAN 4 Pevná ochrana/Fixačná objímka
tr	Ek sayfa ELAN 4 Dura koruması/Asma kovanı

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany
Phone +49 (0) 7461 95-0 | Fax +49 (0) 7461 95-26 00 | www.aesculap.com

Aesculap® – a B. Braun brand

TA014446 2020-03 V6 Change No. AE0060262

ELAN 4 Dura guard/Holding sleeve

Art. no.	Designation
GB941R	ELAN 4 Dura guard, fixed, pediatric
GB942R	ELAN 4 Dura guard, fixed, standard
GB943R	ELAN 4 Dura guard, fixed, long
GB945R	ELAN 4 Holding sleeve for craniotome and multifunction handpiece
GB947R	ELAN 4 Dura guard, rotating, standard

For use with:

Art. no.	Designation	Instructions for use
GA749	ELAN 4 air Craniotome and multifunction handpiece (2-ring)	TA014473/TA014474
GA849	ELAN 4 electro Craniotome and multifunction handpiece (2-ring)	TA014438/TA014439

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