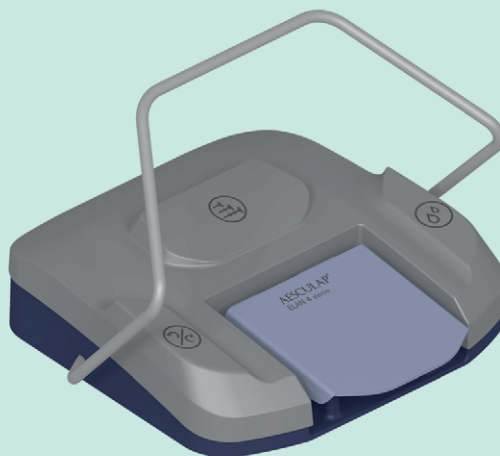
(GB) Instructions for use/Technical description

(USA) ELAN 4 electro Remote foot control GA810

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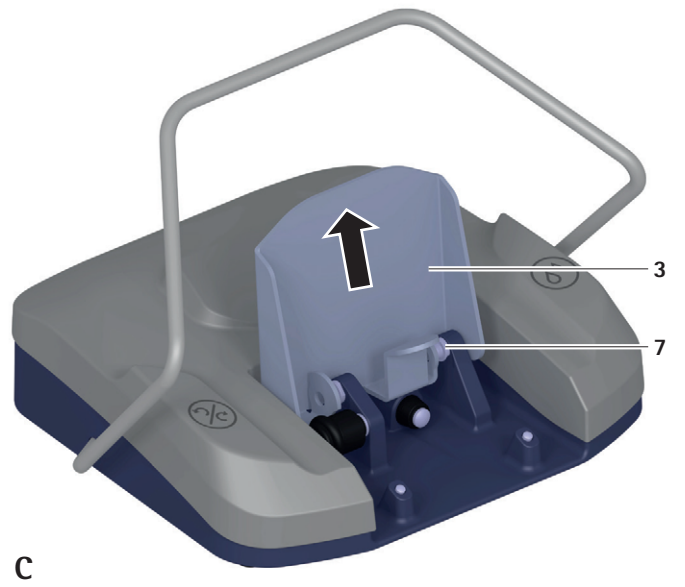
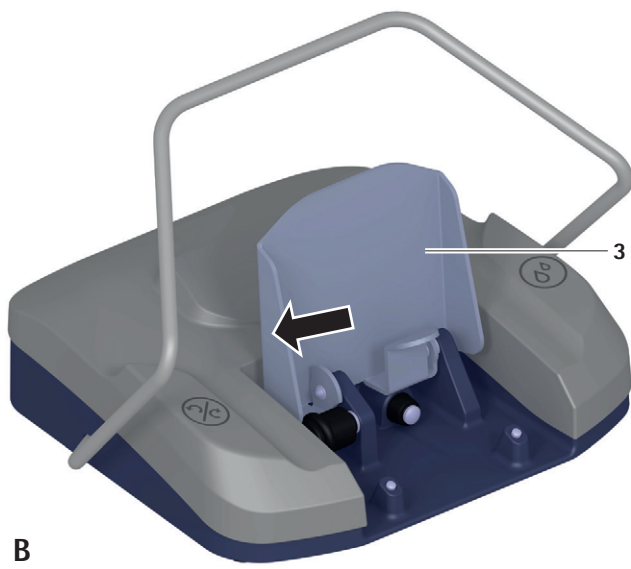
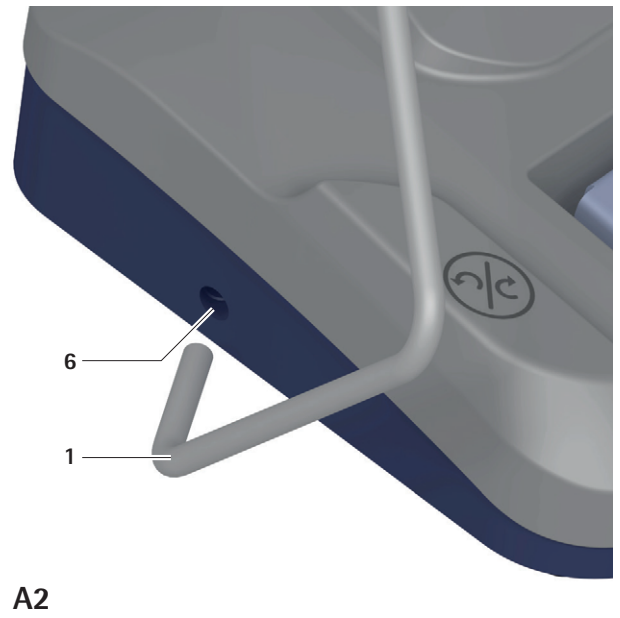
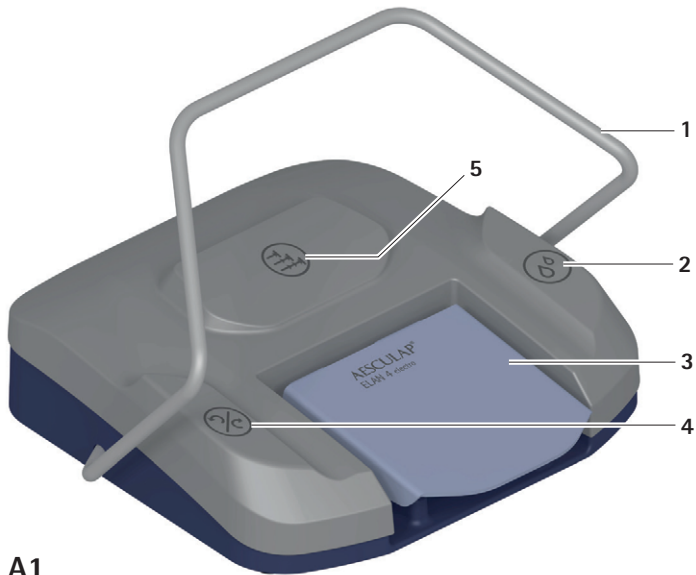
Temperature limits during transport and storage

Air humidity limits during transport and storage

Atmospheric pressure limits during transport and storage

Symbols on product and packages

	Caution Observe important safety information such as warnings and precautions in the instructions for use.
	Manufacturer combined with date of manufacture (year)
	Manufacturer's serial number
	Rotational direction setting for motor
	Pump setting
	Changing the maximum speed
	Device Class AP according to IEC/DIN EN 60601-1
	Labeling of electrical and electronic devices according to directive 2012/19/EU (WEEE), see Disposal
	Date of manufacture
	Manufacturer's batch designation
	Manufacturer's article number

Contents

1.	Intended use	3
1.1	Task/function within system	3
1.2	Application Environment	3
2.	Safe handling	3
3.	Product description	3
3.1	Scope of supply.	3
3.2	Components required for operation	3
3.3	Operating principle.	4
4.	Preparation and setup	4
4.1	First use.	4
5.	Working with the ELAN 4 electrical remote foot control	4
6.	Disassembling the pedal for cleaning.	4
7.	Mounting the pedal (after cleaning).	4
8.	Validated reprocessing procedure.	4
8.1	General safety notes.	4
8.2	Preparation before cleaning	5
8.3	Cleaning/disinfection	5
8.4	Wipe disinfection for electrical devices without sterilization	5
8.5	Inspection, maintenance and checks	6
9.	Maintenance.	6
10.	Troubleshooting list	6
11.	Technical Service	6
12.	Accessories/Spare parts	6
13.	Technical data.	6
13.1	Classification acc. to Directive 93/42/EEC	6
13.2	Basic data, information about standards	7
13.3	Ambient conditions	7
14.	Disposal.	7
15.	Distributor in the US/Contact in Canada for product information and complaints	7

1. Intended use

1.1 Task/function within system

The ELAN 4 electro remote foot control GA810 is an accessory for the ELAN 4 electro motor system.

With the remote foot control:

- the user activates the motor of the applied part and selects the rotational direction,
- the user activates/deactivates the coolant pump,
- the user can change the maximum speed of the motor.

Note

The ELAN 4 electro remote foot control GA810 cannot be used if a manually controlled applied part or a cable-connected foot control is used.

1.2 Application Environment

Application in the unsterile area

Remote foot controls are Class AP devices. The foot control circuit is ignition-safe and approved for operation in medical environments according to IEC /DIN EN 60601-1. The housing is constructed according to Protection Type IPX8.

Set-up location	Floors
-----------------	--------

2. Safe handling

CAUTION

Federal law restricts this device to sale by, or on order of a physician!

WARNING

Risk of injury and material damage due to inappropriate use of the product!

- Use the product only in accordance with the intended purpose.

WARNING

Risk of injury and damage to property due to improper handling of the product!

This product is an accessory for the ELAN 4 electro control unit GA800.

- Follow the instructions for use of the ELAN 4 electro control unit GA800 (TA014401).
- Follow the instructions for use of all products used.

- General risk factors associated with surgical procedures are not described in this documentation.
- It is the operating surgeon's responsibility to ensure that the surgical procedure is performed correctly.
- The operating surgeon must have a thorough understanding of both the hands-on and conceptual aspects of the established operating techniques.
- Prior to use, check that the product is in good working order.
- To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.

3. Product description

3.1 Scope of supply

Art. no.	Designation
GA810	ELAN 4 electro Remote foot control
TA014411	Instructions for use of the remote foot control GA810 (brochure)

3.2 Components required for operation

- ELAN 4 electro control unit GA800
- ELAN 4 electro motor cable for foot control GA806
- ELAN 4 electro applied part
- Tool

Aesculap®

ELAN 4 electro Remote foot control GA810

3.3 Operating principle

With the ELAN 4 electro remote foot control GA810 the active applied part is connected to the control unit without manual control. The coolant pump is activated or deactivated and the maximum speed and direction of the applied part is changed.

Function button 2:

Brief actuation	Activation/deactivation of the coolant pump. The pump only works when a motor is running.
Prolonged Pressing	Flush function The pump operates until activation is ended.

Rotational direction button for motor 4: Selection of right or left rotation.

Pedal 3: The pedal serves as the actuator for the speed setting of the control unit. Depending on the actuating path, the speed of the motor can be selected continuously from 0 to the maximum speed set.

Maximum speed button 5: The maximum speed button is for step-by-step changes to the set maximum speed on the control device GA800 with full operation of the pedal.

4. Preparation and setup

Non-compliance with the following instructions will preclude all responsibility and liability in this respect on the part of Aesculap.

- ▶ When setting up and operating the product, adhere to
 - national regulations for installation and operation,
 - national regulations on fire and explosion protection.

4.1 First use

Assembling/disassembling the transport brace

- ▶ Insert transport brace 1 on one side into the holding receptacle for transport brace 6, see Fig. A2.
- ▶ Widen the transport brace 1 and insert it into the other holding receptacle for transport brace 6. Make certain that the transport brace is correctly mounted.

Note

Disassembly is carried out in the reverse order.

5. Working with the ELAN 4 electrical remote foot control

How to work with the ELAN 4 electro system and its accessory components is described in the instructions for use of the ELAN 4 electro control unit GA800, see TA014401.

6. Disassembling the pedal for cleaning

- ▶ Lift the pedal 3 upwards, see Fig. B.
- ▶ Remove pedal 3 from the guide bolts for pedal 7.
- ▶ Remove the pedal 3 in an upward direction, see Fig. C.

7. Mounting the pedal (after cleaning)

- ▶ Insert pedal 3 onto guide bolts for pedal 7, see Fig. B.
- ▶ Fold pedal 3 down.

8. Validated reprocessing procedure

8.1 General safety notes

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for reprocessing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD, or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

It should be noted that successful reprocessing of this medical device can only be guaranteed following prior validation of the reprocessing method. The operator/reprocessing technician is responsible for this.

The recommended chemistry was used for validation.

8.2 Preparation before cleaning

- ▶ Remove any visible surgical residues as much as possible with a damp, lint-free cloth.
- ▶ Carry out non-fixating/NaCl-free pre-cleaning immediately after use.

Note

The pedal can be removed for cleaning, see *Disassembling the pedal for cleaning*.

8.3 Cleaning/disinfection

Product-specific safety instructions for the reprocessing procedure

CAUTION

Damage to, or destruction of the product caused by mechanical cleaning/disinfection!

- ▶ Only clean and disinfect the product manually.
- ▶ Do not sterilize the product under any circumstances.

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents!

- ▶ Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.

8.4 Wipe disinfection for electrical devices without sterilization

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Wipe disinfection	RT	≥1	-	-	Meliseptol HBV wipes 50 % Propan-1-ol

RT: Room temperature

- ▶ Remove any visible residues with a disposable disinfectant wipe.
- ▶ Wipe all surfaces of the optically clean product with a fresh, disposable disinfectant wipe.
- ▶ Observe the specified application time (1 min minimum).

8.5 Inspection, maintenance and checks

- ▶ Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.
- ▶ Set aside the product if it is damaged.
- ▶ Change battery when the energy level is low:
 - Remove the 4 screws on the bottom of the device with a screw-driver (Torx TX20).
 - Remove the cover of the battery compartment. Replace batteries (3x) (battery type D / LR20 / Mono).
 - Note the polarity of the batteries.
 - Replace the battery compartment cover and secure with 4 screws.

9. Maintenance

To ensure reliable operation, the product must be maintained at least once a year.

For technical service, please contact your national B. Braun/Aesculap agency, see Technical Service.

10. Troubleshooting list

Note

For further information, see instructions for use of ELAN 4 electro control unit GA800 (TA014401).

11. Technical Service

DANGER

Danger to life of patients and users if the product malfunctions and/or protective measures fail or are not used!

- ▶ Do not perform any servicing or maintenance work under any circumstances while the product is being used on a patient.
- ▶ Do not modify the product.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

- ▶ For service and repairs, please contact your national B. Braun/Aesculap agency.

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 (7461) 95 -1601

Fax: +49 (7461) 14 -939

E-Mail: ats@aesculap.de

Or in the US:

Aesculap Inc.

Attn. Aesculap Technical Services

615 Lambert Pointe Drive

Hazelwood

MO, 63042 USA

Aesculap Repair Hotline

Phone: +1 (800) 214 -3392

Fax: +1 (314) 895 -4420

Other service addresses can be obtained from the address indicated above.

12. Accessories/Spare parts

Art. no.	Designation
TA014411	Instructions for use of the ELAN 4 electro remote foot control GA810 (brochure)

13. Technical data

13.1 Classification acc. to Directive 93/42/EEC

Art. no.	Designation	Class
GA810	ELAN 4 electro Remote foot control	I

13.2 Basic data, information about standards

For connection to	ELAN 4 electro control unit GA800
Protection class according to IEC/ DIN EN 60601-1	AP
Housing protection class according to IEC/DIN 60529	IPX8
Weight	3810 g $\pm$ 10 %
Dimensions (L x W x H) without transport brace	271 mm x 266 mm x 85 mm $\pm$ 10 %
Dimensions (L x W x H) with transport brace	271 mm x 310 mm x 200 mm $\pm$ 10 %
Conforming to standard	IEC/DIN EN 60601-1
EMC	IEC/DIN EN 60601-1-2
Battery type	D / LR20 / Mono (3 x)

13.3 Ambient conditions

	Operation	Storage and transport
Temperature	10 °C to 40 °C	-10 °C to 50 °C
Relative humidity	30 % to 75 %	10 % to 90 %
Atmospheric pressure	700 hPa to 1 060 hPa	500 hPa to 1 060 hPa

14. Disposal

Note

The user institution is obliged to process the product before its disposal, see Validated reprocessing procedure.

Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

The recycling pass can be downloaded from the Extranet as a PDF document under the respective article number. (The recycling pass includes disassembling instructions for the product, as well as information for proper disposal of components harmful to the environment.)

Products carrying this symbol are subject to separate collection of electrical and electronic devices. Within the European Union, disposal is taken care of by the manufacturer as a free-of-charge service.

- Detailed information concerning the disposal of the product is available through your national B. Braun/Aesculap agency, see Technical Service.

15. Distributor in the US/Contact in Canada for product information and complaints

Aesculap Inc.
3773 Corporate Parkway
Center Valley, PA, 18034,
USA

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Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany