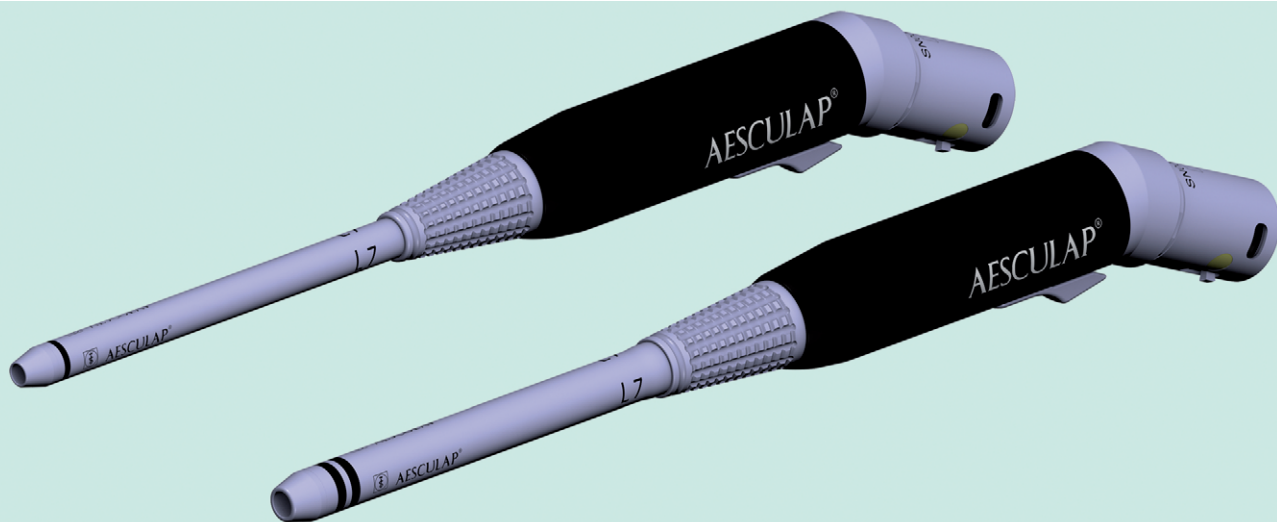
en Instructions for use/Technical description

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USA

Note for U.S. users

This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusa.com. If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.

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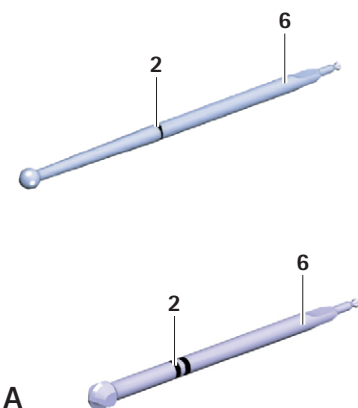
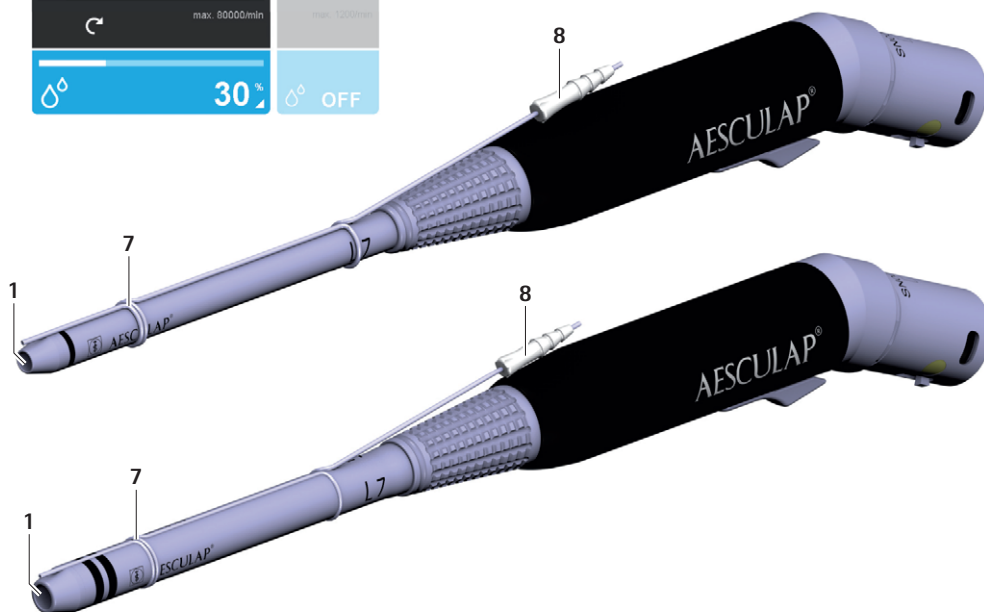
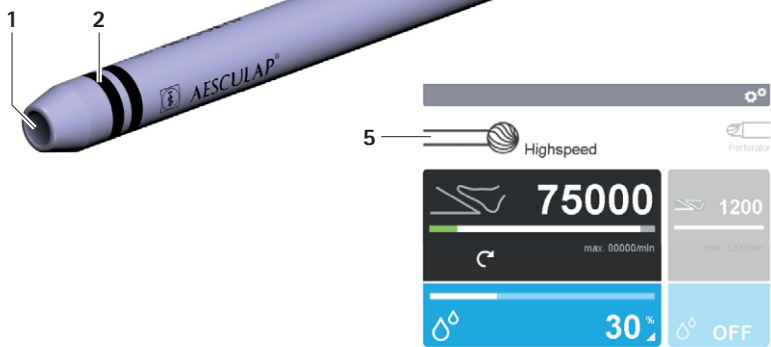
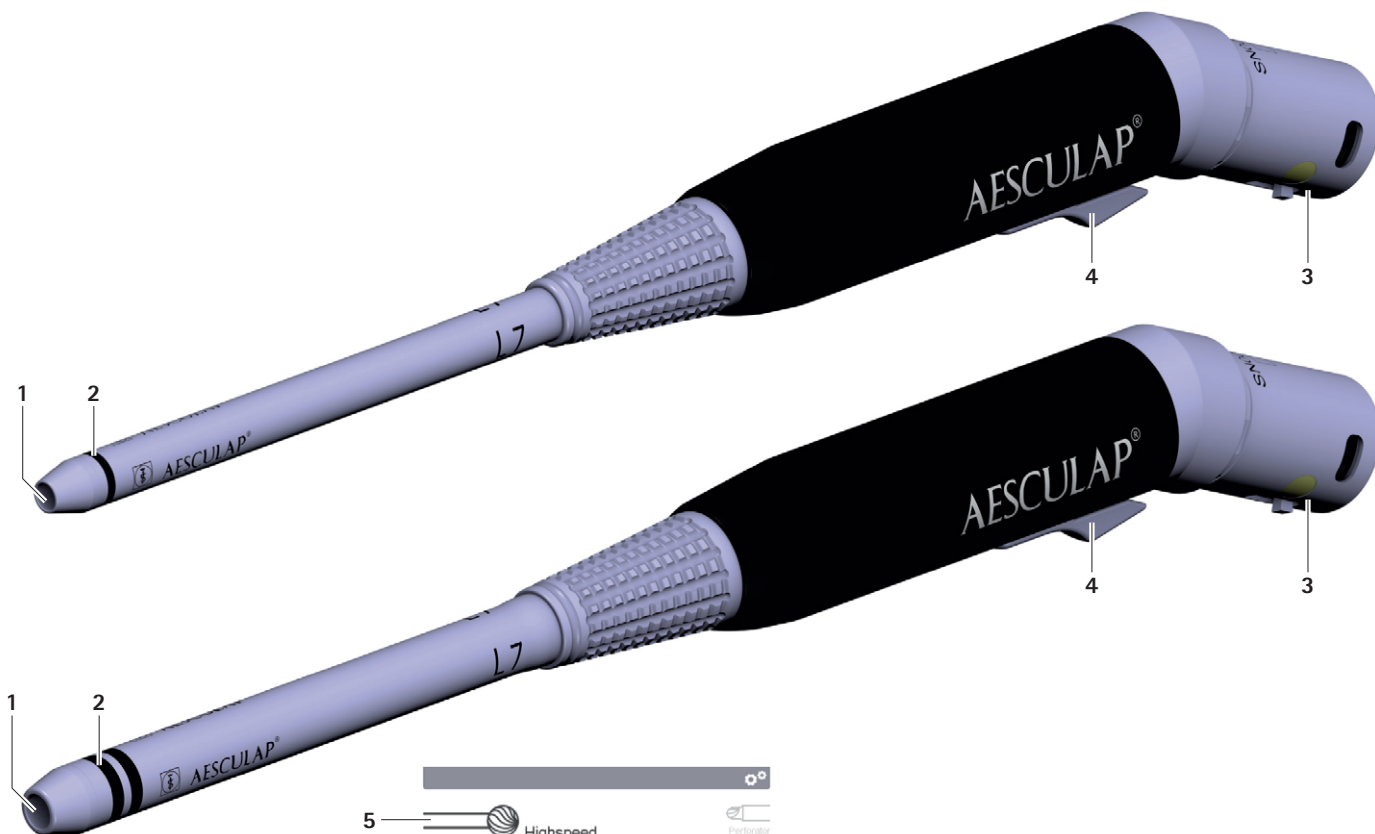
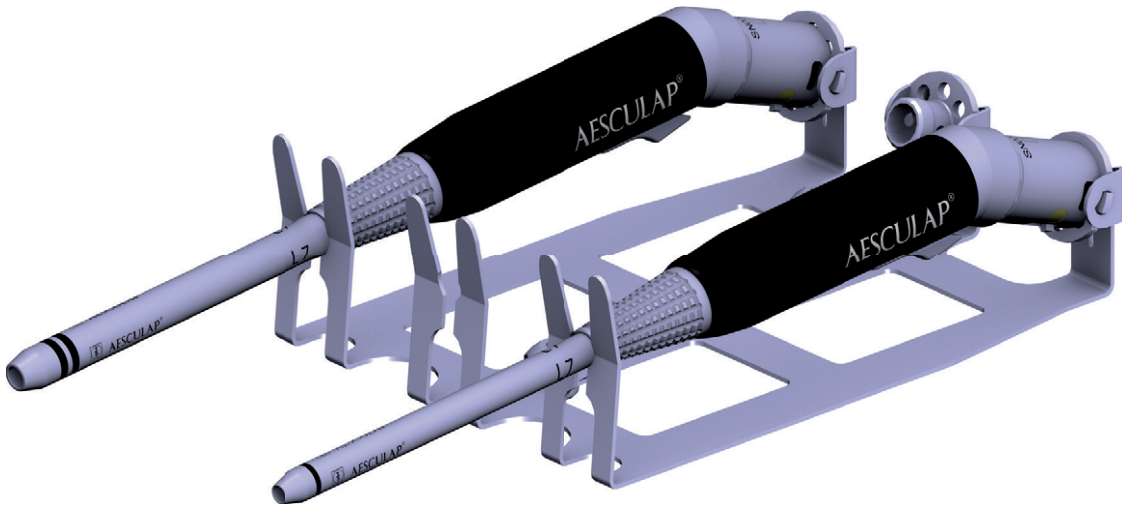
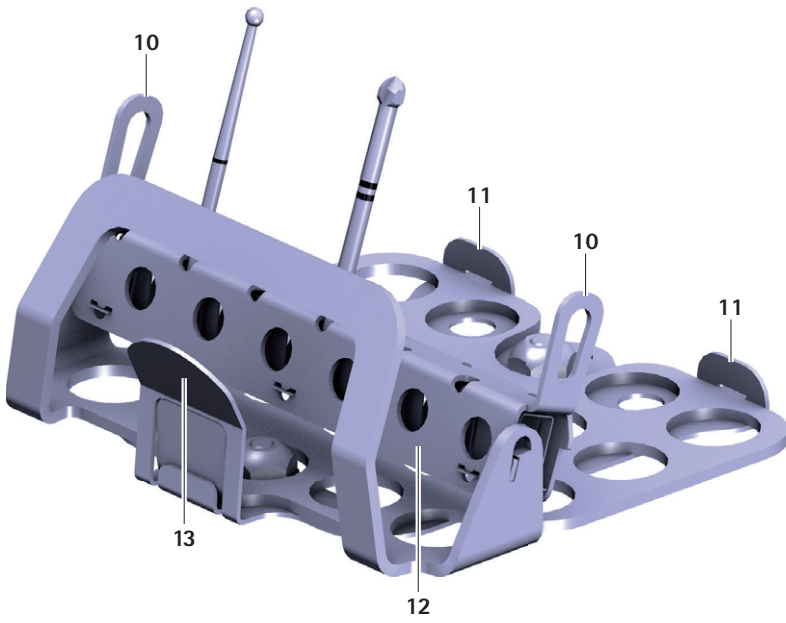
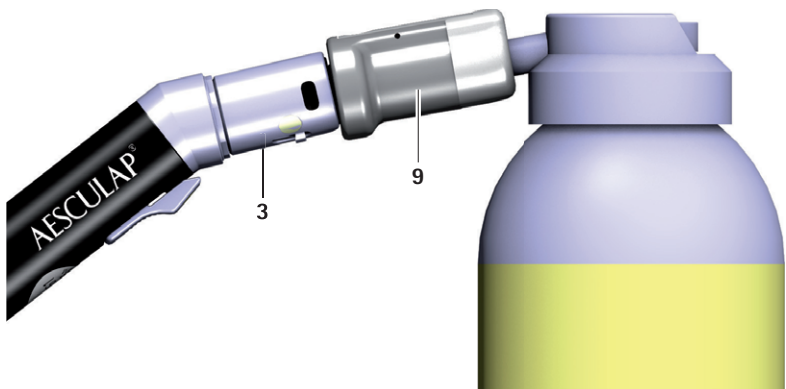
D

Aesculap®

ELAN 4 electro Standard handpiece (1 ring) GA861 to GA864 / ELAN 4 electro HD handpiece (2 ring) GA865 to GA869

Legend

- 1 ELAN 4 tool coupling (1 ring or 2 ring)
- 2 Ring marking
- 3 Connector for motor cable
- 4 Slider for tool release
- 5 Symbol of the applied part type in display of ELAN 4 electro control unit
- 6 ELAN 4 tool (1 ring or 2 ring)
- 7 ELAN 4 single-use spray nozzle for 1 ring or 2 ring handpieces
- 8 Tube connector on spray nozzle
- 9 Oil spray adapter GB600860

ELAN 4 ECCOS holder for 12 reamers GB718R

- 10 Side handles
- 11 Holding nibs
- 12 Tool coupling
- 13 Tab

Symbols on product and packages

	Caution Observe important safety information such as warnings and precautions in the instructions for use.
 YYYY-MM	Maintenance indicator Date of the next recommended servicing (year-month) by the international B. Braun-/Aesculap agency, see Technical Service
	Machine-readable, two-dimensional code The code contains a unique serial number that can be used for electronically tracking individual instruments. The serial number is based on the worldwide sGTIN (GS1) standard.
	Manufacturer
	Date of manufacture
	Sterilization using irradiation
	Single sterile barrier system with inner protective packaging
	Non-sterile medical device
	Not for reuse in intended applications as defined by the manufacturer
	Use by
	Do not use if packaging is damaged

	Manufacturer batch ID
	Manufacturer's serial number
	Manufacturer's article number
	Temperature limits during transport and storage
	Air humidity limits during transport and storage
	Atmospheric pressure limits during transport and storage
Rx only	According to US federal law, this product may only be sold by a physician or on the orders of a physician
	CE mark according to Regulation (EU) 2017/745
	EAC symbol of the Eurasian Economic Union
	Medical device

Contents

1.	About this document.	3
1.1	Scope	3
1.2	Warnings	3
2.	General information	3
2.1	Intended use.	3
2.2	Performance characteristics	3
2.3	Application Environment.	3
2.4	Indications	3
2.5	Absolute contraindications.	3
2.6	Relative contraindications	3
3.	Safe handling.	4
3.1	Sterile ELAN 4 tools.	4
3.2	Sterile ELAN 4 single-use spray nozzles	4
4.	Product description	4
4.1	Available sizes	4
4.2	Package includes	5
4.3	Components required for operation	5
4.4	Operating principle	5
5.	Preparation.	5
6.	Working with the product	5
6.1	System set-up	5
6.2	Function check.	6
6.3	Operation	6

7.	Validated reprocessing procedure	6
7.1	General safety notes	6
7.2	General information	7
7.3	Reusable products	7
7.4	Preparations at the place of use	7
7.5	Preparation before cleaning	7
7.6	Cleaning/disinfection	7
7.7	Manual cleaning with immersion disinfection – applied part ..	8
7.8	Mechanical cleaning/disinfecting with manual pre-cleaning – applied part	9
7.9	Mechanical cleaning/disinfecting with manual pre-cleaning – tools	10
7.10	Inspection, maintenance and checks	11
7.11	Packaging	11
7.12	Steam sterilization	11
7.13	Storage	11
8.	Maintenance	12
9.	Troubleshooting	12
10.	Technical Service	12
11.	Accessories/Spare parts	12
12.	Technical Data	13
12.1	Classification in accordance with Regulation (EC) 2017/745 ..	13
12.2	Performance data, information about standards	13
12.3	Tool speed/Setting	13
12.4	Short interval operation	13
12.5	Ambient conditions	13
13.	Disposal	13

1. About this document

1.1 Scope

These instructions for use apply to the products listed below:

Art. no.	Designation
GA861–GA864	ELAN 4 electro Standard handpiece (1 ring)
GA865–GA869	ELAN 4 electro HD handpiece (2-ring)

- For article-specific instructions for use and material compatibility information, see B. Braun eFUs at eifu.bbraun.com

1.2 Warnings

Warnings indicate hazards to patients, users and/or products that may occur during the use of the product. Warnings are marked as follows:

DANGER

Indicates a potential danger. If it is not prevented, death or severe injuries may result.

WARNING

Indicates a potential danger. If not prevented, minor or moderate injuries may result.

CAUTION

Indicates a potential damage to property. If not prevented, the product may be damaged.

2. General information

2.1 Intended use

The ELAN 4 electro standard handpiece (1-ring) GA861 to GA864 or ELAN 4 electro HD handpiece (2-ring) GA865 to GA869 is an accessory of the ELAN 4 electro motor system.

The highspeed handpiece is connected by an ELAN 4 electro motor cable to the control unit.

The handpiece is used to drive ELAN 4 tools (1-ring or 2-ring).

2.2 Performance characteristics

Speed

min. 0 rpm to max. 80 000 rpm

Direction of rotation

Right and left rotation

Short-time intermittent operation

■ 30 s application, 30 s pause

■ ∞ repetitions at 60 000 rpm

■ Max. temperature 48 °C

Electrical systems generally heat up during continual operation. It is advisable to give the system a break after use to cool down, as indicated below.

Warming depends on the tool used and the load. After a certain number of repetitions, the system should be allowed to cool down. This procedure prevents the system from overheating and protects the patient or users against possible injury.

The user shall be responsible for the implementation and adherence to the interval sequence described.

2.3 Application Environment

Application in the sterile area

2.4 Indications

Modes of application

Separating, resecting and shaping hard tissue, cartilage, similar and bone substitute materials

Surgical discipline/areas of application

Neurosurgery, ENT surgery, OMF surgery, orthopedics and trauma surgery

Note

The type and area of application depend on the tools selected.

2.5 Absolute contraindications

The product is not licensed for use on the central nervous system or central circulatory system.

2.6 Relative contraindications

The safe and effective use of the products greatly depends on influences which can only be controlled by the user. Therefore the specifications provided represent framework conditions only.

The clinically successful use of the product is dependent on the knowledge and experience of the surgeon. The surgeon must decide on which bodily structures to use the product and take into account the safety and warning information contained in these instructions for use.

3. Safe handling

WARNING

If this product is not used as intended, injury and material damage may result!

- Use the product only for its intended purpose.

WARNING

Risk of injury and damage to property due to improper handling of the product!

This product is an accessory of the ELAN 4 electro control unit GA800.

- Follow the instructions for use of the ELAN 4 electro control unit GA800 (TA014401).
- Follow the instructions for use of all products used.

- General risk factors associated with surgical procedures are not described in this documentation.
- It is the operating surgeon's responsibility to ensure that the surgical procedure is performed correctly.
- The operating surgeon must have a thorough understanding of both the hands-on and conceptual aspects of the established operating techniques.
- Remove the transport packaging and clean the new product, either manually or mechanically, prior to its initial sterilization.
- Prior to use, check that the product is in good working order.
- To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.
- Ensure that the electrical installations at the premises where the device is to be used meet all requirements acc. to IEC/DIN EN.
- Do not use the control unit and applied part in potentially explosive areas.
- Sterilize application parts and motor cables before use.
- Observe the relevant instructions for use TA009721 when handling the Aesculap holder systems, see B. Braun eIFU under eifu.bbraun.com

Note

The user agrees to report all serious incidents occurring in connection with the product to the manufacturer and to the competent authority of the country in which the user is registered.

3.1 Sterile ELAN 4 tools

The product is gamma-sterilized and supplied in sterile packaging.

- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Read, observe and store the instructions for use for tools TA012787.
- Do not use products from open or damaged sterile packaging.
- Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Do not use the product after its use-by date.

Note

For further information, see instructions for use for tools TA012787.

Disposable products

DANGER

Reuse can cause infections in patients and/or users, and impairment of product functionality. Contamination and/or impaired function of the products may result in injury, illness and death!

- Do not reprocess the product.

3.2 Sterile ELAN 4 single-use spray nozzles

The spray nozzle is used to supply fluid for rinsing and cooling the tool or the place of application.

DANGER

Risk of infection for patients and/or users and impairment of product functionality due to reuse. Risk of injury, illness or death due to contamination and/or impaired functionality of the product!

- Do not reprocess the product.

The product is gamma-sterilized and supplied in sterile packaging.

- Do not reuse the product.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- read, follow, and keep the instructions for use.
- Use the product only in accordance with its intended use.
- Do not use products from open or damaged sterile packaging.
- Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Do not use the product after its use-by date.

4. Product description

4.1 Available sizes

Note

The shaft length can be seen on the shaft of the drill handpiece.

ELAN 4 electro Standard handpiece (1 ring)

Art. no.	Length of shaft
GA861	L4 (approx. 4 cm)
GA862	L7 (approx. 7 cm)
GA863	L10 (approx. 10 cm)
GA864	L13 (approx. 13 cm)

ELAN 4 electro HD handpiece (2 ring)

Art. no.	Length of shaft
GA865	L7 (approx. 7 cm)
GA866	L10 (approx. 10 cm)
GA867	L13 (approx. 13 cm)
GA868	L17 (approx. 17 cm)
GA869	L22 (approx. 22 cm)

4.2 Package includes

ELAN 4 electro Standard handpiece (1 ring)

Art. no.	Designation
GA861–GA864	ELAN 4 electro Standard handpiece (1-ring)
TA014443	Instructions for use for ELAN 4 electro Standard handpiece (1-ring) GA861 to GA864 / ELAN 4 electro HD handpiece (2-ring) GA865 to GA869 (leaflet)

ELAN 4 electro HD handpiece (2 ring)

Art. no.	Designation
GA865–GA869	ELAN 4 electro HD handpiece (2-ring)
TA014443	Instructions for use for ELAN 4 electro Standard handpiece (1 ring) GA861 to GA864 / ELAN 4 electro HD handpiece (2 ring) GA865 to GA869 (leaflet)

4.3 Components required for operation

- Operation-ready ELAN 4 electro control unit GA800, see TA014401
- ELAN 4 tool (1 ring or 2 ring)

4.4 Operating principle

The ELAN 4 electro handpiece 1-ring or 2-ring is equipped with ELAN 4 a tool coupling (1-ring or 2-ring).

The 1-ring or 2-ring handpiece is used with ELAN 4 electro foot controls or manual controls.

For the 1-ring or 2-ring handpiece, the clamped tool rotates at the set motor speed.

The motor speed of the 1-ring or 2-ring handpiece is infinitely variable by means of the foot/manual control.

The 1-ring or 2-ring handpiece can be operated both clockwise and anti-clockwise.

Note

The ring marking 2 shows the shaft diameter of the tool to be used. ELAN 4 tools can be only coupled to the applied part if the ring markings (1 ring/2 ring) of the applied part and tool match.

5. Preparation

If the rules listed below are not followed, Aesculap will not accept any responsibility.

- Do not use products from open or damaged sterile packaging.
- Prior to use, inspect the product and its accessories for visible damage.
- Use the products and their accessories only if they are in perfect technical condition.

6. Working with the product

⚠ WARNING

Risk of infection and contamination!

Applied parts and motor cable are delivered unsterile!

- Carry out sterile processing of the applied parts and motor cable prior to use, in accordance with the instructions for use.

⚠ WARNING

Risk of injury and material damage due to inadvertent activation of the application part!

- Secure applied parts that are not actually used against inadvertent activation ("Off" position), see instructions for use for ELAN 4 electro control unit GA800 (TA014401).

⚠ WARNING

Risk of injury and material damage due to inappropriate use of tools!

- Always follow the safety advice and information given in the instructions for use.
- When coupling/uncoupling, handle tools with cutting edges with care.

⚠ WARNING

Risk of burns to skin and tissue due to blunt tools/ lack of maintenance of the applied part!

- Use tools only if they are in perfect condition.
- Replace blunt tools.
- Ensure correct maintenance of the applied part, see Maintenance.

⚠ WARNING

Damage to the product if dropped!

- Use the products only if they are in perfect technical condition, see Function check

6.1 System set-up

⚠ WARNING

Risk of injury when coupling/uncoupling tools in the "On" position due to accidental activation of the applied part!

- Only couple/uncouple tools in the "Off" position.

⚠ WARNING

Small parts entering the surgical field can cause injuries!

- Do not couple/uncouple tools over the operation site.

Note

Operating elements on the system components of the ELAN 4 electro motor system are labeled with a gold marking.

Connecting the accessories

Do not use accessories in combinations that are not mentioned in these instructions for use.

- Follow the instructions for use of individual accessories.
- Please address your B. Braun/Aesculap partner or Aesculap Technical Service with any inquiries in this respect; for a contact address, see Technical Service.

Remove the ELAN 4 ECCOS holder for 12 reamers GB718R from the tray

The insert of the ELAN 4 ECCOS holder for 12 reamers GB718R can be removed from the tray in conjunction with the tools.

- Pull at the tab 13 and remove the insert from the holder, see Fig. C.

Coupling the tool to the applied part

- ▶ Lock applied part ("Off" position), see instructions for use of ELAN 4 electro control unit GA800 (TA014401).
- ▶ Push tool shaft **6** as far as it will go into ELAN 4 tool coupling (1 ring or 2 ring) **1**, see Fig. A.
The tool engages. The tool is correctly coupled when the ring marking **2** on the tool is no longer visible.
- ▶ Pull at tool **6** to check that it is securely coupled.

Uncoupling the tool from the applied part

- ▶ Lock applied part ("Off" position), see instructions for use of ELAN 4 electro control unit GA800 (TA014401).
- ▶ Pull back slider for tool release **4** on the applied part and remove tool **6** from ELAN 4 tool coupling (1 ring or 2 ring) **1**, see Fig. A.

Mounting the spray nozzle and tube set

- ▶ Connect tube set GA395SU to pump of ELAN 4 electro control unit GA800, see TA014401.
- ▶ Push the tube end directed at the area of application onto the tube connector of the ELAN 4 the disposable spray nozzle for the 1-ring or 2-ring handpiece **8**.
- ▶ Push single-use spray nozzle for ELAN 4 1 ring or 2 ring handpieces **7** onto the shaft of the applied part and push/turn into the desired position, see Fig. A.

Note

The spray tube of the ELAN 4 single-use spray nozzle for 1 ring or 2 ring handpiece is bendable by hand and can be adapted to the requirements of application (e.g. aligned to tool head).

6.2 Function check

- ▶ Prior to each use, check that all products to be used are in good working order.
- ▶ Check the secure connection of all products to be used.
- ▶ Check that the tool is coupled properly. To do this, pull on the tool.
- ▶ Check that the cutting edges of the tools are not showing any mechanical damage.
- ▶ Briefly run the applied parts at maximum speed.
- ▶ Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

6.3 Operation

⚠ WARNING

Coagulation of patient tissue or risk of burns for patients and users from hot application part/tool!

- ▶ Cool the tool during operation.
- ▶ Keep product/tool out of the reach of patients.
- ▶ Allow the applied part/tool to cool down.
- ▶ When changing the tool, use a cloth as a protection against burns.

⚠ WARNING

Risk of infection from aerosol formation!

Risk of injuries caused by particles coming loose from the tool!

- ▶ Take appropriate protective measures, e.g. watertight protective clothing, face mask and protective goggles, suction extraction.

⚠ WARNING

Risk of injury and/or malfunction!

- ▶ Always carry out a function check prior to using the product.

⚠ WARNING

Risk of injury when using the product beyond the field of view!

- ▶ Apply the product only under visual control.

⚠ WARNING

Risk of injury and damage to the tool/system!

The rotating tool may catch cover drapes (textile or other).

- ▶ Never allow the tool to touch cover drapes (textile or other) during operation.

⚠ WARNING

Danger of injury from foreign body reactions (e.g. inflammation, encapsulation) if particles remain in the body!

When using diamond tools, it is always possible for diamond grains/nickel particles to break away/off.

- ▶ Carefully rinse and vacuum while and after working with diamond tools.

It is only possible to operate the applied part and change the setting parameters on the control unit if:

- the applied part is connected to the control unit,
- there is no second applied part released ("On" position) at the same time and
- the applied part type **5** is shown in the display of the control unit.

Note

For further information, see instructions for use of ELAN 4 electro control unit GA800 (TA014401).

7. Validated reprocessing procedure

7.1 General safety notes

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for reprocessing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD, or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

Mechanical reprocessing should be favored over manual cleaning as it gives better and more reliable results.

Note

It should be noted that successful reprocessing of this medical device can only be guaranteed following prior validation of the reprocessing method. The operator/reprocessing technician is responsible for this.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

See also B. Braun eIFUs at eifu.bbraun.com for up-to-date information on processing and material compatibility

The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

7.2 General information

Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixating pre-cleaning temperatures >45 °C nor fixating disinfecting agents (active ingredient: aldehydes/alcohols) should be used.

Excessive neutralizing agents or basic cleaners may result in a chemical attack and/or fading and the laser marking becoming unreadable either visually or by machine.

Residues containing chlorine or chlorides, e.g., in surgical residues, medicines, saline solutions, and in the service water used for cleaning, disinfection, and sterilization, will cause corrosion damage (pitting, stress corrosion) and result damage to metallic products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Perform additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Material damage such as corrosion, cracks, fracturing, premature aging, or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surface and could cause corrosion
- ▶ For detailed information on hygienically safe and material-friendly/careful processing, please refer to www.a-k-i.org Header "AKI-Brochures", "Red brochure".

7.3 Reusable products

The service life of the product is limited by damage, normal wear, type and duration of the application, handling, storage and transportation of the product.

Careful visual and functional testing prior to next use is the best way to identify a malfunctioning product.

7.4 Preparations at the place of use

- ▶ Separate the products from each other immediately after use.
- ▶ Remove any visible surgical residues as much as possible with a damp, lint-free cloth.
- ▶ Place the dry product in a sealed waste container and forward it on for cleaning and disinfection within 6 hours.

7.5 Preparation before cleaning

Applied part

- ▶ Prior to first mechanical cleaning/disinfection: Mount ECCOS holder GB082R/GB083R in a suitable tray (e.g. JF222R).
- ▶ Place the product in the correct position in the ECCOS holder GB082R/GB083R, see Fig. B.

Tools

- ▶ Thoroughly rinse the product with running, cold water.
- ▶ Prior to first mechanical cleaning/disinfection: Mount the bottom part of the ECCOS holder GB718R in a suitable tray (e.g. JF222R).
- ▶ Align the product in the correct position relative to the tool holder and insert 12 into the ECCOS holder GB718R, see Fig. C.
- ▶ Engage the fixation in the bottom part. In doing so, make sure that the holding lugs 11 on the insert are inserted in the recesses in the bottom part.
- ▶ Lift and fold down the tool holder 12 using the two side handles 10.

7.6 Cleaning/disinfection

Product-specific safety instructions for the reprocessing procedure

⚠ CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

- ▶ Use cleaning and disinfecting agents according to the manufacturer's instructions which
 - are approved for plastic material and high-grade steel,
 - do not attack softeners (e.g. in silicone).
- ▶ Do not use cleaning agents that contain acetone.
- ▶ Observe specifications regarding concentration, temperature and exposure time.
- ▶ Do not exceed the maximum temperature of 60 °C during chemical cleaning and/or disinfection.
- ▶ Do not exceed maximum temperature of 96°C during thermal disinfection using DI water.
- ▶ Dry the product for at least 10 minutes at a maximum of 120 °C.

Note

The indicated drying time is a guide time only. It must be checked taking into account the specific conditions (e.g. load) and if applicable adjusted.

Validated cleaning and disinfection procedure

Product	Validated procedure	Reference
Application part	Manual cleaning with immersion disinfection	see Manual cleaning with immersion disinfection – applied part
	Manual pre-cleaning with brush and subsequent mechanical alkaline cleaning and thermal disinfection	see Mechanical cleaning/disinfecting with manual pre-cleaning – applied part
Tools	Manual pre-cleaning with ultrasound and brush, and subsequent mechanical alkaline cleaning and thermal disinfection	see Mechanical cleaning/disinfecting with manual pre-cleaning – tools

7.7 Manual cleaning with immersion disinfection – applied part

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Pre-cleaning	RT (cold)	-	-	D-W	-
II	Cleaning	RT (cold)	>5	1	D-W	pH-neutral, pH ~ 9*
III	Intermediate rinse	RT (cold)	-	-	D-W	-
IV	Disinfection	RT (cold)	>5	2	D-W	Concentrate-free, aldehyde-free, phenol-free, QUAT-free, pH ~ 9**
V	Final rinse	RT (cold)	-	-	D-W	-
VI	Drying	RT	-	-	-	-

D-W: Drinking water

RT: Room temperature

*Recommended: BBraun Helizyme

**Recommended: BBraun Stabimed fresh

- Follow the instructions for use of the ELAN 4 rinsing devices and rinsing adapters TA014447 and TA014448.
- Do not clean the applied part in an ultrasound bath.

Phase I

- Clean the product under running water, using a suitable cleaning brush until all visible residues have been removed from the surfaces.
- Mobilize non-rigid components, such as release button, unlocking sleeve, etc. during cleaning.
- Rinse the applied part through the rinsing adapter GB698R at least 3 times for 5 seconds with a water gun.

Phase II

- Using a disposable syringe, fill the inside of the applied part with cleaning solution through the rinsing adapter GB698R.
- Place the applied part, with rinsing adapter connected, fully in enzymatic cleaning solution for 5 minutes.

Phase III

- Rinse the entire product (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as release button, unlocking sleeve, etc. during rinsing.
- Rinse the applied part through the rinsing adapter GB698R at least 3 times for 5 seconds with a water gun.

Phase IV

- Prior to manual disinfection, allow rinsing water to drain off the product for a sufficient length of time and use compressed air to blow all liquid out of the product through the rinsing adapter GB698R in order to prevent dilution of the disinfecting solution.
- Using a disposable syringe, fill the inside of the applied part with disinfectant solution through the rinsing adapter GB698R.
- Immerse the application part including the connected rinsing adapter at least 5 min in disinfecting solution.

Phase V

- Rinse the entire product (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as release button, unlocking sleeve, etc. during cleaning.
- Rinse the applied part through the rinsing adapter GB698R at least 3 times for 5 seconds with a water gun.

Phase VI

- Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air).
- After manual cleaning/disinfection, check visible surfaces for residue and repeat the cleaning/disinfection process as required.

7.8 Mechanical cleaning/disinfecting with manual pre-cleaning – applied part

Note

The cleaning and disinfection device must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note

The cleaning and disinfection device used for processing must be serviced and checked at regular intervals.

Manual pre-cleaning with a brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Brushing	RT (cold)	–	–	D–W	–
II	Rinsing	RT (cold)	5	–	D–W	–

D–W: Drinking water
RT: Room temperature

- Follow the instructions for use of the ELAN 4 rinsing devices and rinsing adapters TA014447 and TA014448.
- Do not clean the applied part in an ultrasound bath.

Phase I

- Clean the product with a suitable cleaning brush until all discernible residues have been removed.
- Mobilize non-rigid components, such as release button, unlocking sleeve, etc. during cleaning.

Phase II

- Connect the connector for motor cable 3 with ELAN 4 electro rinsing device GB692R.
- Rinse the applied part:
 - For 5 min with a tap/hose or
 - 3 times for 5 seconds with a water gun

Mechanical alkaline cleaning and thermal disinfection

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemicals
I	Pre-rinse	<25/77	3	D–W	–
II	Cleaning	55/131	10	FD–W	■ Concentrate, alkaline: – pH ~ 13 – <5 % anionic surfactant ■ 0.5 % working solution – pH ~ 11*
III	Intermediate rinse	>10/50	1	FD–W	–
IV	Thermal disinfection	90/194	5	FD–W	–
V	Drying	max. 120/248	min. 10	–	–

D–W: Drinking water

FD–W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)

*Recommended: BBraun Helimatic Cleaner alkaline

- Connect the connector for motor cable 3 with ELAN 4 electro rinsing device GB692R.
- Place the rinsing device on a wire basket suitable for cleaning.
- Connect the rinsing device to the rinsing connection of the rinsing cart.
- After machine cleaning/disinfection process:
 - Use a compressed air gun to remove any remaining rinsing water from the application parts and shafts, see the instructions for use ELAN 4 for rinsing device and rinsing adapter TA014447 or TA014448.
 - Visually check for residue on visible surfaces and repeat the cleaning/disinfection process if necessary.

7.9 Mechanical cleaning/disinfecting with manual pre-cleaning – tools

Note

The cleaning and disinfection device must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note

The cleaning and disinfection device used for processing must be serviced and checked at regular intervals.

Manual pre-cleaning with ultrasound and brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Immersion	RT (cold)	>30	50	D-W	Enzymatic detergent*
II	Rinsing	RT (cold)	-	-	D-W	-
III	Ultrasonic cleaning	55/131	>15	2	D-W	Concentrate-free, aldehyde-free, phenol-free, QUAT-free, pH ~ 9**
IV	Cleaning with brush	RT (cold)	-	-	D-W	-

D-W: Drinking water

RT: Room temperature

*Recommended: BBraun Helizyme

**Recommended: BBraun Stabimed fresh

- Generally perform ultrasound cleaning solely in the ECCOS holder GB718R to prevent any damage to the blades.

Phase I

- Fully immerse the product in enzymatic cleaning solution for at least 30 minutes. Ensure that all accessible surfaces are moistened.

Phase II

- Rinse the entire product (all accessible surfaces) under running water.

Phase III

- Fold up the handles **12** of the ECCOS tool holder GB718R on the side **10**, see Fig C.
- Clean the product in an ultrasonic cleaning bath (frequency 35 kHz) for at least 15 minutes. Ensure that all accessible surfaces are immersed and acoustic shadows are avoided.

Phase IV

- Clean the product with a suitable cleaning brush until all discernible residues have been removed.

Mechanical alkaline cleaning and thermal disinfection

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemicals
I	Pre-rinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	■ Concentrate, alkaline: - pH ~ 13 - <5 % anionic surfactant ■ 0.5 % working solution - pH ~ 11*
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disinfection	90/194	5	FD-W	-
V	Drying	max. 120/248	min. 10	-	-

D-W: Drinking water

FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)

*Recommended: BBraun Helimatic Cleaner alkaline

- Align the product in the correct position relative to the tool holder and insert **12** into the ECCOS holder GB718R, see Fig. C.
- After machine cleaning/disinfection, check visible surfaces for residue and repeat the cleaning/disinfection process as required.

7.10 Inspection, maintenance and checks

- Allow the product to cool down to room temperature.
- Connect the ELAN 4 electro oil spray adapter GB600860 **9** (grey) to the connector for the motor cable **3** and use the STERILIT Power Systems oil spray GB600 to spray through the application part for approx. 2 s, see Fig. D.

Note

Aesculap additionally recommends the occasional spraying of moving parts (such as knobs, couplings) using the Aesculap STERILIT Power Systems oil spray.

- Inspect the product after every cleaning and disinfecting cycle for: cleanliness, damage, function, abnormal operation noise, excessive heat or strong vibration.
- Inspect tools for broken, damaged or blunt edges.
- Set aside the product if it is damaged.

7.11 Packaging

- Observe the instructions for use of the packagings and holders used (e.g. instructions for use TA009721 for Aesculap ECCOS holder system).
- Place the application part in the correct position in the ECCOS holder GB082RGB083R, see Fig. B.
- Insert the tools in the correct position in the ECCOS holder GB718R, see Fig. C.
- Pack trays appropriately for the intended sterilization process (e.g. in Aesculap sterile containers).
- Ensure that the packaging will prevent a recontamination of the product.

7.12 Steam sterilization

- Make sure that all external and internal surfaces of the product are exposed to the sterilizing agent.
- Validated sterilization process
 - Steam sterilization using fractional vacuum process
 - Steam sterilizer according to DIN EN 285 and validated according to DIN EN ISO 17665
 - Sterilization using fractionated vacuum process at 134 °C/holding time 5 min
- When sterilizing several products at the same time in a steam sterilizer, ensure that the maximum load capacity of the steam sterilizer specified by the manufacturer is not exceeded.

7.13 Storage

- Store sterile products in germ-proof packaging, protected from dust, in a dry, dark, temperature-controlled area.

8. Maintenance

Application part

To ensure reliable operation, the product must be maintained in accordance with the maintenance labeling or at least once a year.

e.g. 2016-07

For technical service, please contact your national B. Braun/Aesculap agency, see Technical Service.

Tools

No maintenance required for tools.

- Do not modify, re-sharpen or repair tools.

9. Troubleshooting

Note

For further information, see instructions for use of ELAN 4 electro control unit GA800 (TA014401).

10. Technical Service

⚠ DANGER

Danger to life of patients and users in case of malfunctions and/or failure of protective measures!

- Do not perform any servicing or maintenance work under any circumstances while the product is being used on a patient.

⚠ CAUTION

Modifications to medical equipment may result in loss of guarantee/warranty claims and any approvals.

- Do not modify the product.
- For service and repairs, please contact your national B. Braun/Aesculap representative.

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 (7461) 95 -1601

Fax: +49 (7461) 14 -939

E-Mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

11. Accessories/Spare parts

Art. no.	Designation
GA261SU	ELAN 4 single-use spray nozzle for 1 ring handpiece L4
GA262SU	ELAN 4 single-use spray nozzle for 1 ring handpiece L7
GA263SU	ELAN 4 single-use spray nozzle for 1 ring handpiece L10
GA264SU	ELAN 4 single-use spray nozzle for 1 ring handpiece L13
GA265SU	ELAN 4 single-use spray nozzle for 2 ring handpiece L7
GA266SU	ELAN 4 single-use spray nozzle for 2 ring handpiece L10
GA267SU	ELAN 4 single-use spray nozzle for 2 ring handpiece L13
GA268SU	ELAN 4 single-use spray nozzle for 2 ring handpiece L17
GA269SU	ELAN 4 single-use spray nozzle for 2 ring handpiece L22
GA395SU	ELAN 4 electro single-use tube set
GB082R	ELAN 4 electro ECCOS holder for reamer handpiece GA861 to GA869, 1 handpiece
GB083R	ELAN 4 electro ECCOS holder for reamer handpiece GA861 to GA869, 3 handpieces
GB600	STERILIT Power Systems oil spray
GB600860	ELAN 4 electro oil spray adapter
GB692R	ELAN 4 electro rinsing device
GB698R	ELAN 4 electro rinsing adapter
GB718R	ELAN 4 ECCOS holder for 12 reamers
TA012787	Instructions for use for tools for Aesculap motor systems - disposable products/reusable products
TA014442	Instructions for use for ELAN 4 electro Standard handpiece (1-ring) GA861 to GA864 / ELAN 4 electro HD handpiece (2-ring) GA865 to GA869 (A4 for folder)
TA014443	Instructions for use for ELAN 4 electro Standard handpiece (1 ring) GA861 to GA864 / ELAN 4 electro HD handpiece (2 ring) GA865 to GA869 (leaflet)

12. Technical Data

12.1 Classification in accordance with Regulation (EC) 2017/745

Art. no.	Designation	Class
GA861–GA864	ELAN 4 electro Standard handpiece (1-ring)	Ila
GA865–GA869	ELAN 4 electro HD handpiece (2 ring)	Ila

12.2 Performance data, information about standards

Max. power	approx. 140 W
Max. torque	approx. 2.2 Ncm
Max. motor speed	80 000 rpm
Tool connector	ELAN 4 1 ring (GA861 to GA864) ELAN 4 2 ring (GA865 to GA869)
Application part	Type BF
EMC	IEC/DIN EN 60601-1-2
Compliance with standards	IEC/DIN EN 60601-1

Art. no.	Weight	Dimensions (Ø x L)
GA861	80 g ±10 %	16 mm x 152 mm ±5 %
GA862	83 g ±10 %	16 mm x 177 mm ±5 %
GA863	87 g ±10 %	16 mm x 206 mm ±5 %
GA864	91 g ±10 %	16 mm x 237 mm ±5 %
GA865	89 g ±10 %	16 mm x 176 mm ±5 %
GA866	95 g ±10 %	16 mm x 206 mm ±5 %
GA867	100 g ±10 %	16 mm x 236 mm ±5 %
GA868	108 g ±10 %	16 mm x 276 mm ±5 %
GA869	118 g ±10 %	16 mm x 326 mm ±5 %

After 350 processing cycles, the manufacturer tested the product and it passed.

12.3 Tool speed/Setting

Adjustable motor direction	Clockwise/counterclockwise rotation
Adjustable maximum speed setting	10 000 rpm to 80 000 rpm
Step width for maximum speed setting	5 000 rpm
Default factory setting of upper speed range limit	75 000 rpm, right rotation

12.4 Short interval operation

- 30 s application, 30 s pause
- ∞ repetitions at 60 000 rpm
- Max. temperature 48 °C

12.5 Ambient conditions

Application part

	Operation	Transport and storage
Temperature	10 °C to 27 °C	-10 °C to 50 °C
Relative humidity	30 % to 75 %	10 % to 90 %
Atmospheric pressure	700 hPa to 1 060 hPa	500 hPa to 1 060 hPa

13. Disposal

⚠ WARNING

Risk of infection from contaminated products!

- Observe national regulations when disposing of or recycling the product, its components, and their packaging.

⚠ WARNING

Risk of injury from sharp and/or pointed products!

- When disposing of or recycling the product, make sure the packaging prevents injuries from the product.

Note

The operator must process the product before disposal, see Validated reprocessing procedure.

The recycling passport can be downloaded from the extranet as a PDF document for on the individual part number. (The recycling passport contains instructions on how to disassemble the device, including information on the proper disposal of environmentally harmful components.) Any product labeled with this symbol must be disposed of separately together with electrical and electronic equipment. Within the European Union, the manufacturer will perform the disposal free of charge.

- In case of questions regarding the disposal of the product, contact your local B. Braun/Aesculap representative, see Technical Service.

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