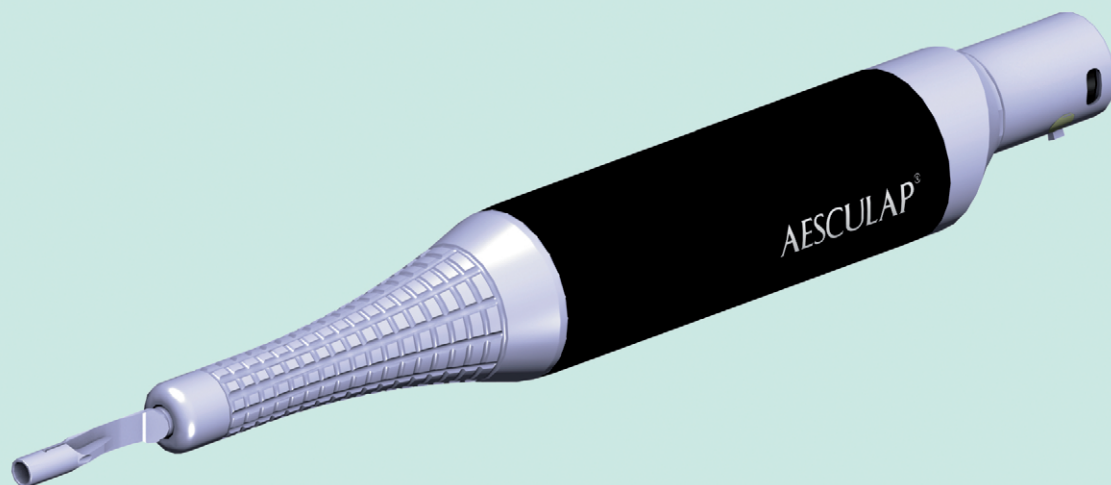
en Instructions for use/Technical description
ELAN 4 electro micro reciprocating saw GA837

USA Note for U.S. users

This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusa.com. If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.

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
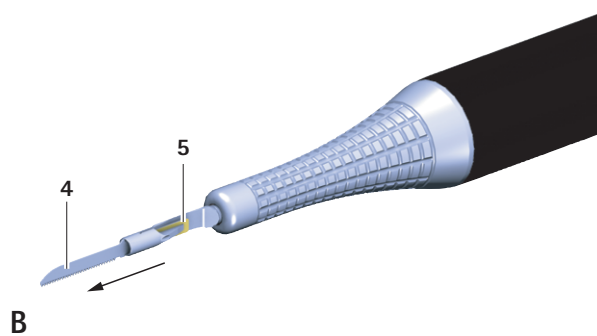
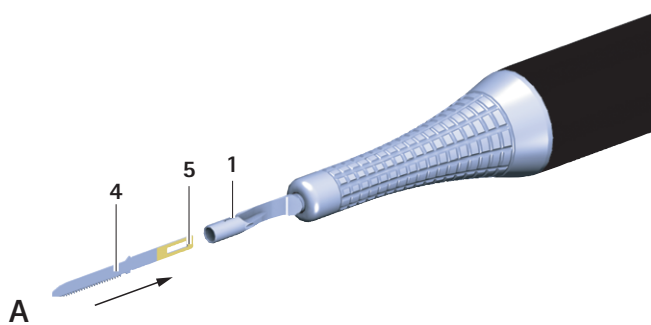
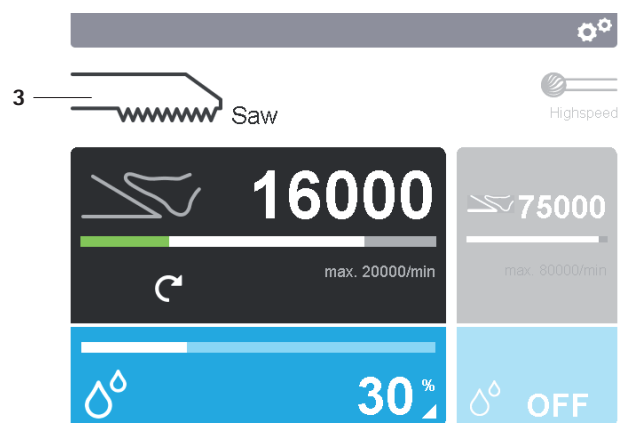
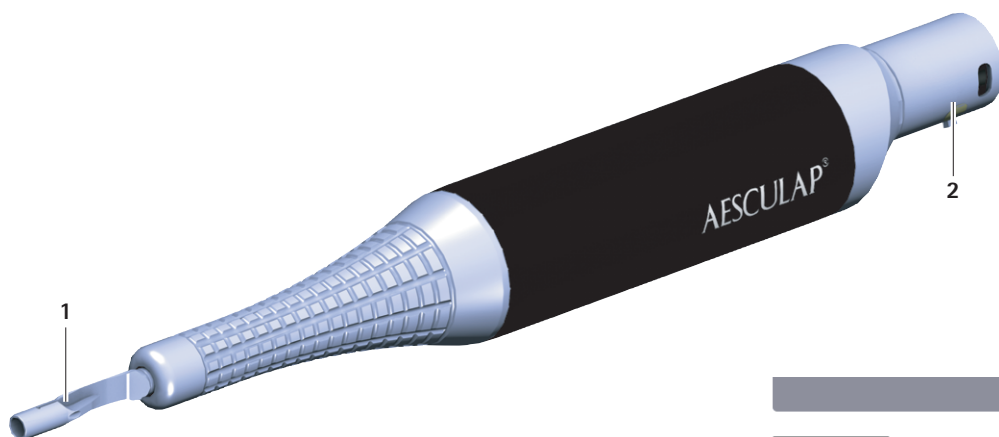
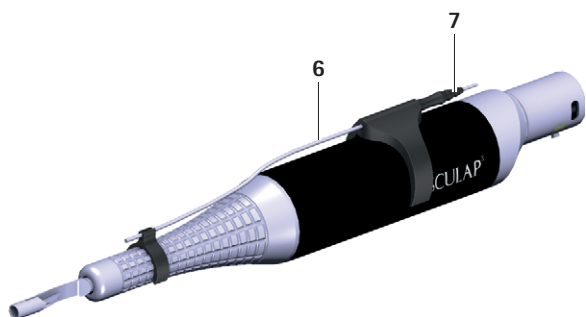
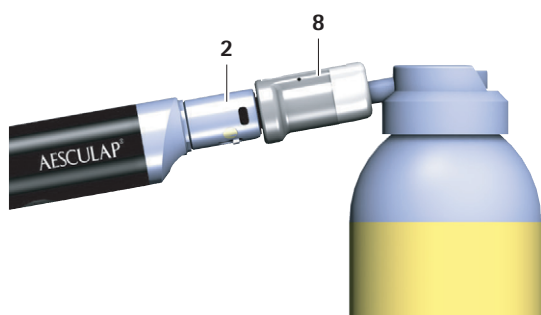
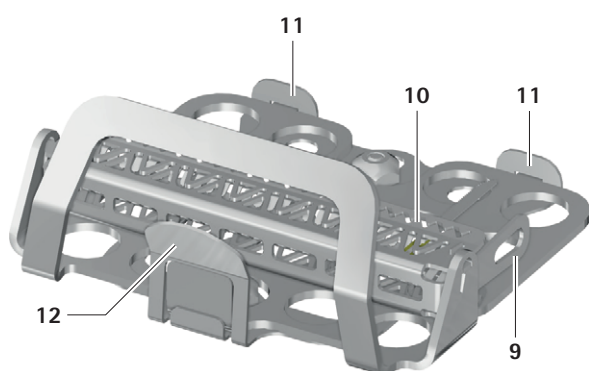
F

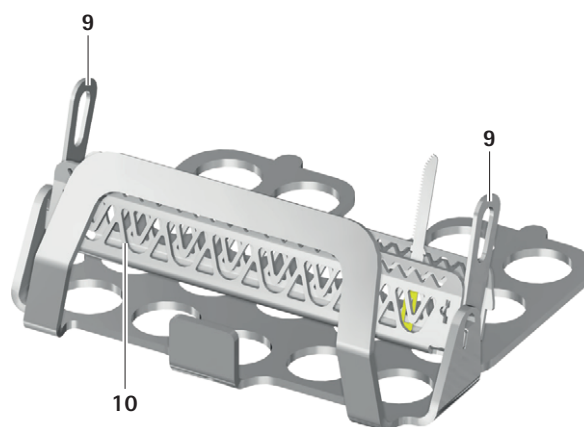Aesculap®

ELAN 4 electro micro reciprocating saw GA837

Legend

- 1 Tool coupling
- 2 Connector for motor cable
- 3 Symbol of the applied part type in display of ELAN 4 electro control unit
- 4 Reciprocating saw blade micro
- 5 Piece on saw blade for tool release
- 6 ELAN 4 disposable spray nozzle for saws
- 7 Tube connection on spray nozzle
- 8 Oil spray adapter GB600860

ECCOS general holder for saw blades GB720R

- 9 Lateral handles
- 10 Tool coupling
- 11 Holding nibs
- 12 Tab

Symbols on product and packages

	Caution Observe important safety information such as warnings and precautions in the instructions for use.
 YYYY-MM	Maintenance indicator Date of the next recommended servicing (year-month) by the international B. Braun-/Aesculap agency, see Technical Service
	Two-dimensional machine readable code The code contains a unique serial number that can be used for electronically tracking individual instruments. The serial number is based on the worldwide sGTIN (GS1) standard.
	Manufacturer
	Date of manufacture
	Sterilization using irradiation
	Single Sterile Barrier System
	Single sterile barrier system with inner protective packaging
	Non-sterile medical device
	Not for reuse in intended applications as defined by the manufacturer
	Use by

	Do not use if packaging is damaged
	Manufacturer batch ID
	Manufacturer's serial number
	Manufacturer's article number
	Temperature limits during transport and storage
	Air humidity limits during transport and storage
	Atmospheric pressure limits during transport and storage
	According to US federal law, this product may only be sold by a physician or on the orders of a physician
	CE mark according to Regulation (EU) 2017/745
	EAC symbol of the Eurasian Economic Union
	Medical device

Contents

1.	About this document.	3
1.1	Scope	3
1.2	Warnings	3
2.	General information	3
2.1	Intended use.	3
2.2	Performance characteristics	3
2.3	Application Environment.	3
2.4	Indications	3
2.5	Absolute contraindications.	3
2.6	Relative contraindications	3
3.	Safe handling	4
3.1	Sterile ELAN 4 tools.	4
3.2	Sterile ELAN 4 disposable spray nozzle for saws GA258SU.	4
4.	Product description	5
4.1	Scope of supply	5
4.2	Components required for operation.	5
4.3	Operating principle	5
5.	Preparation.	5
6.	Working with the product	5
6.1	System set-up	5
6.2	Function checks.	6
6.3	Safe operation	6

7.	Validated reprocessing procedure	6
7.1	General safety notes	6
7.2	General information	7
7.3	Reusable products	7
7.4	Preparations at the place of use	7
7.5	Preparation before cleaning	7
7.6	Cleaning/disinfection	7
7.7	Manual cleaning with immersion disinfection – application part	8
7.8	Mechanical cleaning/disinfection including manual pre-cleaning – application part	9
7.9	Mechanical cleaning/disinfecting with manual pre-cleaning – tools	10
7.10	Inspection, maintenance and checks	11
7.11	Packaging	11
7.12	Steam sterilization	11
7.13	Storage	11
8.	Maintenance	12
9.	Troubleshooting	12
10.	Technical Service	12
11.	Accessories/Spare parts	12
12.	Technical data	12
12.1	Classification in accordance with Regulation (EC) 2017/745	12
12.2	Performance data, information about standards	12
12.3	Tool speed/Setting	12
12.4	Short interval operation	12
12.5	Ambient conditions	13
13.	Disposal	13

1. About this document

1.1 Scope

These instructions for use apply to the products listed below:

Art. no.	Designation
GA837	ELAN 4 electro micro reciprocating saw

- For article-specific instructions for use and material compatibility information, see B. Braun eIFUs at eifu.bbraun.com

1.2 Warnings

Warnings indicate hazards to patients, users and/or products that may occur during the use of the product. Warnings are marked as follows:

DANGER

Indicates a potential danger. If it is not prevented, death or severe injuries may result.

WARNING

Indicates a potential danger. If not prevented, minor or moderate injuries may result.

CAUTION

Indicates a potential damage to property. If not prevented, the product may be damaged.

2. General information

2.1 Intended use

The ELAN 4 electro micro reciprocating saw GA837 is an accessory of the ELAN 4 electro motor system.

The micro reciprocating saw is connected by an ELAN 4 electro motor cable to the control unit.

The micro reciprocating saw is used to drive micro reciprocating saw blades.

2.2 Performance characteristics

Oscillation frequency

min. 0 min⁻¹ to max. 20 000 min⁻¹

Short-time intermittent operation

■ 10 min application, 40 min break

■ ∞ Repetitions

■ Max. temperature 48°C

Electrical systems generally heat up during continual operation. It is advisable to give the system a break after use to cool down, as indicated below.

Warming depends on the tool used and the load. After a certain number of repetitions, the system should be allowed to cool down. This procedure prevents the system from overheating and protects the patient or users against possible injury.

The user shall be responsible for the implementation and adherence to the interval sequence described.

2.3 Application Environment

Application in the sterile area

2.4 Indications

Modes of application

Separating, resecting and shaping hard tissue, cartilage, similar and bone substitute materials

Surgical discipline/areas of application

Neurosurgery, ENT surgery, OMF surgery, orthopedics and trauma surgery

Note

The type and area of application depend on the tools selected.

2.5 Absolute contraindications

The product is not licensed for use on the central nervous system or central circulatory system.

2.6 Relative contraindications

The safe and effective use of the products greatly depends on influences which can only be controlled by the user. Therefore the specifications provided represent framework conditions only.

The clinically successful use of the product is dependent on the knowledge and experience of the surgeon. The surgeon must decide on which bodily structures to use the product and take into account the safety and warning information contained in these instructions for use.

3. Safe handling

WARNING

Risk of injury and material damage if this product is not used as intended!

- Use the product only for its intended purpose.

WARNING

Risk of injury and damage to property due to improper handling of the product!

This product is an accessory of the ELAN 4 electro control unit GA800.

- Follow the instructions for use of the ELAN 4 electro control unit GA800 (TA014401).
- Follow the instructions for use of all products used.

- General risk factors associated with surgical procedures are not described in this documentation.
- It is the operating surgeon's responsibility to ensure that the surgical procedure is performed correctly.
- The operating surgeon must have a thorough understanding of both the hands-on and conceptual aspects of the established operating techniques.
- Remove the transport packaging and clean the new product, either manually or mechanically, prior to its initial sterilization.
- Prior to use, check that the product is in good working order.
- To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.
- Ensure that the electrical installations at the premises where the device is to be used meet all requirements acc. to IEC/DIN EN.
- Do not use the control unit and applied part in potentially explosive areas.
- Sterilize application parts and motor cable before use.
- Observe the relevant instructions for use TA009721 when handling the Aesculap holder systems, see B. Braun eIFU under eifu.bbraun.com

Note

The user agrees to report all serious incidents occurring in connection with the product to the manufacturer and to the competent authority of the country in which the user is registered.

3.1 Sterile ELAN 4 tools

The product is gamma-sterilized and supplied in sterile packaging.

- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Read, observe and store the instructions for use for tools TA012787.
- Do not use products from open or damaged sterile packaging.
- Prior to each use, inspect the product for: Loose, bent, broken, cracked, worn, or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Do not use the product after its use-by date.

Note

For further information, refer to the instructions for use for tools TA012787.

Disposable products

DANGER

Risk of infection for patients and/or users and impairment of product functionality due to reuse. Risk of injury, illness or death due to contamination and/or impaired functionality of the product!

- Do not reprocess the product.

3.2 Sterile ELAN 4 disposable spray nozzle for saws GA258SU

The spray nozzle is used to supply fluid for rinsing and cooling the tool or the place of application.

DANGER

Reuse can cause infections in patients and/or users, and impairment of product functionality. Contamination and/or impaired function of the products may result in injury, illness and death!

- Do not reprocess the product.

The product is gamma-sterilized and supplied in sterile packaging.

- Do not reuse the product.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- read, follow, and keep the instructions for use.
- Use the product only as intended.
- Do not use products from open or damaged sterile packaging.
- Prior to each use, inspect the product for loose, bent, broken, cracked or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Do not use the product after its use-by date.

4. Product description

4.1 Scope of supply

Art. no.	Designation
GA837	ELAN 4 electro micro reciprocating saw
TA014431	Instructions for use for ELAN 4 electro micro reciprocating saw GA837 (leaflet)

4.2 Components required for operation

- Operation-ready ELAN 4 electro control unit GA800, see TA014401
- Reciprocating saw blade micro

4.3 Operating principle

The ELAN 4 electro micro reciprocating saw can be used to drive saw blades.

The micro reciprocating saw is used with ELAN 4 electro foot controls or manual controls.

The oscillation frequency can be adjusted infinitely using the foot/manual control.

No wrenches are required for coupling/uncoupling the saw blades.

5. Preparation

If the rules listed below are not followed, Aesculap will not accept any responsibility.

- Do not use products from open or damaged sterile packaging.
- Prior to use, inspect the product and its accessories for visible damage.
- Use the products and their accessories only if they are in perfect technical condition.

6. Working with the product

⚠ WARNING

Risk of infection and contamination!

Application parts and motor cables are delivered non-sterile!

- Sterilize application parts and motor cables prior to use, in accordance with the instructions for use.

⚠ WARNING

Risk of injury and material damage due to inadvertent activation of the application part!

- Secure applied parts that are not actually used against inadvertent activation ("Off" position), see instructions for use for ELAN 4 electro control unit GA800 (TA014401).

⚠ WARNING

Injury and material damage may result from an inappropriate use of tools!

- Always follow the safety advice and information provided in the instructions for use.
- Handle tools having cutting edges with care when coupling/uncoupling.
- Do not use tools for strutting or levering.

⚠ WARNING

Risk of burns to skin and tissue due to blunt tools/insufficient maintenance of the application part!

- Use tools only if they are in perfect condition.
- Replace blunt tools.
- Maintain the application part properly, see maintenance.

⚠ WARNING

Dropping may damage the product !

- Use the products only if they are in a flawless technical condition, see function check

6.1 System set-up

⚠ WARNING

Risk of injury when coupling/uncoupling tools in the "On" position due to accidental activation of the applied part!

- Only couple/uncouple tools in the "Off" position.

⚠ WARNING

Small parts entering the surgical field can cause injuries!

- Do not couple/uncouple tools over the operation site.

Note

Operating elements on the system components of the ELAN 4 electro motor system are labeled with a gold marking.

Connecting the accessories

Do not use accessories in combinations that are not mentioned in these instructions for use.

- Follow the instructions for use of individual accessories.
- Please address your B. Braun/Aesculap partner or Aesculap Technical Service with any inquiries in this respect; for a contact address, see Technical Service.

Remove the ECCOS general holder for saw blades GB720R from the tray

The insert of the ECCOS holder can be removed from the tray in conjunction with the tools.

- Pull at the tab 12 and remove the insert from the holder, see Fig. F.
- Fold up the tool holder 10.

Coupling the tool to the applied part

- Lock applied part ("Off" position), see instructions for use of ELAN 4 electro control unit GA800 (TA014401).
- Insert connection side of saw blade 4 into the slot in the tool holder 1, see Fig. A. When doing so, make certain that the lateral stops of the saw blade are resting on the tool holder, see Fig. B.
The saw blade engages.
- Pull at saw blade 4 to check that it is securely coupled.

Uncoupling the tool from the applied part

- Lock applied part ("Off" position), see instructions for use of ELAN 4 electro control unit GA800 (TA014401).
- Gently press the gold piece on saw blade for tool release 5 downwards and keep pressed down, see Fig. B.
- Remove the saw blade 4 from the tool holder 1.

Install spray nozzle and tube set

The ELAN 4 disposable spray nozzle for sawing GA258SU is equipped with a small and large front retainer clip for optional use.:

- The small retainer clip is only for use in conjunction with the ELAN 4 micro saws GA836, GA837.
- The large retainer clip is only for use in conjunction with the big ELAN 4 saw GA832.
- ▶ Connect tube set GA395SU to pump of ELAN 4 electro control unit GA800, see instructions for use of ELAN 4 electro control unit GA800 TA014401.
- ▶ Remove the large front retainer clip from the spray nozzle **6**.
- ▶ Push the tube end directed at the area of application onto the tube connector **7** of the spray nozzle.
- ▶ Clip the spray nozzle **6** onto the black handle area of the application part and push/turn into the desired position, see Fig. C.
- ▶ Push the small front retainer clip into position and clip onto the application part.
 - or –
 - Remove the small front retainer clip from the spray nozzle.

Note

The spray tube of the spray nozzle can be easily bent by hand and can be adapted to the requirements of the application (e.g. following the contour of the housing).

6.2 Function checks

- ▶ Prior to each use, check that all products to be used are in good working order.
- ▶ Check the secure connection of all products to be used.
- ▶ Check that the saw blade is coupled properly: Pull at the saw blade.
- ▶ Check that the cutting edges of the saw blades are not showing any mechanical damage.
- ▶ Briefly run the applied part at maximum speed.
- ▶ Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

6.3 Safe operation

⚠ WARNING

Coagulation of patient tissue or risk of burns for patients and users from hot application part/tool!

- ▶ Cool the tool during operation.
- ▶ Keep product/tool out of the reach of patients.
- ▶ Allow the applied part/tool to cool down.
- ▶ When changing the tool, use a cloth as a protection against burns.

⚠ WARNING

Risk of infection from aerosol formation!

Risk of injuries caused by particles coming loose from the tool!

- ▶ Take appropriate protective measures, e.g. watertight protective clothing, face mask and protective goggles, suction extraction.

⚠ WARNING

Risk of injury and/or malfunction!

- ▶ Always carry out a function check prior to using the product.

⚠ WARNING

Risk of injury when using the product beyond the field of view!

- ▶ Apply the product only under visual control.

⚠ WARNING

Risk of injury if the tool comes loose accidentally!

- ▶ Do not press the saw blade tool release during operation.
- ▶ Check for secure fixation of the tool after any tool change.

Note

The recommended maximum oscillation frequency for operation is 16 000 rpm.

A higher oscillation frequency can cause the product to heat more quickly and adversely affect the lifetime of the product.

It is only possible to operate the applied part and change the setting parameters on the control unit if:

- the applied part is connected to the control unit,
- there is no second application part activated ("On" position) at the same time and
- the application part type **3** is shown on the display of the control unit.

Note

For further information, see instructions for use of ELAN 4 electro control unit GA800 (TA014401).

7. Validated reprocessing procedure

7.1 General safety notes

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for reprocessing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD, or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

Mechanical reprocessing should be favored over manual cleaning as it gives better and more reliable results.

Note

It should be noted that successful reprocessing of this medical device can only be guaranteed following prior validation of the reprocessing method. The operator/reprocessing technician is responsible for this.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

See also B. Braun eIFUs at eifu.bbraun.com for up-to-date information on processing and material compatibility

The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

7.2 General information

Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixating pre-cleaning temperatures >45 °C nor fixating disinfecting agents (active ingredient: aldehydes/alcohols) should be used.

Excessive neutralizing agents or basic cleaners may result in a chemical attack and/or fading and the laser marking becoming unreadable either visually or by machine.

Residues containing chlorine or chlorides, e.g., in surgical residues, medicines, saline solutions, and in the service water used for cleaning, disinfection, and sterilization, will cause corrosion damage (pitting, stress corrosion) and result damage to metallic products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Perform additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Material damage such as corrosion, cracks, fracturing, premature aging, or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surface and could cause corrosion
- ▶ For detailed information on hygienically safe and material-friendly/careful processing, please refer to www.a-k-i.org Header "AKI-Brochures", "Red brochure".

7.3 Reusable products

The service life of the product is limited by damage, normal wear, type and duration of the application, handling, storage and transportation of the product.

Careful visual and functional testing prior to next use is the best way to identify a malfunctioning product.

7.4 Preparations at the place of use

- ▶ Separate the products from each other immediately after use.
- ▶ Remove any visible surgical residues as much as possible with a damp, lint-free cloth.
- ▶ Place the dry product in a sealed waste container and forward it on for cleaning and disinfection within 6 hours.

7.5 Preparation before cleaning

Applied part

- ▶ Prior to first mechanical cleaning/disinfection: Install the ECCOS holder GB087R in a suitable tray (e.g. JF222R).
- ▶ Place the product in the correct position in the ECCOS holder GB087R, see Fig. D.

Tools

- ▶ Thoroughly rinse the product with running, cold water.
- ▶ Prior to first mechanical cleaning/disinfection: Install the bottom part of the ECCOS holder GB720R in a suitable tray (e.g. JF222R).
- ▶ Position the reciprocating saw blade micro 4 in the correct attitude relative to the tool holder 10 and insert it in the ECCOS holder GB720R, see Fig. F.
- ▶ Engage the holder in the bottom part. In doing so, make sure that the holding lugs 11 on the insert are inserted in the recesses in the bottom part.
- ▶ Lift and fold down the tool holder 10 using the two side handles 9.

7.6 Cleaning/disinfection

Product-specific safety instructions for the reprocessing procedure

⚠ CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

- ▶ Use cleaning and disinfecting agents according to the manufacturer's instructions which
 - are approved for plastic material and high-grade steel,
 - do not attack softeners (e.g. in silicone).
- ▶ Do not use cleaning agents that contain acetone.
- ▶ Observe specifications regarding concentration, temperature and exposure time.
- ▶ Do not exceed the maximum temperature of 60 °C during chemical cleaning and/or disinfection.
- ▶ Do not exceed maximum temperature of 96°C during thermal disinfection using DI water.
- ▶ Dry the product for at least 10 minutes at a maximum of 120 °C.

Note

The indicated drying time is a guide time only. It must be checked taking into account the specific conditions (e.g. load) and if applicable adjusted.

Validated cleaning and disinfection procedure

Product	Validated procedure	Reference
Application part	Manual cleaning with immersion disinfection	see Manual cleaning with immersion disinfection – application part
	Manual pre-cleaning with brush and subsequent mechanical alkaline cleaning and thermal disinfecting	see Mechanical cleaning/disinfection including manual pre-cleaning – application part
Tools	Manual pre-cleaning with ultrasound and brush, and subsequent mechanical alkaline cleaning and thermal disinfecting	see Mechanical cleaning/disinfecting with manual pre-cleaning – tools

7.7 Manual cleaning with immersion disinfection – application part

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Pre-cleaning	RT (cold)	–	–	D–W	–
II	Cleaning	RT (cold)	>5	1	D–W	pH neutral, pH ~ 9*
III	Intermediate rinse	RT (cold)	–	–	D–W	–
IV	Disinfection	RT (cold)	>5	2	D–W	Concentrate-free, aldehyde-free, phenol-free, QUAT-free, pH ~ 9**
V	Final rinse	RT (cold)	–	–	D–W	–
VI	Drying	RT	–	–	–	–

D–W: Drinking water

RT: Room temperature

*Recommended: BBraun Helizyme

*Recommended: B. Braun Stabimed fresh

- Observe instructions for use for ELAN 4 rinsing devices and rinsing adapter TA014447 or TA014448.
- Do not clean the applied part in an ultrasound bath.

Phase I

- Clean the product under running water, using a suitable cleaning brush until all visible residue has been removed from the surface.
- Move non-rigid components, such as unlocking buttons, unlocking sleeves, etc., during cleaning.
- Rinse the applied part through the rinsing adapter GB698R at least 3 times for 5 seconds with a water gun.

Phase II

- Using a disposable syringe, fill the inside of the applied part with cleaning solution through the rinsing adapter GB698R.
- Immerse the application part with the connected rinsing adapter at least 5 minutes in an enzymatic cleaning solution.

Phase III

- Rinse product completely (all accessible surfaces) under running water.
- Move non-rigid components, such as unlocking buttons, unlocking sleeves, etc., during rinsing.
- Rinse the applied part through the rinsing adapter GB698R at least 3 times for 5 seconds with a water gun.

Phase IV

- Prior to manual disinfection, allow rinsing water to drain off the product for a sufficient length of time and use compressed air to blow all liquid out of the product through the rinsing adapter GB698R in order to prevent dilution of the disinfecting solution.
- Using a disposable syringe, fill the inside of the applied part with disinfectant solution through the rinsing adapter GB698R.
- Immerse the application part with the connected rinsing adapters in the disinfectant solution for at least 5 minutes.

Phase V

- Rinse product completely (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as release button, unlocking sleeve, etc. during cleaning.
- Rinse the applied part through the rinsing adapter GB698R at least 3 times for 5 seconds with a water gun.

Phase VI

- During the drying phase, dry the product using the appropriate equipment (e.g. wipes, compressed air).
- After manual cleaning/disinfection, check visible surfaces for residue and repeat the cleaning/disinfection process as required.

7.8 Mechanical cleaning/disinfection including manual pre-cleaning – application part

Note

The cleaning and disinfecting machine must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note

The cleaning and disinfection machine used for processing must be serviced and checked at regular intervals.

Manual pre-cleaning using brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Brushing	RT (cold)	–	–	D–W	–
II	Rinsing	RT (cold)	5	–	D–W	–

D–W: Drinking water
RT: Room temperature

- Follow the instructions for use of the ELAN 4 rinsing devices and rinsing adapters TA014447 and TA014448.
- Do not clean the application part in an ultrasonic bath.

Phase I

- Clean the product using a suitable cleaning brush until no residue is visible on the surface.
- Mobilize non-rigid components, such as release button, unlocking sleeve, etc. during cleaning.

Phase II

- Connect motor cable connector 2 to ELAN 4 electro rinsing device GB692R.
- Rinse application part:
 - For 5 min using a tap/hose or
 - 3 times for 5 s using a water gun

Mechanical alkaline cleaning and thermal disinfection

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemical
I	Pre-rinse	<25/77	3	D–W	–
II	Cleaning	55/131	10	FD–W	■ Concentrate, alkaline: – pH ~ 13 – <5 % anionic surfactant ■ Working solution 0.5 % – pH ~ 11*
III	Intermediate rinse	>10/50	1	FD–W	–
IV	Thermal disinfection	90/194	5	FD–W	–
V	Drying	max. 120/248	min. 10	–	–

D–W: Drinking water

FD–W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)

*Recommended: BBraun Helimatic Cleaner alkaline

- Connect motor cable connector 2 to ELAN 4 electro rinsing device GB692R.
- Place the rinsing device in a wire basket suitable for cleaning.
- Connect the rinsing device to the rinsing port of the rinsing cart.

- After machine cleaning/disinfection process:
 - Use a compressed air gun to remove the remaining rinsing water from the application parts and shafts, see the instructions for use for ELAN 4 rinsing system and rinsing adapter TA014447 or TA014448.
 - Visually check for residue on visible surfaces and repeat the cleaning/disinfection process if necessary.

7.9 Mechanical cleaning/disinfecting with manual pre-cleaning – tools

Note

The cleaning and disinfection device must be certified (e.g., FDA approval or CE mark in accordance with DIN EN ISO 15883).

Note

The cleaning and disinfection device used must be serviced and tested at regular intervals.

Manual pre-cleaning with ultrasound and brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemical
I	Immersion	RT (cold)	>30	50	D-W	Enzymatic detergent*
II	Rinsing	RT (cold)	–	–	D-W	–
III	Ultrasonic cleaning	55/131	>15	2	D-W	Concentrate-free, aldehyde-free, phenol-free, QUAT-free, pH ~ 9**
IV	Cleaning with brush	RT (cold)	–	–	D-W	–

D-W: Drinking water

RT: Room temperature

*Recommended: BBraun Helizyme

*Recommended: B.Braun Stabimed fresh

- As a general rule, only perform ultrasound cleaning in the ECCOS holder GB720R to prevent any damage to the blades.

Phase I

- Fully immerse the product in enzymatic cleaning solution for at least 30 minutes. Ensure that all accessible surfaces are moistened.

Phase II

- Rinse product completely (all accessible surfaces) under running water.

Phase III

- Clean the product in an ultrasonic cleaning bath (frequency 35 kHz) for at least 15 minutes. Ensure that all accessible surfaces are immersed and acoustic shadows are avoided.

Phase IV

- Clean the product using a suitable cleaning brush until no residue is visible on the surface.

Alkaline machine cleaning and thermal disinfection

Type of device: Single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemical
I	Pre-rinse	<25/77	3	D-W	–
II	Cleaning	55/131	10	FD-W	■ Concentrate, alkaline: – pH ~ 13 – <5 % anionic surfactant ■ Working solution 0.5 % – pH ~ 11*
III	Intermediate rinse	>10/50	1	FD-W	–
IV	Thermal disinfection	90/194	5	FD-W	–
V	Drying	max. 120/248	min. 10	–	–

D-W: Drinking water

FD-W: Deionized water (demineralized, microbiologically at least in compliance with potable water regulations)

*Recommended: B. Braun Helimatic Cleaner alkaline

- Place product in the ECCOS holder GB720R on a wire basket suitable for cleaning.
- After machine cleaning/disinfection, check visible surfaces for residue and repeat the cleaning/disinfection process as required.

7.10 Inspection, maintenance and checks

- Allow the product to cool down to room temperature.
- Connect the ELAN 4 electro oil spray adapter GB600860 8 (grey) to the connector for the motor cable 2 and use the STERILIT Power Systems oil spray GB600 to spray through the application part for approx. 2 s, see Fig. E.

Note

Aesculap additionally recommends the occasional spraying of moving parts (such as knobs, couplings) using the Aesculap STERILIT Power Systems oil spray.

- Inspect the product after every cleaning and disinfecting cycle for: cleanliness, damage, function, abnormal operation noise, excessive heat or strong vibration.
- Inspect tools for broken, damaged or blunt edges.
- Set aside the product if it is damaged.

7.11 Packaging

- Observe the instructions for use of the packagings and holders used (e.g. instructions for use TA009721 for Aesculap ECCOS holder system).
- Place the application part in the correct position in the ECCOS holder GB087R, see Fig. D.
- Insert the reciprocating saw blades in the correct position in the ECCOS holder GB720R, see Fig. F.
- Pack trays appropriately for the sterilization process (e.g. in Aesculap sterile containers).
- Ensure that the packaging will prevent a recontamination of the product.

7.12 Steam sterilization

- Make sure that all external and internal surfaces of the product are exposed to the sterilizing agent.
- Validated sterilization process
 - Steam sterilization using fractional vacuum process
 - Steam sterilizer according to DIN EN 285 and validated according to DIN EN ISO 17665
 - Sterilization using fractionated vacuum process at 134 °C/holding time 5 min
- When sterilizing several products at the same time in a steam sterilizer, ensure that the maximum load capacity of the steam sterilizer specified by the manufacturer is not exceeded.

7.13 Storage

- Store sterile products in germ-proof packaging, protected from dust, in a dry, dark, temperature-controlled area.

8. Maintenance

Application part

In order to ensure reliable operation, servicing must be performed in accordance with the maintenance labeling or at least once annually.

e.g. 2016-07

Please contact your national B. Braun/Aesculap representative, see Technical Service if you require such services.

Tools

No maintenance required for tools.

- Do not modify, re-sharpen or repair tools.

9. Troubleshooting

Note

For further information, see instructions for use of ELAN 4 electro control unit GA800 (TA014401).

10. Technical Service

⚠ DANGER

Danger to life of patients and users in case of malfunctions and/or failure of protective measures!

- Do not perform any servicing or maintenance work under any circumstances while the product is being used on a patient.

⚠ CAUTION

Modifications to medical equipment may result in loss of guarantee/warranty claims and any approvals.

- Do not modify the product.
- For service and repairs, please contact your national B. Braun/Aesculap representative.

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 (7461) 95 -1601

Fax: +49 (7461) 14 -939

E-Mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

11. Accessories/Spare parts

Art. no.	Designation
GA258SU	ELAN 4 disposable spray nozzle for saws
GA395SU	ELAN 4 electro disposable tube set
GB087R	ELAN 4 electro ECCOS holder for micro reciprocating saw GA837
GB600	STERILIT Power Systems oil spray
GB600860	ELAN 4 electro oil spray adapter
GB692R	ELAN 4 electro rinsing device
GB698R	ELAN 4 electro rinsing adapter
GB720R	ECCOS general holder for saw blades
TA012787	Instructions for use Tools for Aesculap motor systems
TA014430	Instructions for use for ELAN 4 electro micro reciprocating saw GA837 (A4 for folder)
TA014431	Instructions for use for ELAN 4 electro micro reciprocating saw GA837 (leaflet)

12. Technical data

12.1 Classification in accordance with Regulation (EC) 2017/745

Art. no.	Designation	Class
GA837	ELAN 4 electro micro reciprocating saw	Ila

12.2 Performance data, information about standards

Max. power	approx. 180 W
Max. oscillation frequency	20 000 rpm
Weight	198 g ±10 %
Dimensions (∅ x L)	22.8 mm x 184 mm ±5 %
Application part	Type BF
EMC	IEC/EN 60601-1-2
Compliance with standards	IEC/DIN EN 60601-1

After 350 processing cycles, the manufacturer tested the product and it passed.

12.3 Tool speed/Setting

Adjustable maximum speed setting (oscillation frequency)	8 000 rpm to 20 000 rpm
Step width for maximum speed setting	1 000 rpm
Default factory setting of upper speed range limit	16 000 rpm
Tool connector	Reciprocating saw blade micro

12.4 Short interval operation

- 10 min application, 40 min break
- ∞ Repetitions
- Max. temperature 48 °C

12.5 Ambient conditions

Application part

	Operation	Transport and storage
Temperature	10 °C to 27 °C	-10 °C to 50 °C
Relative humidity	30 % to 75 %	10 % to 90 %
Atmospheric pressure	700 hPa to 1 060 hPa	500 hPa to 1 060 hPa

13. Disposal

WARNING

Risk of infection from contaminated products!

- Observe national regulations when disposing of or recycling the product, its components, and their packaging.

WARNING

Risk of injury from sharp and/or pointed products!

- When disposing of or recycling the product, make sure the packaging prevents injuries from the product.

Note

The operator must process the product before disposal, see Validated reprocessing procedure.

The recycling passport can be downloaded from the extranet as a PDF document for on the individual part number. (The recycling passport contains instructions on how to disassemble the device, including information on the proper disposal of environmentally harmful components.)

Any product labeled with this symbol must be disposed of separately together with electrical and electronic equipment. Within the European Union, the manufacturer will perform the disposal free of charge.

- In case of questions regarding the disposal of the product, contact your local B. Braun/Aesculap representative, see Technical Service.

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