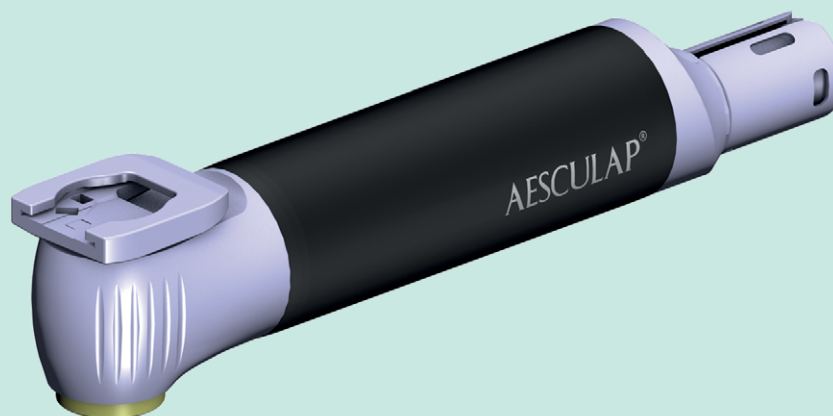
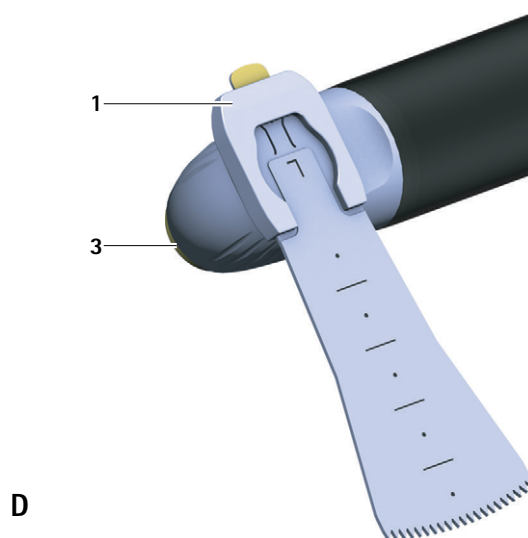
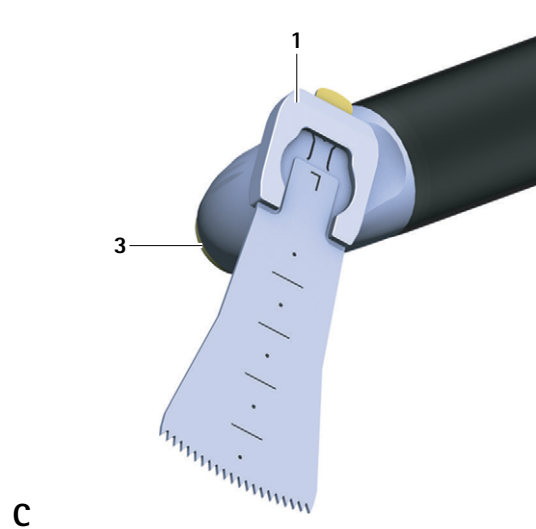
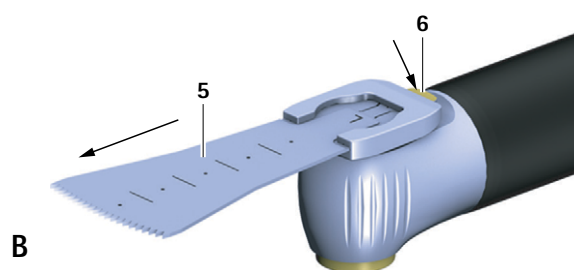
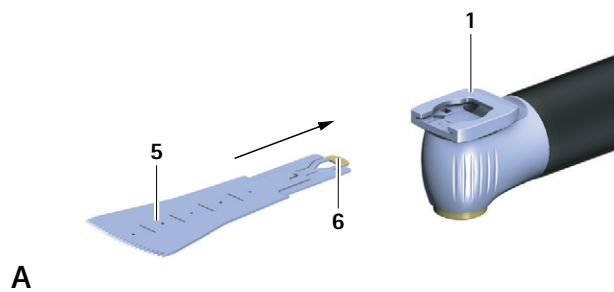
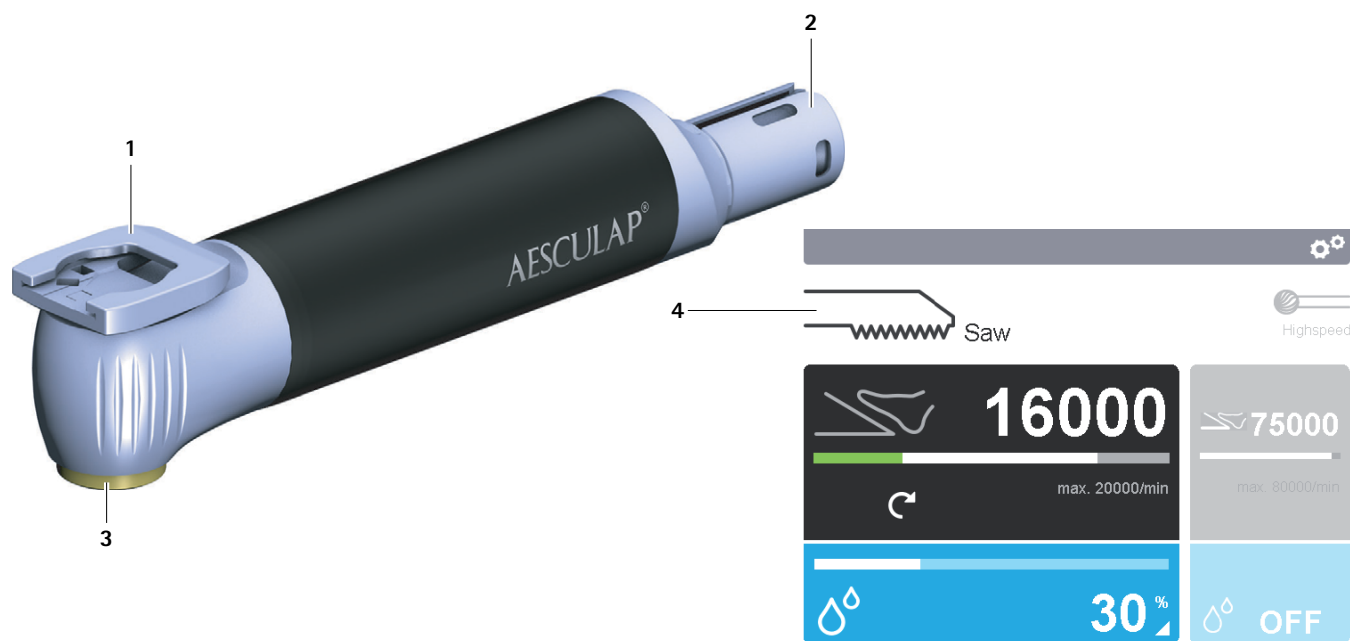
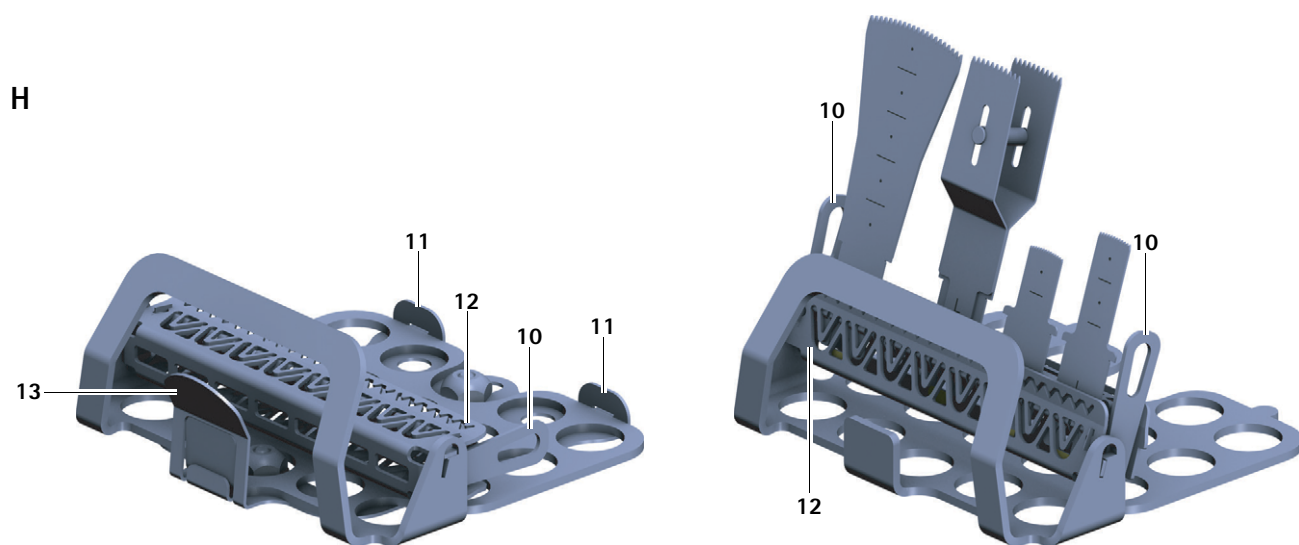
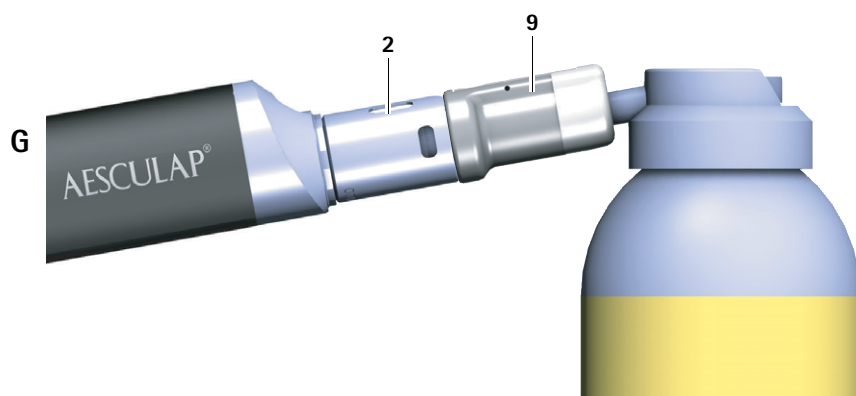
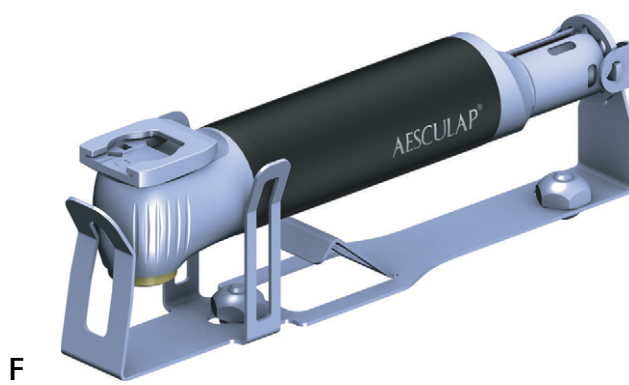
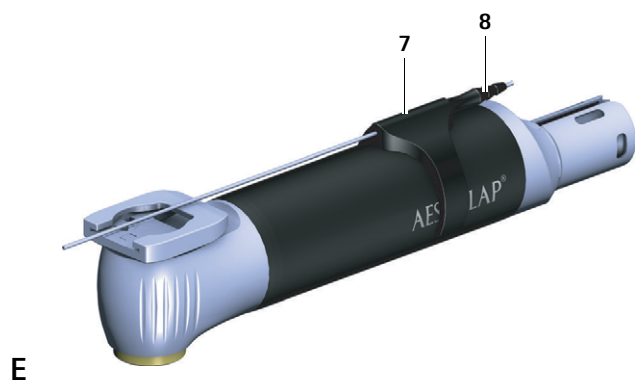
- (GB)** Instructions for use/Technical description
- (USA)** ELAN 4 electro sagittal saw GA831
- (D)** Gebrauchsanweisung/Technische Beschreibung
ELAN 4 electro Sagittalsäge GA831
- (F)** Mode d'emploi/Description technique
Scie sagittale ELAN 4 electro GA831
- (E)** Instrucciones de manejo/Descripción técnica
Sierra sagital ELAN 4 electro GA831
- (I)** Istruzioni per l'uso/Descrizione tecnica
Sega sagittale ELAN 4 electro GA831
- (P)** Instruções de utilização/Descrição técnica
Serra sagital ELAN 4 electro GA831
- (NL)** Gebruiksaanwijzing/Technische beschrijving
ELAN 4 electro sagittaalzaag GA831

- (S)** Bruksanvisning/Teknisk beskrivning
ELAN 4 electro sagittalsågar GA831
- (RUS)** Инструкция по применению/Техническое описание
Сагиттальная пила ELAN 4 electro GA831
- (CZ)** Návod k použití/Technický popis
Sagitální pila ELAN 4 electro GA831
- (PL)** Instrukcja użytkowania/Opis techniczny
Piła strzałkowa ELAN 4 electro GA831
- (SK)** Návod na použitie/Technický opis
Sagitálna pila ELAN 4 electro GA831
- (TR)** Kullanım Kılavuzu/Teknik açıklama
ELAN 4 electro sagital testere GA831
- (KR)** 사용 설명서 / 기술 설명
ELAN 4 electro 시상 톱 GA831

Aesculap®

ELAN 4 electro sagittal saw GA831

Legend

- 1 Tool holder with ELAN 4 L sagittal tool coupling
- 2 Connector for motor cable
- 3 Button for adjusting the tool holder
- 4 Symbol of the applied part type in display of ELAN 4 electro control unit
- 5 Saw blade L sagittal
- 6 Piece on saw blade for tool release
- 7 Spray nozzle for sawing GA258SU
- 8 Tube connection on spray nozzle
- 9 Oil spray adapter GB600860

ELAN 4 ECCOS universal holder for saw blades GB720R

- 10 Side handles
- 11 Holding nibs
- 12 Tool coupling
- 13 Tab

Symbols on product and packages

	Caution Observe important safety information such as warnings and precautions in the instructions for use.
 YYYY-MM	Maintenance label Recommended date of next service (format: YYYY-MM) by international B. Braun-/Aesculap agents, see Technical Service
	Machine-readable, two-dimensional code The code contains a unique serial number which can be used for electronic tracking of the individual instrument. The serial number is based on the global standard sGTIN (GS1).
	Manufacturer
	Date of manufacture
	Sterilization using irradiation
	Not for reuse in intended applications as defined by the manufacturer
	Use by
	Manufacturer's batch designation
	Manufacturer's serial number
	Manufacturer's article number
	Temperature limits during transport and storage
	Air humidity limits during transport and storage
	Atmospheric pressure limits during transport and storage

Contents

1.	Applicable to	3
2.	Intended use	3
2.1	Task/function within system	3
2.2	Application Environment	3
2.3	Indications	3
2.4	Contraindications	4
3.	Safe handling	4
3.1	Sterile ELAN 4 tools	4
3.2	ELAN 4 single-use spray nozzle for sawing GA258SU	4
4.	Product description	5
4.1	Scope of supply	5
4.2	Components required for operation	5
4.3	Operating principle	5
5.	Preparation	5
6.	Working with the applied part	5
6.1	System set-up	6
6.2	Function checks	6
6.3	Safe operation	7
7.	Validated reprocessing procedure	7
7.1	General safety notes	7
7.2	General information	8
7.3	Preparations at the place of use	8
7.4	Preparation before cleaning	8
7.5	Cleaning/disinfection	8
7.6	Manual cleaning with immersion disinfection – applied part ..	10
7.7	Mechanical cleaning/disinfecting with manual pre-cleaning – applied part	11
7.8	Mechanical cleaning/disinfecting with manual pre-cleaning – tools	13
7.9	Inspection, maintenance and checks	14
7.10	Packaging	14
7.11	Steam sterilization	14
7.12	Sterilization for the US market	15
7.13	Storage	15
8.	Maintenance	15
9.	Troubleshooting list	15
10.	Technical Service	15
11.	Accessories/Spare parts	16
12.	Technical data	16
12.1	Classification acc. to Directive 93/42/EEC	16
12.2	Performance data, information about standards	16
12.3	Tool speed/Setting	16
12.4	Short interval operation	16
12.5	Ambient conditions	16
13.	Disposal	17

14.	Distributor in the US/Contact in Canada for product information and complaints	17
-----	---	----

1. Applicable to

- For item-specific instructions for use and information on material compatibility, see also the Aesculap Extranet at <https://extranet.bbraun.com>

2. Intended use

2.1 Task/function within system

The hand-held ELAN 4 electro sagittal saw is an accessory of the ELAN 4 electro motor system.

It is connected by an ELAN 4 electro motor cable to the control unit.

The ELAN 4 electro sagittal saw is used for driving ELAN 4 sagittal saw blades.

Oscillation frequency	min. 0 min ⁻¹ to max. 20 000 min ⁻¹
Short interval operation	30 second application, 60 second pause, 4 repetitions at 16 000 min ⁻¹

2.2 Application Environment

Application in the sterile area

2.3 Indications

The ELAN 4 electric sagittal saw is used in neurosurgery, ENT surgery, OMF surgery, orthopedics and trauma surgery. It is used for separating, removing and shaping hard tissue, cartilage and similar, as well as for sawing bone and bone substitute materials.

Note

The type and area of application depend on the tools selected.

Aesculap®

ELAN 4 electro sagittal saw GA831

2.4 Contraindications

The ELAN 4 electro motor system is not authorized for use in the central nervous system or central circulatory system.

Note

The safe and effective use of electrically driven applied parts depends heavily on influences that only the user can control him or herself. Therefore the specifications provided represent framework conditions only.

Note

The clinical success of the use of the ELAN 4 electro motor system is dependent on the knowledge and experience of the surgeon. The surgeon must decide which structures it is sensible to treat and take into account the safety and warning information contained in these instructions for use.

3. Safe handling

CAUTION

Federal law restricts this device to sale by, or on order of a physician!

WARNING

Risk of injury and material damage due to inappropriate use of the product!

- Use the product only in accordance with the intended purpose.

WARNING

Risk of injury and damage to property due to improper handling of the product!

This product is an accessory of the ELAN 4 electro control unit GA800.

- Follow the instructions for use of the ELAN 4 electro control unit GA800 (TA014401).
- Follow the instructions for use of all products used.

- General risk factors associated with surgical procedures are not described in this documentation.
- It is the operating surgeon's responsibility to ensure that the surgical procedure is performed correctly.
- The operating surgeon must have a thorough understanding of both the hands-on and conceptual aspects of the established operating techniques.
- Remove the transport packaging and clean the new product, either manually or mechanically, prior to its initial sterilization.
- Prior to use, check that the product is in good working order.
- Observe "Notes on Electromagnetic Compatibility (EMC)", see TA022130.

- To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- Keep the instructions for use accessible for the user.
- Always adhere to applicable standards.
- Ensure that the electrical installations at the premises where the device is to be used meet all requirements acc. to IEC/DIN EN.
- Do not use the control unit and applied part in potentially explosive areas.
- Carry out sterile processing in any reusable applied parts, motor cables and tools prior to use.
- When handling, observe instructions for use that are relevant to Aesculap holder systems TA009721, see Aesculap Extranet at <https://extranet.bbraun.com>

3.1 Sterile ELAN 4 tools

The product is gamma-sterilized and supplied in sterile packaging.

- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- read, follow, and keep the instructions for use.
- Use the product only in accordance with its intended use, see Intended use.
- Do not use products from open or damaged sterile packaging.
- Prior to each use, inspect the product for loose, bent, broken, cracked or fractured components.
- Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- Do not use the product after its use-by date.

3.2 ELAN 4 single-use spray nozzle for sawing GA258SU

DANGER

Risk of infection for patients and/or users and impairment of product functionality due to reuse. Risk of injury, illness or death due to contamination and/or impaired functionality of the product!

- Do not reprocess the product.

The product is gamma-sterilized and supplied in sterile packaging.

The product must not be reused.

- Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- read, follow, and keep the instructions for use.
- Use the product only in accordance with its intended use, see Intended use.

- ▶ Do not use products from open or damaged sterile packaging.
- ▶ Prior to each use, inspect the product for: loose, bent, broken, cracked, worn, or fractured components.
- ▶ Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- ▶ Do not use the product after its use-by date.

4. Product description

4.1 Scope of supply

Art. no.	Designation
GA831	ELAN 4 electro sagittal saw
TA014423	Instructions for use of ELAN 4 electro sagittal saw GA831 (leaflet)

4.2 Components required for operation

- Operation-ready ELAN 4 electro control unit GA800, see TA014401
- ELAN 4 saw blade L sagittal (e.g. GP501R to GP539R)

4.3 Operating principle

The oscillating movement of the saw blade cuts through bone and cartilage.

The oscillation frequency can be adjusted continuously by the manual/foot control.

The recommended maximum oscillation frequency for sawing is 16 000 min⁻¹.

The tool holder can be locked in five different positions without requiring a wrench (-90°/-45°/0°/45°/90°).

No wrenches are required for coupling/uncoupling the saw blades.

5. Preparation

Non-compliance with the following instructions will preclude all responsibility and liability in this respect on the part of Aesculap.

- ▶ Do not use products from open or damaged sterile packaging.
- ▶ Prior to use, inspect the product and its accessories for any visible damage.
- ▶ Use the products and their accessories only if they are in perfect technical condition.

6. Working with the applied part

WARNING

Risk of infection and contamination!

Applied parts, motor cable and sometimes also tools are delivered unsterile!

- ▶ Carry out sterile processing of the applied parts, motor cable and unsterile tools prior to use, in accordance with the instructions for use.

WARNING

Risk of injury and material damage due to inadvertent activation of the applied part!

- ▶ Secure applied parts that are not actually used against inadvertent activation ("Off" position), see instructions for use for ELAN 4 electro control unit GA800 (TA014401).

WARNING

Risk of injury and material damage due to inappropriate use of tools!

- ▶ Always follow the safety advice and information given in the instructions for use.
- ▶ When coupling/uncoupling, handle tools with cutting edges with care.

WARNING

Damage to the product if dropped!

- ▶ Use the products only if they are in perfect technical condition, see Function check

WARNING

Risk of burns to skin and tissue due to blunt tools/lack of maintenance of the applied part!

- ▶ Use tools only if they are in perfect condition.
- ▶ Replace blunt tools.
- ▶ Ensure correct maintenance of the applied part, see Maintenance.

6.1 System set-up

Note

Operating elements on the system components of the ELAN 4 electro motor system are labeled with a gold marking.

Connecting the accessories

The sagittal saw is fitted with an ELAN 4 L sagittal tool coupling.

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

All configurations must comply with basic standard IEC/DIN EN 60601-1. Any individual connecting devices with one another is responsible for such configuration and must ensure compliance with basic standard IEC/DIN EN 60601-1 or applicable national standards.

- ▶ Follow the instructions for use of the accessories (e.g. TA012787: Tools for Aesculap motor systems).
- ▶ Please address your B. Braun/Aesculap partner or Aesculap Technical Service with any inquiries in this respect; for a contact address, see Technical Service.

Removing the universal ELAN 4 ECCOS holder for saw blades GB720R from the tray

The insert of the ELAN 4 ECCOS holder can be removed from the tray together with the tools.

- ▶ Pull the tab 13 and remove the insert of the holder, see Fig. H.

Coupling the tool to the applied part

WARNING

Risk of injury when coupling/uncoupling tools in the "On" position due to accidental activation of the applied part!

- ▶ Only couple/uncouple tools in the "Off" position.

- ▶ Lock applied part ("Off" position), see instructions for use of ELAN 4 electro control unit GA800 (TA014401).
- ▶ Insert saw blade 5 into the slot in the tool holder 1 with the "L" marking facing upward, see Fig. A. When doing so, make certain that the lateral stops of the saw blade are resting on the tool holder, see Fig. B. The saw blade engages.
- ▶ Pull at saw blade 5 to check that it is securely coupled.

Uncoupling the tool from the applied part

WARNING

Risk of injury when coupling/uncoupling tools in the "On" position due to accidental activation of the applied part!

- ▶ Only couple/uncouple tools in the "Off" position.

Note

For easier uncoupling of the tool, position tool holder in the 45° position (see Fig. C) or in the 90° position (see Fig. D), see Adjusting the tool holder.

- ▶ Lock applied part ("Off" position), see instructions for use of ELAN 4 electro control unit GA800 (TA014401).
- ▶ Gently press the gold piece on saw blade for tool release 6 downwards and keep pressed down, see Fig. B.
- ▶ Remove the saw blade 5 from the tool holder 1.

Adjusting the tool holder

- ▶ Press button for adjusting the tool holder 3 and keep pressed down.
- ▶ Turn tool holder 1 to the desired position (-90°/-45°/0°/45°/90°), see Fig. C and Fig. D.
- ▶ Release button for adjusting the tool holder 3 and lock tool holder 1 by engaging it in place. If necessary, slightly move the tool holder back and forth.

Mounting the spray nozzle and tube set

- ▶ Connect tube set GA395SU to pump of ELAN 4 electro control unit GA800, see instructions for use of ELAN 4 electro control unit GA800 (TA014401).
- ▶ Remove the front holding clip of spray nozzle GA285SU 7 (is only used in combination with ELAN 4 micro saws).
- ▶ Push application-side tube end onto tube connection 8 of the ELAN 4 single-use spray nozzle for sawing.
- ▶ Clip spray nozzle GA258SU 7 onto the black handle area of the applied part and push/turn into the desired position, see Fig. E.

Note

The spray tube of the ELAN 4 single-use spray nozzle is bendable by hand and can be adapted to the requirements of application (e.g. along the edges of the housing).

6.2 Function checks

- ▶ Prior to each use, check that all products to be used are in good working order.
- ▶ Check the secure connection of all products to be used.
- ▶ Check that the tool holder 1 is securely locked in place. Turn tool holder.
- ▶ Check that the saw blade is coupled properly. Pull at the saw blade.
- ▶ Check that the cutting edges of the saw blades are not showing any mechanical damage.
- ▶ Briefly run the applied part at maximum speed.
- ▶ Do not use the product if it is damaged or defective. Set aside the product if it is damaged.

6.3 Safe operation

WARNING

Coagulation of patient tissue or risk of burns for patients and users from hot applied part/tool!

- ▶ Cool the tool during operation.
- ▶ Put down the applied part/tool beyond reach of the patient.
- ▶ Allow the applied part/tool to cool down.
- ▶ Use a cloth to protect against burns when changing the tool.

WARNING

Risk of injury if the tool comes loose accidentally!

- ▶ Do not press the piece on saw blade for tool release during operation.
- ▶ Check for secure fixation of the tool after any tool change.

WARNING

Risk of infection from aerosol formation!

Risk of injuries caused by particles coming loose from the tool!

- ▶ Take appropriate protective measures, e.g. watertight protective clothing, face mask and protective goggles, suction extraction.

WARNING

Risk of injury and/or malfunction!

- ▶ Always carry out a function check prior to using the product.

Note

A speed in excess of the recommended $16\,000\text{ min}^{-1}$ can cause the product to heat up more quickly and adversely affect the lifetime of the product.

It is only possible to operate the applied part and change the setting parameters on the control unit if:

- the applied part is connected to the control unit,
- there is no second applied part released at the same time and
- the applied part type 4 is shown in the display of the control unit.

Note

For further information, see instructions for use of ELAN 4 electro control unit GA800 (TA014401).

7. Validated reprocessing procedure

7.1 General safety notes

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for reprocessing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD, or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

Mechanical reprocessing should be favored over manual cleaning as it gives better and more reliable results.

Note

It should be noted that successful reprocessing of this medical device can only be guaranteed following prior validation of the reprocessing method. The operator/reprocessing technician is responsible for this.

The recommended chemistry was used for validation.

Note

If there is no final sterilization, then a virucidal disinfectant must be used.

Note

For up-to-date information about reprocessing and material compatibility, see also the Aesculap Extranet at <https://extranet.bbraun.com>

The validated steam sterilization procedure was carried out in the Aesculap sterile container system.

7.2 General information

Dried or affixed surgical residues can make cleaning more difficult or ineffective and lead to corrosion. Therefore the time interval between application and processing should not exceed 6 h; also, neither fixating pre-cleaning temperatures >45 °C nor fixating disinfecting agents (active ingredient: aldehydes/alcohols) should be used.

Excessive neutralizing agents or basic cleaners may result in a chemical attack and/or fading and the laser marking becoming unreadable either visually or by machine.

Residues containing chlorine or chlorides, e.g., in surgical residues, medicines, saline solutions, and in the service water used for cleaning, disinfection, and sterilization, will cause corrosion damage (pitting, stress corrosion) and result damage to metallic products. These must be removed by rinsing thoroughly with demineralized water and then drying.

Perform additional drying, if necessary.

Only process chemicals that have been tested and approved (e.g. VAH or FDA approval or CE mark) and which are compatible with the product's materials according to the chemical manufacturers' recommendations may be used for processing the product. All the chemical manufacturer's application specifications must be strictly observed. Failure to do so can result in the following problems:

- Material damage such as corrosion, cracks, fracturing, premature aging, or swelling.
- ▶ Do not use metal cleaning brushes or other abrasives that would damage the product surface and could cause corrosion
- ▶ For further detailed information on hygienically safe and material-preserving/value-preserving reprocessing, see www.a-k-i.org, link to Publications, Red Brochure – Proper maintenance of instruments.

7.3 Preparations at the place of use

- ▶ Separate the products from each other immediately after use.
- ▶ Remove any visible surgical residues as much as possible with a damp, lint-free cloth.
- ▶ Place the dry product in a sealed waste container and forward it on for cleaning and disinfection within 6 hours.

7.4 Preparation before cleaning

Applied part

- ▶ Prior to first mechanical cleaning/disinfection process: Assemble ECCOS holder GB088R in a suitable tray (e.g. JF222R).
- ▶ Place the product in the correct position in the ECCOS holder GB088R, see Fig. F.

Tools

- ▶ Thoroughly rinse the product with running, cold water.
- ▶ Prior to first mechanical cleaning/disinfection process: Mount the bottom part of the ECCOS holder GB720R in a suitable tray (e.g. JF222R).
- ▶ Place the product in the correct position in the tool holder 12 of the ECCOS holder GB720R, see Fig. H.
- ▶ Engage the insert of the holder in the bottom part. When doing so, ensure that the holding nibs 11 on the insert are inserted in the recesses in the bottom part.

7.5 Cleaning/disinfection

Product-specific safety instructions for the reprocessing procedure

CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents and/or excessive temperatures!

- ▶ Use cleaning and disinfecting agents according to the manufacturer's instructions which
 - are approved for plastic material and high-grade steel,
 - do not attack softeners (e.g. in silicone).
- ▶ Do not use cleaning agents that contain acetone.
- ▶ Observe specifications regarding concentration, temperature and exposure time.
- ▶ Do not exceed the maximum temperature of 60 °C during chemical cleaning and/or disinfection.
- ▶ Do not exceed the maximum temperature of 90 °C during thermal disinfection with DI water.
- ▶ Dry the product for at least 10 minutes at a maximum of 120 °C.

Note

The indicated drying time is a guide time only. It must be checked taking into account the specific conditions (e.g. load) and if applicable adjusted.

Validated cleaning and disinfection procedure

Product	Validated procedure	Reference
Applied part	Manual cleaning with immersion disinfection	see Manual cleaning with immersion disinfection - applied part
	Manual pre-cleaning with brush and subsequent mechanical alkaline cleaning and thermal disinfecting	see Mechanical cleaning/disinfecting with manual pre-cleaning - applied part
Tools	Manual pre-cleaning with ultrasound and brush, and subsequent mechanical alkaline cleaning and thermal disinfecting	see Mechanical cleaning/disinfecting with manual pre-cleaning - tools

7.6 Manual cleaning with immersion disinfection – applied part

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Pre-cleaning	RT (cold)	-	-	D-W	-
II	Cleaning	RT (cold)	>5	1	D-W	pH-neutral, pH ~ 9*
III	Intermediate rinse	RT (cold)	-	-	D-W	-
IV	Disinfection	RT (cold)	>15	2	D-W	Concentrate-free, aldehyde-free, phenol-free, QUAT-free, pH ~ 9**
V	Final rinse	RT (cold)	-	-	D-W	-
VI	Drying	RT	-	-	-	-

D-W: Drinking water

RT: Room temperature

*Recommended: BBraun Helizyme

**Recommended: BBraun Stabimed

- Follow the instructions for use of the ELAN 4 rinsing devices and rinsing adapters TA014447 and TA014448.
- Do not clean the applied part in an ultrasound bath.

Phase I

- Clean the product under running water, using a suitable cleaning brush until all visible residues have been removed from the surfaces.
- Mobilize non-rigid components, such as release button, unlocking sleeve, etc. during cleaning.
- Rinse the applied part through the rinsing adapter GB698R at least 3 times for 5 seconds with a water gun.

Phase II

- Using a disposable syringe, fill the inside of the applied part with cleaning solution through the rinsing adapter GB698R.
- Place the applied part, with rinsing adapter connected, fully in enzymatic cleaning solution for 5 minutes.

Phase III

- Rinse the entire product (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as release button, unlocking sleeve, etc. during rinsing.
- Rinse the applied part through the rinsing adapter GB698R at least 3 times for 5 seconds with a water gun.

Phase IV

- Prior to manual disinfection, allow rinsing water to drain off the product for a sufficient length of time and use compressed air to blow all liquid out of the product through the rinsing adapter GB698R in order to prevent dilution of the disinfecting solution.
- Using a disposable syringe, fill the inside of the applied part with disinfectant solution through the rinsing adapter GB698R.
- Place the applied part, with rinsing adapter connected, fully in disinfectant solution for 15 minutes.

Phase V

- Rinse the entire product (all accessible surfaces) under running water.
- Mobilize non-rigid components, such as release button, unlocking sleeve, etc. during cleaning.
- Rinse the applied part through the rinsing adapter GB698R at least 3 times for 5 seconds with a water gun.

Phase VI

- Dry the product in the drying phase with suitable equipment (e.g. cloth, compressed air).
- After manual cleaning/disinfection, check visible surfaces visually for residues.
- Repeat the cleaning/disinfection process if necessary.

7.7 Mechanical cleaning/disinfecting with manual pre-cleaning – applied part

Note

The cleaning and disinfecting machine must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note

The cleaning and disinfection machine used for processing must be serviced and checked at regular intervals.

Manual pre-cleaning with a brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Brushes	RT (cold)	–	–	D–W	–
II	Rinsing	RT (cold)	5	–	D–W	–

D–W: Drinking water

RT: Room temperature

- ▶ Follow the instructions for use of the ELAN 4 rinsing devices and rinsing adapters TA014447 and TA014448.
- ▶ Do not clean the applied part in an ultrasound bath.

Phase I

- ▶ Clean the product with a suitable cleaning brush until all discernible residues have been removed.
- ▶ Mobilize non-rigid components, such as release button, unlocking sleeve, etc. during cleaning.

Phase II

- ▶ Connect the connector for motor cable 2 with ELAN 4 electro flushing device GB692R.
- ▶ Rinse the applied part:
 - For 5 min with a tap/hose or
 - 3 times for 5 seconds with a water gun

Mechanical alkaline cleaning and thermal disinfection

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemicals
I	Pre-rinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	■ Concentrate, alkaline: - pH ~ 13 - <5 % anionic surfactant ■ 0.5 % working solution - pH ~ 11*
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disinfection	90/194	5	FD-W	-
V	Drying	-	-	-	In accordance with the program for the cleaning and disinfecting machine

D-W: Drinking water

FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)

*Recommended: BBraun Helimatic Cleaner alkaline

- ▶ Connect the connector for motor cable 2 with ELAN 4 electro rinsing device GB692R.
- ▶ Place the rinsing device on a wire basket suitable for cleaning.
- ▶ Connect the rinsing device to the rinsing connection of the rinsing cart.
- ▶ After the mechanical cleaning/disinfection process:
 - Remove any residual rinsing water from the applied part using a compressed-air gun, see instructions for use ELAN 4 rinsing device and rinsing adapter TA014447 and TA014448.
 - Check visible surfaces for residues.

7.8 Mechanical cleaning/disinfecting with manual pre-cleaning – tools

Note

The cleaning and disinfecting machine must be of tested and approved effectiveness (e.g. FDA approval or CE mark according to DIN EN ISO 15883).

Note

The cleaning and disinfection machine used for processing must be serviced and checked at regular intervals.

Manual pre-cleaning with ultrasound and brush

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Immersion	RT (cold)	>30	50	D-W	Enzymatic detergent*
II	Rinsing	RT (cold)	-	-	D-W	-
III	Ultrasonic cleaning	55/131	>15	2	D-W	Concentrate-free, aldehyde-free, phenol-free, QUAT-free, pH ~ 9**
IV	Cleaning with brush	RT (cold)	-	-	D-W	-

D-W: Drinking water

RT: Room temperature

*Recommended: BBraun Helizyme

**Recommended: BBraun Stabimed

- Only perform ultrasound cleaning in the ECCOS holder GB720R to prevent any damage to the cutting edges.

Phase I

- Fully immerse the product in enzymatic cleaning solution for at least 30 minutes. Ensure that all accessible surfaces are moistened.

Phase II

- Rinse the entire product (all accessible surfaces) under running water.

Phase III

- Clean the product in an ultrasonic cleaning bath (frequency 35 kHz) for at least 15 minutes. Ensure that all accessible surfaces are immersed and acoustic shadows are avoided.

Phase IV

- Clean the product with a suitable cleaning brush until all discernible residues have been removed.

Mechanical alkaline cleaning and thermal disinfection

Machine type: single-chamber cleaning/disinfection device without ultrasound

Phase	Step	T [°C/°F]	t [min]	Water quality	Chemicals
I	Pre-rinse	<25/77	3	D-W	-
II	Cleaning	55/131	10	FD-W	■ Concentrate, alkaline: - pH ~ 13 - <5 % anionic surfactant ■ 0.5 % working solution - pH ~ 11*
III	Intermediate rinse	>10/50	1	FD-W	-
IV	Thermal disinfection	90/194	5	FD-W	-
V	Drying	-	-	-	In accordance with the program for the cleaning and disinfecting machine

D-W: Drinking water

FD-W: Fully desalinated water (demineralized, low microbiological contamination: drinking water quality at least)

*Recommended: BBraun Helimatic Cleaner alkaline

- ▶ Place the product on a wire basket suitable for cleaning.
- ▶ Check visible surfaces for residues after mechanical cleaning/disinfecting.

- ▶ Pack trays appropriately for the sterilization process (e.g. in Aesculap sterile containers).
- ▶ Ensure that the packaging will prevent a recontamination of the product.

7.9 Inspection, maintenance and checks

- ▶ Allow the product to cool down to room temperature.
- ▶ Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.
- ▶ Connect ELAN 4 electro oil spray adapter GB600860 9 (gray) to connector for motor cable 2 and spray through the applied part for approx. 2 seconds with Aesculap-STERILIT oil spray GB600, see Fig. G.
- ▶ Check the product for any damage, abnormal running noise, overheating or excessive vibration.
- ▶ Inspect tools for broken, damaged or blunt edges.
- ▶ Set aside the product if it is damaged.

7.10 Packaging

- ▶ Always follow the instructions for use of the respective packaging and holders (e.g. instructions for use TA009721 for Aesculap-ECCOS holder system).
- ▶ Place the applied part in the correct position in the ECCOS holder GB088R, see Fig. F.
- ▶ Place the tools on a tray in such a way that it is protected against damage.

7.11 Steam sterilization*Note*

For further information, see Technical data.

- ▶ Check to ensure that the sterilizing agent will come into contact with all external and internal surfaces (e.g. by opening any valves and faucets).
- ▶ Validated sterilization process
 - Steam sterilization using fractional vacuum process
 - Steam sterilizer according to DIN EN 285 and validated according to DIN EN ISO 17665
 - Sterilization using fractionated vacuum process at 134 °C/holding time 5 min
- ▶ When sterilizing several products at the same time in a steam sterilizer, ensure that the maximum load capacity of the steam sterilizer specified by the manufacturer is not exceeded.

7.12 Sterilization for the US market

- Aesculap advises against sterilizing the device by flash sterilization or chemical sterilization.
- Sterilization may be accomplished by a standard prevacuum cycle in a steam autoclave.

To achieve a sterility assurance level of 10^{-6} , Aesculap recommends the following parameters:

Aesculap Orga Tray/Sterile container (perforated bottom) Minimum cycle parameters*			
Sterilization method	Temp.	Time	Minimum drying time
Prevacuum	270 °F/275 °F	4 min	20 min

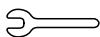
*Aesculap has validated the above sterilization cycle and has the data on file. The validation was accomplished in an Aesculap sterile container cleared by FDA for the sterilization and storage of these products. Other sterilization cycles may also be suitable, however individuals or hospitals not using the recommended method are advised to validate any alternative method using appropriate laboratory techniques. Use an FDA cleared accessory to maintain sterility after processing, such as a wrap, pouch, etc.

7.13 Storage

- Store sterile products in germ-proof packaging, protected from dust, in a dry, dark, temperature-controlled area.

8. Maintenance

To ensure reliable operation, the product must be maintained in accordance with the maintenance labeling or at least once a year.

e.g. 2016-07

For technical service, please contact your national B. Braun/Aesculap agency, see Technical Service.

9. Troubleshooting list

Note

For further information, see instructions for use of ELAN 4 electro control unit GA800 (TA014401).

10. Technical Service

Danger to life of patients and users if the product malfunctions and/or protective measures fail or are not used!

- Do not perform any servicing or maintenance work under any circumstances while the product is being used on a patient.
- Do not modify the product.

Modifications carried out on medical technical equipment may result in loss of guarantee/warranty rights and forfeiture of applicable licenses.

- For service and repairs, please contact your national B. Braun/Aesculap agency.

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 (7461) 95 -1601

Fax: +49 (7461) 14 -939

E-Mail: ats@aesculap.de

Or in the US:

Aesculap Inc.

Attn. Aesculap Technical Services

615 Lambert Pointe Drive

Hazelwood

MO, 63042 USA

Aesculap Repair Hotline

Phone: +1 (800) 214 -3392

Fax: +1 (314) 895 -4420

Other service addresses can be obtained from the address indicated above.

11. Accessories/Spare parts

Art. no.	Designation
GA258SU	ELAN 4 single-use spray nozzle for sawing
GA395SU	ELAN 4 electro single-use tube set
GB088R	ELAN 4 electro ECCOS holder for sagittal saw GA831
GB600	STERILIT Power Systems oil spray
GB600860	ELAN 4 electro oil spray adapter
GB692R	ELAN 4 electro rinsing device
GB698R	ELAN 4 electro rinsing adapter
GB720R	ECCOS universal holder for saw blades
GP501R– GP539R	ELAN 4 saw blade L sagittal
TA012787	Instructions for use for tools for Aesculap motor systems.
TA014422	Instructions for use of ELAN 4 electro sagittal saw GA831 (A4 for ring binder)
TA014423	Instructions for use of ELAN 4 electro sagittal saw GA831 (leaflet)

12. Technical data

12.1 Classification acc. to Directive 93/42/EEC

Art. no.	Designation	Class
GA831	ELAN 4 electro sagittal saw	Ila
GP501R– GP539R	ELAN 4 saw blade L sagittal	Ila

12.2 Performance data, information about standards

Max. power	approx. 180 W
Max. oscillation frequency	20 000 min ⁻¹
Weight	220 g ± 10 %
Dimensions (Ø x L)	23 mm x 129 mm ± 5 %
Applied part	Type BF
EMC	IEC/DIN EN 60601-1-2
Conforming to standard	IEC/DIN EN 60601-1

The product can be reprocessed 350 times. This was proved after 350 reprocessing cycles by a high voltage test at 1.6 kV.

12.3 Tool speed/Setting

Adjustable maximum speed setting (oscillation frequency)	8 000 min ⁻¹ to 20 000 min ⁻¹
Step width for maximum speed setting	1 000 min ⁻¹
Factory pre-setting for maximum speed setting	16 000 min ⁻¹
Tool connection	ELAN 4 L sagittal

12.4 Short interval operation

Ambient temperature	10 °C to 27 °C
Application cycle	30 second application 60 second pause
Max. application cycles	4 at 16 000 min ⁻¹
Max. temperature	48 °C
Cooling time	30 min

12.5 Ambient conditions

	Operation	Storage and transport
Temperature	10 °C to 27 °C	-10 °C to 50 °C
Relative humidity	30 % to 75 %	10 % to 90 %
Atmospheric pressure	700 hPa to 1 060 hPa	500 hPa to 1 060 hPa

13. Disposal

Note

The user institution is obliged to process the product before its disposal, see Validated reprocessing procedure.

Adhere to national regulations when disposing of or recycling the product, its components and its packaging!

The recycling pass can be downloaded from the Extranet as a PDF document under the respective article number.

(The recycling pass includes disassembling instructions for the product, as well as information for proper disposal of components harmful to the environment.)

Products carrying this symbol are subject to separate collection of electrical and electronic devices. Within the European Union, disposal is taken care of by the manufacturer as a free-of-charge service.

- Detailed information concerning the disposal of the product is available through your national B. Braun/Aesculap agency, see Technical Service.

14. Distributor in the US/Contact in Canada for product information and complaints

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