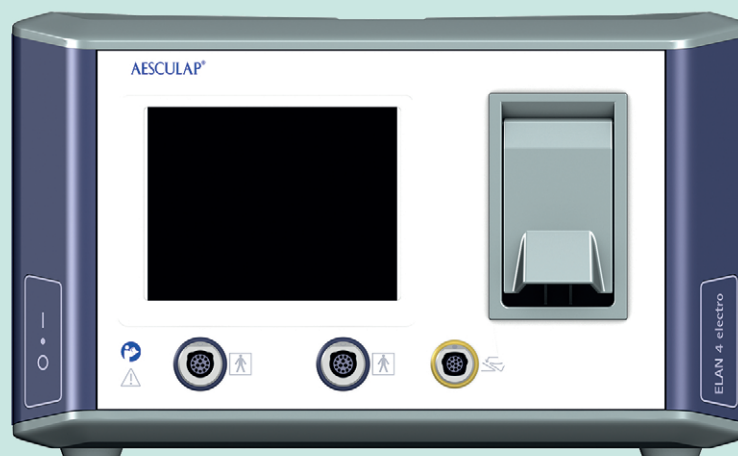
en Instructions for use/Technical description

ELAN 4 electro control unit GA800

USA Note for U.S. users

This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at www.aesculapusa.com and clicking the "Products" menu. If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost.

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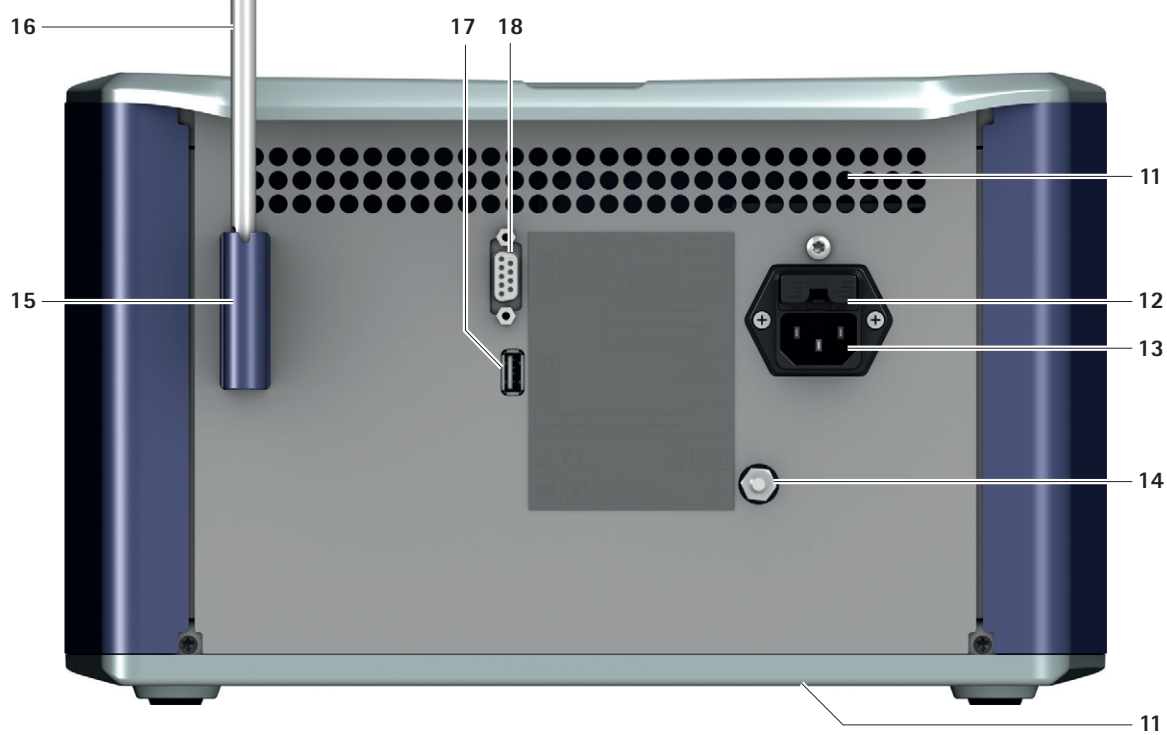
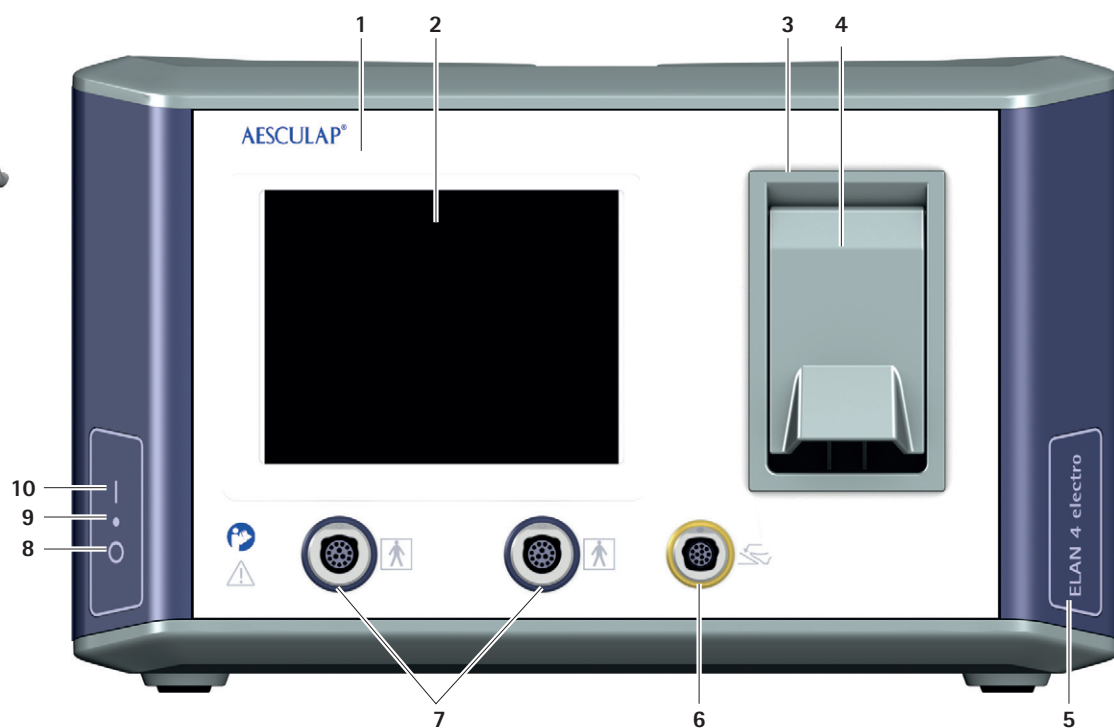
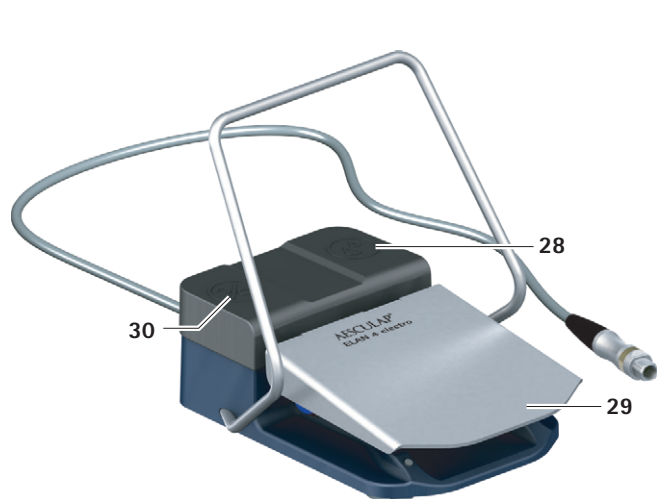
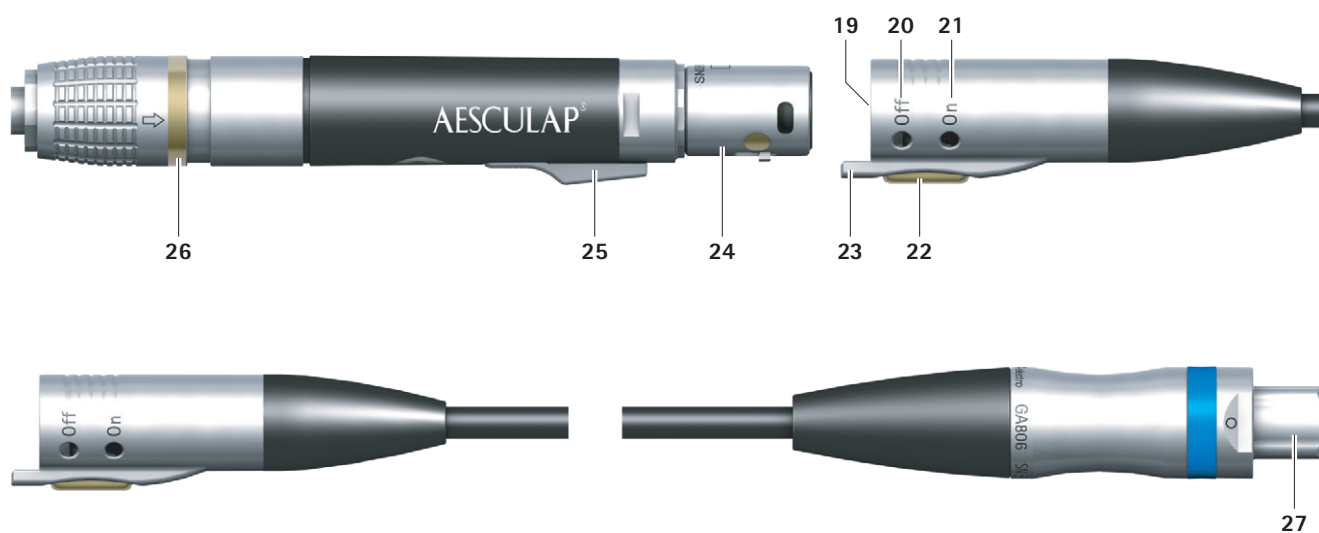
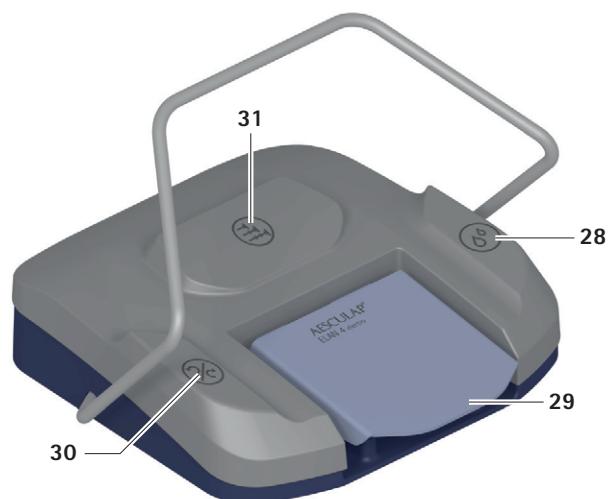
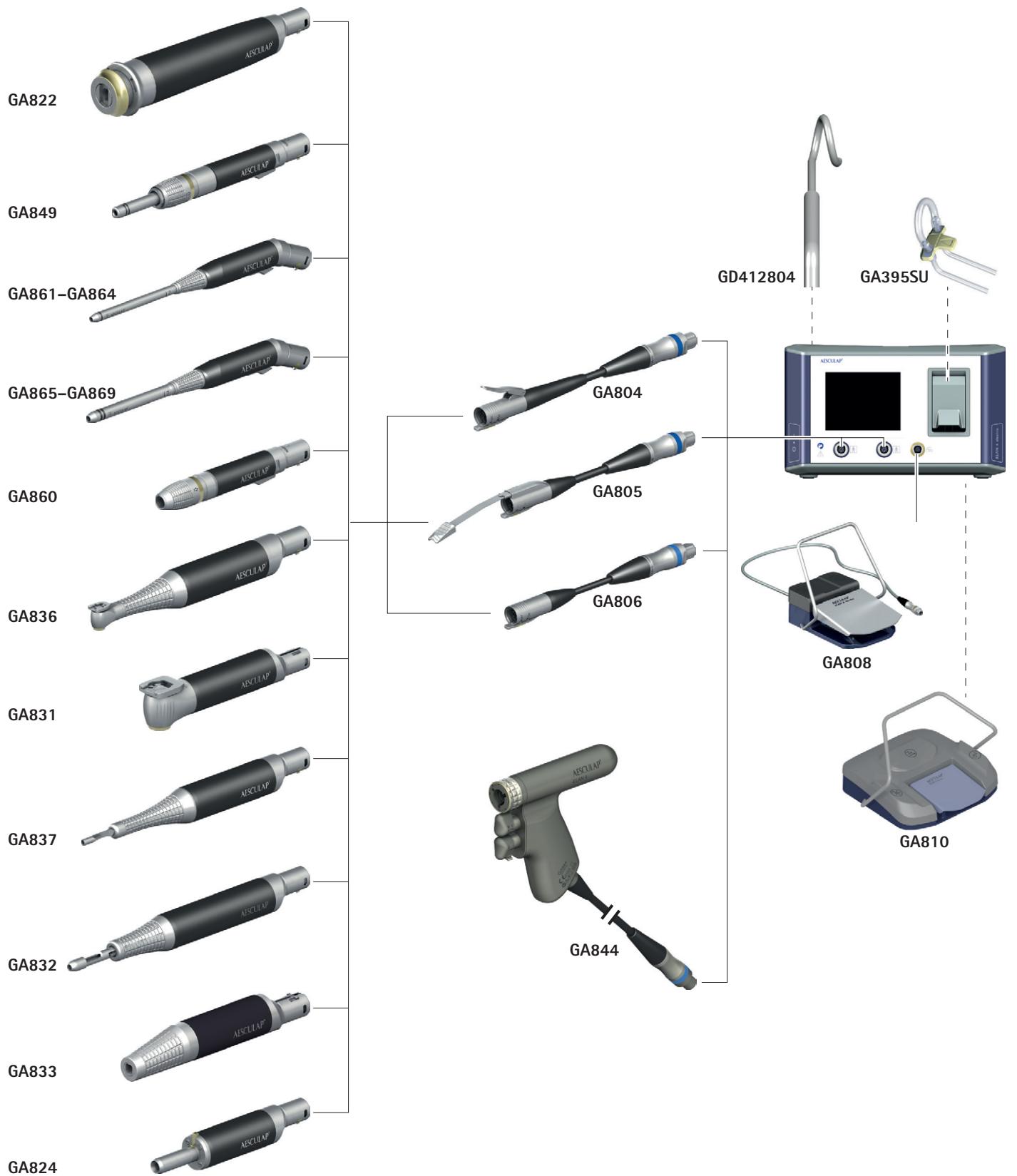
ELAN 4 electro control unit GA800

Legend

- 1 ELAN 4 electro control unit GA800
- 2 Display with touch control panel
- 3 Coolant and irrigation pump
- 4 Flap
- 5 Indicator light
- 6 Connection socket for foot control
- 7 Connection sockets for motor cable
- 8 Power OFF switch
- 9 Power ON indicator
- 10 Power ON switch
- 11 Ventilation grate
- 12 Fuse holder
- 13 Power socket
- 14 Terminal for equipotential bonding conductor
- 15 Bottle holder mount
- 16 Bottle holder
- 17 USB interface: Only intended for use by the manufacturer or by Service technicians authorized by Aesculap.
- 18 Interface RS232: Only intended for use by the manufacturer.

Motor cable/applied part

- 19 Connection for applied part
- 20 Visual field "Off"
- 21 Visual field "On"
- 22 Release button
- 23 Lug
- 24 Connection for motor cable on applied part
- 25 Slider for tool release
- 26 Pull back unlocking sleeve
- 27 Connection for control unit

Foot control

- 28 Function button
- 29 Pedal
- 30 Rotational direction button for motor
- 31 Speed range limit button (only for wireless foot control GA810)

Symbols on product and packaging

	Caution Observe important safety information such as warnings and precautions in the instructions for use.
	Follow the instructions for use
	"OFF" (power supply)
	"ON" (power supply)

	Type BF applied part
	Foot control
	Terminal for equipotential connector according to IEC/DIN EN 60601-1
	Fuse
	Alternating current
	Manufacturer
	Labeling of electrical and electronic devices according to directive 2012/19/EU (WEEE), see Chapter 13.
	Date of manufacture
	Manufacturer's batch designation
	Manufacturer's serial number
	Manufacturer's article number
	Delivery quantity
	Temperature limits during transport and storage
	Air humidity limits during transport and storage
	Atmospheric pressure limits during transport and storage
	Non sterile medical device
	CE marking according to Directive 93/42/EEC
	Federal law restricts this device to sale by or on order of a physician

Applied part types

Symbol	Text	Art. no.	Designation
	Perforator	GA822	ELAN 4 electro perforator driver
	Intra	GA824	ELAN 4 electro Lowspeed motor with intra coupling
	Saw	GA831	ELAN 4 electro sagittal saw
		GA832	ELAN 4 electro reciprocating saw
		GA833	ELAN 4 electro transverse saw
		GA836	ELAN 4 electro micro sagittal saw
		GA837	ELAN 4 electro micro reciprocating saw
	Highspeed	GA849	ELAN 4 electro craniotome and multifunctional handpiece (2-ring)
		GA860	ELAN 4 electro MIS hand piece
		GA861	ELAN 4 electro Standard handpiece (1-ring) L4
		GA862	ELAN 4 electro Standard handpiece (1-ring) L7
		GA863	ELAN 4 electro Standard handpiece (1-ring) L10
		GA864	ELAN 4 electro Standard handpiece (1-ring) L13
		GA865	ELAN 4 electro Standard handpiece (2-ring) L7
		GA866	ELAN 4 electro Standard handpiece (2-ring) L10
		GA867	ELAN 4 electro Standard handpiece (2-ring) L13
	Drill	GA868	ELAN 4 electro Standard handpiece (2-ring) L17
		GA869	ELAN 4 electro Standard handpiece (2-ring) L22
	Drill	GA844	ELAN 4 electro power drill

Display/control elements in the applied part control panel

Symbol	Designation
	Activation mode of motor via foot control GA808
	Activation of motor via wireless foot control GA810
	Activation of motor via hand control GA804/GA805/GA844
	Motor direction indicator for clockwise rotation preselected The display depends on whether the motor is activated or disabled.
	Motor direction indicator for counterclockwise rotation preselected The display depends on whether the motor is activated or disabled.
	Reduce the maximum speed setting
	Increase the maximum speed setting

Display/control elements in the pump control panel

Symbol	Designation
	Identification of the control panel for the coolant and irrigation pump Display in active applied part control panel
	Switch on the pump
	Switch off the pump
	Activate the "Flush" function (continuous flushing)
	Reduce the delivery rate
	Increase the delivery rate

Display/control elements in the system settings menu

Symbol	Designation
	Call up the system settings menu
	Leave the system settings menu
	User profiles
	Applied part settings
	Control unit settings
	Wireless foot control settings
	Control unit information
	Maximum speed/stroke rate
	Speed up
	Slow down
	Oscillation mode
	Oscillation angle
	Flow rate
	Increase the value
	Reduce the value
	System volume
	Display brightness
	System language

Symbol	Designation
	Reset device to factory settings
	Go back to menu structure
	Call up the submenu
	Move down list
	Move back up list
	Add new user profile
	Delete existing user profile
	Edit existing user profile
	Save user profile

Contents

1.	About this document	5
1.1	Scope	5
1.2	Warnings	5
2.	General information	6
2.1	Intended use	6
2.2	Application Environment	6
2.3	Indications	6
2.4	Absolute contraindications	6
2.5	Relative contraindications	6
3.	Safe handling	6
4.	Product description	7
4.1	Scope of supply	7
4.2	Components required for operation	7
4.3	Operating principle	7
4.3.1	Control unit	7
4.3.2	Display/control concept	7
4.3.3	Explanation with an example	7
4.3.4	Applied part types	8
4.3.5	Identification of control elements on system components	8
4.3.6	Recognition of connected motor cable and applied part	8
4.3.7	Overload protection	8
4.3.8	Coolant and irrigation pump	8
5.	Preparation and setup	8
5.1	Set-up environment/set-up location	8
5.2	Stacking of units	8
6.	Working with the ELAN 4 electro system	9
6.1	System set-up	9
6.1.1	Connecting the accessories	9
6.1.2	Connecting the power supply	9

6.1.3	Switching on the control unit	9	11.1	ELAN 4 electro motor cables, applied parts and foot controls	21
6.1.4	Switching off the control unit	9	11.2	Coolant and irrigation pump	22
6.1.5	Putting out of operation	9	11.3	Power cord.	22
6.1.6	Connecting the ELAN 4 electro to wireless foot control GA810	9	11.4	Equipotential bonding leads	22
6.1.7	Connecting the ELAN 4 electro foot control GA808 to the control unit	9	11.5	Spare parts	22
6.1.8	Connecting the ELAN 4 electro motor cable GA804/GA805/GA806 or ELAN 4 electro drill tool GA844 to control unit	9	12.	Technical data	22
6.1.9	Connecting the ELAN 4 electro single-use tube set GA395SU	10	12.1	Classification according to Directive 93/42/EEC	22
6.1.10	Connecting the applied part to the motor cable.	10	12.2	Performance data, information about standards	22
6.1.11	Releasing the applied part for operation ("On" position)	11	12.3	Ambient conditions.	23
6.1.12	Locking the applied part ("Off" position)	11	13.	Disposal	23
6.1.13	Disconnecting the applied part from the motor cable	11			
6.1.14	Disconnecting the ELAN 4 electro foot control GA808 from the control unit	11			
6.1.15	Disconnecting the ELAN 4 electro motor cable GA804/GA805/GA806 or ELAN 4 electro drill tool GA844 from control unit	11			
6.2	Function checks	11			
6.3	Safe operation	11			
6.3.1	Activating the motor/pump control panel	12			
6.3.2	Changing the maximum speed setting/rotational direction of the applied part	12			
6.3.3	Activating the coolant and irrigation pump/"Flush" function (continuous flushing)	12			
6.3.4	Deactivating the coolant and irrigation pump and changing the delivery rate	12			
6.3.5	Enable application part using foot control GA808 or wireless foot control GA810	13			
6.3.6	Activating the applied part via hand control GA804/GA805	13			
6.3.7	Activating the coolant pump via foot control	13			
6.4	System settings menu	13			
6.4.1	User profiles	14			
6.4.2	Motor settings	14			
6.4.3	Device settings	15			
6.4.4	Wireless foot control	15			
6.4.5	Device information	15			
7.	Reprocessing procedure	16			
7.1	General safety notes	16			
7.2	Reusable products	16			
7.3	Preparation before cleaning	16			
7.4	Cleaning/disinfection	16			
7.4.1	Product-specific safety instructions for the reprocessing procedure	16			
7.5	Wipe disinfection for electrical equipment	16			
7.6	Inspection, maintenance and checks	16			
8.	Maintenance	16			
9.	Troubleshooting list	17			
9.1	Display error messages	17			
9.1.1	System error	17			
9.1.2	Accessory error	17			
9.1.3	Operating error	18			
9.1.4	Other faults in control unit	19			
9.1.5	Faults when working with the applied part	19			
9.2	Fuse replacement	21			
10.	Technical Service	21			
11.	Accessories/Spare parts	21			

1. About this document

Note

General risk factors associated with surgical procedures are not described in these instructions for use.

- For article-specific instructions for use and material compatibility information, see B. Braun eIFUs at eifu.bbraun.com

1.1 Scope

This document describes all necessary instructions and steps for the provision, setting and safe operation of the ELAN 4 electro system and its accessory components.

Additional instructions and steps regarding the accessory components, in particular on their connection and reprocessing, can be found in the respective instructions for use or supplement for the components.

1.2 Warnings

Warnings indicate hazards to patients, users and/or products that may occur during the use of the product. Warnings are marked as follows:

DANGER

Indicates a potential danger. If it is not prevented, death or severe injuries may result.

WARNING

Indicates a potential danger. If not prevented, minor or moderate injuries may result.

CAUTION

Indicates a potential damage to property. If not prevented, the product may be damaged.

2. General information

2.1 Intended use

The ELAN 4 electro control unit GA800 together with the accessory forms an electrically operated motor system.

The ELAN 4 electro control unit GA800 supplies the energy and monitors the motors in the ELAN 4 electro applied parts. The control unit receives the speed request via the hand or foot control. The direction of rotation is preselected by means of the foot control buttons, by means of the display when using a hand control and by means of the pushers for the drill tool.

The purpose of the integrated coolant and irrigation pump is to deliver coolant or irrigation fluid to the operating field for cooling the tool and tissue or irrigating the operating field. It is activated via the buttons on the foot control or via the display.

Maximum delivery rate of the pump

65 ml/min

2.2 Application Environment

The motor system fulfills the requirements of Type BF in accordance with IEC/DIN EN 60601-1.

For use in operating rooms, outside of the explosive hazard zone (e.g. areas with highly pure oxygen or anesthetic gases).

Control unit	
Application Environment	In the unsterile area
Set-up location	Table, ceiling supply unit, equipment cart, or similar

2.3 Indications

Modes of application	Separation, removal and modeling of hard tissue, cartilage, related matter and bone substitutes
Surgical discipline/areas of application	Neurosurgery, ENT surgery, OMF surgery, orthopedics and trauma surgery

Note

Mode and area of application depend on the applied parts and tools selected.

2.4 Absolute contraindications

The product is not approved for use in the central nervous system or central circulatory system.

2.5 Relative contraindications

The safe and effective use of the product strongly depends on influences only the user can assess. For that reason, the above information only outlines general conditions.

The clinically successful use of the product depends on the knowledge and experience of the surgeon. The surgeon has to decide which structures can be properly treated taking into account the safety and warning notices listed in these instructions for use.

3. Safe handling

⚠ DANGER

Risk of death by electric shock!

- ▶ Do not open the product.
- ▶ Connect the product only to a grounded power supply

⚠ WARNING

Risk of injury and material damage due to inappropriate use of the product!

- ▶ Use the product only in accordance with the intended use.

⚠ WARNING

Risk of injury and damage to property due to improper handling of the product!

The ELAN 4 electro control unit GA800 together with the accessory forms an electronically operated motor system.

- ▶ Follow the instructions for use of ELAN 4 electro accessories.
- ▶ Follow the instructions for use of all products used.

- General risk factors associated with surgical procedures are not described in this documentation.
- It is the operating surgeon's responsibility to ensure that the surgical procedure is performed correctly.
- The operating surgeon must have a thorough understanding of both the hands-on and conceptual aspects of the established operating techniques.
- The ELAN 4 electro control unit GA800 fulfills the requirements of CISPR11, Class A.
- ▶ Inspect the new product after removing its transport packaging and prior to first use to ensure it is in good working order.
- ▶ Observe "Notes on Electromagnetic Compatibility (EMC) for ELAN 4 electro control unit GA800" TA022452, see B. Braun eIFU at eifu.bbraun.com
- ▶ To prevent damage caused by improper setup or operation, and in order not to compromise warranty and manufacturer liability:
 - Use the product only according to these instructions for use.
 - Follow the safety and maintenance instructions.
 - Only combine Aesculap products with each other.
- ▶ Ensure that the product and its accessories are operated and used only by persons with the requisite training, knowledge, or experience.
- ▶ Keep the instructions for use accessible for the user.
- ▶ Always adhere to applicable standards.
- ▶ Do not pull any of the cables by the actual cable but only by the connector.

Note

The user agrees to report all serious incidents occurring in connection with the product to the manufacturer and to the competent authority of the country in which the user is registered.

4. Product description

4.1 Scope of supply

Art. no.	Designation
GA800	ELAN 4 electro control unit
GD412804	Bottle holder
TA014401	Instructions for use of ELAN 4 electro control unit
TA014482	Supplement for ELAN 4 electro motor system

4.2 Components required for operation

- ELAN 4 electro control unit GA800
- Power cord, see Chapter 11.
- ELAN 4 electro motor cable for foot control GA806 and ELAN 4 electro foot control GA808
 - or –
- ELAN 4 electro motor cable with hand lever GA804 (from software version 2.00)
 - or –
- ELAN 4 electro motor cable with hand control GA805 (from software version 2.00)
 - or –
- ELAN 4 electro wireless foot control GA810 (software version 3.00 and higher)
- ELAN 4 electro applied part

Note

The installed software version is displayed in the system settings menu in the device information sub-item.

Contact the national B. Braun/Aesculap representative to upgrade the control unit to the current software version, see Chapter 10.

Note

The term "Applied part" comprises all handpieces and motors of the ELAN 4 electro motor system, see Chapter 11.

When using the coolant and irrigation pump:

- Bottle holder GD412804
- Single-use tube set GA395SU
- Coolant or irrigation fluid: Physiological saline solutions up to 1 000 ml

Note

The coolant/irrigation fluid is not an Aesculap accessory.

4.3 Operating principle

4.3.1 Control unit

The ELAN 4 electro control unit 1 is designed for main voltage ranges of 100 V~ to 120 V~ and 220 V~ to 240 V~ from 50 Hz to 60 Hz. The mains voltage is converted into safety extra low voltage for supplying the micro motors in the applied parts.

The control unit features two connection sockets for applied parts for connecting two different applied parts, and has one connection socket for a foot control. Only one applied part can be operated at a time.

Note

The essential performance features are defined by the "speed" and "rotational direction" specifications. The exception is a conditionally defined motor stop to a safe state caused by a detected error condition

4.3.2 Display/control concept

The display 2 indicates the current device status (operating and error status) at all times. The display is divided into the applied part control panel and the pump control panel.

The display shows the group of the applied part currently connected.

The display shows information about the speed, rotational direction, activation and delivery rate of the coolant and irrigation pump. By activating the relevant panel, the control elements appear. Settings can then be changed. If the control elements are not activated, they will disappear again after a short time.

4.3.3 Explanation with an example

Note

If two applied parts are connected to the control unit, the display is divided into 2/3 for the active applied part and 1/3 for the inactive applied part.

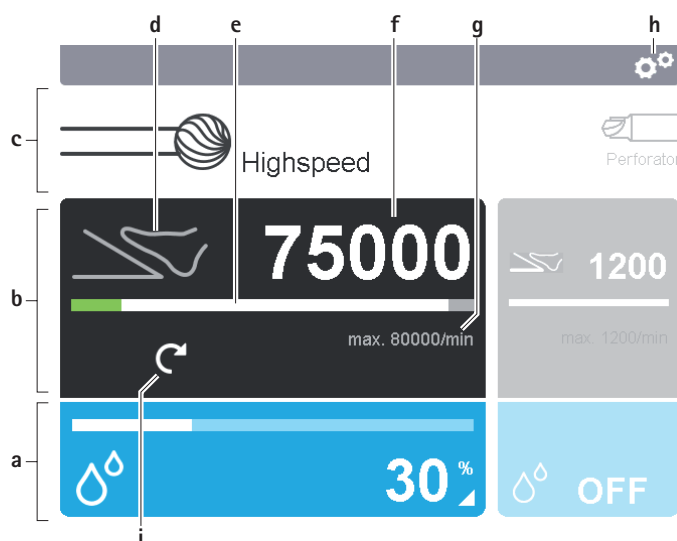
Fig. 1 Example of control concept

Legend

- a Pump control panel
Selected delivery rate % (here 30 %)
- b Applied part control panel
- c Applied part type (here ELAN 4 electro high speed motor)
- d Activation mode: (for foot control)
- e Qualitative bar display:
Display of the maximum set speed (white bar).
The actual current speed in the range from 0 to the maximum set speed (green bar)
Difference between the maximum set speed and the upper limit of the maximum speed (gray bar)
- f Maximum set speed: (here 75 000 rpm)
- g Maximum speed (uppermost limit) with this group: max. 80 000 rpm
- h Call up the system settings menu
- i Rotational direction (here clockwise rotation)

4.3.4 Applied part types

The control unit recognizes the different types of applied parts (motors and handpieces). These are shown in the display as a combination of symbol and text. The size and display depend on which socket the applied part is connected to and whether the applied part is active or locked.

4.3.5 Identification of control elements on system components

Control elements on the system components of the ELAN 4 electro motor system are labeled with a gold marking.

4.3.6 Recognition of connected motor cable and applied part

The settings (upper speed limit, pump status and delivery volume) last selected with this application part type on this bushing are retrieved upon inserting the same application part type.

When several different hand/foot controls are connected to the control unit, they are prioritized in the following order (higher items have priority over lower):

- Hand controls GA804/GA805 or drill tool GA844
- Foot control GA808
- GA810 wireless foot control

4.3.7 Overload protection

To protect the micro motors in the applied parts against damage from overheating, the motor temperature is monitored. If the temperature is too high a warning signal sounds and a thermometer symbol appears in the display **2**.

If the temperature continues to be too high, the applied part is switched off. The following message appears in the display **2**: "The motor currently being used is overheating. Please let the motor cool down or use another suitable motor".

After a cooling phase the applied part can be reactivated.

It is recommended to have a second applied part ready at hand.

4.3.8 Coolant and irrigation pump

The control unit is equipped with a coolant and irrigation pump **3**.

The pump can be activated either in the corresponding pump control panel or via the corresponding button on the foot control. It can be started by activating the motor or via the "Flush" function (continuous flushing). The delivery rate can only be adjusted via the pump control panel.

5. Preparation and setup

Non-compliance with the following instructions will preclude all responsibility and liability in this respect on the part of Aesculap.

- When setting up and operating the product, adhere to
 - national regulations for installation and operation,
 - national regulations on fire and explosion protection.

Note

For the safety of patients and users it is essential that the mains power cord and, especially, the protective earth connection are intact. In many cases defective or missing protective earth connections are not registered immediately.

- Connect the device via the potential equalization terminal at the rear panel of the device to the potential equalization system of the room used for medical purposes.

Note

The potential equalization lead can be ordered from the manufacturer as art. no. GK535 (4 m length) or TA008205 (0.8 m length).

5.1 Set-up environment/set-up location

DANGER

Fire and explosion hazard!

- Use the product outside of explosion hazard zones (e.g. areas with highly pure oxygen or anesthetic gases).

The ELAN 4 electro control unit is authorized for operation in the operating room.

Note

After being installed and put into operation, the control unit must not be transported or moved to a different set-up location.

Note

The control unit must not be placed on an Aesculap mobile stand (GA415, GA416 and GD416M).

- Make certain that the ventilation slots in the bottom of the housing and in the rear panel of the control unit are not covered, e.g. by an OR cloth.
- Make certain that the control elements, mains power switch and power socket **13** are freely accessible for the user.
- Make certain the system is set up on a sufficiently stable support (e.g. a table, ceiling support, equipment cart, etc.).
- Follow the instructions for use of the support.

5.2 Stacking of units

- Do not exceed a stack height of 415 mm.
- Place the units in a stable position.
- Aesculap devices must be stacked on top of each other facing identically.
- Never move stack devices.

6. Working with the ELAN 4 electro system

6.1 System set-up

WARNING

Risk of injury and material damage due to inadvertent activation of the applied part!

- Secure applied parts that are not actually used against inadvertent activation ("Off" position), see section on locking applied part ("Off" position).

6.1.1 Connecting the accessories

Combinations of accessories that are not mentioned in the present instructions for use may only be employed if they are specifically intended for the respective application, and if they do not compromise the performance and safety characteristics of the products.

Also note that any equipment connected at the interfaces must demonstrably meet the respective IEC standards (e.g. IEC 60950 for data processing equipment, IEC/DIN EN 60601-1 for electromedical devices).

All configurations must fulfill the basic standard IEC/DIN EN 60601-1. Any individual connecting devices with one another is responsible for such configuration and must ensure compliance with basic standard IEC/DIN EN 60601-1 or applicable national standards.

- Follow the instructions for use of individual accessories.
- In case of questions, contact your B. Braun/Aesculap partner or Aesculap Technical Service, Address see Chapter 10.

6.1.2 Connecting the power supply

DANGER

Risk of death by electric shock!

- Connect the product only to a grounded power supply.

Note

The mains voltage must correspond to the voltage indicated on the type plate at the back of the unit.

- Plug in the power cord at power cord socket 13.
- Connect power supply by inserting into a wall socket.

6.1.3 Switching on the control unit

- Activate the power ON switch 10.

The power ON indicator 9 and indicator light 5 light up.

The control unit 1 always performs a power-on self-test when the unit is switched on.

If a malfunction is detected, an error message will appear in the display 2, see system errors.

6.1.4 Switching off the control unit

- Activate the power OFF switch 8.

The power ON indicator 9, the indicator light 5 and the display with touch control panel 2 will fade out.

6.1.5 Putting out of operation

Note

The safe and all-pole disconnection of the product from the main power supply is only guaranteed when the power cord is unplugged.

- Switching off the product: Activate the power OFF switch 8.
- Unplug the power cord from the power socket 13.
The operation of the device is safely terminated.

6.1.6 Connecting the ELAN 4 electro to wireless foot control GA810

The system configuration menu is used to connect the wireless foot control to the control unit, see Chapter 6.4.4

6.1.7 Connecting the ELAN 4 electro foot control GA808 to the control unit

Note

The plug connection of the foot control has a yellow coded ring and a filled-in dot.

- Position connector of foot control c so that marking b on the connector is aligned with marking a on the connection socket for foot control 6, see Fig. 2.
- Plug the connector for foot control c firmly into the connection socket for foot control 6.

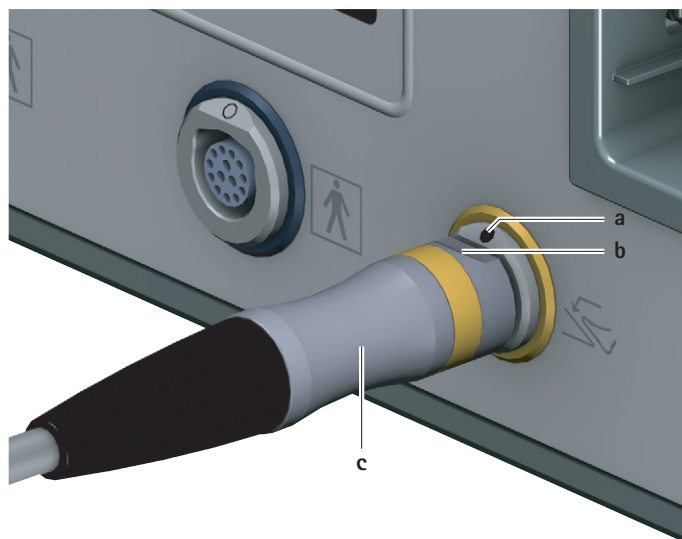
Fig. 2 Connecting the foot control

Legend

- a Marking for connection socket
- b Marking for connector
- c Connector for foot control

6.1.8 Connecting the ELAN 4 electro motor cable GA804/GA805/GA806 or ELAN 4 electro drill tool GA844 to control unit

Note

The motor cable is sterile. Sterile separation occurs at the motor cable.

Note

The plug connection of the motor cable has a blue coded ring and a hollow dot.

- Position connector for control unit **c** on the motor cable so that marking **b** on the connector is aligned with marking **a** on the connection socket for applied parts **7**, see Fig. 3.
- Plug connector for control unit **c** on the motor cable firmly into one of the two connection sockets for applied parts **7**.

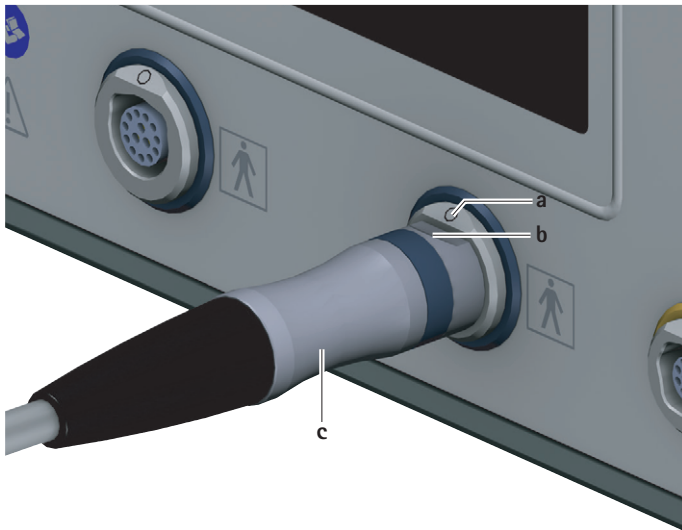
Fig. 3 Connecting the motor cable

Legend

- a** Marking for connection socket
- b** Marking for connector
- c** Connector for control unit

Note

The motor cable must be connected to the control unit without an applied part or with the applied part locked ("Off" position).

Otherwise the control unit will not recognize the applied part, and a message will appear on the display (except for drill tool GA844).

- If the applied part is not recognized;
 - Lock the applied part, see Chapter 6.1.12.
 - Release the applied part again, see Chapter 6.1.11.

6.1.9 Connecting the ELAN 4 electro single-use tube set GA395SU

Note

The tube set is sterile. Sterile separation occurs at the tube set.

- Open flap **a** of the coolant and irrigation pump **3**, see Fig. 4.
- Insert single-use tube set **c**:
 - Place pump tube with loop over the roller wheel **b**.
 - Push nib **e** of the single-use tube set under the pull slide **d** until the nib engages.
- Close flap of the coolant and irrigation pump **a**. When doing this, make certain the pump tube does not get clamped in.
- Insert the bottle holder **16** for the sterile fluid bottle in bottle holder mount **15**.
- Insert the punching spike into the sterile fluid bottle.
- When using sterile fluid bottles made from glass: Open the ventilation flap on the punching spike.
- Hang the sterile fluid bottle from the bottle holder **16**.
- Fasten the tube to the motor cable using fixation clamps.
- Shorten the length of the tubes to correspond with the applied part being used and connect to the spray nozzle.

Fig. 4 Connecting the single-use tube set

Legend

- a** Flap of the coolant and irrigation pump
- b** Roller wheel
- c** Single-use tube set
- d** Pull slide
- e** Nib of the single-use tube set

6.1.10 Connecting the applied part to the motor cable

Note

All ELAN 4 electro motor cables (GA804/GA805/GA806) can be used in combination with all applied parts that do not have a fixed connection with a cable.

- Connect connector for motor cable **24** to connector for application part **19** of the motor cable. Ensure that the nose **23** on the motor cable is aligned with the groove on the coupling of the application part. The applied part engages. When the visual field is "Off" **20** on the motor cable, a gold-colored marking is visible. The control unit **1** recognizes the applied part type and displays this type in the corresponding applied part control panel in the display **2**. The most recent settings for this applied part type at the same motor connection socket are shown on the display **2**.

Note

The applied part connected to this motor cable is only ready for operation if a gold-colored marking is visible in the visual field "On" **21** on the motor cable.

6.1.11 Releasing the applied part for operation ("On" position)

Note

The operation of the drill tool GA844 is different. Refer to drill tool instruction manual GA844 (TA014436).

- ▶ Activate the release button **22** on the motor cable and push the applied part further onto the motor cable.
The applied part engages. When the visual field is "On" **21** on the motor cable, a gold-colored marking is visible.

Note

For applied parts with slider for tool release **25**, in the "On" position **21** the nib **23** on the motor cable blocks the slider. Uncoupling of the tools is therefore only possible in "Off" position **20**.

For applied parts with unlocking sleeve **26**, in the "On" position **21** the connection for applied part **19** on the motor cable blocks the unlocking sleeve. Uncoupling of the attachments is therefore only possible in "Off" position **20**.

For applied parts without slider for tool release, it is possible to couple/uncouple tools in the "On" position, but this must not be done due to the risk of injury from unintentional activation of the applied part.

6.1.12 Locking the applied part ("Off" position)

Note

The operation of the drill tool GA844 is different. Refer to drill tool instruction manual GA844 (TA014436).

- ▶ Activate the release button **22** on the motor cable and detach the motor cable from the applied part.
The applied part engages. When the visual field is "Off" **20** on the motor cable, a gold-colored marking is visible.

6.1.13 Disconnecting the applied part from the motor cable

- ▶ Activate the release button **22** on the motor cable and pull at the connection for applied part **19** to disconnect the motor cable from the applied part.

6.1.14 Disconnecting the ELAN 4 electro foot control GA808 from the control unit

- ▶ Unplug connector for foot control **c** from the connection socket for foot control **6**, see Fig. 2.

6.1.15 Disconnecting the ELAN 4 electro motor cable GA804/GA805/GA806 or ELAN 4 electro drill tool GA844 from control unit

- ▶ Unplug the motor cable on the connector for control unit **c** from the connection socket for applied parts **7**, see Fig. 3.

6.2 Function checks

- ▶ Prior to each use and after every change of applied part, check that all products to be used are in good working order.
- ▶ Check the secure connection of all products to be used.
- ▶ Ensure that the parameters are set and the unit is operated according to the instructions for use and the safety information for the applied parts or tools.
- ▶ Check that the cutting edges of the tools are not showing any mechanical damage.
- ▶ Check that the correct applied part type for the applied parts connected is showing in the corresponding applied part control panel in the display.
- ▶ Do not use the product if it is damaged or defective. Set aside the product if it is damaged.
- ▶ Release the applied part for operation.
- ▶ Activate the foot or hand control down to the limit stop.
The applied part starts running in the preselected rotational direction and reaches the maximum speed as indicated in the applied part control panel in the display.
The applied part runs quietly at a constant speed.
The qualitative bar display for the actual current applied part speed in the applied part control panel is fully illuminated.
- ▶ If necessary, release the coolant and irrigation pump in the corresponding applied part control panel or by pressing the function button on the foot control.
The coolant and irrigation pump starts as soon as the applied part is operated.

6.3 Safe operation

⚠ WARNING

Risk of injury and/or malfunction!

- ▶ Always carry out a function check prior to using the product.

⚠ WARNING

Risk of injury and material damage due to inadvertent activation when repositioning/moving the foot control!

- ▶ To reposition the foot control: Use a transport brace.
- ▶ Prior to repositioning: Secure the applied part against inadvertent activation ("Off" position).

It is only possible to operate the applied part and change the setting parameters on the control unit if:

- the applied part is connected to the control unit by a motor cable,
- a foot control is connected to the connection socket for foot control **6** (when using the ELAN 4 electro motor cable GA806),
- there is no second applied part released at the same time and
- the applied part type is shown on the display of the control unit.

The following motor settings of the applied part cannot be changed when the applied part is running:

- Rotational direction (except for GA844, via pusher for rotational direction)
- Maximum speed

6.3.1 Activating the motor/pump control panel

Note

Motor settings of applied parts that are not connected to a cable cannot be changed while the applied part is running.

- ▶ Changing the setting parameters of the applied part: Activate the applied part control panel **b** in the display 2, see Fig. 5.
- ▶ Changing the setting parameters of the coolant and irrigation pump: Activate the pump control panel **a** in the display 2.

The activated control panel switches to the Settings mode. Now the setting parameters listed below can be changed.

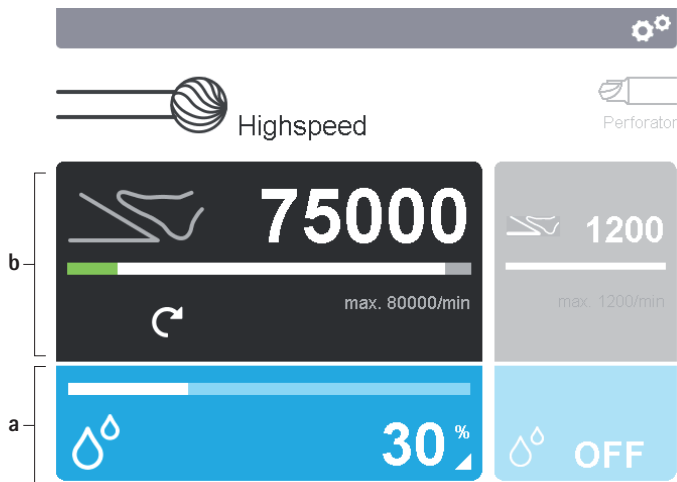
Fig. 5 Activating the control panel

Legend

- a Pump control panel
- b Applied part control panel

6.3.2 Changing the maximum speed setting/rotational direction of the applied part

- ▶ Activate the applied part control panel, see Chapter 6.3.1.
- ▶ Change direction of rotation (except for drill tool GA844): Actuate the inactive, grey clockwise/counterclockwise rotation button **a/b**, see Fig. 6.

The rotational direction is changed from clockwise rotation to counterclockwise rotation and vice versa.

- ▶ Change speed range limit: Press the buttons to reduce/increase the speed range limit **c/d** or the speed range limit switch **31** on the wireless foot switch.

The maximum speed setting is reduced/increased by increments.

Note

The increments for changing the maximum speed setting depend on the applied part connected.

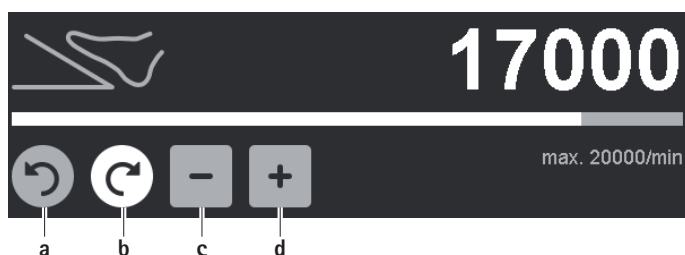
Fig. 6 Changing the maximum speed setting/rotational direction

Legend

- a Rotational direction button for counterclockwise rotation
- b Rotational direction button for clockwise rotation
- c Button for reducing the maximum speed setting
- d Button for increasing the maximum speed setting

6.3.3 Activating the coolant and irrigation pump/"Flush" function (continuous flushing)

- ▶ Activate the pump control panel, see Chapter 6.3.1.
- ▶ Activate the coolant and irrigation pump: Activate the button for switching on the irrigation pump **a**, see Fig. 7.
The coolant and irrigation pump is active and delivers coolant or irrigation fluid at the currently set delivery rate.
- ▶ Activate the "Flush" function: Activate and hold down the "Flush" button **b**.
The "Flush" function is now active. The coolant and irrigation pump delivers coolant or irrigation fluid at the maximum delivery rate until the "Flush" button **b** is released.

Note

The activation of the "Flush" function does not depend on whether an applied part is connected or activated.

Fig. 7 Activating the coolant and irrigation pump/"Flush" function

Legend

- a Button for switching the coolant and irrigation pump on/off
- b "Flush" button
- c Button for reducing the delivery rate
- d Button for increasing the delivery rate

6.3.4 Deactivating the coolant and irrigation pump and changing the delivery rate

Note

The delivery rate of the coolant and irrigation pump can only be changed when the irrigation pump is activated.

- ▶ Activate the pump control panel, see Chapter 6.3.1.
- ▶ Deactivate the coolant and irrigation pump: Activate the button for switching off the irrigation pump **a**, see Fig. 7.
The coolant and irrigation pump is inactive and no longer delivers coolant.
- ▶ Change the delivery rate: Activate the buttons for reducing/increasing the delivery rate **c/d**.
The delivery rate of the coolant and irrigation pump is reduced/increased by increments.

The delivery rate can be adjusted in the following increments:

- 1 % to 5 %: 1 % increments
- 5 % to 100 %: 5 % increments

6.3.5 Enable application part using foot control GA808 or wireless foot control GA810

Note

Applied parts connected to a hand control cable cannot be operated via the foot control.

Note

When an applied part is plugged in, clockwise rotation is always pre-set.

To activate clockwise rotation:

- Set the rotational direction button for motor **30** to clockwise rotation. The motor direction indicator for clockwise rotation is illuminated in the applied part control panel.
- Activate the pedal **29**. The applied part runs clockwise.

To activate counterclockwise:

- Set the rotational direction button for motor **30** to counterclockwise rotation. The motor direction indicator for counterclockwise rotation is illuminated in the applied part control panel.
- Activate the pedal **29**. The applied part runs counterclockwise. The control unit emits an acoustic signal.

Set rotary speed range limit (for wireless foot control GA810):

- Actuate the switch for speed range limit **31**.

6.3.6 Activating the applied part via hand control GA804/GA805

Note

When an applied part is plugged in, clockwise rotation is always pre-set.

To activate clockwise rotation:

- Set the motor direction to clockwise rotation on the control unit. The motor direction indicator for clockwise rotation is illuminated in the applied part control panel.
- Activate the control element of the hand control. The applied part rotates clockwise.

To activate counterclockwise rotation:

- Set the motor direction to counterclockwise rotation on the control unit. The motor direction indicator for counterclockwise rotation is illuminated in the applied part control panel.
- Activate the control element of the hand control. The applied part rotates counterclockwise. The control unit emits an acoustic signal.

6.3.7 Activating the coolant pump via foot control

- Switch the coolant and irrigation pump on/off: Briefly activate function button **28**.
- Activate the "Flush" function (continuous flushing): Activate the function button **28** for a longer period and keep pressed down.

6.4 System settings menu

Note

The system settings menu can only be called up when no applied part is being operated.

While the system settings menu is active, the operation of the applied part is blocked.

- Call up the system settings menu: Activate the button for the system settings menu **h**, see Chapter Fig. 1. The system settings menu opens, see Fig. 8.

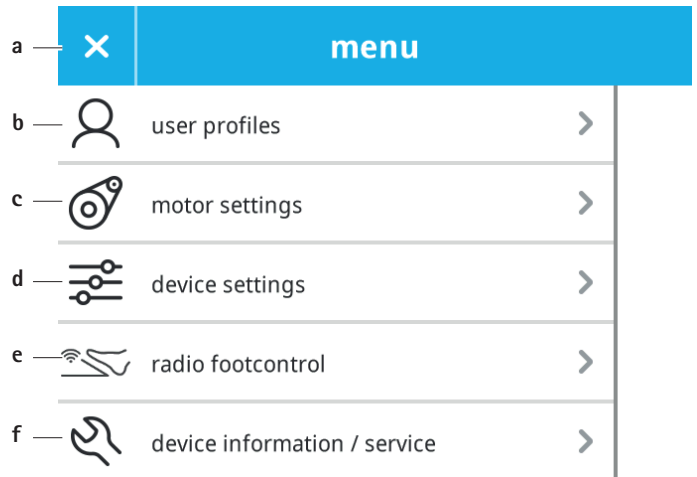
Fig. 8 System settings menu

Legend

- a "Exit system settings" button
- b "user profiles" button
- c "motor settings" button
- d "device settings" button
- e "radio footcontrol" button
- f "device information" button

Menu	Description
user profiles	Manage user profiles
motor settings	Display and change the settings for the individual applied part types
device settings	Display and change the basic settings for the control unit
radio footcontrol	Connect/disconnect the GA810 wireless foot control
device information	Display information about the control unit

- Call up menu: Activate the menu button.
- Exit system configuration menu: Press "Exit configuration menu" **a** button.

6.4.1 User profiles

User profiles can be created and edited in the user profiles menu.

Note

When the control unit is restarted, the default profile is loaded automatically.

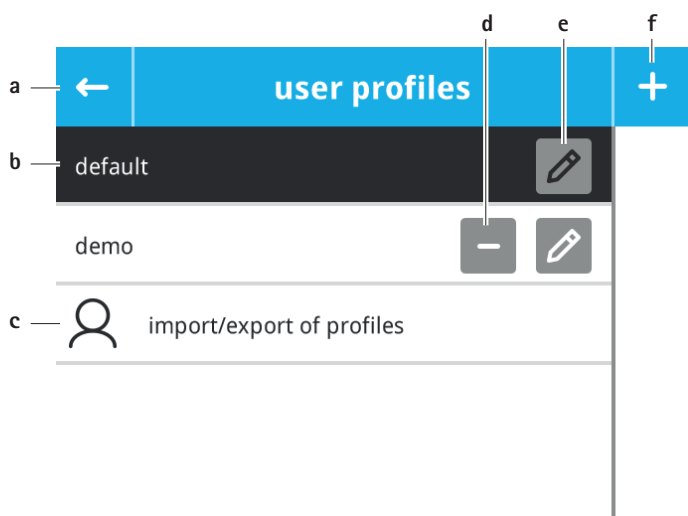
Fig. 9 User profiles

Legend

- a Exit menu
- b User profile button
- c Import/export user profiles from/to USB stick
- d Delete user profile
- e Modify user profile
- f Create user profile

- Exit menu: Press "Exit menu" **a** button.
- Select user profile: Press the user profile **b** button.
The active user profile is highlighted in black.
- Create user profile:
 - Press button "Create user profile" **f**.
 - Enter the name of the user profile.
 The settings below can be made for every user profile:
 - Motor settings
 - Device settings
 - Oscillation settings (only for drill tool GA844)
- Modify user profile: Press button "Modify user profile" **e**.
- Delete user profile: Press button "Delete user profile" **d**.
- Import/export user profiles from/to USB stick:
 - Inset USB stick into USB port of the control unit.
 - Press button "import/export of profiles" **c**.
 - Press button "import" or "export".
 - Follow the instructions on the display.

Note

Profiles must be saved after creation or modification.

- Save Profile: Press button

6.4.2 Motor settings

The motor settings menu displays the applied part types, see Fig. 10.

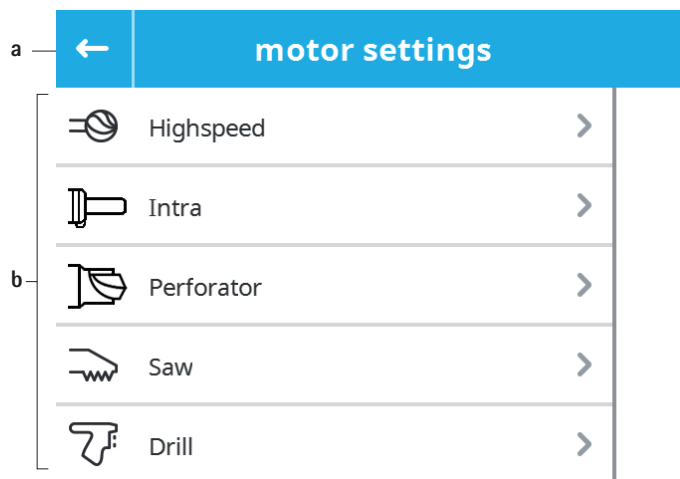
Fig. 10 Motor settings – Overview of applied part types

Legend

- a Exit menu
- b Buttons for the types of applied parts

- Exit menu: Press "Exit menu" **a** button.
- View/change the motor settings of a type of applied part: Press the type of applied part **b** button.
The submenu for the applied part type selected opens, see Fig. 11.

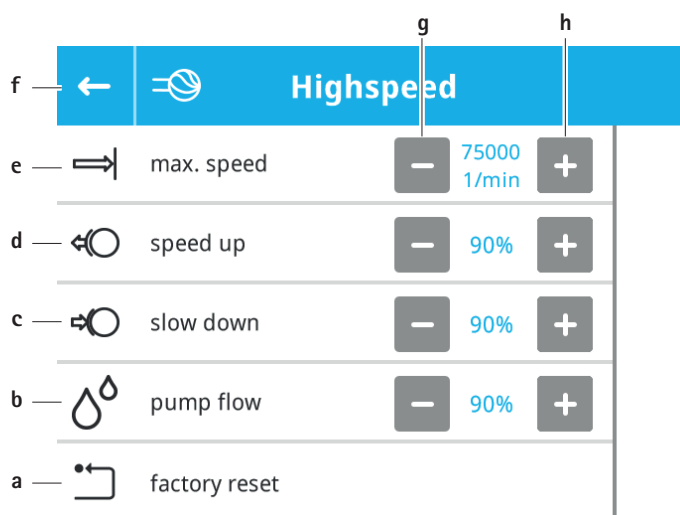
Fig. 11 Motor settings – Selected applied part type

Legend

- a Factory reset
- b Flow rate of the coolant and irrigation pump
- c Slow down rate of the applied part
- d Speed up rate of the applied part
- e Maximum speed (maximum speed/stroke rate)
- f Leave menu
- g Reduce value
- h Increase value

- Leave menu: Press button "Leave menu" **f**.
- Change motor settings: Press button "Reduce value" **h** or "Increase value" **g**.
The change is implemented immediately. The set value is displayed.
- Reset motor settings to default settings
 - Press button "Factory reset" **a**.
 - Confirm message.

Drill tool GA844

The settings below are additionally available for the drill tool GA844:

Setting	Description
Oscillation mode	Determines whether the oscillation function on the drill can be activated.
Oscillation angle	Total oscillation angle
Thread cutting operation angle left	Thread cutting operation angle counter-clockwise
Thread cutting operation angle right	Thread cutting operation angle clockwise

6.4.3 Device settings

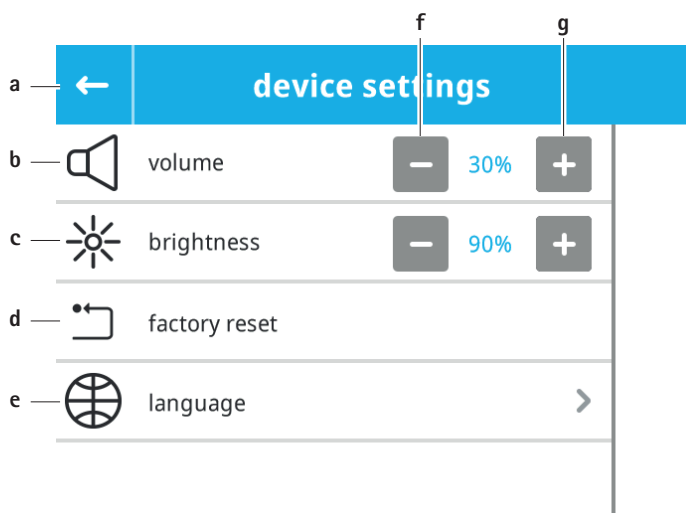
Fig. 12 Device settings

Legend

- a** Exit menu
 - b** Set system volume
 - c** Setting display brightness
 - d** Reset system volume and brightness to default settings
 - e** Set system language
 - f** Reduce value
 - g** Increase value
- Exit menu: Press Exit menu **a** button.
 - Change system volume/brightness: Press button "Increase value" **g** or "Reduce value" **f**.
The change is implemented immediately. The set value is displayed.
 - Set system language:
 - Press button "language" **e**.
 - Select desired language.
 - Reset system volume and brightness to default settings
 - Press button "factory reset" **a**.
 - Confirm message.

6.4.4 Wireless foot control

The wireless foot control GA810 can be connected to the control unit or disconnected from the control unit via the menu "radio footcontrol".

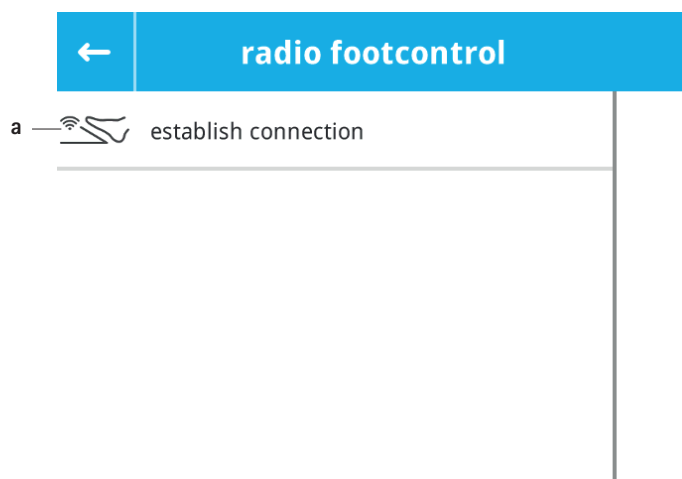
Fig. 13 Wireless foot control settings

Legend

- a** Connect wireless foot control (or disconnect if a wireless foot control is connected)
- Connect the GA810 wireless foot control:
 - Select Connect to GA810 button and
 - follow the instructions on the display. Compare the serial number in the control unit display to the
 - serial number of the wireless foot control and select "Yes" to confirm the message.
 - Disconnect the GA810 wireless foot control
 - by pressing the "Disconnecting from GA810" button
 - or –
 - switch off the control unit for approx. 10 min.

6.4.5 Device information

The "device information" menu displays general information about the device and the device software.

- Leave the menu: Activate the "Leave menu" **e** button.

7. Reprocessing procedure

7.1 General safety notes

Note

Adhere to national statutory regulations, national and international standards and directives, and local, clinical hygiene instructions for reprocessing.

Note

For patients with Creutzfeldt-Jakob disease (CJD), suspected CJD, or possible variants of CJD, observe the relevant national regulations concerning the reprocessing of products.

Note

It should be noted that successful reprocessing of this medical device can only be guaranteed following prior validation of the reprocessing method. The operator/reprocessing technician is responsible for this.

7.2 Reusable products

The service life of the product is limited by damage, normal wear, type and duration of the application, handling, storage and transportation of the product.

Careful visual and functional testing prior to next use is the best way to identify a malfunctioning product.

7.3 Preparation before cleaning

- ▶ Separate the products from each other immediately after use.
- ▶ Shut down control unit 1 immediately after use, see Chapter 6.1.5.
- ▶ Remove any visible surgical residues as much as possible with a damp, lint-free cloth.

7.4 Cleaning/disinfection

7.4.1 Product-specific safety instructions for the reprocessing procedure

⚠ DANGER

Risk of electric shock and fire hazard!

- ▶ Unplug the device before cleaning.
- ▶ Do not use flammable or explosive cleaning or disinfecting solutions.
- ▶ Ensure that no fluids will penetrate the product.

⚠ CAUTION

Damage to, or destruction of the product caused by mechanical cleaning/disinfection!

- ▶ Clean the product with wipe disinfection only.
- ▶ Do not sterilize the product under any circumstances.

⚠ CAUTION

Damage to the product due to inappropriate cleaning/disinfecting agents!

- ▶ Only use cleaning/disinfecting agents approved for surface cleaning. Follow the manufacturer's instructions for the respective cleaning/disinfecting agent.
- ▶ Do not clean the product in a ultrasonic bath and do not immerse the product in any fluids.

7.5 Wipe disinfection for electrical equipment

Phase	Step	T [°C/°F]	t [min]	Conc. [%]	Water quality	Chemicals
I	Cleaning	RT	1	-	-	17 % Propan-1-ol, 0.23 % Didecyltrimethylammonium chloride*
II	Wipe disinfection	RT	≥ 1	-	-	17 % Propan-1-ol, 0.23 % Didecyltrimethylammonium chloride*

RT: Room temperature

* Recommended: Meliseptol® wipes sensitive (B. Braun)

Phase I

- ▶ Remove any visible residues with a disposable disinfectant wipe.

Phase II

- ▶ Wipe all surfaces of the optically clean product with a fresh, disposable disinfectant wipe.
- ▶ Observe contact time requirements (at least 1 min).

7.6 Inspection, maintenance and checks

- ▶ Inspect the product after each cleaning and disinfecting cycle to be sure it is: clean, functional, and undamaged.
- ▶ Set aside the product if it is damaged.

8. Maintenance

To ensure reliable operation, the product must be maintained at least once a year.

Contact your national B. Braun/Aesculap representative if you need these services, see Chapter 10.

9. Troubleshooting list

- Have defective products repaired by Aesculap Technical Service, see Chapter 10.

9.1 Display error messages

Faults that are recognized by the control unit are shown as error messages in the display.

There are three types of error messages:

- System error (text in red field): It is not possible to work with the control unit or system.
- Accessory error (text in yellow field): It is possible to work with a different component.
- Operating error (text in blue field): It is possible to work with the system after rectifying the cause of the error.

Note

Many faults cannot be clearly categorized. They can be caused by an operating error or an accessory error. In such cases, an operating error is assumed first, to avoid any unnecessary changing or sending in of new products.

9.1.1 System error

Message displayed	Cause	Remedy
System error Switch the control unit off and on. If the error re-occurs, replace the control unit.	The internal monitoring of the control unit recognizes an error or fault.	Switch the control unit off and on again. If the message appears again: Replace the control unit.

9.1.2 Accessory error

Message displayed	Cause	Remedy
Motor or motor cable could be damaged	Motor cable or applied part defective	Replace motor cable or applied part.
Malfunction of foot control If this message is displayed frequently, please perform a technical inspection on the foot control	Foot control defective	Replace foot control.
Critical Foot Control Error Unexpected error occurred in the area of the foot control. If this message does not disappear automatically, please restart the device.	Foot control defective	Replace foot control.
Critical error in manual control module Unexpected error occurred in the area of the hand control. If this occurs repeatedly, please replace the hand control.	Unexpected error in the area of the hand control	Restart device
	Hand control defective	Replace hand control.
Prohibited combination of accessories A prohibited combination of motor and cable coding was found on socket 1/socket 2.	A prohibited combination of application part and motor cable is connected to the control unit.	Check inserted products and replace with a permissible combination of application part and motor cable as applicable.
Motor not recognized It was impossible to recognize the motor type. Please replace the motor. If this does not work, replace the motor cable as well.	Wrong applied part Applied part defective	Change applied part.
	Motor cable defective	Replace motor cable.
Unknown motor type Please verify that the software version of the device is suitable for this motor type.	Software version of the control unit is incompatible with the application part used.	Verify software version of the control unit. Have control unit software updated by Technical Service.
Motor overheated The motor currently being used is overheating. Please let the motor cool down or use another suitable motor.	Motor of the applied part has overheated	Allow the applied part to cool down. If the applied part overheats excessively: Replace the applied part.
Motor blocked Stop motor actuation and release blockage If the error re-occurs, replace the product.	Applied part blocked	Stop applied part actuation and release blockage. If the error occurs when the applied part is running idle: Replace the applied part.

9.1.3 Operating error

Message displayed	Cause	Remedy
Motor turned off in operation. The motor was turned to the off-position n operation. Stop the engine before turning off the engine	The motor was turned to the off-position n operation.	Stop the engine before turning off the engine
Motor not recognized Switch the motor to "Off" position and then back to "On" position	Applied part connected to control unit in "On" position	Lock the applied part ("Off" position). The control unit recognizes the applied part type. In order to work: Release the applied part ("On" position).
Two motors simultaneously in on position Please move one of the motors to the Off position.	Foot control is activated while two applied parts are released on the motor cable ("On" position) Note: Only one applied part can be operated at a time.	Only release the applied part intended to be operated ("On" position). Lock the applied part which isn't to be operated ("Off" position).
Activation in off-position Activation of the motor in Off position Switch the motor into the On position before activation.	Foot control is activated while the applied part is locked on the motor cable ("Off" position)	Release the applied part ("On" position).
Activation of foot control without motor Connect a motor to the control unit.	Foot control is activated while the applied part is not connected to the control unit	Connect the motor cable to the control unit. Connect the applied part to the motor cable.
Activation of foot control during initialization The pedal or a foot control button was activated during initialization. Please release pedal and buttons. If this message still does not disappear, please subject the foot control to a technical check.	Pedal or button on the foot control is activated during automatic self-test.	Release the activation. If the pedal or button is not activated, the foot control is defective. If necessary, replace foot control.
Multiple control elements of the foot control activated Multiple control elements were activated simultaneously. Please release all control elements. If this message does not disappear, please have the foot control inspected.	Multiple control elements were activated simultaneously.	Release all foot control operating elements. If this message continues to appear: foot control must undergo a technical inspection.
Extended button activation on foot control A foot control button was activated for longer than 20 seconds without interruption. Please release all buttons. If this message does not disappear, please have the foot control inspected.	A foot control operating element was activated for longer than 20 seconds without interruption.	Release all foot control operating elements. If this message continues to appear: foot control must undergo a technical inspection.
Activation of manual control without motor Connect a motor to the control unit.	Manual control was activated without an application part being connected to the control unit.	Connect application part to control unit.
Manual control activated upon connecting Please move the manual control lever/slider to an unactivated position one time.	Manual control lever/slider was activated during connection.	Move manual control level/slider to an unactivated position one time.
Touchscreen timeout The touchscreen was activated for longer than 20 seconds without interruption. Please release the touchscreen. If this message continues to display, the device may be damaged.	The touchscreen was activated for longer than 20 seconds without interruption.	Release touchscreen. If this message continues to display, the control unit may be damaged. Replace the control unit.

9.1.4 Other faults in control unit

Malfunction	Cause	Detection	Remedy
Control unit cannot be switched on.	No power to control unit	Control unit not connected to mains power or not switched on (power ON indicator dark, black display)	Connect control unit to the main power. Switch on the control unit.
	Fuses blown	Power ON indicator does not light up, black display	Replace the fuses.
No coolant/irrigation fluid flow	Coolant/irrigation fluid container empty	Coolant/irrigation fluid container empty	Replace coolant/irrigation fluid container
	Tubing inserted incorrectly	Tubing inserted incorrectly	Insert tubing correctly.
	Tubing leaking	Coolant/irrigation fluid is leaking out	Replace tubing.
	Spray nozzle blocked	Coolant and irrigation pump running. Coolant/irrigation fluid is not being delivered.	Replace spray nozzle.
	Coolant and irrigation pump motor defective	Coolant and irrigation pump not running.	Replace the control unit.

9.1.5 Faults when working with the applied part

Malfunction	Cause	Detection	Remedy
Applied part not recognized	Cable not connected to control unit	Cable not connected to control unit	Connect the cable to the control unit. Connect the applied part to the motor cable.
	Cable defective	Cable defective	Replace motor cable.
Disconnecting/connecting the applied part from/with the motor cable not possible (change from "On" to "Off" position and vice versa)	Release button on the motor cable is not released between connecting/disconnecting and releasing/locking the applied part	Coupling processes cannot be performed	Release the release button on the motor cable between connecting/disconnecting and releasing/locking the applied part and activate again.
	Motor coupling defective		Change the applied part or motor cable.
Tool cannot be coupled/decoupled	ELAN 4 electro High speed: Automatic tool lock blocked	High speed tool doesn't engage	Activate slider for tool release and hold, then couple tool.
	ELAN 4 electro High speed: Slider for tool release locked	Applied part released for operation ("On" position)	Lock the applied part ("Off" position).
	Incompatible tool	Wrong tool	Select an appropriate tool according to the instructions for use of the applied part.
	Tool connection or coupling deformed, defective	Tool is difficult to couple or cannot be coupled	Use a new tool. Replace the applied part.
Attachment for GA849 (craniotome) cannot be coupled or uncoupled	Automatic attachment lock blocked	Attachment doesn't engage	Pull back unlocking sleeve and hold, then couple the attachment.
	Unlocking sleeve blocked	Applied part released for operation ("On" position)	Lock the applied part ("Off" position).
	Attachment coupling defective	Attachment is difficult to couple or cannot be coupled	Use a new attachment. Replace the applied part.
	Attachment coupling is dirty		Clean the attachment or use a new attachment. Clean the applied part.

Malfunction	Cause	Detection	Remedy
Pivotable dura protection GB947R is difficult to turn	Pivotable dura protection is difficult to move	Bearing dirty or worn	Follow the instructions for use (TA014438/TA014439) (reprocessing, maintenance). Change turnable dura guard.
Loud noise coming from applied part	Loud, irregular noise	Gearings/ball bearings defective Ball bearings of attachments defective	Change applied part. Preventive measure: Ensure that the applied part is lubricated at regular intervals.
High speed applied part vibrates excessively	Shaft of applied part bent Applied part defective Reprocessing carried out incorrectly	Loud running noise / vibrations	Change applied part. Follow instructions for use of the applied part (reprocessing, maintenance)
Applied part is getting too hot	Blunt tool Applied part defective Excessive use Reprocessing/maintenance carried out incorrectly Shaft of applied part bent Accessory defective	Tool overheats The applied part becomes hot Loud running noise Vibration	Change tool. Change accessory. Preventive: Oil the applied part regularly and observe the instructions for use of the accessory. Follow instructions for use of the applied part (interval operation). Follow instructions for use of the applied part (reprocessing, maintenance) Change applied part.
Applied part not running	Applied part defective Foot control defective Applied part in "Off" position (for GA844) Hand control defective	Tool not moving Pedal not moving Safety catch is in "Off" position Slider/hand lever not moving	Change applied part. Change foot control. Switch safety catch to "On" position. Change hand control.
Insufficient power	Blunt tool Applied part operated in counterclockwise rotation No central alignment of the craniotomy reamer to the dura protector bracket Applied part defective Accessory defective	Cutting edge worn Cutting edges coated with substances, e.g. from too little rinsing Serrated tool run in counterclockwise rotation mode Dura protector bracket bent Poor progress during craniotomy Insufficient power of applied part Steep rise in temperature after a short period of time Large amount of heat generated after a short time	Change tool. Operate serrated tool in clockwise rotation. Follow instructions for use (TA014438/TA014439). Change dura guard Follow instructions for use of the applied part (reprocessing, maintenance) Change applied part. Observe instructions for use of the accessory (reprocessing, maintenance). Change accessory.
Rinsing adapter not adaptable	Rinsing adapter not compatible Port on applied part deformed Rinsing adapter deformed/defective	Rinsing adapter not adaptable	Use matching rinsing adapter Change applied part. Replace the rinsing adapter.

Malfunction	Cause	Detection	Remedy
Oil spray adapter cannot be plugged in	Oil spray adapter not compatible	Oil spray adapter cannot be plugged in	Use the matching oil spray adapter.
Counterclockwise rotation pusher cannot be pressed (for GA844).	Counterclockwise rotation pusher jammed/defective	Counterclockwise rotation pusher cannot be actuated.	Change applied part.
Speed control safety catch cannot be actuated (for GA844)	Applied part in secured off position	Safety catch is in "Off" position	Switch the safety catch to "On" position.
	Speed control safety catch jammed/defective	Speed control safety catch cannot be actuated	Replace the applied part.

9.2 Fuse replacement

DANGER

Risk of death by electric shock!

- Unplug the device before changing the fuses!

Specified fuse set: 2 units IEC 127 – T 6.3 A switching capacity H (1 500 A at 250 V/50 Hz)

- Use a small screwdriver to release the clip on the fuse holder **12**.
- Pull out the fuse holder **12**.
- Replace both fuses.
- Reinsert the fuse holder **12** in such a way that it audibly snaps into place.

Note

If the fuses regularly blow, the unit is defective and has to be repaired. see Chapter 10.

10. Technical Service

DANGER

Danger to life of patients and users in case of malfunctions and/or failure of protective measures!

- Do not perform any servicing or maintenance work under any circumstances while the product is being used on a patient.

CAUTION

Modifications to medical equipment may result in loss of guarantee/warranty claims and any approvals.

- Do not modify the product.
- For service and repairs, please contact your national B. Braun/Aesculap representative.

Service addresses

Aesculap Technischer Service

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78532 Tuttlingen / Germany

Phone: +49 (7461) 95 -1601

Fax: +49 (7461) 14 -939

E-Mail: ats@aesculap.de

Other service addresses can be obtained from the address indicated above.

11. Accessories/Spare parts

11.1 ELAN 4 electro motor cables, applied parts and foot controls

Art. no.	Designation
GA804	ELAN 4 electro motor cable with hand lever (from software version 2.00)
GA805	ELAN 4 electro motor cable with hand control (from software version 2.00)
GA806	ELAN 4 electro motor cable for foot control
GA808	ELAN 4 electro foot control
GA810	ELAN 4 electro wireless foot control (from software version 3.00)
GA822	ELAN 4 electro perforator driver
GA824	ELAN 4 electro Lowspeed motor with intra coupling
GA831	ELAN 4 electro sagittal saw
GA832	ELAN 4 electro reciprocating saw
GA833	ELAN 4 electro transverse saw
GA836	ELAN 4 electro micro sagittal saw
GA837	ELAN 4 electro micro reciprocating saw
GA844	ELAN 4 electro drill tool (from software version 3.00)
GA849	ELAN 4 electro craniotome and multifunctional handpiece (2-ring)
GA860	ELAN 4 electro MIS hand piece
GA861	ELAN 4 electro Standard handpiece (1-ring) L4
GA862	ELAN 4 electro Standard handpiece (1-ring) L7
GA863	ELAN 4 electro Standard handpiece (1-ring) L10
GA864	ELAN 4 electro Standard handpiece (1-ring) L13
GA865	ELAN 4 electro Standard handpiece (2-ring) L7
GA866	ELAN 4 electro Standard handpiece (2-ring) L10
GA867	ELAN 4 electro Standard handpiece (2-ring) L13
GA868	ELAN 4 electro Standard handpiece (2-ring) L17
GA869	ELAN 4 electro Standard handpiece (2-ring) L22
TA014401	Instructions for use of ELAN 4 electro control unit GA800 (A4 for collection binder)

11.2 Coolant and irrigation pump

Art. no.	Designation
GA395SU	ELAN 4 electro single-use tube set
GD412804	Bottle holder
–	Physiological saline solutions up to 1 000 ml Note: No Aesculap accessories

11.3 Power cord

Art. no.	Approvals	Color	Length
TE780	Europe	black	1.5 m
TE730	Europe	black	5 m
TE734	Great Britain	black	5 m
TE735	USA, Canada, Japan	gray	3.5 m

11.4 Equipotential bonding leads

Art. no.	Designation
GK535	Equipotential bonding lead (4 m)
TA008205	Equipotential bonding lead (0.8 m)

11.5 Spare parts

Art. no.	Designation
TA021473	Fuse: Fusible cartridge T 6.3 AH

12. Technical data

12.1 Classification according to Directive 93/42/EEC

Art. no.	Designation	Class
GA800	ELAN 4 electro control unit	Ila
GA804	ELAN 4 electro motor cable with hand lever	I
GA805	ELAN 4 electro motor cable with hand control	I
GA806	ELAN 4 electro motor cable for foot control	I
GA808	ELAN 4 electro foot control	I
GA810	ELAN 4 electro wireless foot control	I
GA822	ELAN 4 electro perforator driver	Ila
GA824	ELAN 4 electro Low-speed motor with intra coupling	Ila
GA831	ELAN 4 electro sagittal saw	Ila
GA832	ELAN 4 electro reciprocating saw	Ila
GA833	ELAN 4 electro transverse saw	Ila
GA836	ELAN 4 electro micro sagittal saw	Ila
GA837	ELAN 4 electro micro reciprocating saw	Ila
GA844	ELAN 4 electro power drill	Ila

Art. no.	Designation	Class
GA849	ELAN 4 electro craniotome and multifunctional handpiece (2-ring)	Ila
GA860	ELAN 4 electro MIS hand piece	Ila
GA861	ELAN 4 electro Standard handpiece (1-ring) L4	Ila
GA862	ELAN 4 electro Standard handpiece (1-ring) L7	Ila
GA863	ELAN 4 electro Standard handpiece (1-ring) L10	Ila
GA864	ELAN 4 electro Standard handpiece (1-ring) L13	Ila
GA865	ELAN 4 electro Standard handpiece (2-ring) L7	Ila
GA866	ELAN 4 electro Standard handpiece (2-ring) L10	Ila
GA867	ELAN 4 electro Standard handpiece (2-ring) L13	Ila
GA868	ELAN 4 electro Standard handpiece (2-ring) L17	Ila
GA869	ELAN 4 electro Standard handpiece (2-ring) L22	Ila
GA395SU	ELAN 4 electro single-use tube set	Ila

12.2 Performance data, information about standards

Protection class (acc. to IEC/DIN EN 60601-1)	I
Housing protection rating according to IEC/DIN EN 60529	IP20
Applied part	Type BF
Mains voltage ranges	100 V~ to 120 V~ ±10 % 220 V~ to 240 V~ ±10 %
Current consumption (ready for operation)	0.2 A (at 100 V~ to 120 V~) 0.3 A (at 220 V~ to 240 V~)
Current consumption (maximum load)	5.4 A to 4.4 A (at 100 V~ to 120 V~) 2.3 A to 2.2 A (at 220 V~ to 240 V~)
Frequency	50 Hz to 60 Hz
Operating mode	Continuous operation
Device fuse (acc. to IEC 60127-1)	T 6.3 AH, 250 V Form factor: 5 x 20 mm
Maximum delivery rate of the coolant and irrigation pump	65 ml/min ± 15 %
Weight	9.5 kg ± 10 %
Dimensions (L x W x H)	380 mm x 330 mm x 201 mm ± 5 %
Dimensions (L x W x H) with bottle holder	380 mm x 379 mm x 427 mm ± 5 %
Conforming to standard	IEC/DIN EN 60601-1
EMC	IEC/DIN EN 60601-1-2
CISPR11	Class A
ISM band	2403.0 MHz to 2480.0 MHz
Radio standards	ETSI EN 300 440 ETSI EN 301 489-1 ETSI EN 301 489-17 IEC/DIN EN 62311

12.3 Ambient conditions

	Operation	Storage and transport
Temperature	10 °C to 40 °C	-10 °C to 50 °C
Relative humidity	30 % to 75 %	10 % to 90 %
Atmospheric pressure	700 hPa to 1 060 hPa	500 hPa to 1 060 hPa

13. Disposal

WARNING

Risk of infection due to contaminated products!

- Adhere to national regulations when disposing of or recycling the product, its components and its packaging.

Note

The operator has to treat the product before disposal, see Chapter 7.

The recycling pass can be downloaded from the Extranet as a PDF document under the respective article number. (The recycling pass includes disassembling instructions for the product, as well as information for proper disposal of components harmful to the environment.)

Products carrying this symbol are subject to separate collection of electrical and electronic devices. Within the European Union, disposal is taken care of by the manufacturer as a free-of-charge service.

- In case of questions regarding the disposal of the product, contact your local B. Braun/Aesculap representative, see Chapter 10.

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